

## Instructions for service on Invacare walking frames

The following check-ups and controls are to be done when maintaining Invacare walking frames.

### FOLDING MECHANISM

	Examine that/if:	To do:	Comments:	OK
1	The walking frame locks and stays in the unfolded position.	Replace the folding mechanism. Otherwise scrap the product.		

### HAND GRIPS

	Examine that/if:	To do:	Comments:	OK
1	The hand grips are not damaged.	When hand grips are available as a spare part, replace the hand grips. Otherwise scrap the product.		
2	The hand grips can not be easily turned.	When hand grips are available as a spare part, replace the hand grips. Otherwise scrap the product.		

### FRAME

	Examine that/if:	To do:	Comments:	OK
1	There are no mechanical damages or rust on the frame.	Scrap the product.		
2	Anti raffles at the tube ends are undamaged	Replace antiraffles		
3	The locking device for the height adjustment locks properly.	Change the spring button.		
4	The adjustment holes are not elongated or deformed.	Scrap the product.		
5	All four legs have ground contact on a plane surface.	Scrap the product.		



**SCREW CONNECTIONS**

	<b>Examine that/if:</b>	<b>To do:</b>	<b>Comments:</b>	<b>OK</b>
I	All screw connections are properly tightened.	Tighten the screw connections.		

**FERRULES**

	<b>Examine that/if:</b>	<b>To do:</b>	<b>Comments:</b>	<b>OK</b>
I	The ferrules are undamaged / unworn.	Change all the ferrules.		

**CONTROL OF THE WALKING FRAME AS ABOVE ROUTINES SHALL BE CARRIED OUT REGULARLY.  
THE FREQUENCY SHALL BE RELATED TO THE INTENSITY OF USE AND WHAT STRESS THE  
WALKING FRAME HAS BEEN EXPOSED TO.**

**Service made by:**

**Name:** \_\_\_\_\_

**Walking frame model:** \_\_\_\_\_

**Date:** \_\_\_\_\_

**Serial number:** \_\_\_\_\_

**Sign:** \_\_\_\_\_