viaplus V12 / 1520058.01

Konfiguration des viaplus

Configuration of the viaplus

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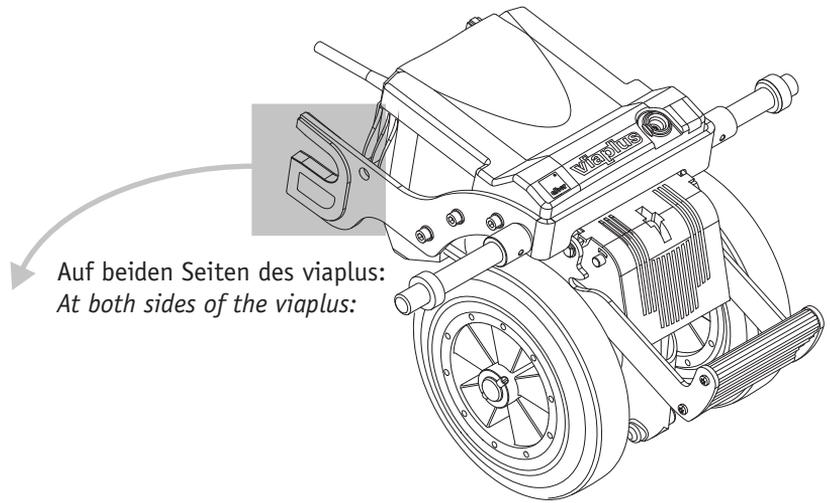
*= Based on the tolerance of the weld seams at the wheelchair, it is possible that you have to use this mounting type

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
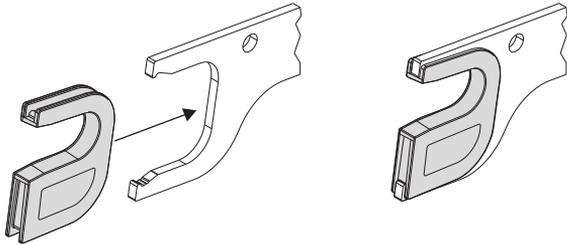
Front adapter, Positon 1 (see also column 5 of the chart at page 1)

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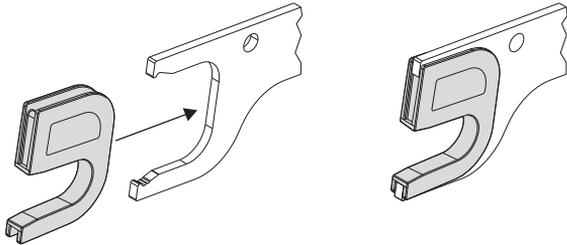Front adapter, Positon 2 (see also column 5 of the chart at page 1)

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For further information about the mounting at the wheelchair, see the "Mounting Instruction" enclosed to the viaplus V12

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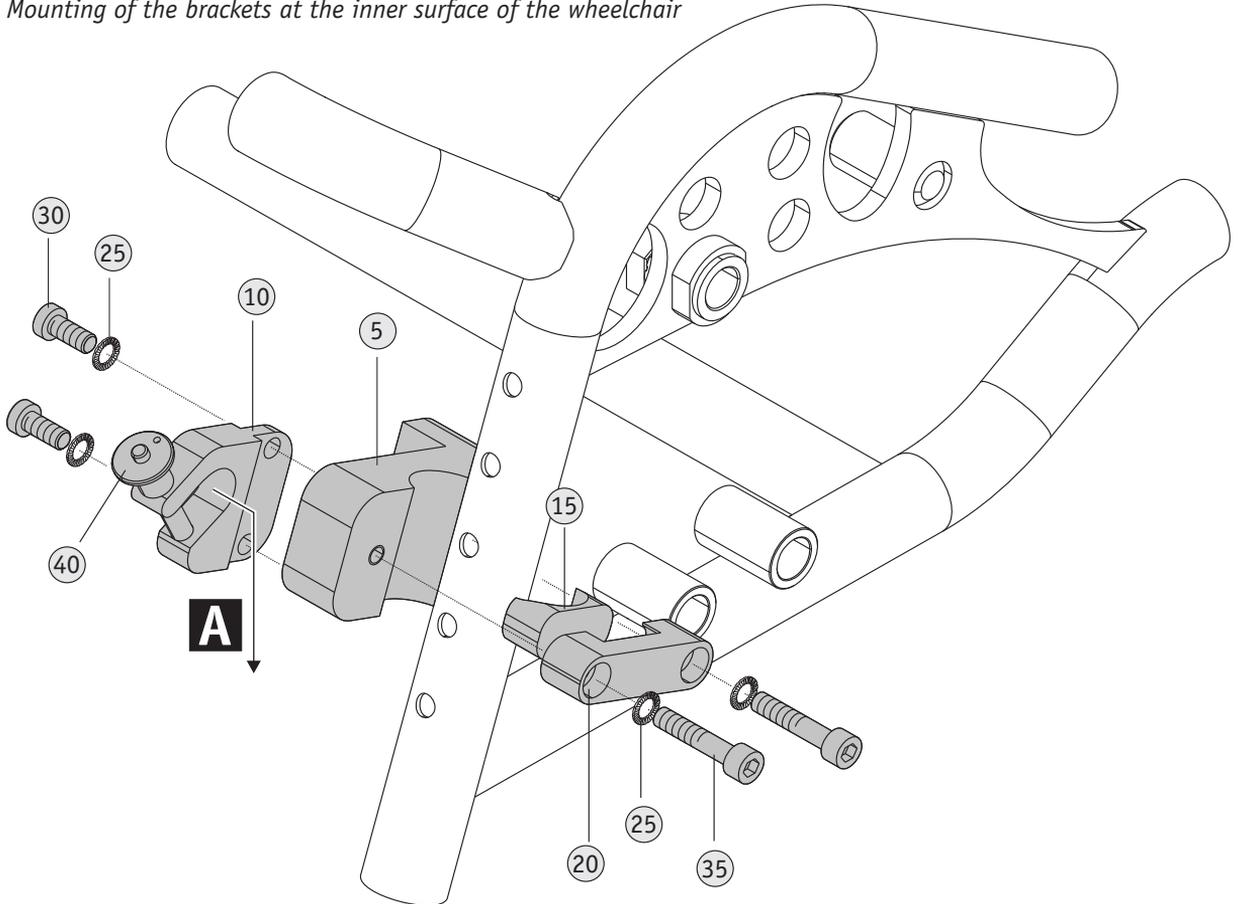
*Distance from the middle of the mounting
fork to bottom: 240 mm*

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It is strongly recommended that anti-tippers are mounted to your viaplus or wheelchair for drives on flat surfaces! The use of paired anti-tippers is mandatory on drives on uphill slopes over 8%.

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Please follow the instructions to stick together screws and screw nuts at page 7

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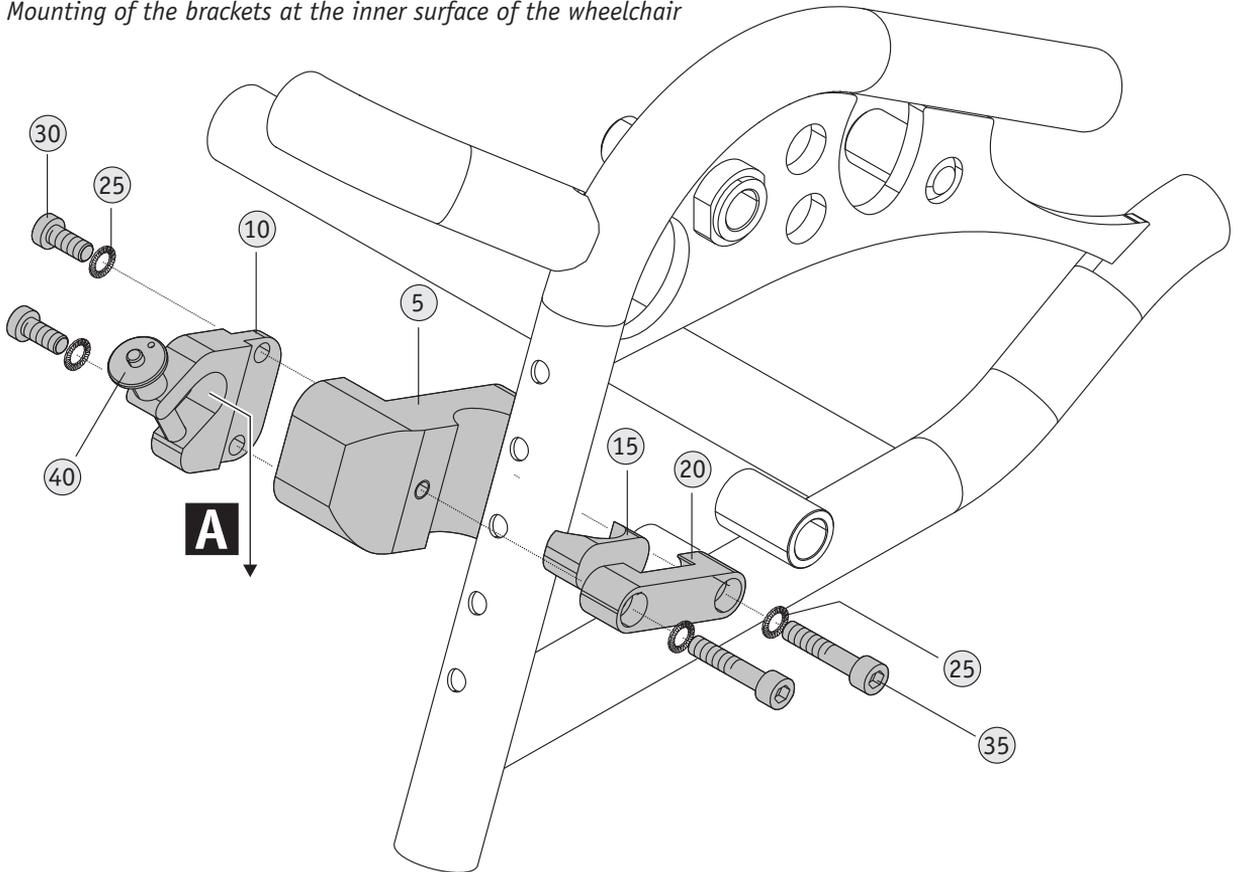
Distance from the middle of the mounting fork to bottom: 240 mm

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Mounting of the brackets at the inner surface of the wheelchair

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
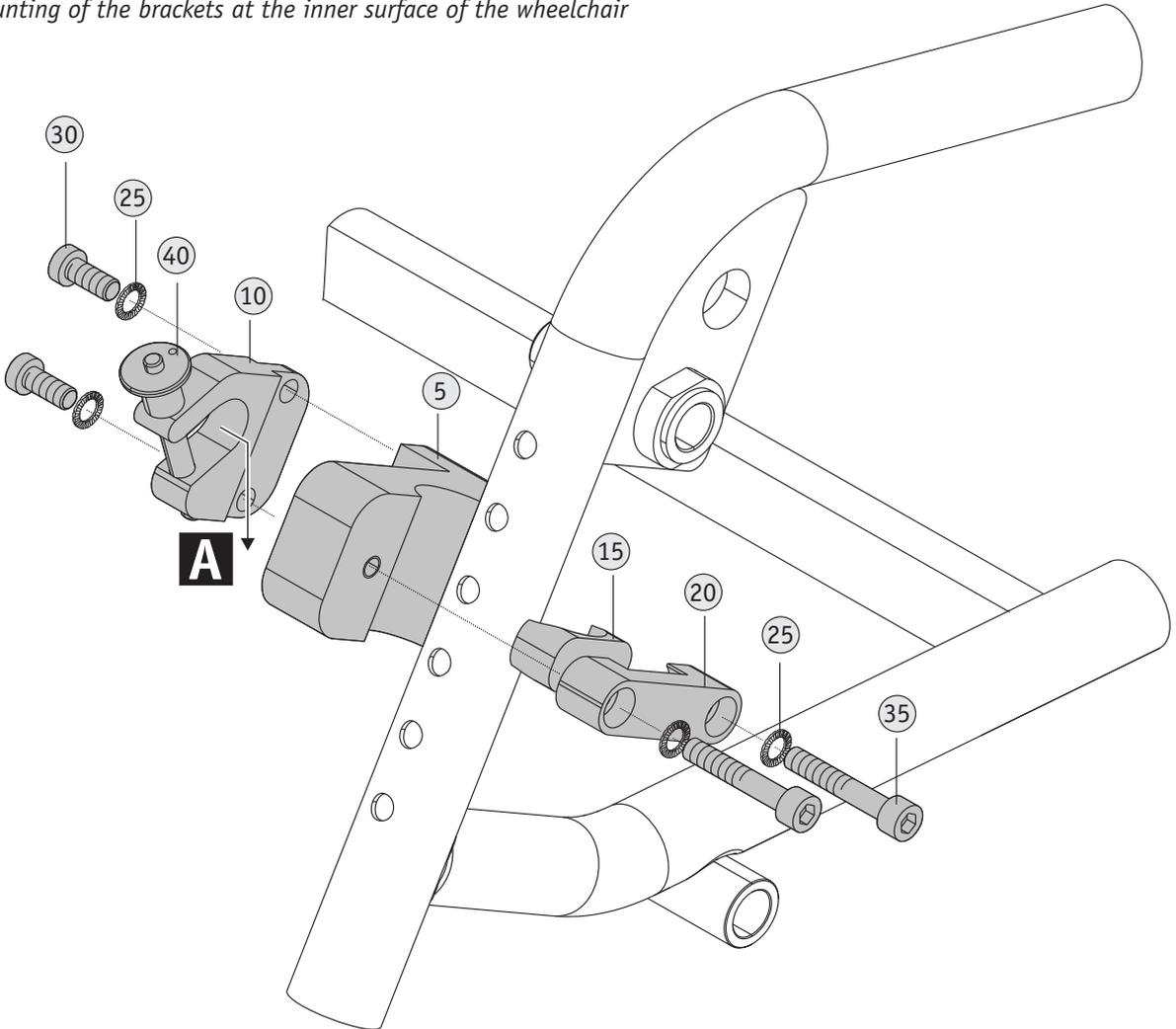
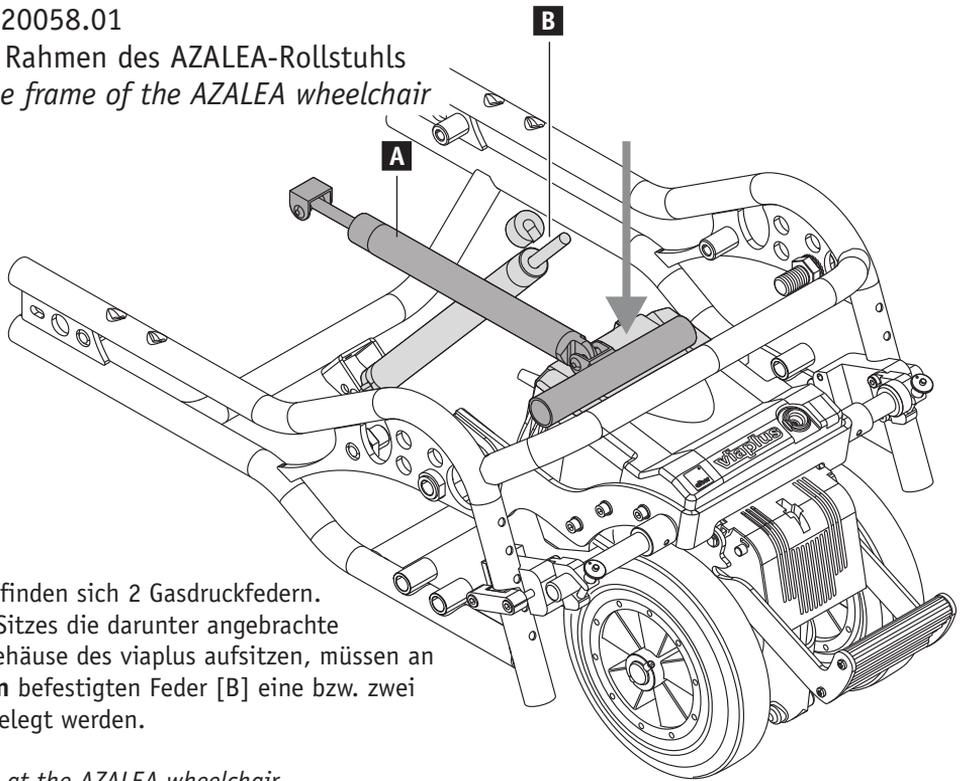
Mounting steps at the frame of the AZALEA wheelchair

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There are two gas springs at the AZALEA wheelchair.

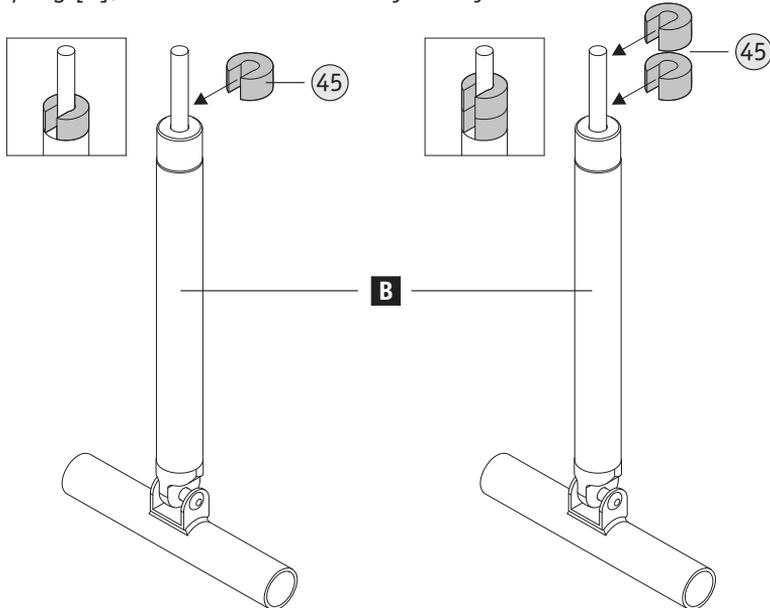Tip over the seat and look, whether the gas spring [A] (which is mounted under the seat) press on the cover of the viaplus or not. If the gas spring [A] press on the cover of the viaplus, then mount one or two distance spacers [45] into the gas spring [B], which is mounted at the frame of the wheelchair.

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
Locking of screws and screw nuts

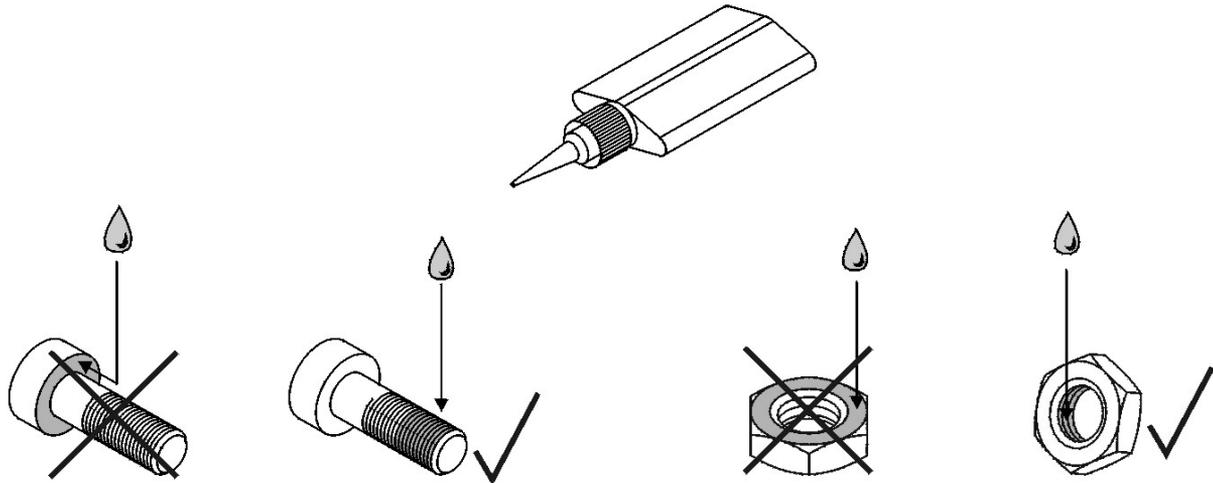If there is no other specification in our mounting instruction: Save all screw fittings which have no locking washer with screw locking adhesive (see pictures below). Use WEICON adhesive AN 302-43 (blue) / LOCTITE 243 adhesive to stick adapter sockets and screws. Use WEICON adhesive AN 306-01 (green) / LOCTITE 601 adhesive to stick the mechanical fasteners of the e-fix control-unit.

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Fastening torques for screw fittings

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