



Active wheelchair users make full use of their wheelchairs in their everyday life. They also frequently transfer in and out of their wheelchair. To them, a wheelchair cushion has to provide comfort and stability, it has to prevent them from developing pressure ulcers, and all of this at the lowest weight possible...

Vicair® Academy Active

Vicair's solution is the Academy Active. This wheelchair cushion is a combination of the unique Vicair technology and ergonomically-shaped foam. The Academy Active weighs only 1,400 grams.

Foam: The front of the Active is made of ergonomically-shaped foam. The foam enables the user to make an easier transfer and provides a great sense of stability.

Air: The cushion's back side has three compartments with air-filled SmartCells. This part of the cushion provides comfort while sitting and helps to prevent pressure ulcers. It is also a great addition to a decubitus treatment plan.

Cover

The cover is made from a comfortable, watertight material. It also includes a storage compartment for keys, a wallet or a mobile phone.

Two Heights

The Academy Active is available in two heights, 6 cm/2.5 inch and 9 cm/3.5 inch.

Cleaning

Treat light stains by wiping the cushion cover with a damp cloth, using a mild detergent.




Difficult spots or stains can be cleaned by first saturating the spot and then using a soft brush to loosen the dirt, or by machine washing the emptied cushion cover at 60 °C.

Before washing the cover, remove the SmartCells and foam block from the cover. The SmartCells are placed in three bags (SmartCell liners) to facilitate this. Follow the washing instructions on the label inside the cushion cover.



The Vicair principle

The Vicair principle is based on the unique Vicair technology. Each Vicair cushion is filled with hundreds of air-filled SmartCells. The cells move independently of each other and continuously adjust to fit the shape of the user's body. This results in the optimum distribution of pressure over the entire area and ensures stable support for the user.

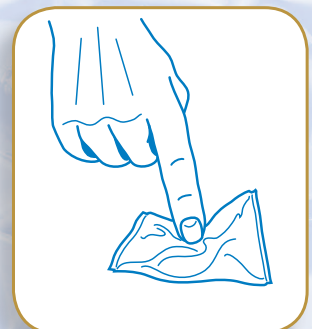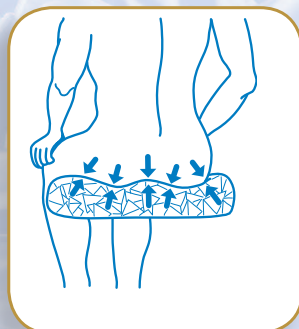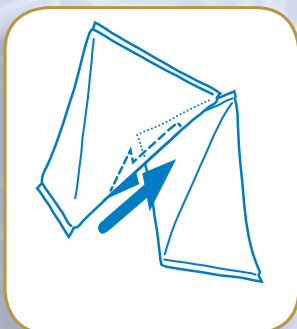

Maintenance

We recommend checking the SmartCells every three months.

Warranty

Vicair offers a limited warranty against defects in workmanship and materials for a period of two years. Please refer to the User Manual for the applicable terms and conditions.

Academy Wheelchair Cushions




1. Superior pressure distribution
2. Optimal positioning
3. Easiest to use

MODELCHART ACADEMY ACTIVE

		WIDTH											
		12	14	15	16	17	18	19	20	22	24		
		inch	cm	30	35	38	40	43	45	48	50	55	60
DEPTH	12	30											
	14	35											
	15	38											
	16	40		Standard		Standard		Standard					
	17	43											
	18	45				Standard		Standard					
	19	48											
	20	50											
	22	55											
	24	60											

 Standard; available from stock

 Special; three week delivery time; upcharges apply

Sizes not mentioned in the model chart: availability and prices on request.



VICAIR®
the best in body support

Any questions?

Visit www.vicair.com for contact details and the Vicair Live chat.

Or contact your Vicair distributor.