

Invacare® | **Vicair® Liberty Range**



**Warranty  
2 years**




**Comfort, support and stability**

The *Invacare Vicair Liberty range* comprises of a seat cushion and back, suitable for those 'At Risk' of developing pressure ulcers. Providing comfort, reliability, pressure reduction and ease of use, both the cushion and back automatically adjust to the clients weight and body contours without the need to remove or add **SoftCells™**.



**Yes, you can.®**

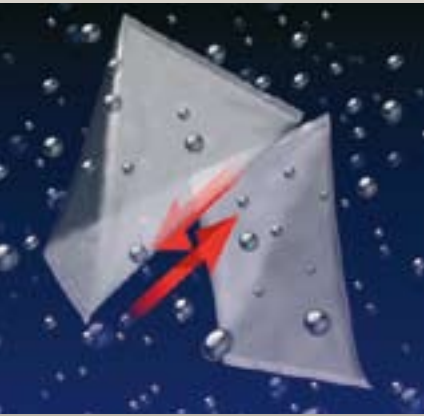

The Liberty Range

**Innovative technology**

The **Liberty range** is made up of air filled **SoftCells™** surrounded by a small quantity of silicon gel. This allows the cells to move smoothly and freely against each other and conform to the body's contours.


**Ease of use**

This innovative design results in a comfortable and easy to use seating solution. To set up the back and cushion, simply open the valve, sit the client in the chair and leave for a couple of minutes. The air volume will automatically adjust itself to the clients weight and body shape. Finally close the valve and the cushion and back are ready for use.



**Liberty PT**

This entry level cushion has two compartments, one at the front and one to the rear. The rear section has fewer cells which helps to reduce shear forces and eliminates the tendency to slide forward. For increased positioning, this cushion is available with a pelvic stabiliser.

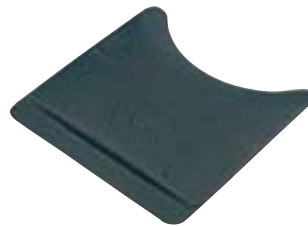

**Pelvic Stabiliser**

By adding the Pelvic Stabiliser to the **Liberty PT** cushion, it will raise the seat surface under the clients thighs, reducing the tendency of the pelvis to rotate backwards. This will in turn reduce shear forces whilst providing an upright, comfortable seating surface.



**Liberty Back**

With the same features as the **Liberty PT**, the **Liberty Back** provides comfort, stability and support for the client. Available with or without back straps, it is an ideal replacement for a wheelchair with a standard sling back. The lower section provides lumbar support whilst the top gives thoracic support. The back rest can also be adjusted to follow the contours of the spinal column.

**Technical data**



Size:

**Liberty Back:**

360-400mm, 400-440mm and 440-480mm

**Liberty PT:**

440x440mm, 400x450mm. 430x430mm, 450x440mm. 450x450mm and 500x450mm



Weight:

0.7kg



Max User Weight:

23.5 stone/ 150kg



Outer cover:

washable at 60°.

Inner cushion:

hand wash

**Vicair Liberty**

Invacare Limited  
Pencoed Technology Park - Pencoed -  
Bridgend CF35 5AQ - United Kingdom  
Tel: +44 1 656 776 222  
Fax: +44 1656 776 220  
E-mail: uk@invacare.com  
Sales Order E-mail:  
ordersuk@invacare.com  
www.invacare.co.uk

Invacare Ireland Ltd  
Unit 5 - Seatown Business Campus  
Seatown Road - SWORDS - County  
Dublin - Ireland  
Tel. +353 1 8107084  
Fax +353 1 8107085  
Email: Ireland@invacare.com  
www.invacare.ie

© 2012 Invacare International Sàrl  
All rights reserved.

All information quoted is believed to be correct at time of print.  
Invacare reserves the right to alter product specifications without prior consultation.

Vicair Liberty - UK - 03/2012 - 1554356

**Cover options**

**Comfair:** Multi-stretch breathable top fabric allows air to circulate, provides comfort and reduces the risk of moisture/ heat build up.

**Incontince:** Multi-stretch water resistant and vapour permeable fabric with an easy to maintain wipedown surface.

\* Essential nursing care is pivotal in pressure ulcer prevention. This product will positively contribute to the outcome of a pressure ulcer prevention care plan. Education, clinical judgement and action based planning based on vulnerability are fundamental factors in the prevention of pressure ulcers. A range of assessment scales can be used as a formal method of assessing risk from pressure ulcer development, and should be used in conjunction with an informal assessment (informed nursing judgement). Informal assessment is considered to be of greater importance and clinical value.



**Yes, you can.®**