



By Appointment to
HM Queen Elizabeth The Queen Mother
Wheelchair Manufacturer
Invacare
Bridgend



Bulletin No: FSN008R
Date: 5 December 2008

Field Safety Notice

THIS IS A RE-ISSUE OF THE FOLLOWING NOTICE **Action 3 Wheelchairs fitted with ratchet reclining backrest**

Ref: 2007/003/006/081/010

**THIS IS A REMINDER REGARDING THE ORIGINAL CIRCULATION OF THIS FIELD SAFETY NOTICE.
PLEASE CHECK YOUR RECORDS AND ENSURE ALL YOUR CUSTOMERS HAVE BEEN MADE AWARE & FOLLOW THE INFORMATION BELOW.**

Important Safety Information:

In addition to Field Safety Notice FSN006 dated 16th October 2007, re failures of the ratchet indexing mechanism used on the Action 3 manual reclining backrest.

Incidents have occurred when the locating pin fitted in the backrest tube has become disengaged with the teeth in the ratchet housing. A contributory factor in these incidents has been the removal of the back-brace. The back-brace is essential to the safe usage of the product and provides rigidity to the back-rest, it should only be disengaged for folding when transporting the wheelchair, storage etc. **The back-brace must be fitted when the wheelchair is in use.**

It is also advised that inspection of the ratchet mechanism is carried out on a regular basis. The inspection period depends on the frequency of use; however, it is recommended that the unit should be checked once a month as a minimum. Where the wheelchair is subjected to frequent / heavy use the inspection should be completed weekly. In the event of any damage being identified please refrain from using the wheelchair until adequate maintenance / repair has been completed.

Please note that Invacare advise against hanging bags etc from the back-rest posts; this results in extra force being exerted on the backrest and also has an adverse effect on the stability of the wheelchair which could result in the chair tipping backwards.

Information, instructions and warnings relating to this FSN will be added to the user manual.

Please ensure that the information in this notice is made available to all relevant personnel within your organisation and / or customer base, and that awareness is maintained for an appropriate period.

If you have any questions relating to this bulletin, or should you require any additional information, please contact;

Technical Services Department
Tel. 01656 647327 Fax. 01656 753299

INVACARE UK OPERATIONS LIMITED

South Road Bridgend Industrial Estate Bridgend CF31 3PY
Sales/Technical Service Tel: 01656 647327 Fax: 01656 649016 Reception Tel: 01656 664321
Registered in the UK No. 3281202