



# Footplate

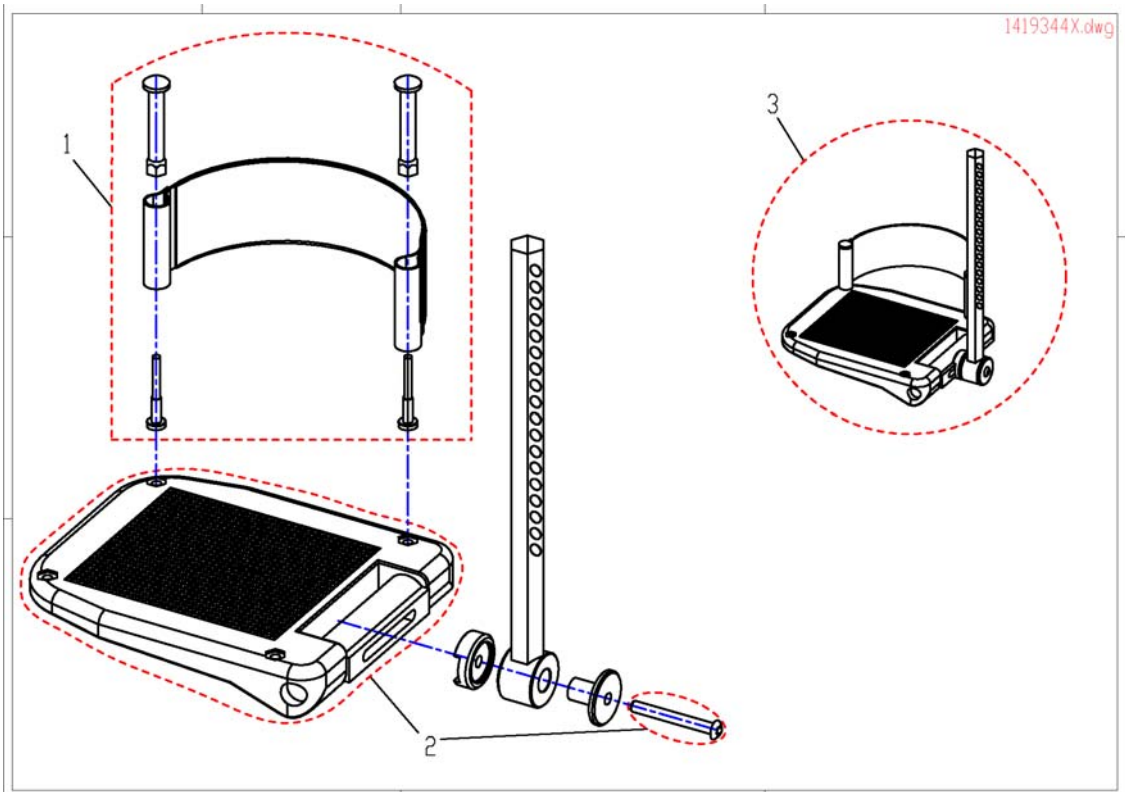


Pos.	Part number	Description	Valid from → until	Qty
1	SP5321579	Heel Strap		1
2	SP5106376	Footplate		1
3	SP1419344	Footplate LH Anlge And Depth Adjustable, With Support Tube cpl.		1

## Footplate with support tube fix RH/LH

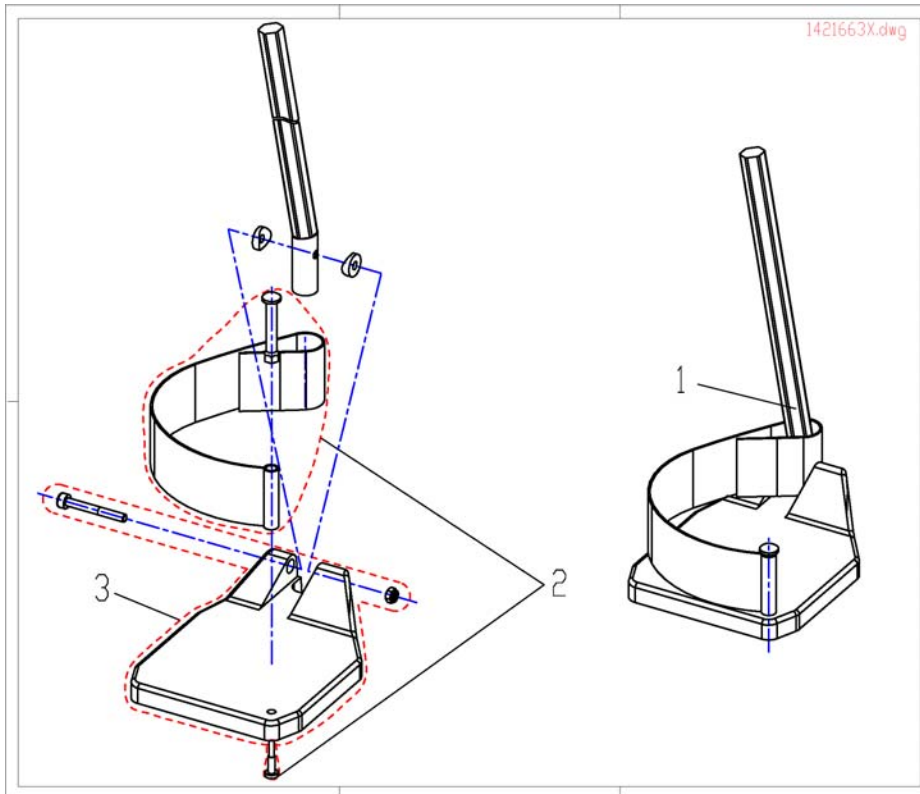

Pos.	Part number	Description	Valid from → until	Qty
1	SP1421662	Footplate RH fix, with Support Tube cpl.		1
1	SP1421663	Footplate LH fix, with Support Tube cpl.		1
2	SPB273011	Heel Strap		1
3	SPZ173217	Footplate		1
3	SPZ173218	Footplate		1


# Footplate with support tube angle adjust. RH/LH

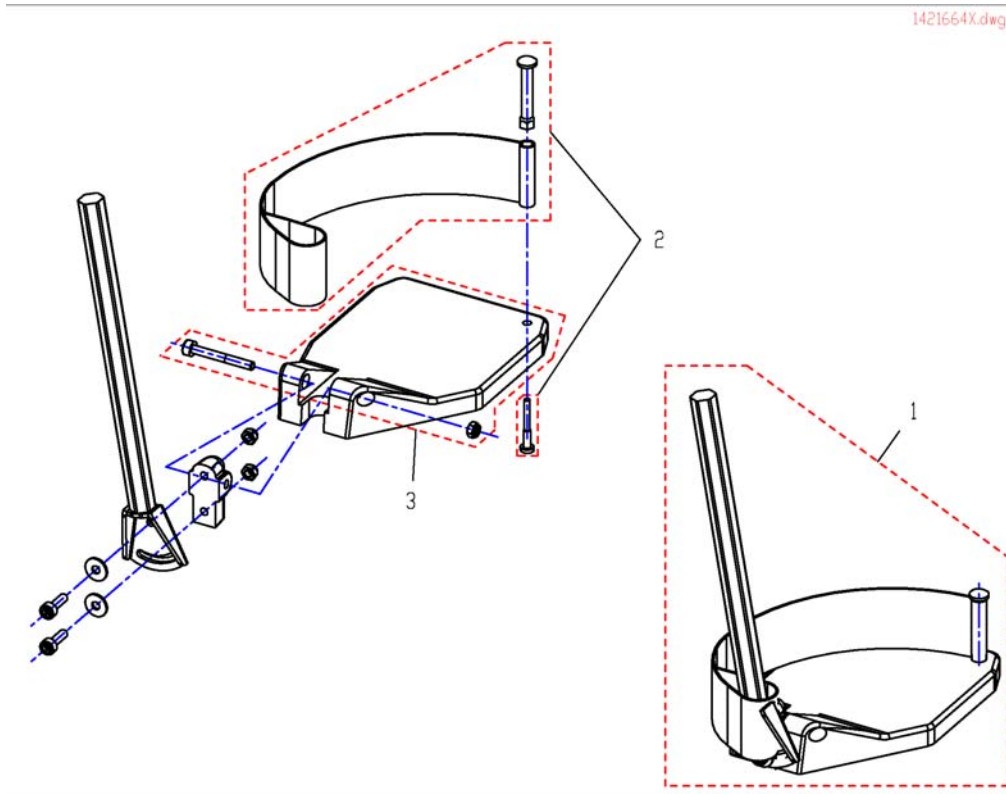

Pos.	Part number	Description	Valid from → until	Qty
1	SP1421664	Footplate RH anlge adjustable, with Support Tube cpl.		1
1	SP1421665	Footplate RH anlge adjustable, with Support Tube cpl.		1
2	SPB273011	Heel Strap		1
3	SPZ173217	Footplate		1
3	SPZ173218	Footplate		1


## Support Tube with Footplate for central Legrest

1440290X.dwg

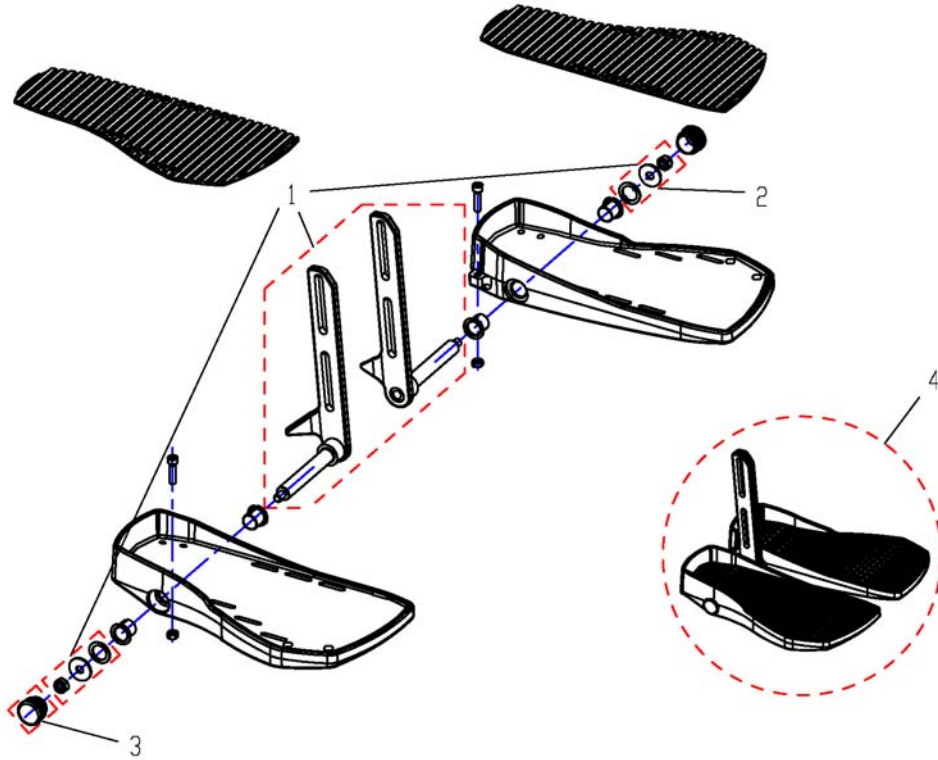

Pos.	Part number	Description	Valid from → until	Qty
1	SP1440442	Central Lower Legrest, small		1
1	SP1440443	Central Lower Legrest, medium		1
1	SP1440444	Central Lower Legrest, large		1
2	SPF00002	Fixation kit		1
3	SP1440517	Darning		1
4	SP1440290	Central Lower Legrest, small (32-38cm), cpl.		1
4	SP1440291	Central Lower Legrest, medium (38-44cm), cpl.		1
4	SP1440292	Central Lower Legrest, large (44-50cm), cpl.		1

## Footrest Lower Parts

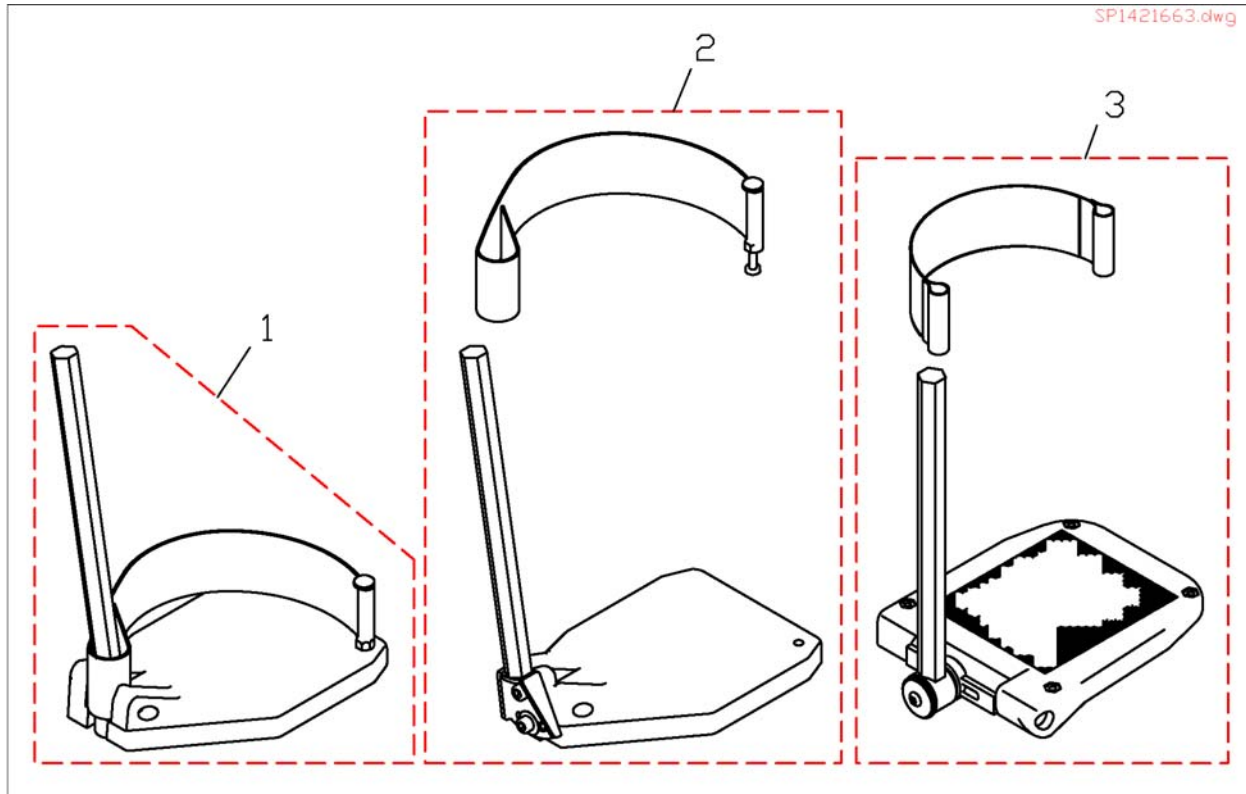

Pos.	Part number	Description	Valid from → until	Qty
1	SP1421663	Footplate LH fix, with Support Tube cpl.		1
1	SP1421662	Footplate RH fix, with Support Tube cpl.		1
2	SP1421664	Footplate RH anlage adjustable, with Support Tube cpl.		1
2	SP1421665	Footplate RH anlage adjustable, with Support Tube cpl.		1
3	SP1419344	Footplate LH Anlage And Depth Adjustable, With Support Tube cpl.		1
3	SP1419345	Foot Plate RH Angle And Depth Adjustable cpl.		1
4	SP1446549	Part not for sale		1
5	SP1446548	Part not for sale		1