



Penny & Giles Controls

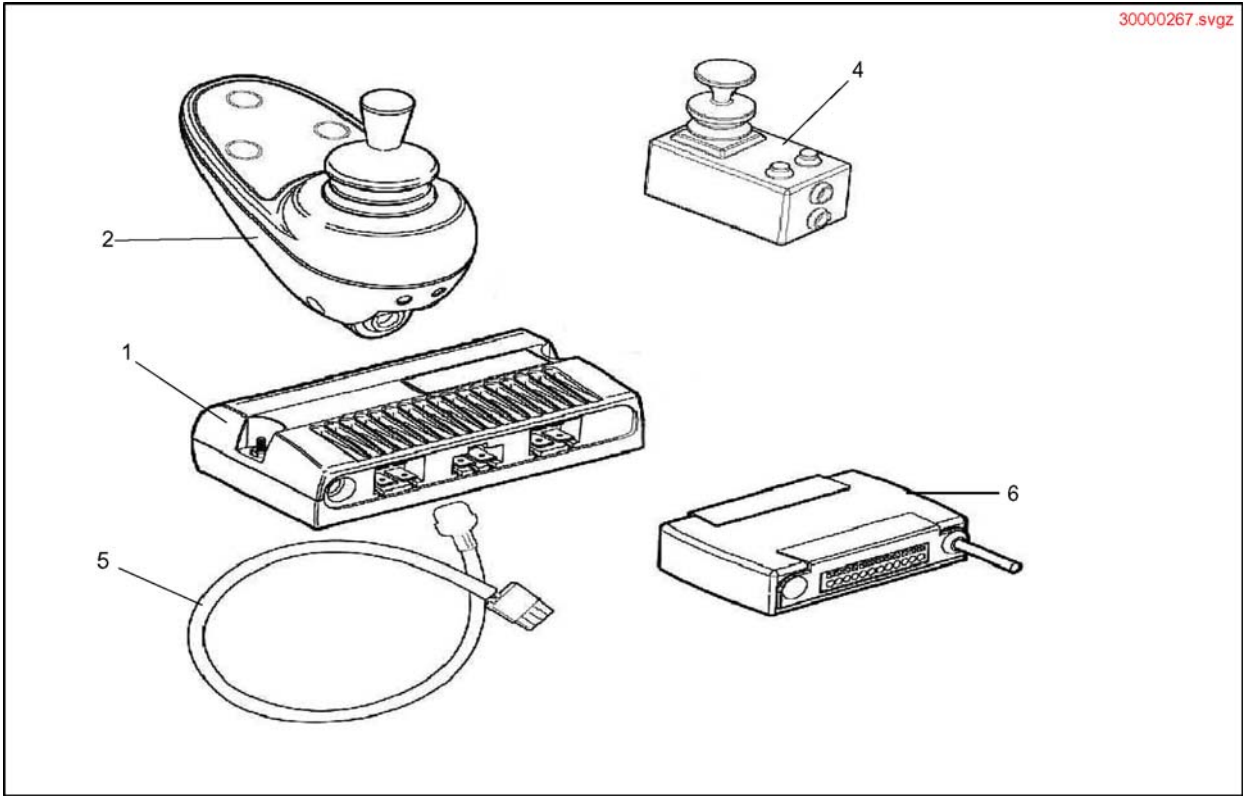

Pos.	Part number	Description	Valid from → until	Qty
1	30000265	Pilot+ PM50 Module (D49859)		1
2	30000267	Pilot+ Remote (D49671)		1
4	30000270	Pilot+ Dual Control D49682		1
6	30000271	Pilot+ Clam ALM-5L (D49664)		1
5	1484386	Actuator Loom (Pilot+)		1


Holder Pilot+, swing away, high end

1528311X.dwg

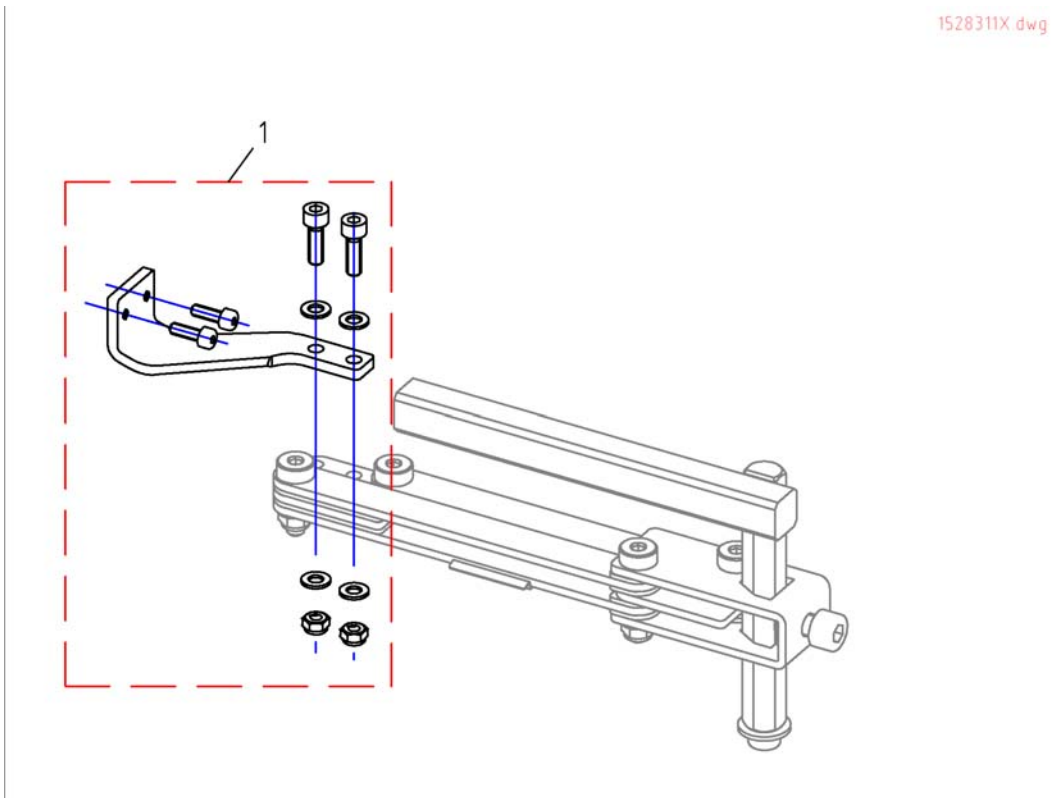


Pos.	Part number	Description	Valid from → until	Qty
1	SP1528311	Holder Pilot+, swing away, high end RH		1
1	SP1528312	Holder Pilot+, swing away, high end LH		1

ACS Control System

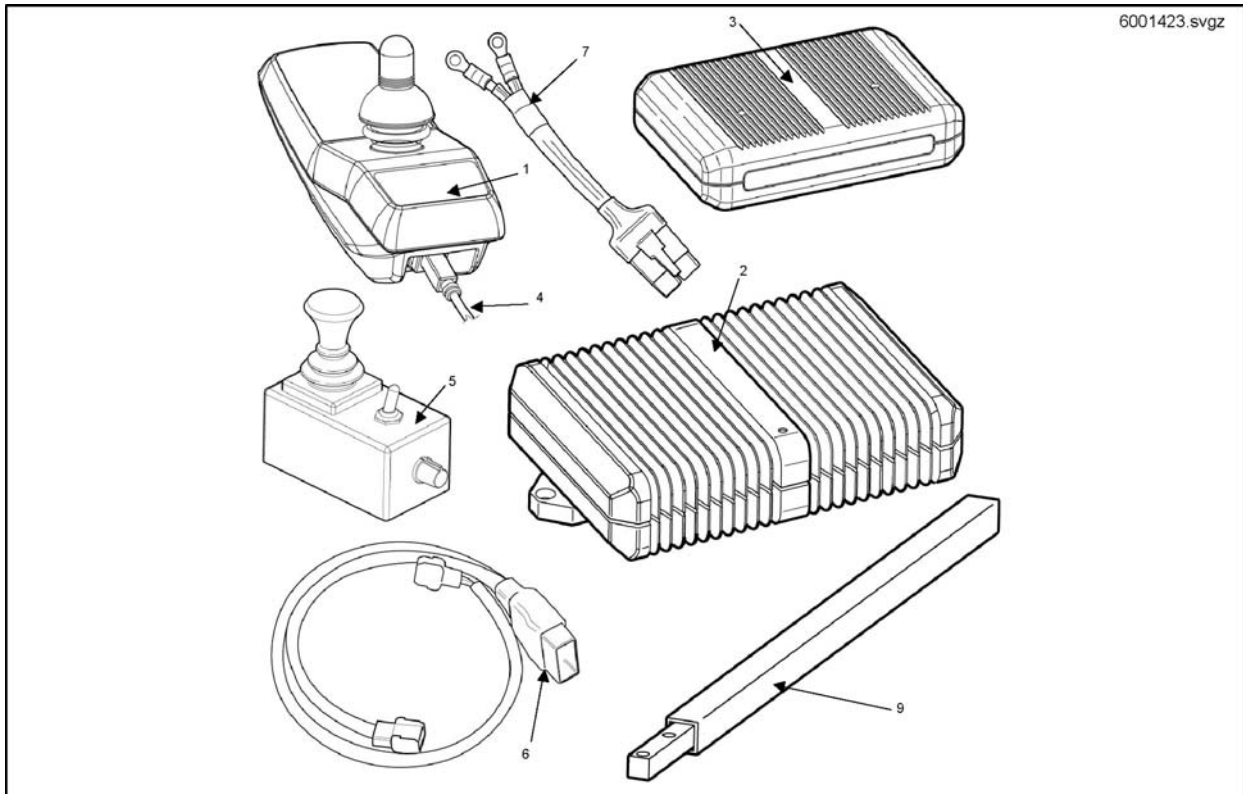

Pos.	Part number	Description	Valid from → until	Qty
1	1436541	Remote REM 24 SD		1
2	6001423	ACS Power Module DX-PMB AS60A		1
3	P10081	ACS Light/Actuator Module (Clam)		1
4	GSM63011	ACS Bus-Cable 1,1M		1
5	6001425	ACS Dual Control		1
6	1422381	Actuator Loom Forb/Rests - Seat - L/Rests		1
7	6001413	Power Lead (DX)		1
9	1441778-0205	Joystick Tube Assy Black		1


G90 & Holder

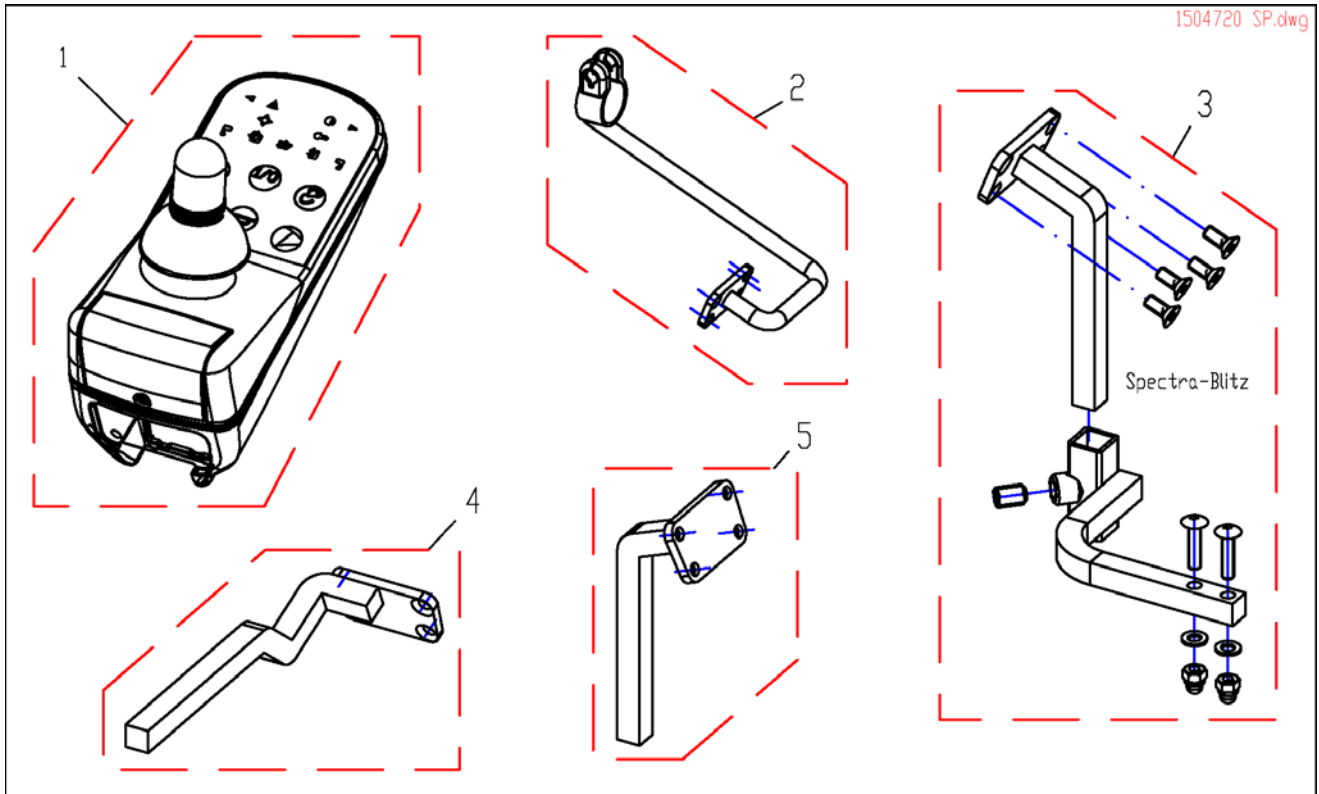

Pos.	Part number	Description	Valid from → until	Qty
1	1504720	Remote G90A		1
2	SP1509696	Holder Attendant Control G90A RH		1
2	SP1509702	Holder Attendant Control G90A LH		1
3	1518367	Remote Holder Swing-Away (Spectra-Blitz)		1
4	SP1518449	Holder G90A, Standard, RH		1
4	SP1518450	Holder G90A, Standard, LH		1
5	SP1517352	Remote Holder Swing-Away		1


G91 & Holder

1505187.dwg

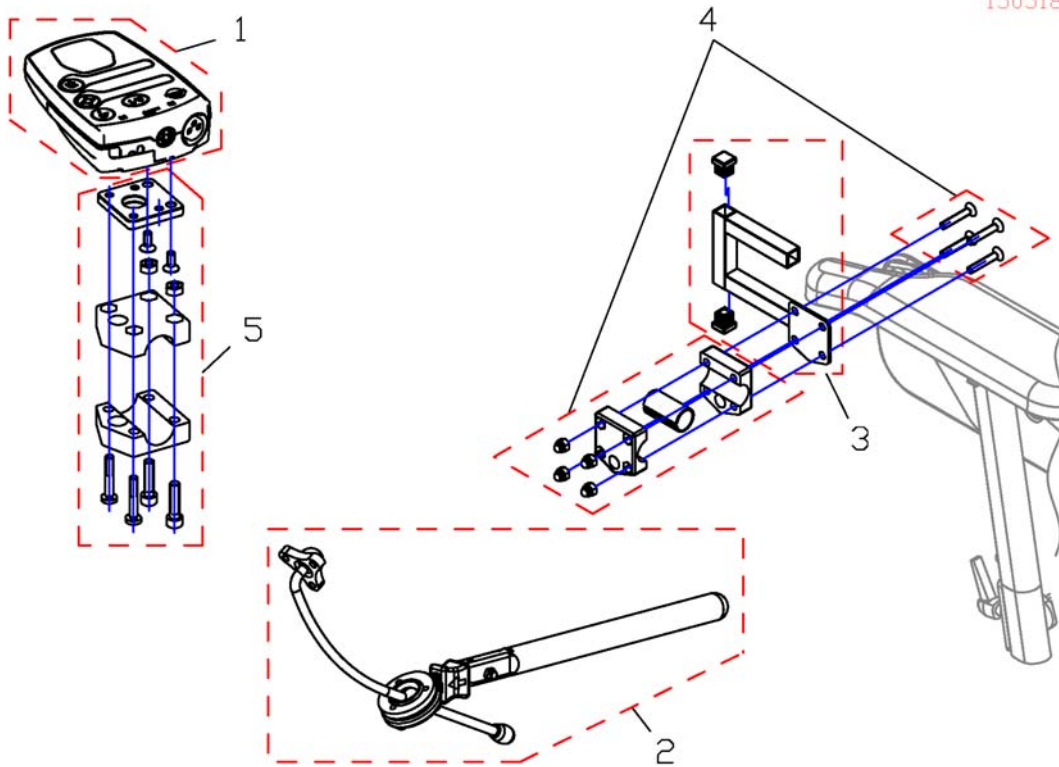


Pos.	Part number	Description	Valid from → until	Qty
1	1505187	Remote Control G91		1
2	SP1527504	Swan Neck Remote Bracket with Adapter Plate		1
3	SP1509711	Adapter G91 RH		1
3	SP1509716	Adapter G91 LH		1
4	SP1527503	Gripper Clamp cpl.		1
5	SP1528271	Holder for Backresttube		1

Dynamic Mounting Options

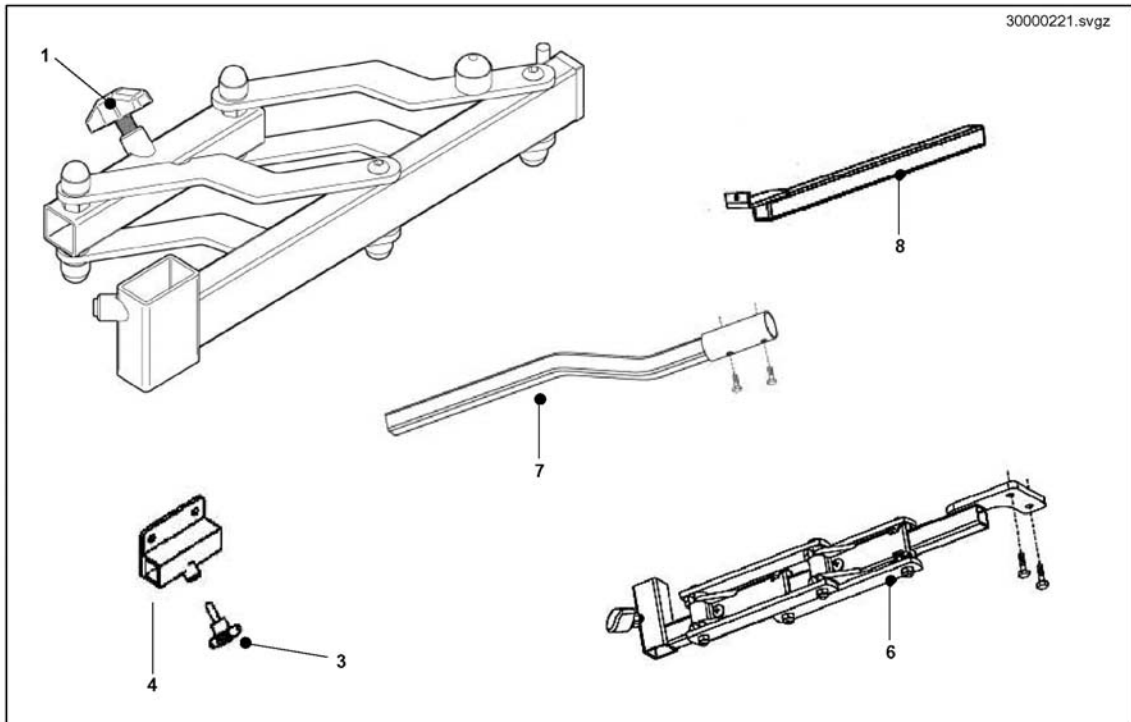

Pos.	Part number	Description	Valid from → until	Qty
	1436867-0216	LH Swing Away Joystick Mounting Arm Bracket		1
	1433312-0216	RH Swing Away Joystick Mounting Arm Bracket		1
1	30000221	S/Away Arm LH		1
1	30000220	S/Away Arm RH		1
2	1441778-0205	Joystick Tube Assy Black		1
3	256099	Wing Screw M8 X 20 639		1
4	23008273-0205	Part not for sale		1
5	SP1509904	Swingaway Bracket Pilotplus RH		1
5	SP1509903	Swingaway Bracket Pilotplus LH		1
6	SP1441636	Part not for sale		1
6	1441637	LH S/Away Arm Complete (Shark 1 & 2)		1
7	1441631	J/Stick Mtg Arm (Fixed)		1
8	1442252-0216	Mounting Brkt Pilot+ Blk		1
9	SP1513568	VR2 Remote Bracket swing away LH		1
9	SP1513567	VR2 Remote Bracket swing away RH		1

Attendant Control Mounting Hardware

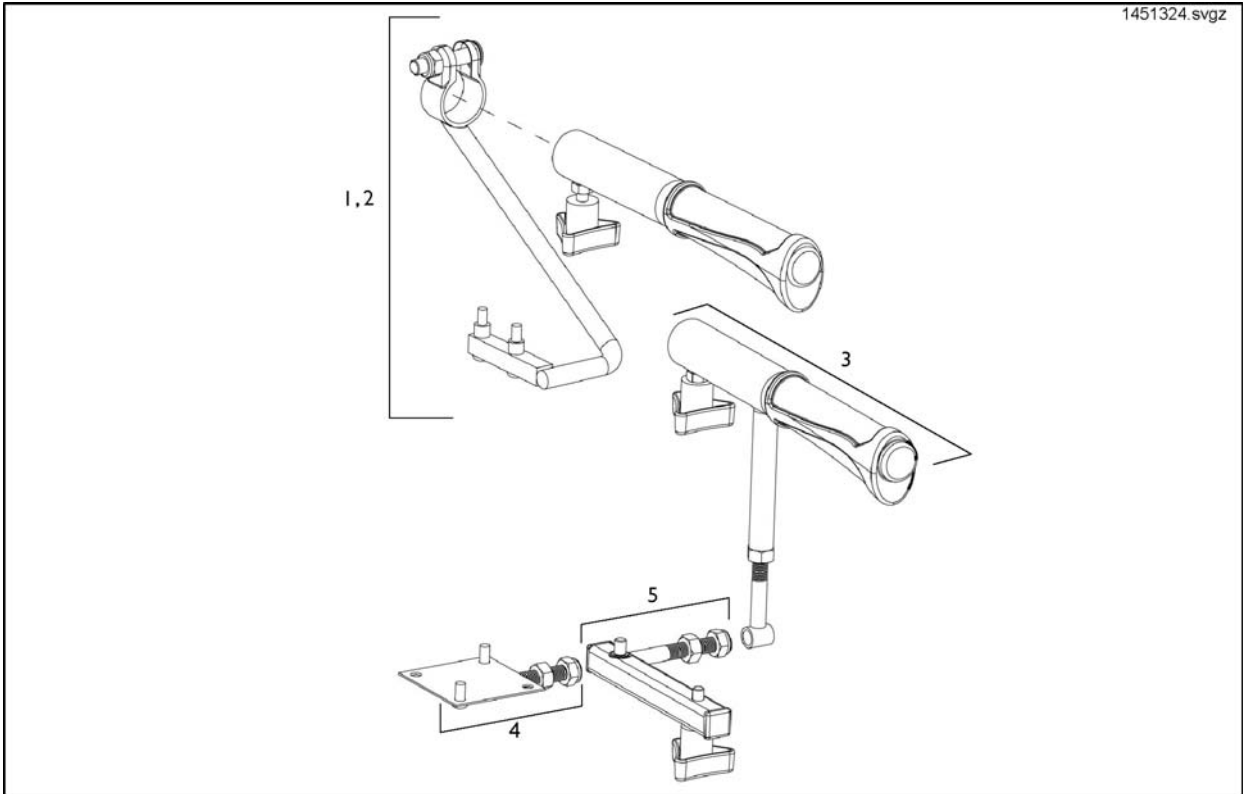

Pos.	Part number	Description	Valid from → until	Qty
1	1451324	ACS/Shark - Joystick Attendant Mounting Arm Right		1
2	1484794	ACS/Shark - Joystick Attendant Mounting Arm Left		1
3	1451325	Push Handle Extension Kit		1
4	1451326	Pilot+ Dual Control Mount Handle Extension Kit		1
5	1451319	P&G Attendant/Dual Control Unit Fixing Kit		1
6	1451320	ACS Dual Control Fixing Kit		1