


Drive Wheels

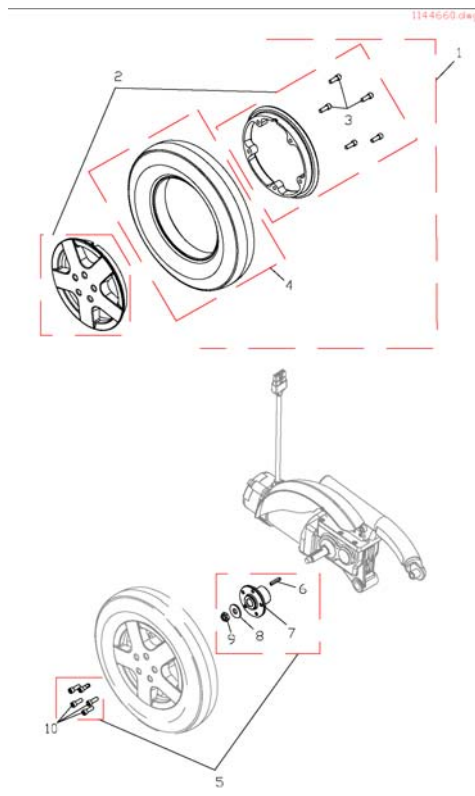

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523161	Drive Wheel Assembly, w/ Flat Free (14"x3")		1
2	SP1523013	Kit,8" Rim with Hardware (14"x3"Wheel)		1
3	SP1522975	Screw, Socket Head Cap w/Patch (5/16-18 x7/8")		1
4	SP1522940	Tire, Pneumatic (14"x 3")		1
4	SP1523019	Tire with Flat Free (14"x3")		1
5	SP1523157	Kit, Drive Wheel Mounting Hardware (14"x3")		1
6	SP1522934	Key (3/16 x1-1/4")		1
7	SP1523092	Wheel Hub		1
8	SP1522939	Washer (1/2 x1-9/32 x1/8")		1
10	SP1522975	Screw, Socket Head Cap w/Patch (5/16-18 x7/8")		1