





**SAFETY**

- The rollator must be fully unfolded when used as a walking support. NB. Only to be used as a walking support.
- Exercise caution on sloping ground.
- Do not go up or down stairs with the rollator.
- Do not use the rollator on an escalator.
- Exercise caution when there is a heavy load in the basket.
- Loads should be transported solely in the basket, bag or on the tray. Max. overall load in the basket and on the tray is 5 kg. Max. load in the bag 5 kg.
- The handlebars must not be angled outside the marked position as this reduces the rollator's lateral stability, see figure A.

**HANDLEBARS**

NB. The handlebars must point directly backwards when using the rollator. Under no circumstances should they be angled outwards. The vertical line in the knurling must be located in the middle of the indentation in the frame, see picture A.

To adjust the height of the handles: Undo the locking levers and adjust the handles to a suitable height. NB. The handles must never be set above their highest position, indicated by the knurling on the tubes.

- Tighten the locking levers, which should then be adjusted so that they point downwards. (The locking levers can be freely turned by first pulling them outwards). Check that the handles are set at the same height. The handles should normally be positioned at wrist height.

**ACCESSORIES**


- B) Fitting the basket.
- C) Fitting the tray.
- D) Fitting the bag.

**ASSEMBLING**

Before the rollator is assembled remove plastic caps in the side tubes.

Open the rollator into operating position (see below).

1. Fit the locking lever with the bolt. The locking lever with the washer shall be installed on the outside. Ensure that the head of the bolt slots into the recess on the fitting.
2. Insert the handle tubes into the side uprights and tighten the locking levers. These must be pointing downwards during use.

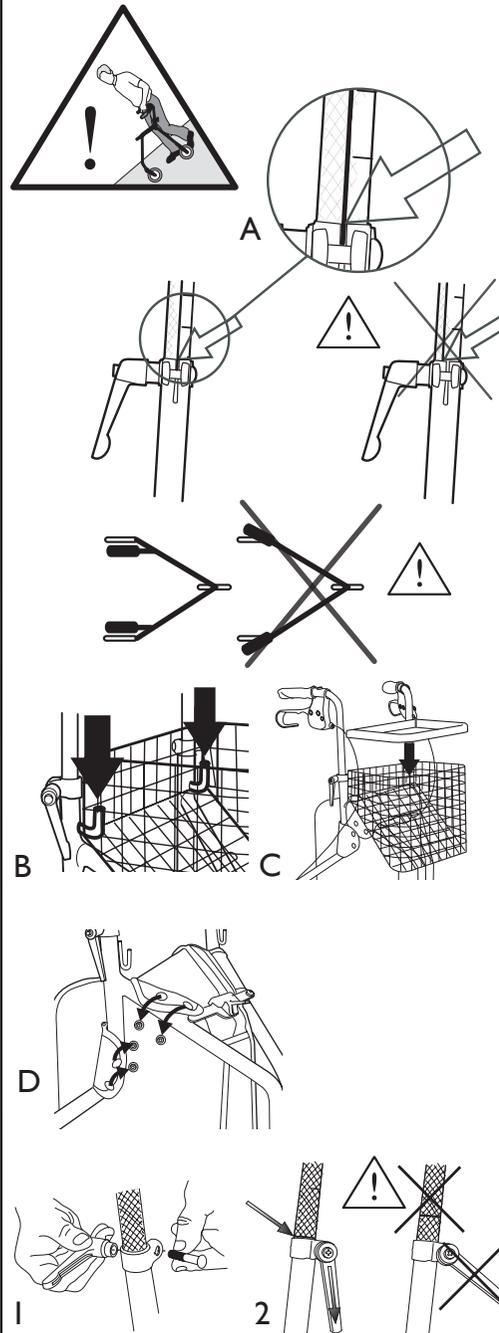

**OPERATING POSITION**

Open the rollator into operating position. NB. Always check that the safety catch is in locked position

**FOLDING**

1. Pull the safety catch upwards - backwards.
2. Fold the rollator. NB. Be aware of the risk of crushing between the parts of the frame.

**BRAKES**

Apply the brakes by pulling both brake handles upwards.

**PARKING BRAKE**

Press the brake handles carefully downwards to apply the parking brakes. A distinct click indicates that the brake is in the correct position. It is important that both the brakes are put into parking position when they are to be used. To release the parking brake, pull the brake handle upwards.

**ADJUSTMENT OF BRAKE EFFECT**

Two setting screws, one in the brake lever and one in the middle of the brake cable adjust the tension of the brake and the application of the brake to the user's capability. The braking power increases and the movement required in the brake lever is reduced when the setting screws are loosened.

NB. Make sure to tighten the lock nuts after adjustment.

