

Appliance number: V14 _____

Specialist dealer: _____



Ulrich Alber GmbH
Vor dem Weißen Stein 21
72461 Albstadt
Telephone 07432/2006-0
Fax 07432/2006-299

Work	Check	Comments
Drive unit		
Check tyres for depth of profile	<input type="checkbox"/>	Diameter of wheel min.: 194.0 mm
Check running surface of wheel for even profile wear	<input type="checkbox"/>	
Check pedal function - extend, retract drive	<input type="checkbox"/>	
Check drive for running noises	<input type="checkbox"/>	
Check that bolts and adjusting rings sit securely	<input type="checkbox"/>	
Check supporting contacts of battery pack	<input type="checkbox"/>	
Handle unit		
Control knob: Check that on/off and speed are functioning	<input type="checkbox"/>	
Control knob: Check parking brake with steerer pressed	<input type="checkbox"/>	
Control knob: Check rolling function	<input type="checkbox"/>	
Steerer: Check forwards/reverse function	<input type="checkbox"/>	
Check LED status display	<input type="checkbox"/>	
Check pin contacts	<input type="checkbox"/>	
Battery pack		
Check fuses	<input type="checkbox"/>	
Check contacts	<input type="checkbox"/>	
Check releases and locks	<input type="checkbox"/>	
Final test		
General function test of whole system with wheelchair and occupant	<input type="checkbox"/>	
Check tip support setting	<input type="checkbox"/>	
Check position of mount and that the locking bar moves easily	<input type="checkbox"/>	
Check housing and bracket for tears	<input type="checkbox"/>	

Safety Checks

viamobil V14

Appliance number: V14 _____

Specialist dealer: _____



Ulrich Alber GmbH
 Vor dem Weißen Stein 21
 72461 Albstadt
 Telephone 07432/2006-0
 Fax 07432/2006-299

Work

Check

Comments

Documentation

Replace "Safety Check" label on device



(1)

(1) Write label text with permanent marker.
 In the field "Next Test": Mark date (interval of 24 months)

! The safety-related checks on the viamobil may only be carried out by trained specialist personnel.

Date

Name

Signature of technician