



Yes, you can.®



Invacare®
Etude Duo

***Spare parts catalogue
EN 1970***



CE

Information about using this list.....	3
3-sectioned mattress support without tilt EN 1970 / with tilt “EN 1970”.	4
4-sectioned mattress support without tilt EN 1970 / with tilt “EN 1970”.	6
Option. 4-sectioned mattress support without tilt EN 1970 / with tilt “EN 1970”.	10
Mattress support extension with cable.....	12
Bedend Duo steel.....	14
Bedend supplement for Duo wood	16
Siderail steel Scala Basic, Medium, Decubi	18
Siderail steel - 5529	20
Support handles	22
Transport fitting - a pair.....	24
Lifting pole – model 5760	26

When placing an order for motors and control unit, please see the photos below.

MOTOR



CONTROL UNIT FRONT



CONTROL UNIT REAR



Information about using this list**Order of wearing parts/exchange package:**

In the table of contents on page 2 you can see where in the list you can find the wanted part/package.

Wearing parts are sold in exchange packages with an order number for the entire package.

Order of spare parts:

In the table of contents on page 2 you can see where in the list you can find the wanted part.

Single numbers

<i>Pos.</i>	<i>Ident. No.</i>	<i>Pcs.</i>	<i>Description</i>
13	1423845	1	Hand control box with tilt
14	1423814	1	Hand control box without tilt

Order of a complete unit:

In the table of contents on page 2 you can see where in the list you can find the wanted unit of the bed.

Backrest Motor package

Ident. No.: 1432658

<i>Pos.</i>	<i>Pcs.</i>	<i>Description</i>
2	1	Lock cam for Control unit
5	2	Pipe pin ø10x50
6	1	Motor 24V – backrest
16	1	Cable – 200 mm

Example:

Needed: New cable for backrest motor (identical for 2- and 4-sectioned mattress supports).

Action: This cable is only available in the “Allround cable package” (pg. 4, 8, 12, 18 or 22) or in the “Backrest Motor package” (pg. 4, 8 or 12).

Information about using this list

If you choose the “Allround cable package” you will get all cables available for the Invacare® Etude bed.

If the “Backrest Motor package” is chosen you will get the cable together with the backrest motor, a lock cam and two pipe pins.

NOTE: Only the 4 sectioned mattress support is applicable to the UK

3-sectioned mattress support without tilt EN 1970 / with tilt "EN 1970".

Side 1 af 2

Ident. No.: 3-sectioned mattress support without tilt EN 1970/ with tilt "EN 1970": ETUDE.CX100.M0

January 2006

Control unit package

Ident. No.: I423805

Pos.	Qty.	Description
2	1	Lock cam for control unit
3	1	Control unit 230V – EU, UK, AUS/NZ
4	1	Net cable 4300 mm – EU Mains

Allround cable package

Ident.No.: I450495

Pos.	Qty.	Description
2	1	Lock cam for control unit
4	1	Net cable 4300 mm – EU Mains
	2	Cable – 1900 mm
16	1	Cable – 200 mm
	1	Cable – 1250 mm

Allround cable package

Ident.No.: I450494

Pos.	Qty.	Description
2	1	Lock cam for control unit
4	1	Net cable 4300 mm – UK Mains
	2	Cable – 1900 mm
16	1	Cable – 200 mm
	1	Cable – 1250 mm

Motor package

Ident.No.: I432658

Pos.	Qty.	Description
2	1	Lock cam for control unit
5	2	Pipe pin ø10x50
6	1	Motor 24V LA27
16	1	Cable 200 mm

Raise fitting package

Ident.No.: I427773

Pos.	Qty.	Description
8	1	Pipe pin ø6x32
9	1	Raise fitting
10	1	Pipe pin ø8x40

Plastic package

Ident.No.: I427774

Pos.	Qty.	Description
1	2	Handle for mattress, low
2	1	Lock cam for control unit
7	1	Plastic handle
15	2	Butt knob, grey


Insertion tube package

Ident.No. I427775

Pos.	Qty.	Description
11	4	Thumb screw M8x13
12	2	Insertion tube

Single numbers

Pos.	Ident.No.	Qty.	Description
13	I423814	1	Hand control box with tilt
14	I423845	1	Hand control box without tilt
13	I434048	1	Soft control with tilt
14	I434049	1	Soft control without tilt

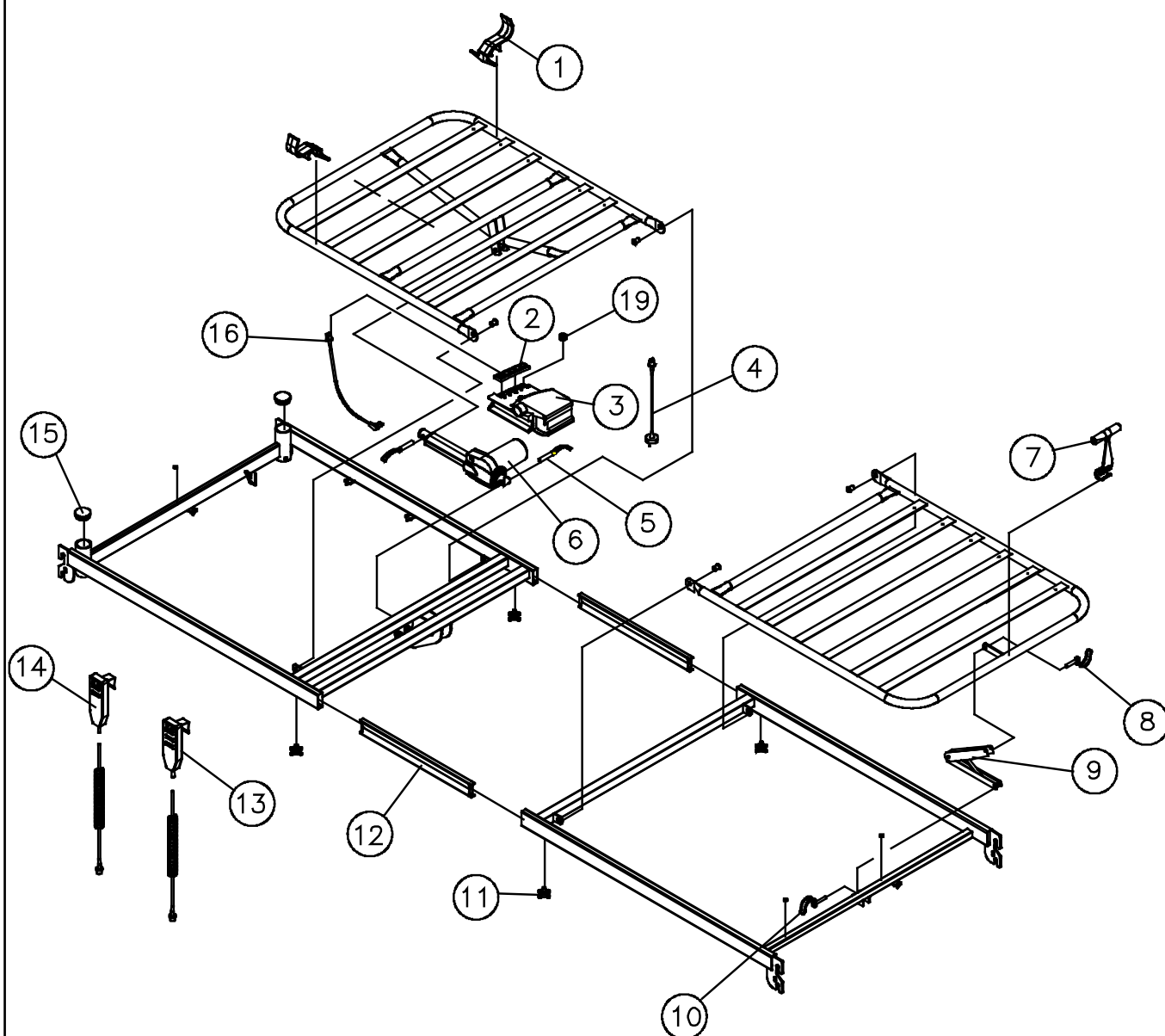

4-sectioned mattress support without tilt EN 1970 / with tilt “EN 1970”.

Page 1 of 4

Ident. No.: 4-sectioned mattress support without tilt EN 1970/ with tilt “EN 1970”: ETUDE.CX100.M0

January 2006

Control unit package

Ident. No.: I423805

Pos.	Qty.	Description
2	1	Lock cam for control unit
3	1	Control unit 230V – EU, UK, AUS/NZ
4	1	Net cable 4300 mm – EU Mains

Allround cable package - EU

Ident.No.: I450495

Pos.	Qty.	Description
2	1	Lock cam for control unit
4	1	Net cable 4300 mm – EU Mains
	2	Cable – 1900 mm
16	1	Cable – 200 mm
11	1	Cable – 1250 mm

Allround cable package - UK

Ident.No.: I450494

Pos.	Qty.	Description
2	1	Lock cam for control unit
4	1	Net cable 4300 mm – UK Mains
	2	Cable – 1900 mm
16	1	Cable – 200 mm
11	1	Cable – 1250 mm

Motor package

Ident.No.: I432674

Pos.	Qty.	Description
2	1	Lock cam for control unit
5	2	Pipe pin ø10x50
10	1	Motor 24V LA27
11	1	Cable 200 mm

Backup battery package

Ident.No.: I449687-XXXX

Motor package


Ident.No.: I432658

Pos.	Qty.	Description
2	1	Lock cam for control unit
5	2	Pipe pin ø10x50
6	1	Motor 24V LA27
16	1	Cable 200 mm

Raise fitting package

Ident.No.: I427773

Pos.	Qty.	Description
8	1	Pipe pin ø6x32

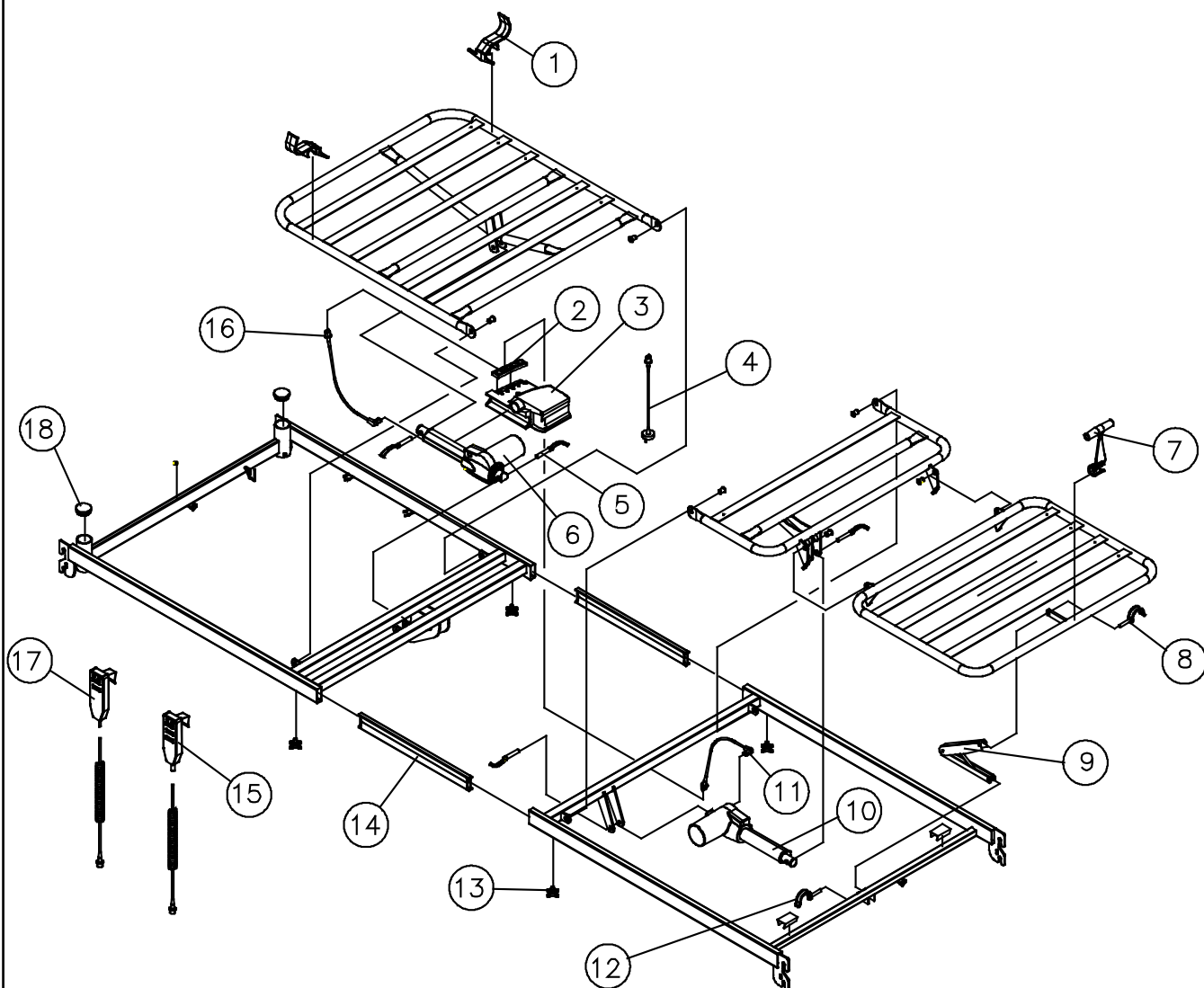

4-sectioned mattress support without tilt EN 1970 / with tilt "EN 1970".

Page 3 of 4

Ident. No.: 4-sectioned mattress support without tilt EN 1970/ with tilt "EN 1970": ETUDE.CX100.M0

January 2006

Plastic package

Ident.No.: I427774

Pos.	Qty.	Description
1	2	Handle for mattress, low
2	1	Lock cam for control unit
7	1	Plastic handle
18	2	Butt knob, grey


Insertion tube package

Ident.No. I427775

Pos.	Qty.	Description
13	4	Thumb screw M8x13
14	2	Insertion tube

Single numbers

Pos.	Ident.No.	Qty.	Description
15	I423814	1	Hand control box with tilt
17	I423845	1	Hand control box without tilt
15	I434046	1	Soft control with tilt
17	I434047	1	Soft control without tilt

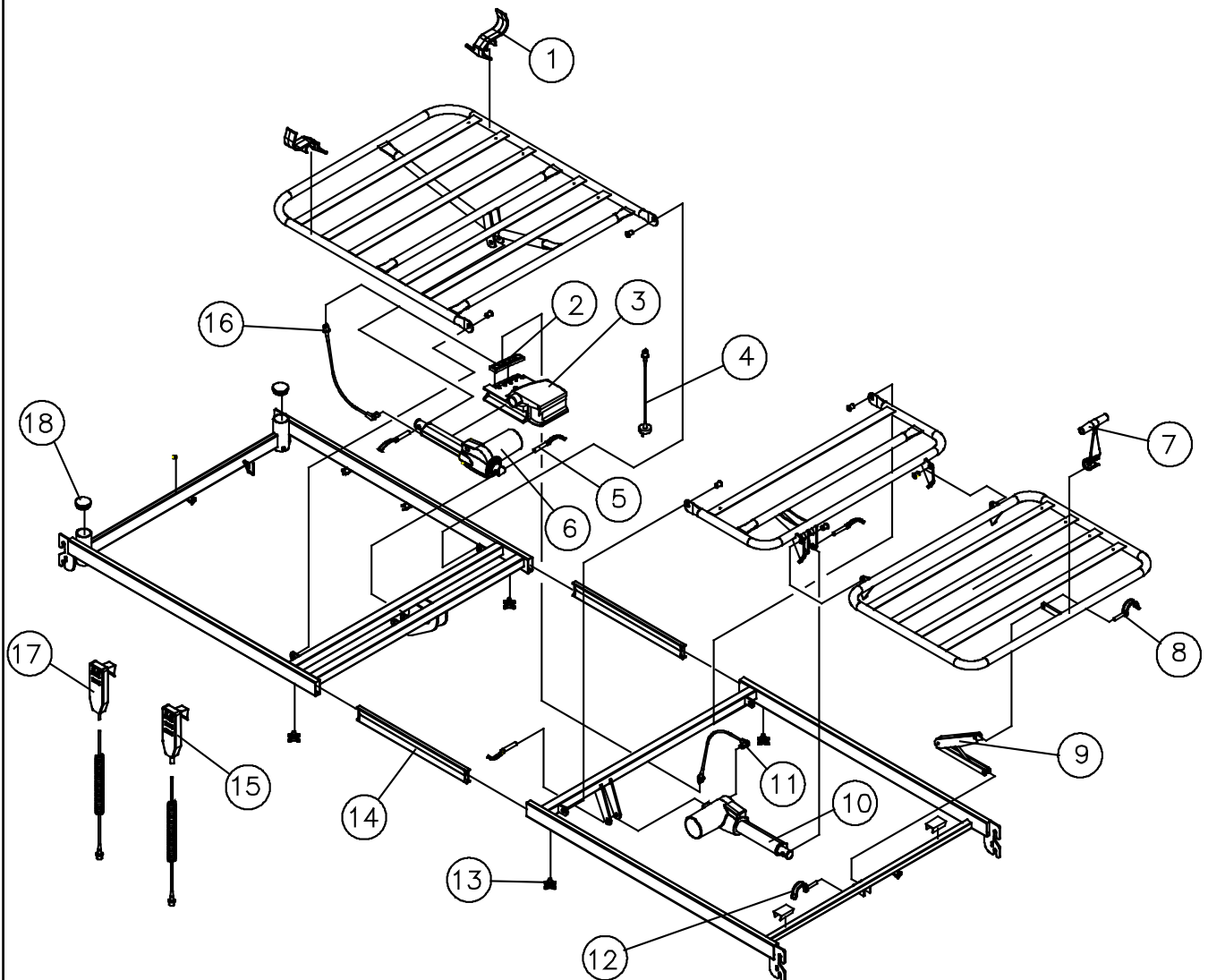

Option.

Page 1 of 2

4-sectioned mattress support without tilt EN 1970 / with tilt "EN 1970". . . .

Ident. No.: 4-sectioned mattress support without tilt EN 1970/ with tilt "EN 1970": ETUDE.Gx100.M0

January 2006

Control unit package

Ident. No.: 1432805

Pos.	Qty.	Description
2	1	Lock cam for control unit
3	1	Control unit 230V – EU, UK, AUS/NZ
4	1	Net cable 4300 mm – EU Mains

Allround cable package EU

Ident.No.: 1450495

Pos.	Qty.	Description
2	1	Lock cam for control unit
4	1	Net cable 4300 mm – EU Mains
	2	Cable – 1900 mm
11	1	Cable – 200 mm
14	1	Cable – 1250 mm

Allround cable package UK

Ident.No.: 1450494

Pos.	Qty.	Description
2	1	Lock cam for control unit
4	1	Net cable 4300 mm – UK Mains
	2	Cable – 1900 mm
11	1	Cable – 200 mm
14	1	Cable – 1250 mm

Backrest Motor package

Ident. No.: 1432658

Pos.	Pcs.	Description
2	1	Lock cam for Control unit
5	2	Pipe pin ø10x50
6	1	Motor 24V – backrest
11	1	Cable – 200 mm

Motor package - thigh and leg section

Ident. No.: 1432674

Pos.	Pcs.	Description
2	1	Lock cam for control unit
12	2	Pipe pin ø10x50
13	1	Motor 24V – leg section
14	1	Cable – 1250 mm

Plastic package

Ident. No.: 1427774

Pos.	Pcs.	Description
1	2	Handle for mattress, low
2	1	Lock cam for Control unit
10	2	Butt knob, grey
15	1	Plastic handle

Insertion tube package

Ident. No.	Pos.	Pcs.	Description
1427775	8	4	Thumb screw M8x20
	7	2	Insertion tube
1432764	8	4	Thumb screw M8x20 UK
	7	2	Insertion tube


Single numbers

Pos.	Ident. No.	Pcs.	Description
9	1423815	1	Hand control box without tilt
9	1434047	1	Soft control without tilt

4-sectioned mattress support without tilt EN 1970 / with tilt "EN 1970".

Ident. No.: 4-sectioned mattress support without tilt EN 1970/ with tilt "EN 1970": ETUDE.Gx100.M0

January 2006

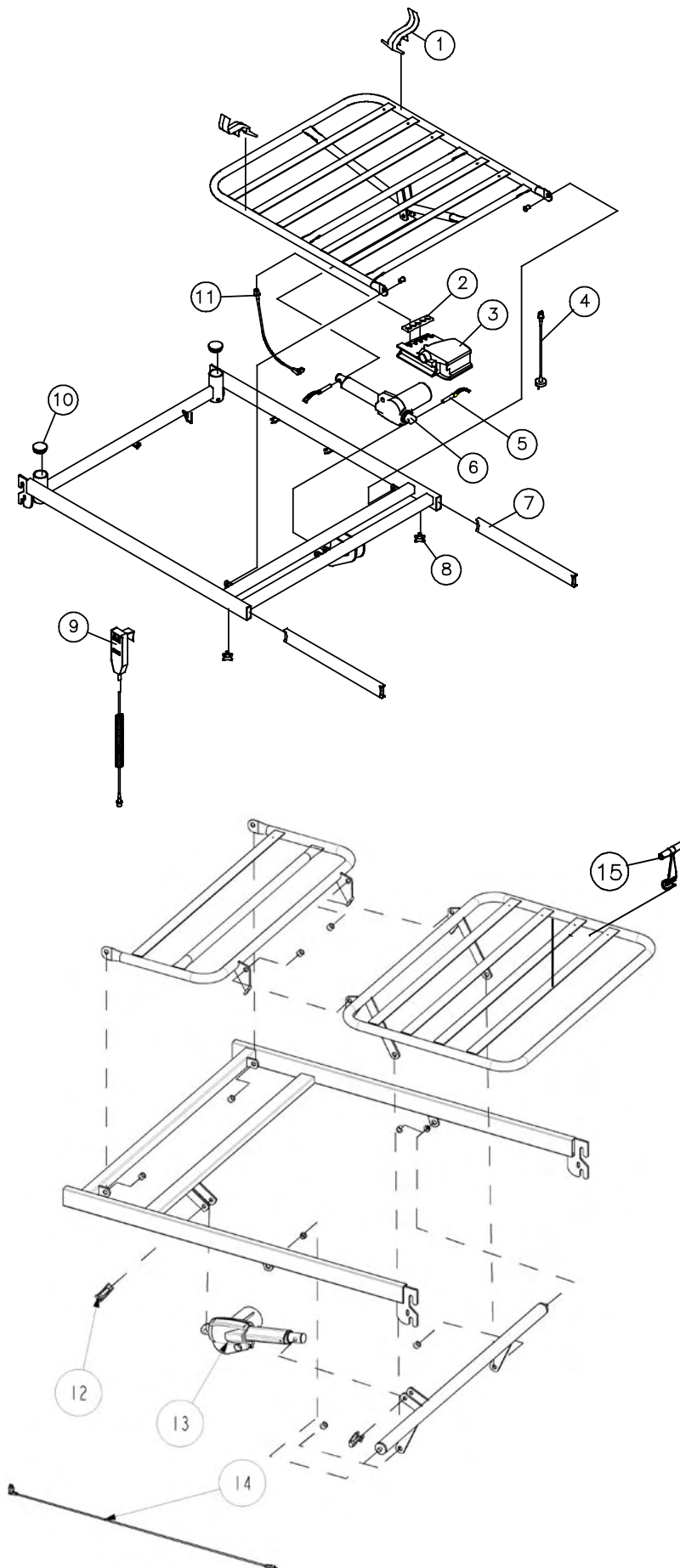

Mattress support extension with cable

Page 1 of 2

Ident. No.: Mattress support extension – 10 cm: 14466379-0152

Ident. No.: Mattress support extension – 20 cm: 14466380-0152

January 2006




Single numbers

<i>Pos.</i>	<i>Ident. no. 10 cm</i>	<i>Ident. no. 20 cm</i>	<i>Qty.</i>	<i>Description</i>
1	1435325-9005	1435325-9005	2	Thumb screw
2	1423811		1	Cable - 1250mm + 125 spiral cable

Mattress support extension with cable

Ident. No.: Mattress support extension – 10 cm: I4466379-0152

Ident. No.: Mattress support extension – 20 cm: I4466380-0152

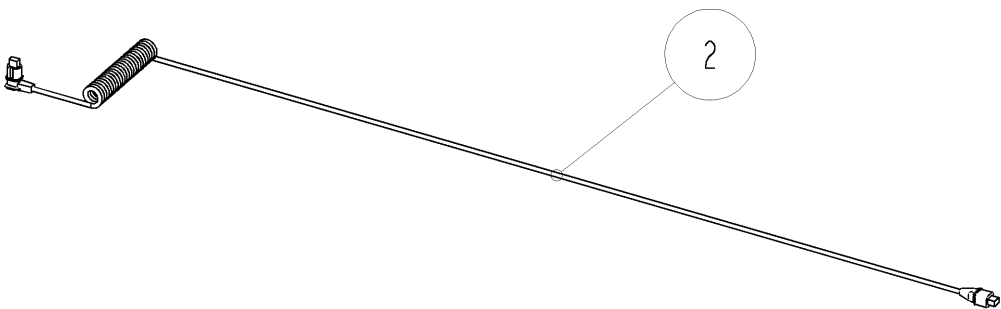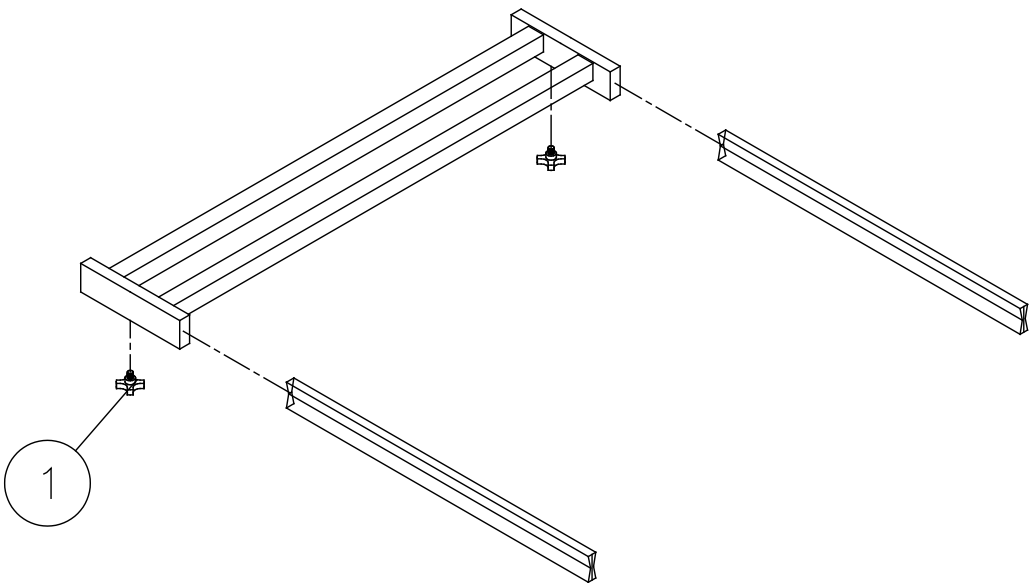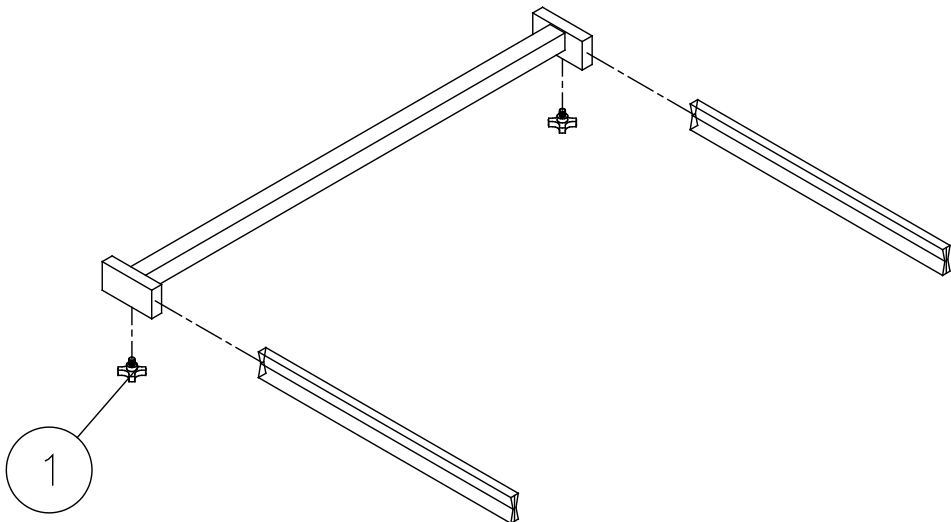

Universal cable package, GB

Ident. No.: I450494

Pos.	Qty.	Description
	1	Lock cam for control unit
	1	Net cable 4300 mm – UK plug
7	2	Cable – 1900 mm
	1	Cable – 200 mm
	1	Cable – 1250 mm

Universal cable package, EU, AUS, NZ

Ident. No.: I450495

Pos.	Qty.	Description
	1	Lock cam for control unit
	1	Net cable 4300 mm – EU, AUZ, NZ plug
7	2	Cable – 1900 mm
	1	Cable – 200 mm
	1	Cable – 1250 mm

Plastic package

Ident. No.: I443960

Pos.	Qty.	Description
8	4	RohrEndstopfen

Motor cover package

Ident. No.: I448792

Pos.	Qty.	Description
3	2	Screw, taptite M4x6 mm
9	2	Cover

Motor package

Ident. No.: I443986

Pos.	Qty.	Description
5	2	Pipe pin ø10x50
6	1	Motor 24V LA27
7	1	Cable 1900 mm

Lock package


Ident. No.: I432683

Pos.	Qty.	Description
1	4	Lock ø52xø37.5x24

Castor package

Ident. No.: I432684

Pos.	Qty.	Description
2	4	Screw M12x30
4	4	Castor ø100 mm TENTE

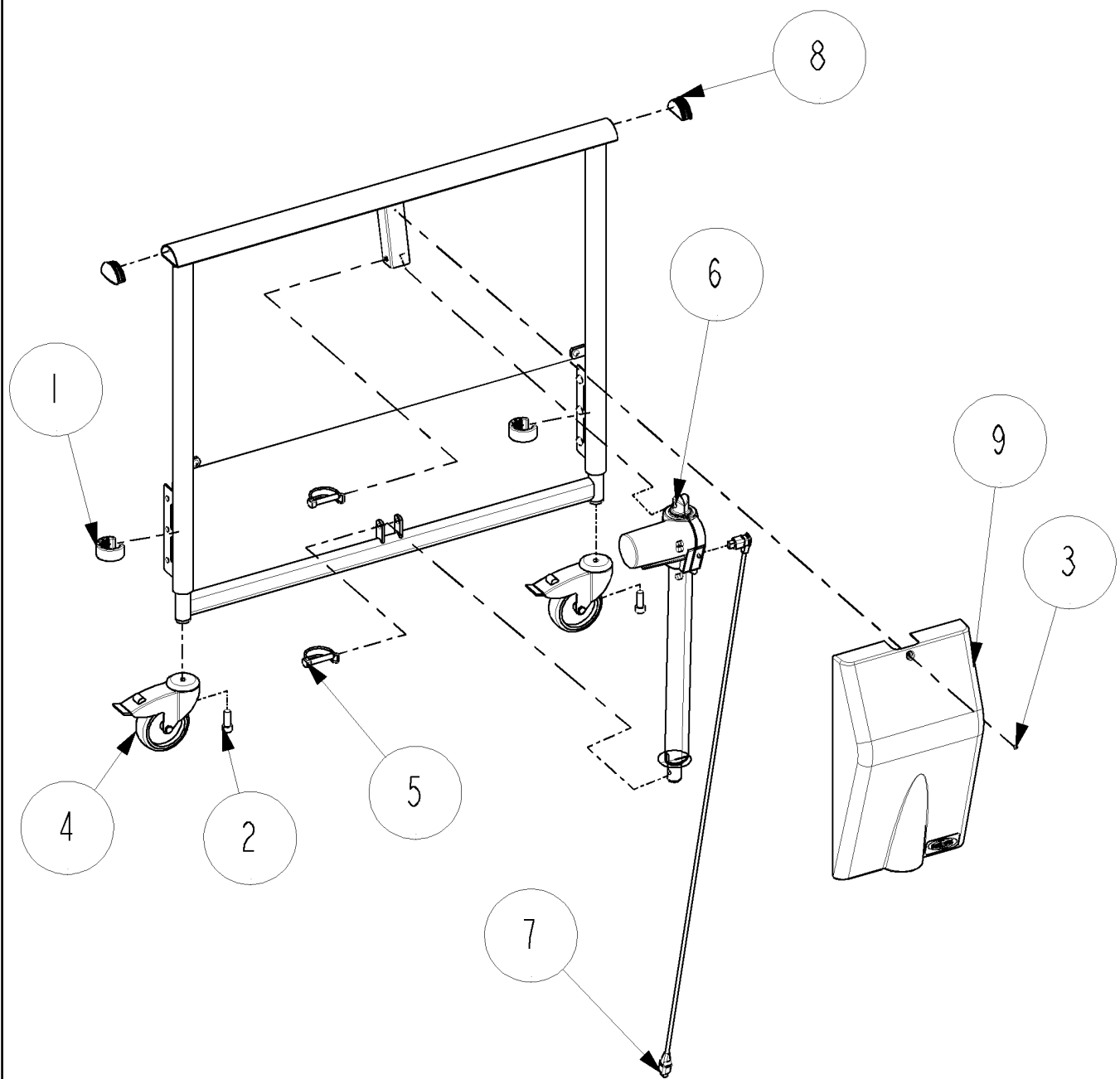

Bedend supplement for Duo wood		Before 20/05	After 21/05	Page 1 of 2
Wood package complete for Mona/Lisa: Ident. No.: Siderail wood - Scale Mona: Ident. No.:		1441497-0101 1443811-0101	1478731-0101 1478732-0101	January 2006
Duo wood supplement package				
Ident. No.: 1443957				
Pos.	Qty.	Description		
4	2	Cross tube		
5	4	End stop		
6	8	Screw M6x25		
Plastic package				
Ident. No.: 1443959				
Pos.	Qty.	Description		
5	4	End stop		
6	8	Screw M6x25		
Single numbers.				
Pos.	Before 20/05	After 21/05	Qty.	Description
2	1441216-0101	1446289-0101	1	Guiding stock, Rechts
3	1441221-0101	1446290-0101	1	Guiding stock, Links
Slide shoe package				
Pos.	Before 20/05	After 21/05	Qty.	Description
1	1443958	1446293	2	Slide shoe

Bedend supplement for Duo wood

Before 20/05

After 21/05


Page 2 of 2

Wood package complete for Mona/Lisa: Ident. No.:
Siderail wood - Scale Mona: Ident. No.:

1441497-0101
1443811-0101

1478731-0101
1478732-0101

January 2006

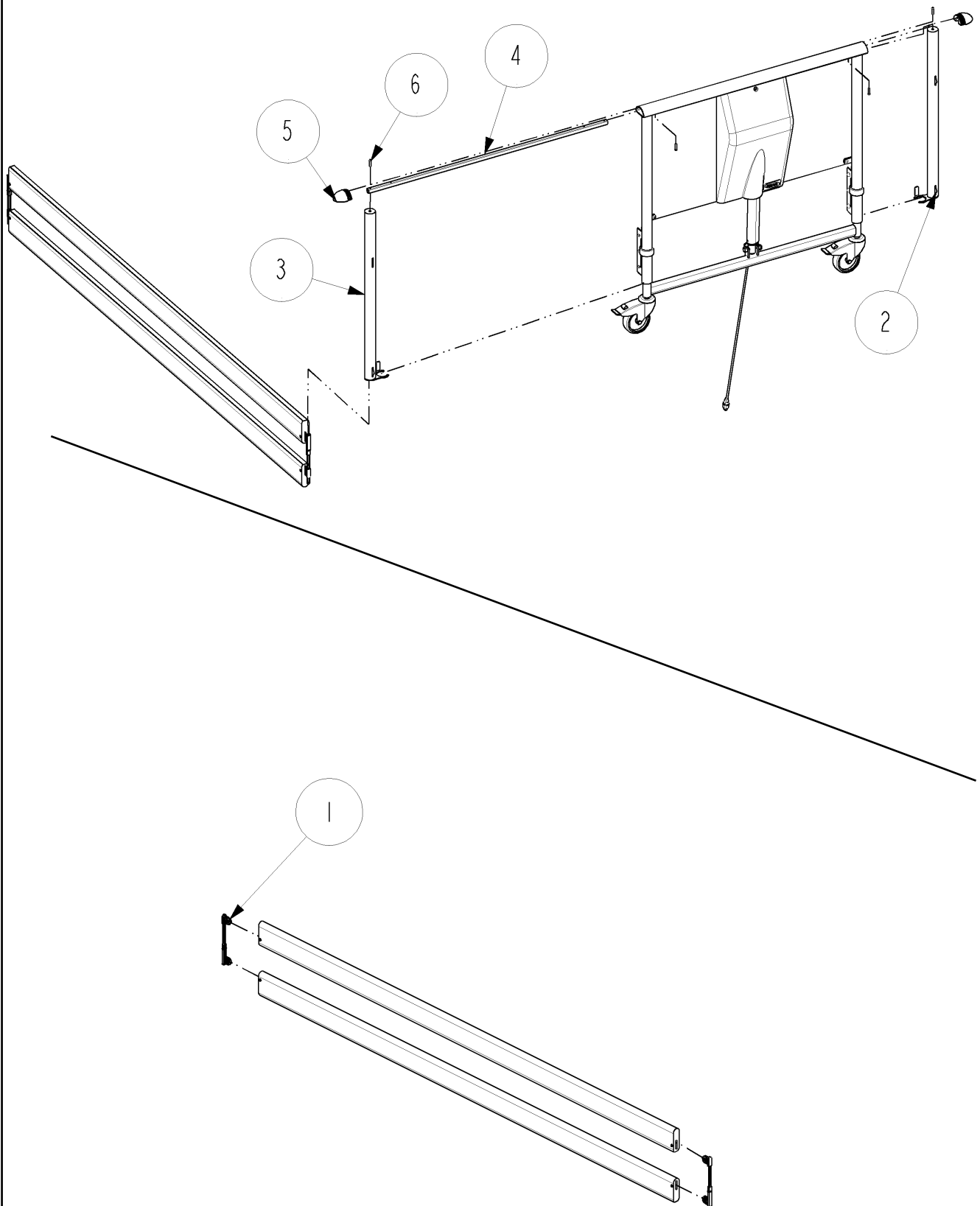

Siderail steel Scala Basic, Medium, Decubi

Page 1 of 2

dent. No.: Siderail steel Scala Basic: I432784-0152 Left/Right, Single: I432782-0152L / I432783-0152R

Ident. No.: Siderail steel Scala Medium: I432781-0152 Left/Right, Single: I432779-0152L / I432780-0152R

Ident. No.: Siderail steel Scala Decubi: I432793-0152 Left/Right, Single: I432791-0152L / I432792-0152R

January 2006

Release package

Ident. No.: 020026

Pos.	Qty.	Description
1	2	Release button
2	2	Spring
3	2	Ratchet

Screw and endstop package

Ident. No.: I443977

Pos.	Qty.	Description
4	3	Ferrule
5	2	Endstop
6	2	Endstop
7	2	Thumb screw M6x30
8	1	Endstop

Siderail steel, Scala Basic, Medium, Decubi


Page 2 of 2

Ident. No.: Siderail steel Scala Basic: 1432784-0152 Left/Right, Single: 1432782-0152L / 1432783-0152R


Ident. No.: Siderail steel Scala Medium: 1432781-0152 Left/Right, Single: 1432779-0152L / 1432780-0152R

Ident. No.: Siderail steel Scala Decubi: 1432793-0152 Left/Right, Single: 1432791-0152L / 1432792-0152R


January 2006

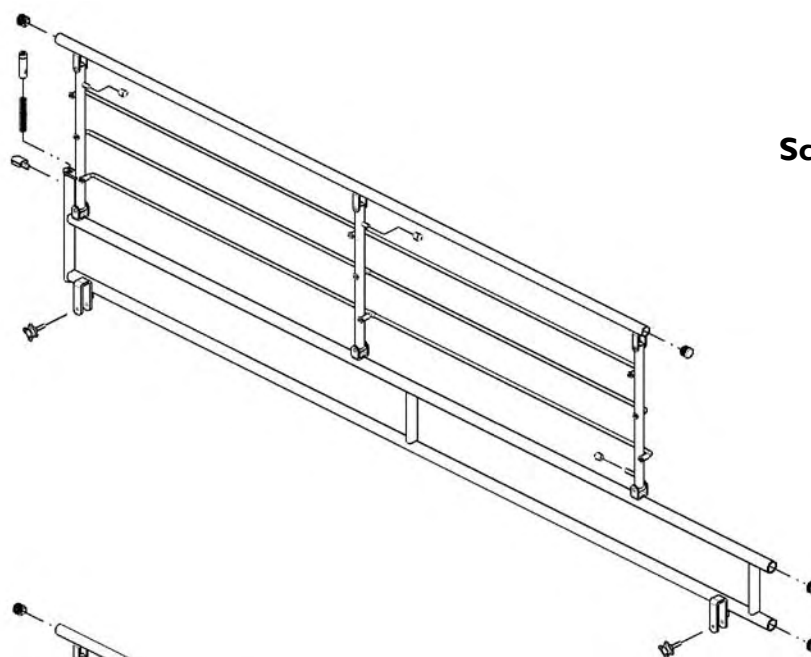

Scala Decubi

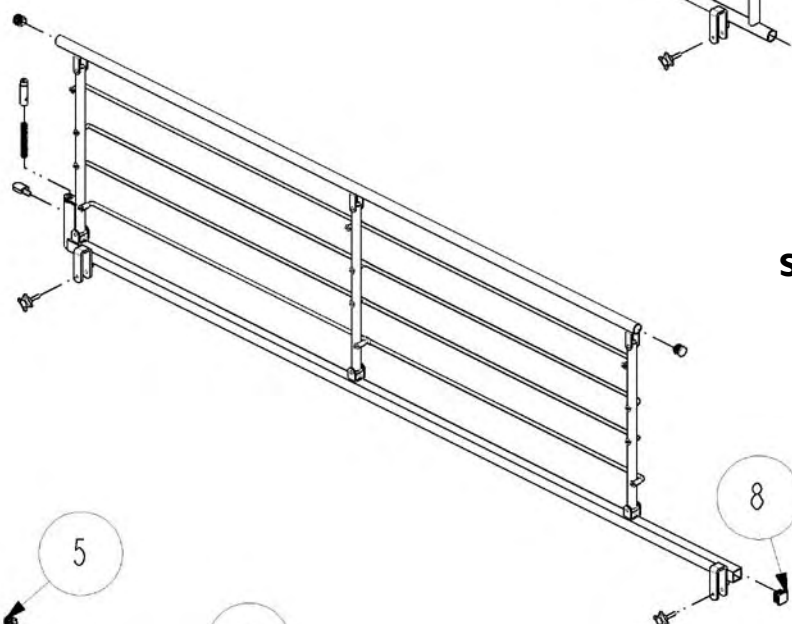

Scala Medium

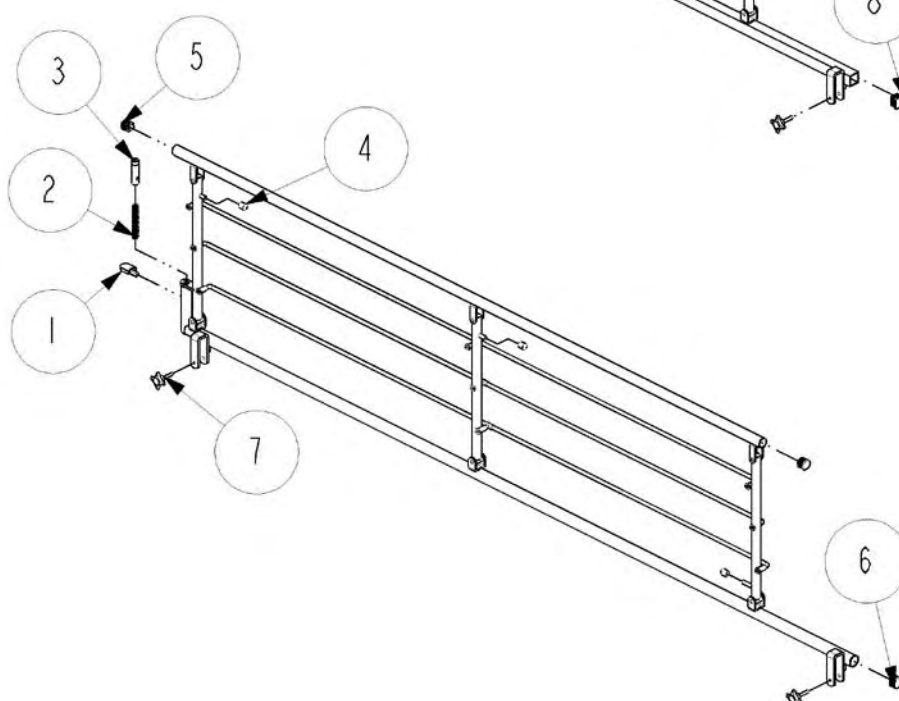

Scala Basic

Siderail steel – 5529

Model 5529: Ident. No.: 50.55290.XX/L+R

Page 1 of 2

January 2006

Locking package

Ident. No.: 020026

Pos.	Qty.	Description
-------------	-------------	--------------------

2	2	Release button
---	---	----------------

3	2	Rachet
---	---	--------

4	2	Spring
---	---	--------


Single numbers

Pos.	Ident. No.. 5529	Qty.	Description
-------------	-----------------------------	-------------	--------------------

1	804399	6	End stop
---	--------	---	----------

5	804386	6	Ferrule
---	--------	---	---------

6	821268	4	Thumb screw
---	--------	---	-------------

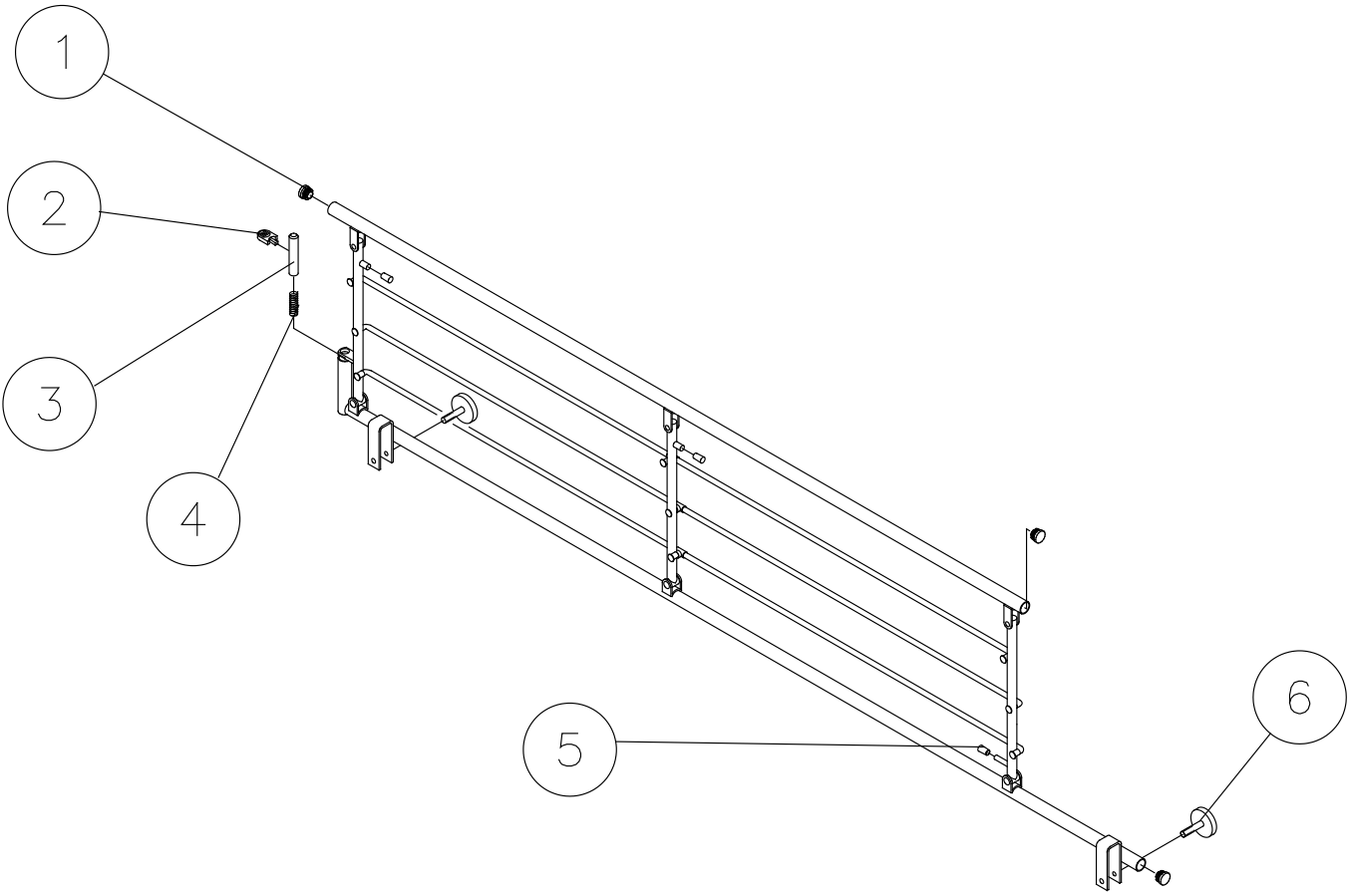

Page 1 of 2

Ident. No.: Support, high 40x95cm |4|75|2-XXXX

January 2006

Ident. No: 020026

Pos.	Qty.	Description
2	2	Thumb screw
3	2	Plastic insert
4	2	Ferrule

Support handles

Page 2 of 2

Ident. No.: Support, high 25x30cm 021964.XX





Ident. No.: Support, high 25x80cm 1417510-XXXX


Ident. No.: Support, high 40x30cm 1417511-XXXX

Ident. No.: Support, high 40x50cm 021963.XX

Ident. No.: Support, high 40x95cm 1417512-XXXX

January 2006

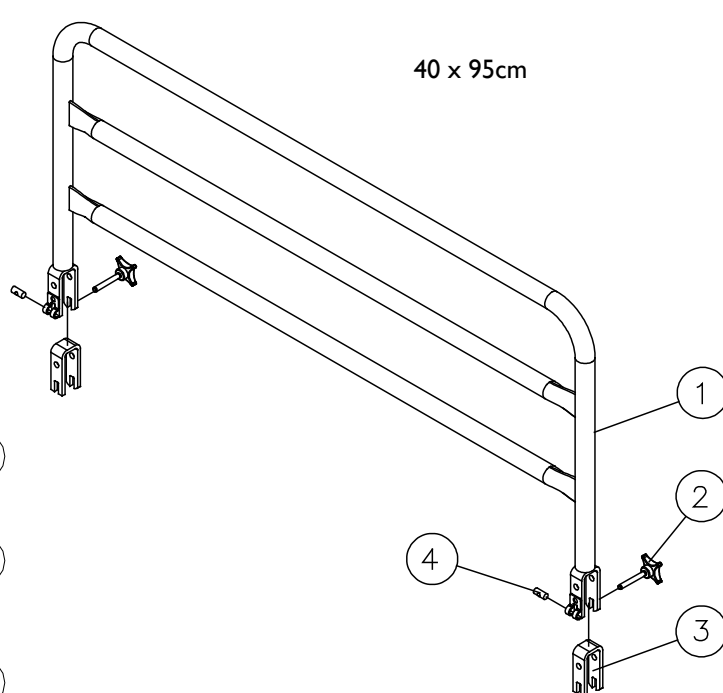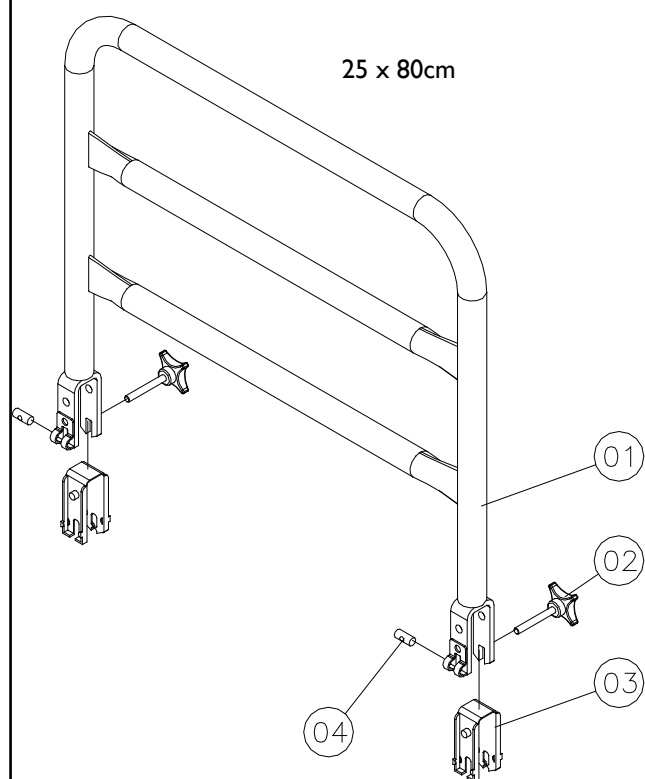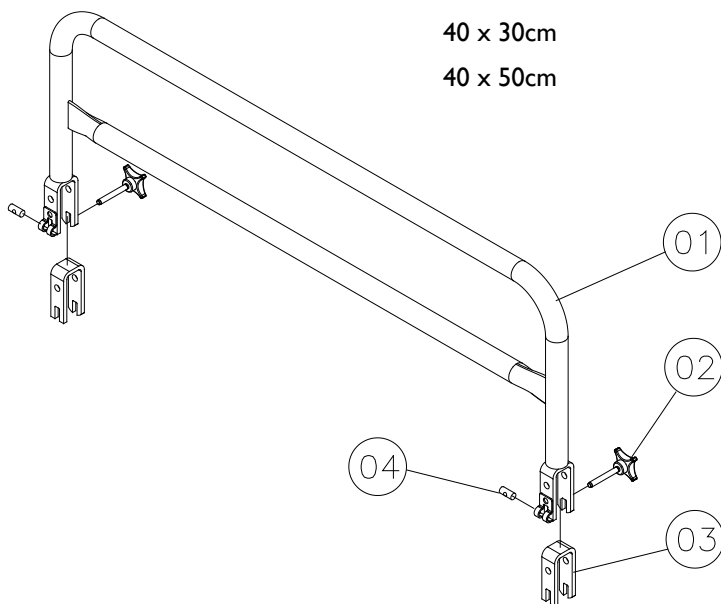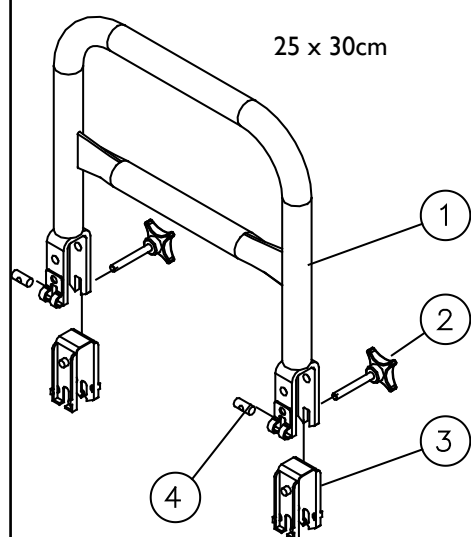

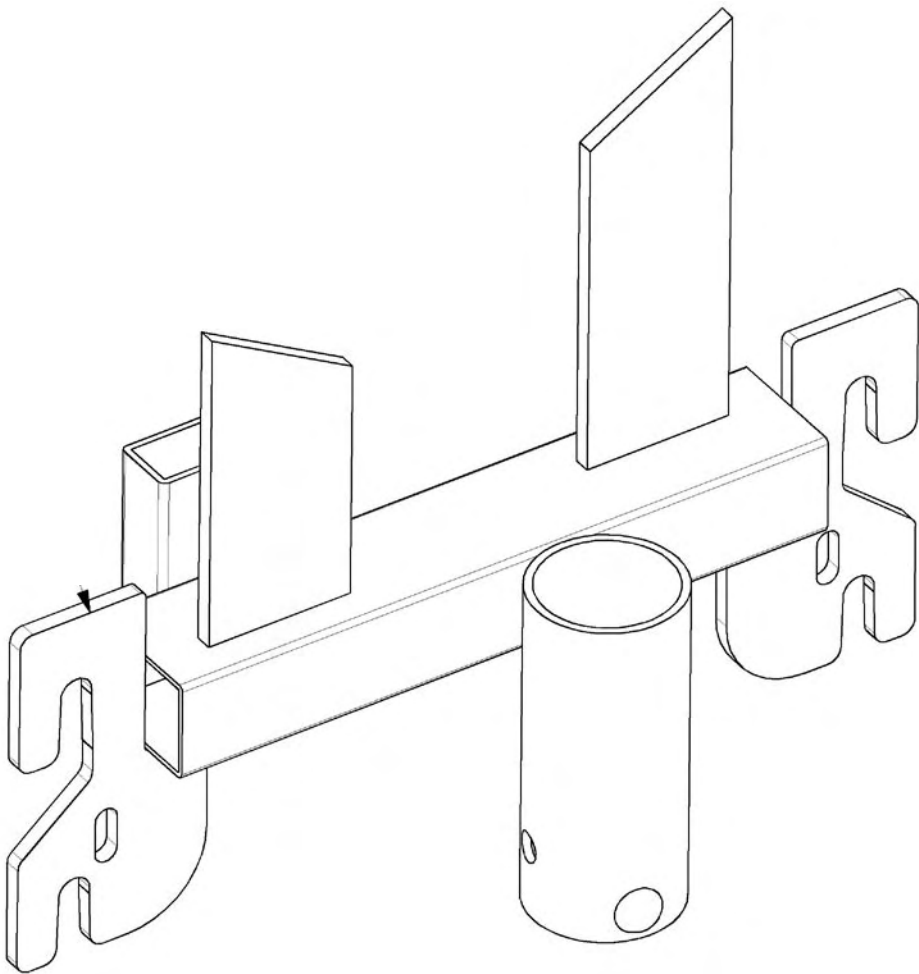

Lifting pole – model 5760

Ident. No.: 50.57600.XX


Page 1 of 2

January 2006

Lifting pole package

Ident. No.: 020028

Pos.	Qty.	Description
1	1	Thumb screw M6x16
2	1	Handle for lifting pole incl. suspension and lock
3	1	Ferrule for lifting pole=

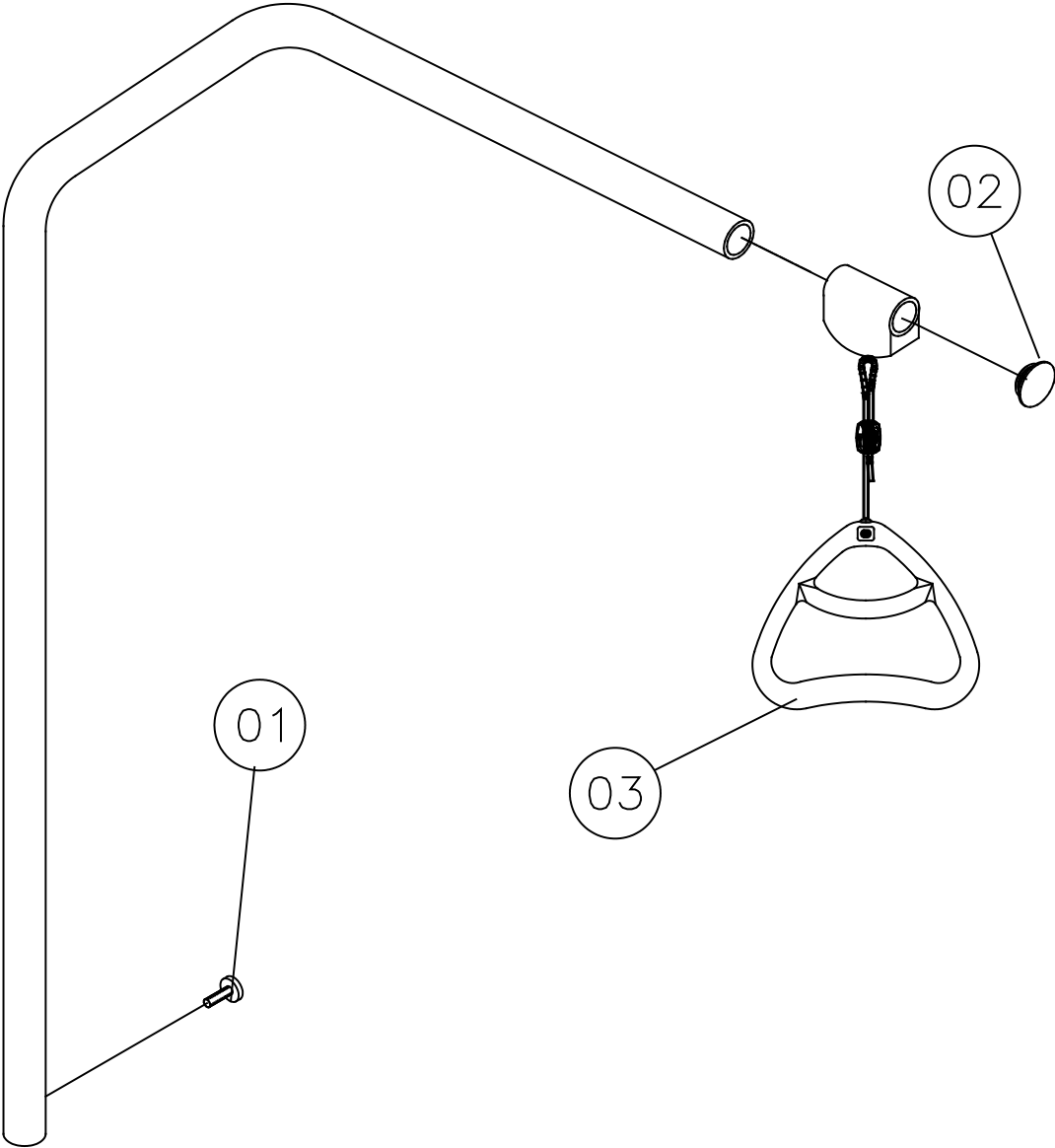



Customer Sales and Service

INVACARE A/S

Sdr. Ringvej 39
DK-2605 Brøndby
Phone: +45 36 90 00 00
Fax: +45 36 90 00 01
www.invacare.dk

INVACARE N.V.

Autobaan 14
B-8210 Loppem, Brügge
Phone: +32 50 83 10 10
Fax: +32 50 83 10 11
www.invacare.be

INVACARE Poirier S.A.S

Route de St Roch (RD 36)
F-37230 Fondettes
Phone: +33 2 47 62 64 66
Fax: +33 2 47 42 12 24
www.invacare.fr

INVACARE AB

Fagerstagatan 9 / Box 66
S-163 91 Spånga
Phone: +46 8 761 70 90
Fax: +46 8 761 81 08
www.invacare.se

INVACARE B.V.

Celsiusstraat 46
NL-6716 BZ Ede
Phone: +31 318 69 5 757
Fax: +31 318 69 5 758
www.invacare.nl

INVACARE MECCSAN S.R.L.

Via dei Pini 62
I-36016 Thiene (VI)
Phone: +39 0445 380059
Fax: +39 0445 380034

INVACARE AS

Grensesvingen 9
P.O. Box 6230 / Etterstad
N-0603 Oslo
Phone: +47 22 57 95 00
Fax: +47 22 57 95 01
www.invacare.no

INVACARE Deutschland GmbH

Kleiststraße 49
D-32457 Porta Westfalica
Phone: +49 (0) 57 31 / 754-0
Fax: +49 (0) 57 31 / 754-52 191
www.invacare.de

INVACARE LTD

South Road
Bridgend Industrial Estate
UK-Bridgend, CF31 3PY
Phone: +44 1 656 664 321
Fax: +44 1 656 667 532

INVACARE S.A.

Cl Areny s/n
Poligon industrial de Celrà
17460 Celrà (Girona-Spain)
Tel: +34 972 493200
Fax: +34 972 493220

INVACARE Lda

Rua Senhora de Campanhã 105
4369-001 Porto
Phone: +351 22 510 59 46/47
Fax: +351 22 510 57 39

Manufacturer:

INVACARE EC-Høng A/S

Østergade 3
DK-4270 Høng
www.invacarebeds.dk

Invacare® EC-Høng A/S
Ident. no.: 1449716
Version 02 01.2006



QUALITY SYSTEM
DS/EN ISO 9001
DS/EN ISO 13485