

Softform® Mattresses



Yes, you can.®

Instructions for Use

- Remove all packaging before use.
- Place the Softform mattress directly onto a bed frame.
- To ease storage and handling the Softform mattresses are compact rolled. Whilst this in no way effects the performance or longevity of the product, it is advisable that the product be unrolled within 21 days of receipt. When the mattress is unrolled for the first time, allow to recover for a minimum of 1 hour before use (depending on room temperature), to optimise performance.
- Apart from bed sheets, where possible avoid the use of additional covers or padding between the patient and mattress. This may reduce the product's effectiveness.
- The inner foams should be regularly inspected. If the cover becomes damaged then the mattress should not be used until a new cover is fitted.
- When fitting a cover, care must be taken to ensure that the corners of the foam core are positioned correctly into the corners of the cover.

Product Description

The Invacare® range of Softform mattresses offer exceptional pressure reducing qualities, specific to the individual needs of the body. By promoting even weight distribution, it reduces tissue pressure at potentially vulnerable areas.

The heavy duty water-resistant cover provides a vapour-permeable, two-way stretch surface, to promote patient comfort and to maximise the effectiveness of the foam core.

If you require more information on the Softform mattress range please telephone the sales department at Invacare on +44 (0) 1656 647 327 who will be happy to help you.

Repositioning

It is very important for the patient to reposition themselves, or to be repositioned, on a regular basis. This relieves pressure, helps prevent both tissue compression and potential pressure ulcer formation.

Cleaning and Care

The cover may be wiped down using a solution of neutral detergent and hot water applied with a single-use wipe, or it can be laundered at 71°C. Cleaning agents containing phenols or alcohols should not be used.

The cover should be thoroughly drip-dried before being refitted to the mattress. Do not tumble dry. For heavy soiling, the cover may be laundered at 71°C with a dilute detergent solution.

In the event of the cover becoming contaminated, please ask your Infection Control Officer for advice. Alternatively, contact Invacare for information on decontamination protocol.

Quality and Flame Retardancy

Quality is fundamental to the company's operation, working within the disciplines of ISO 9001:2000 and ISO 13485:2003. All Invacare® products feature the CE mark, in compliance with the Medical Devices Directive.

The foam used to manufacture the Invacare® Softform range of mattresses is independently tested and certified in accordance with BS 5852: 1990 ignition sources 0, 1 and crib 5. The cover is tested to BS7175:1989 and composite test BS7177:1996 ignition sources 0, 1 and crib 5.



Yes, you can.®

Manufactured by Invacare Ltd.

South Road, Bridgend Industrial Estate, Bridgend CF31 3PY UK

Tel: +44 (0) 1656 647327 Fax: +44 (0) 1656 649016 Email: uk@invacare.com

www.invacare.co.uk



In keeping with our policy of continuous improvement, we reserve the right to modify designs without prior notice.
Softform is a registered trademark of Invacare Ltd.

L002032-C