


# DOLOMITE MAXI +

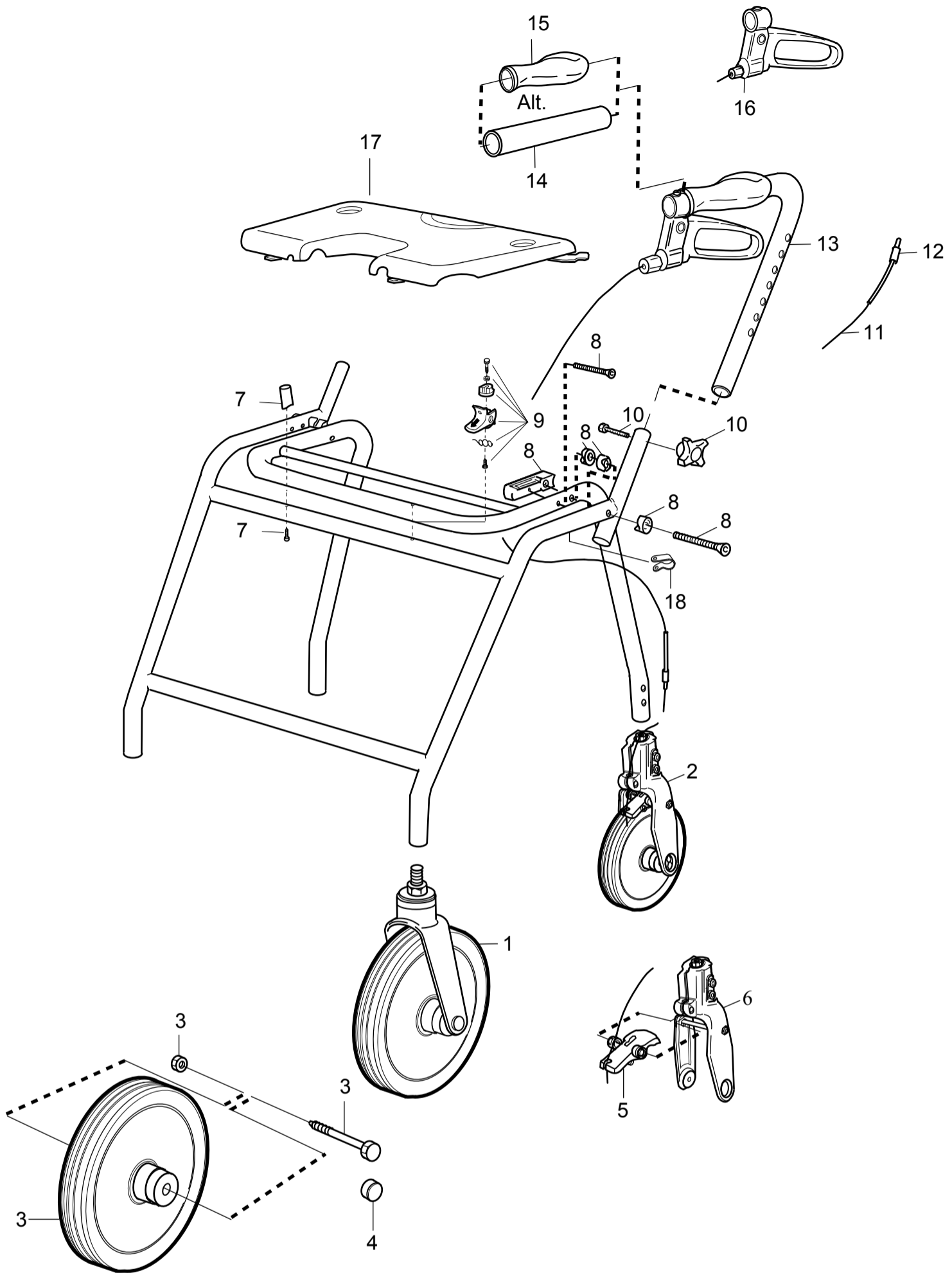

- |    |          |   |
|----|----------|---|
| 1  | 13565-25 | SL II-wheel complete in front fork (pcs)          |
| 2  | 12590-25 | SL II-wheel complete in rear fork (pcs)           |
| 3  | 12609    | SL II-wheel kit, complete (pcs)                   |
| 4  | 70050    | Cover (pcs)                                       |
| 5  | 12620    | Brake flap, complete DABSII (pcs) grey            |
| 6  | 13096    | Rear fork, DABSII (pcs)                           |
| 7  | 12708-M  | Cross bar spacer                                  |
| 8  | 12627-10 | Folding hinge kit (pair)                          |
| 9  | 12707-M  | Folding lock - kit (pcs)                          |
| 10 | 12665-M  | Knob and screw for heightadjustment (pair)        |
| 11 | 80060    | Wire (pcs)  |
| 12 | 80094    | Wire cover L=820 mm (pcs)                         |
| 13 | 13238-L  | Handle kit DABS, left. Incl. grip and wire (pcs)  |
| 13 | 13238-R  | Handle kit DABS, right. Incl. grip and wire (pcs) |
| 14 | 12677    | Soft grip, L=220 mm (pair)                        |
| 15 | 12509    | Anatomic grips, incl. Cover (pair)                |
| 16 | 13464-L  | Brake handle kit, left incl. wire (pcs)           |
| 16 | 13464-R  | Brake handle kit, right incl. wire (pcs)          |
| 17 | 12721-M  | Flip-up seat, compl w. attachmentparts (pcs)      |
| 18 | 70043    | Cable holder (pcs)                                |
|    | 82249    | Operating instructions (pcs)                      |