



Essential[®] Basic

Instructions for Use

- Please read the user instructions carefully before use.
- Remove all packaging before use.
- Place the cushion on wheelchair or chair with grey non-slip base facing down.
- The two-way stretch cover is breathable, waterproof and has a water repellent feature; this allows fluid to ‘roll off’ the cushion fabric, however with sustained periods of moisture the top layer of the cover may absorb fluids. To avoid this, wipe away any liquid as soon as it is identified.
- The cover has a non-slip base which is designed to inhibit the cushions from slipping once positioned correctly.
- The inner foam should be regularly inspected, if the cover becomes damaged, this will allow fluid to soil the inner foam core. Damaged covers and/or soiled foam should be replaced. Replacement foam and covers can be ordered from an Authorised Invacare Dealer.
- When fitting a cover, care should be taken to ensure that the corners of the foam are positioned correctly into the corners of the cover.

Product Description

Invacare Essential Basic Cushions are constructed from a high density foam core (2” or 3” depth) and a two-way stretch, breathable, waterproof and water repellent cover. These cushions are designed to offer additional comfort and support to wheelchair or static seating.

Invacare Essential Basic cushions are not intended for use by individuals who are 'At Risk' of developing pressure ulcers.

Cleaning and Care

The product should be stored in a clean and dry environment. The cover can be machine washed. Use only a dilute detergent solution. Ensure the cover is thoroughly dried before refitting to the foam.

It is necessary to re-activate the fluorocarbon finish after laundry to ensure that it remains effective. This is best achieved by laundering the cover at a temperature in excess of 80°C, then tumble drying on a low heat setting.

Quality and Flame Retardancy

Quality is essential to the company's operation, working within the disciplines of ISO 9001:2000 and ISO 13485:2003. The Invacare Essential Basic Cushion has been fire safety tested to BS 7176-16.

Warranty

Essential Basic 2" cushion – 1 year warranty

Essential Basic 3" cushion – 2 year warranty



Invacare Ltd.

South Road, Bridgend Industrial Estate, Bridgend CF31 3PY UK

Tel: +44 (0) 1656 647327 Fax: +44 (0) 1656 649016

Email: uk@invacare.com Orders: ordersuk@invacare.com Web: www.invacare.co.uk



In keeping with our policy of continuous improvement, we reserve the right to modify designs without prior notice.

Essential Basic is a trademark of Invacare Ltd

This is a retail user manual only.