

Spare Parts Catalogue

Version: 2015/08/11a EN



Invacare[®] **STREAM**



Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

Information & User Manuals

User Manuals REM34 & REM41 Tray/Centre Control	5
User Manuals Stream	6
User Manuals A-Series	7
User Manuals Shark2	8

Chassis

Chassis	9
Shrouds	10
Anti-Tipper	11

Seat

Seatcushion	12
Seat Frame & Plate	13
Seat Tilt manually settable	14
Seat angle adjustment, manual	15

Backrest

Backrest	16
----------------	----

Legrests

Legrest Supports	17
Legrest Supports	18
Legrests 80°	19
Legrests 80° (=> 31.05.2015)	20
Hanger Std. 80° (01.06.2015 =>)	21
Footplate, Standard (01.06.2015 =>)	22
Legrest, manually height adjustable (01.06.2015 =>)	23

Armrests

Armrest	24
---------------	----

Castors

Frontwheel	25
------------------	----

Drive Wheels

Drive Wheel	26
Drive Wheel	27

Propulsion

Motor & Decoupling 6/9 km/h	28
-----------------------------------	----

Electronic

Electronic	29
Powermodules	30
Powermodul	31



Yes, you can.[®]

TABLE OF CONTENTS

Remotes & Brackets	32
Remotes	33
Remotes	34
Remote Brackets	35
Lightboard	36
Lightboard	37
Cables & Accessories	38
Bus-Cables Shark	39
Programming Kits	40

Cables, Chargers & Batteries

Cables	41
Charger	42
Batteries	43

Safety Options

Retaining Belts	44
Lightsystem	45
Lightsystem	46
Rear View Mirror	47
Crash-Kit	48

Seat Accessories

Headrest	49
----------------	----

Options & Accessories

Caneholder	50
------------------	----

User Manuals REM34 & REM41 Tray/Centre Control

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
DA	SP1538022	User Manual REM41 Tray Control Danish		1
EN	SP1538023	User Manual REM41 Tray Control English		1
ES	SP1538024	User Manual REM41 Tray Control Spanish		1
FR	SP1538025	User Manual REM41 Tray Control French		1
IT	SP1538026	User Manual REM41 Tray Control Italian		1
NL	SP1538027	User Manual REM41 Tray Control Dutch		1
NO	SP1538028	User Manual REM41 Tray Control Norwegian		1
PT	SP1538029	User Manual REM41 Tray Control Portuguese		1
SV	SP1538030	User Manual REM41 Tray Control Swedish		1
CS	SP1538031	User Manual REM41 Tray Control Czech		1
SL	SP1538032	User Manual REM41 Tray Control Slovenian		1
DE	SP1541402	User Manual REM41 Tray Control German		1

User Manuals Stream

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DE 1529250	User Manual Stream German		1
	ES 1530285	User Manual Stream Spanish		1
	FR 1530286	User Manual Stream Francais		1
	CZ 1531148	User Manual Stream Czech		1
	PL 1531149	User Manual Stream Polish		1
	HR 1531150	User Manual Stream Croatian		1
	LT 1531151	User Manual Stream Lithuanian		1
	ET 1531152	User Manual Stream Estonian		1
	HU 1531153	User Manual Stream Hungarian		1
	RO 1531154	User Manual Stream Romanian		1
	EN 1534065	User Manual Stream English		1
	SV 1534066	User Manual Stream Swedish		1
	BG 1546031	Stream user manual bulgarian		1
	RU SP1549083	User Manual Stream Russian		1
	FI SP1567725	User Manual Stream Finnish		1

User Manuals A-Series

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		


Pos.	Part number	Description	Valid from → until	Qty
	DE 1530207	User Manual A-Series German		1
	ES 1530283	User Manual A-Series Spanish		1
	FR 1530284	User Manual A-Series Francais		1
	PT 1533505	User Manual A-Series Portuguese		1
	EN 1536135	User Manual A-Series English		1

User Manuals Shark2

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		


Pos.	Part number	Description	Valid from → until	Qty
	DE 1451346	User Manual Shark II German		1
	FR 1451347	User Manual Shark II French		1
	SV 1451348	User Manual Shark II Swedish		1
	DA 1451349	User Manual Shark II Danish		1
	NO 1451350	User Manual Shark II Norwegian		1
	NL 1451351	User Manual Shark II Dutch		1
	ES 1451352	User Manual Shark II Spanish		1
	PT 1451353	User Manual Shark II Portuguese		1
	IT 1451354	User Manual Shark II Italian		1
	FI 1451355	User Manual Shark II Finnish		1
	EN 1451356	User Manual Shark II English		1
	SL 1451584	User Manual Shark II Slovenian		1
	CS SP1533297	User Manual Shark2 CS		1
	ET SP1533298	User Manual Shark2 EE		1
	HU SP1533299	User Manual Shark2 HU		1
	LT SP1533300	User Manual Shark2 LT		1
	PL SP1533301	User Manual Shark2 PL		1
	BG SP1546061	User Manual Shark II Bulgarian		1
	RU SP1570186	User Manual Shark II Russian		1
	HR SP1574028	User Manual Shark II Croatian		1
	LV SP1576075	User Manual Shark II Latvian		1

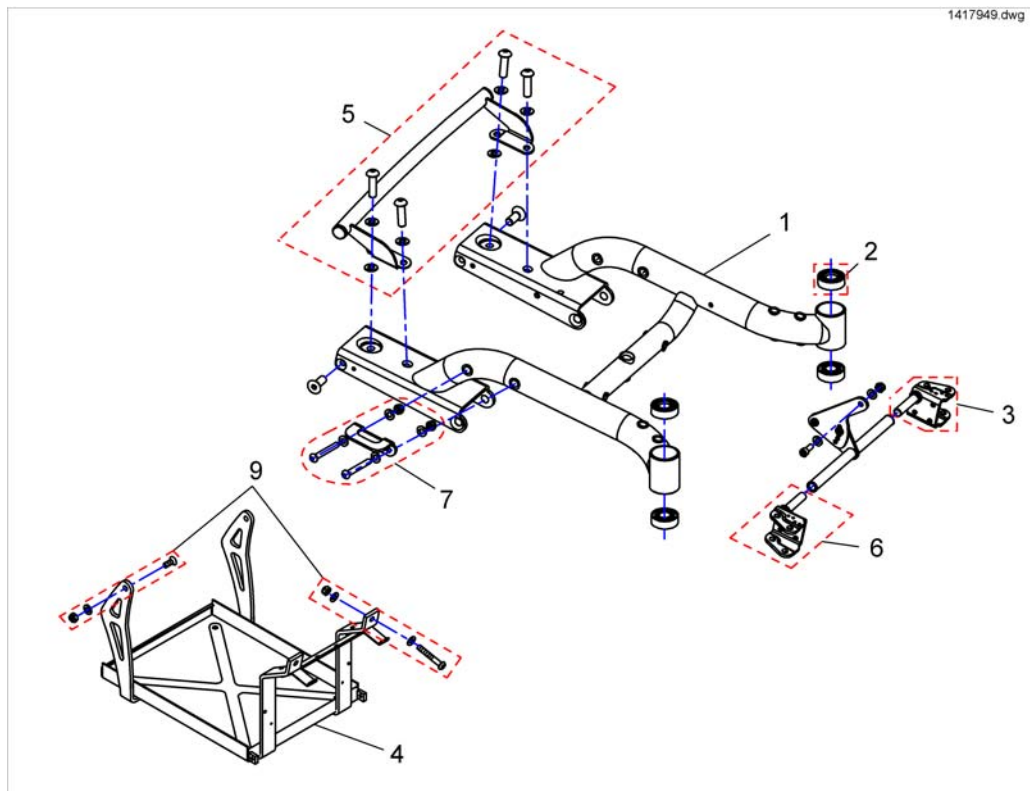Chassis

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530604	Chassis, silver, cpl.		1
2	1419254	Angular Ball Bearing 7202B 2Rs		1
3	1418547-9005	Bearing Plate LH (black)		1
4	1492498	Battery Frame ACS		1
5	SP1530649	Carrier, cpl.		1
6	1418544-9005	Bearing Plate RH (black)		1
7	SP1492647	Taxi Fixation cpl.		1
8	1442551	Fixing Kit Luggage Carrier cpl.		1
9	1442601	Battery Clamp Fixing Kit		1
-	SP1552603	Touch-up paint, silver		1

Shrouds


1442067STR.dwg

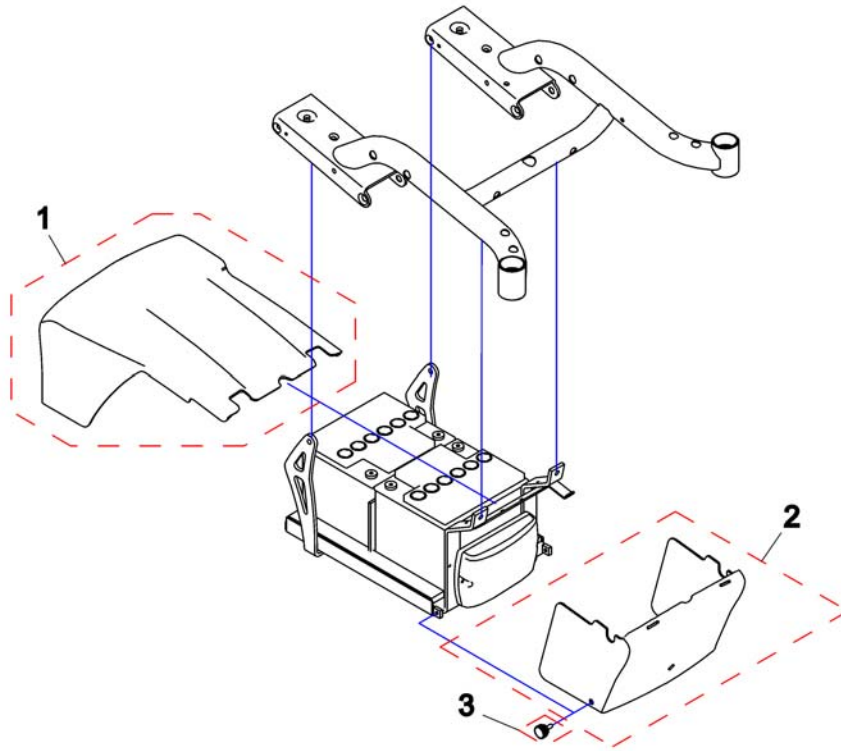

Pos.	Part number	Description	Valid from → until	Qty
1	SP1442599	Battery Cover Rear Complete		1
2	1440684-9006	Battery Cover Front <i>Only usable with SHARK Powermodule - used since CTM Serial-No. 10AMB0201</i>		1
3	1440769	Hand wheel		1


Anti-Tipper

1530701X.dwg

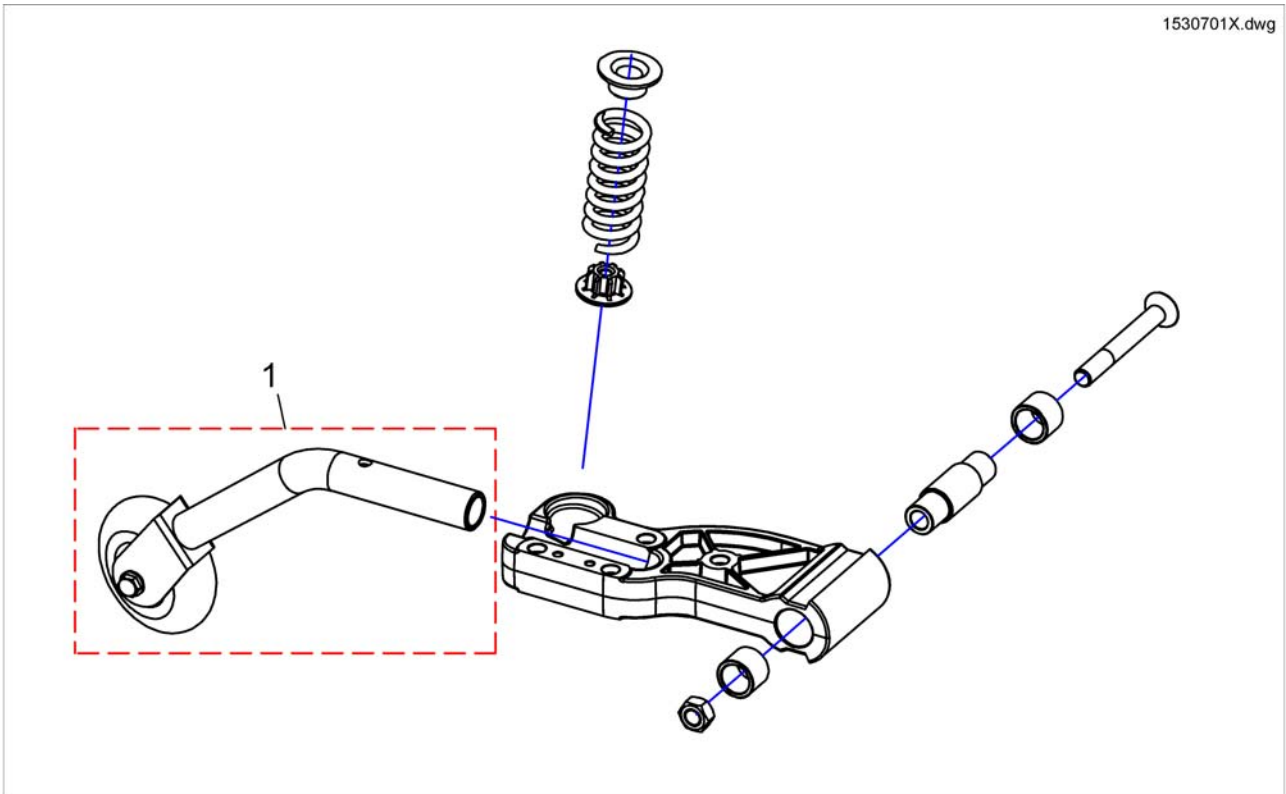


Pos.	Part number	Description	Valid from → until	Qty
1	SP1530701	Anti-Tipper		1

Seatcushion

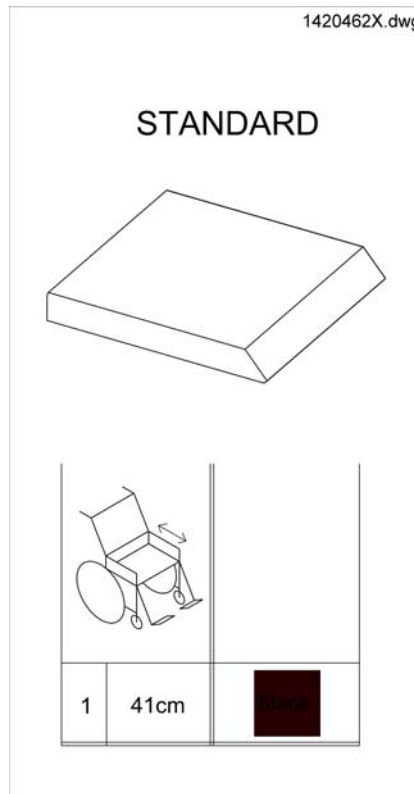


Pos.	Part number	Description	Valid from → until	Qty
1	1420460	Seat Cushion Standard SD 410mm		1

Seat Frame & Plate

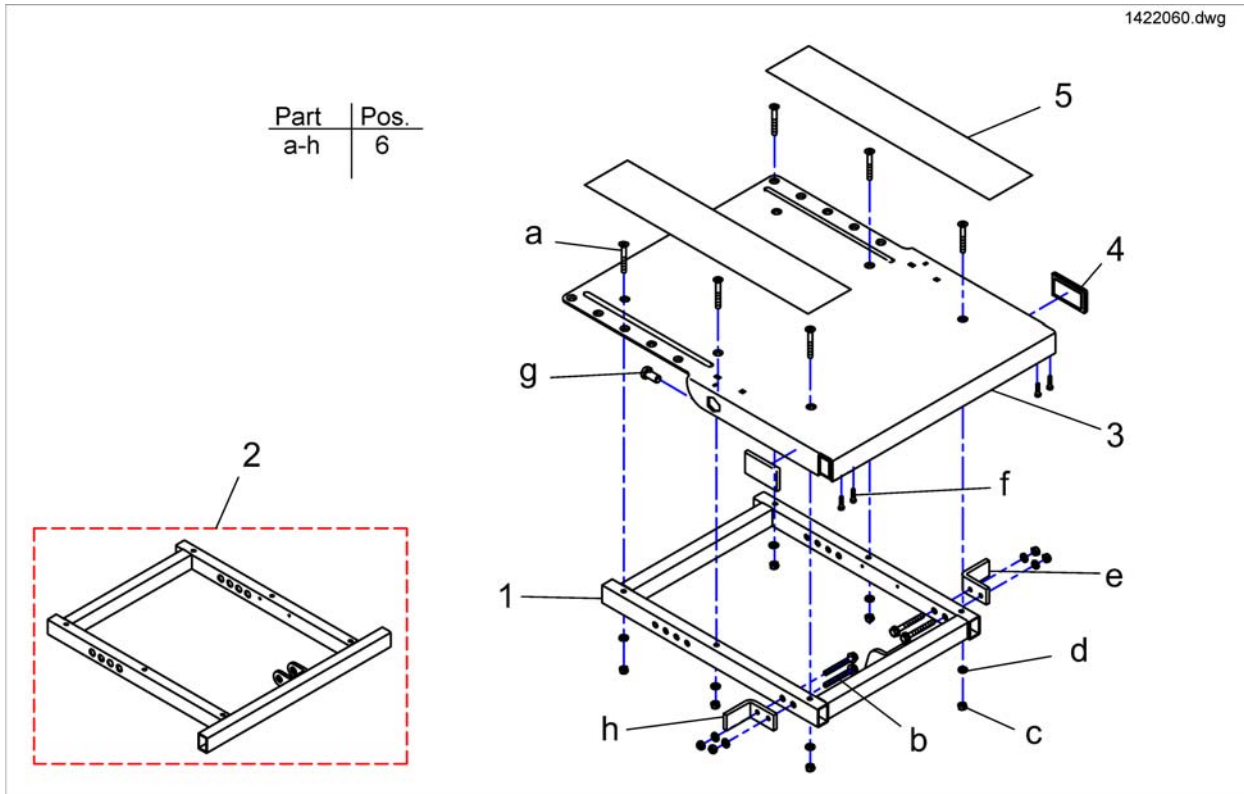

Pos.	Part number	Description	Valid from → until	Qty
2	1442655-9005	Seat Frame For Electrical Tilt		1
3	1419999-9005	Seat Plate		1
4	P31298	Reflector, yellow		1
5	U21087	Velcro Fastener Strip 38 X 400mm Adhesive		1
6	SP1423205	Seat Mounting Kit (One-Piece Version)		1


Seat Tilt manually settable

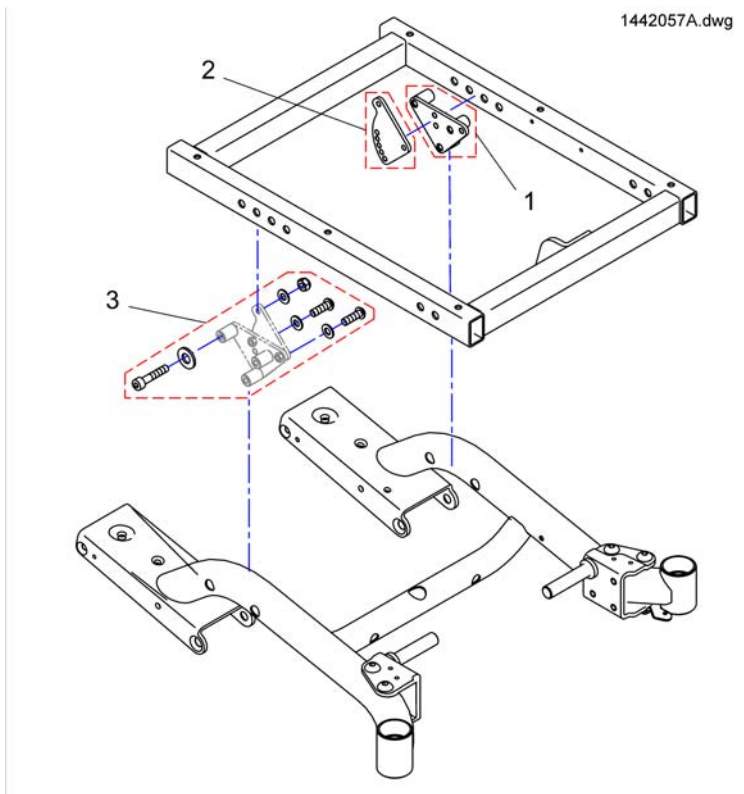

Pos.	Part number	Description	Valid from → until	Qty
1	1440733-9005	Plate Black Right		1
1	1440763-9005	Plate Black Left		1
2	1440721-9005	Fixing Plate		1
3	1442594	Mounting Kit Lifter And Rear Seat Fixation		1


Seat angle adjustment, manual

1442057D.dwg


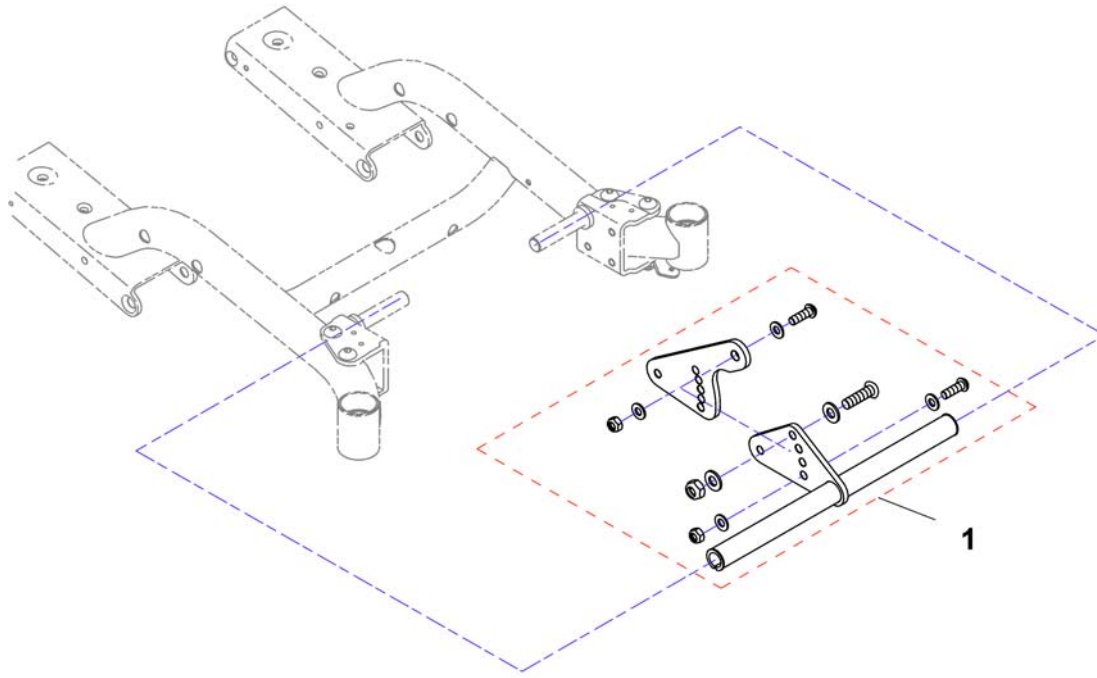


Pos.	Part number	Description	Valid from → until	Qty
1	1449107	Seat Fixation front with manual Seat Tilt, cpl.		1

Backrest

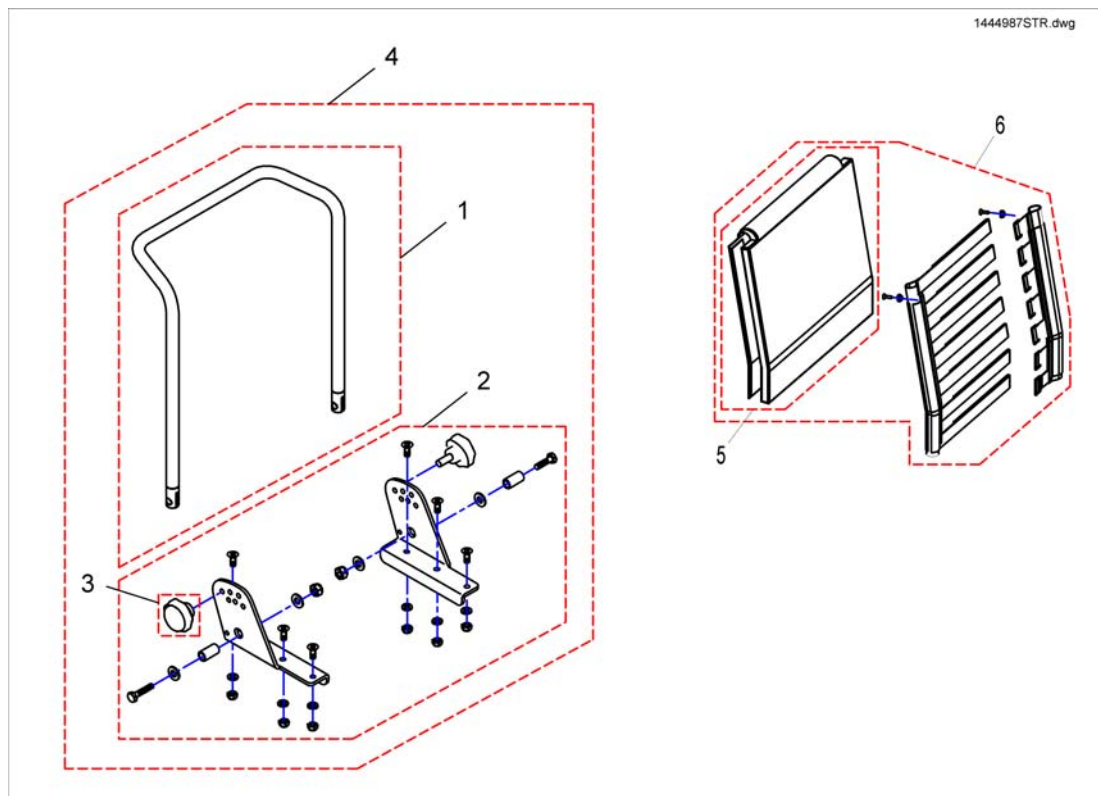




Pos.	Part number	Description	Valid from → until	Qty
1	1419996-9005	Back Tube 480 mm		1
2	SP1423208	Std. Backrest Assembly Kit cpl.		1
3	SP1420444	Star Grip M8		1
4	SP1444987	Back Adjuster Standard, Back Height 48 cm, With Backrest Upholstery,		1
5	1420445	Backrest upholstery, 480mm		1
6	1422062	Sling Back with Backrest upholstery 480 mm		1

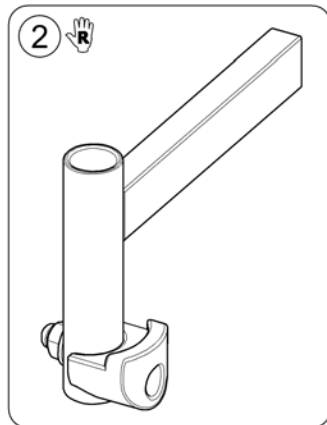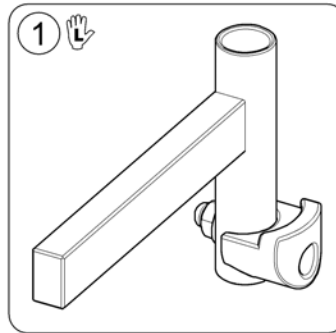

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530703	Legrest Support, Stream 80° - LH		1
2	SP1530704	Legrest Support, Stream 80° - RH		1


Legrest Supports

1530703X.iss

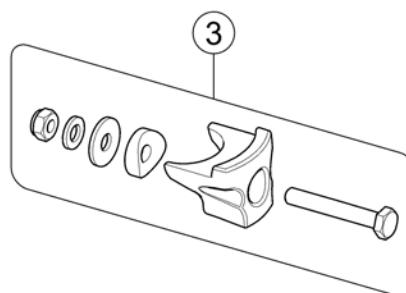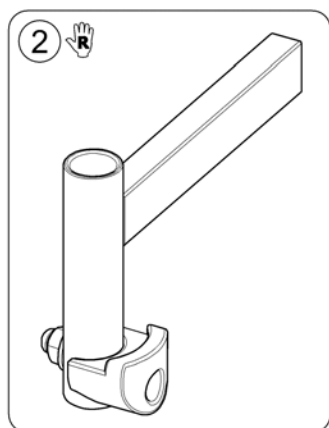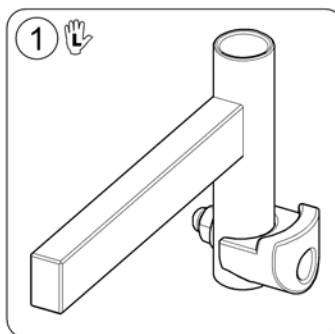


Pos.	Part number	Description	Valid from → until	Qty
1	SP1530703	Legrest Support, Stream 80° - LH	→ 2015-05-31	1
1	SP1441526	Legrest Support Std. 80° / Elevating - LH	2015-06-01 →	1
2	SP1530704	Legrest Support, Stream 80° - RH	→ 2015-05-31	1
2	SP1441527	Legrest Support Std. 80° / Elevating - RH	2015-06-01 →	1
3	SP1588726	Kit, Support Nose	2015-06-01 →	1

Legrests 80°

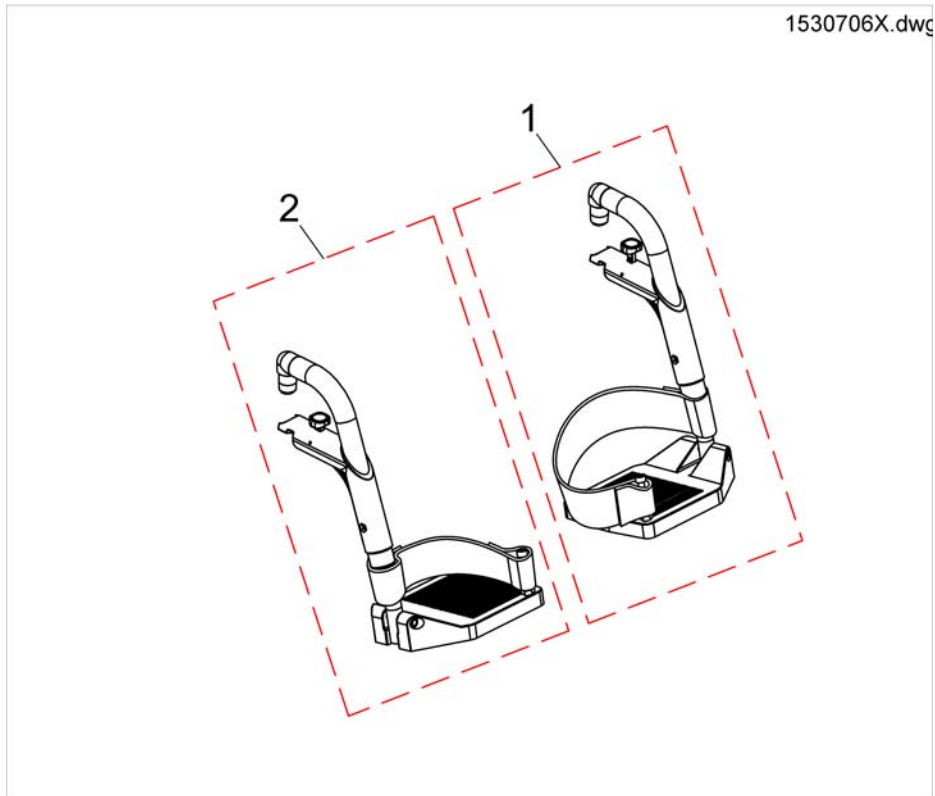

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530706	Legrest LH		1
2	SP1530707	Legrest RH		1


Legrests 80° (=> 31.05.2015)

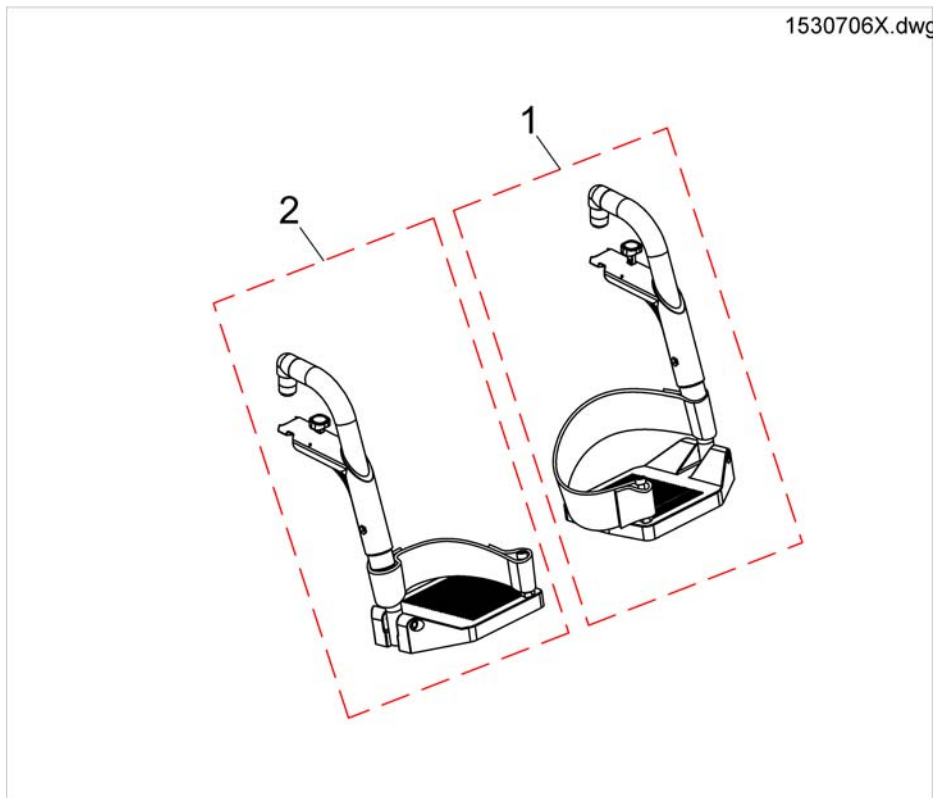

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530706	Legrest LH	→ 2015-05-31	1
2	SP1530707	Legrest RH	→ 2015-05-31	1

Hanger Std. 80° (01.06.2015 =>)


1588852 Rev

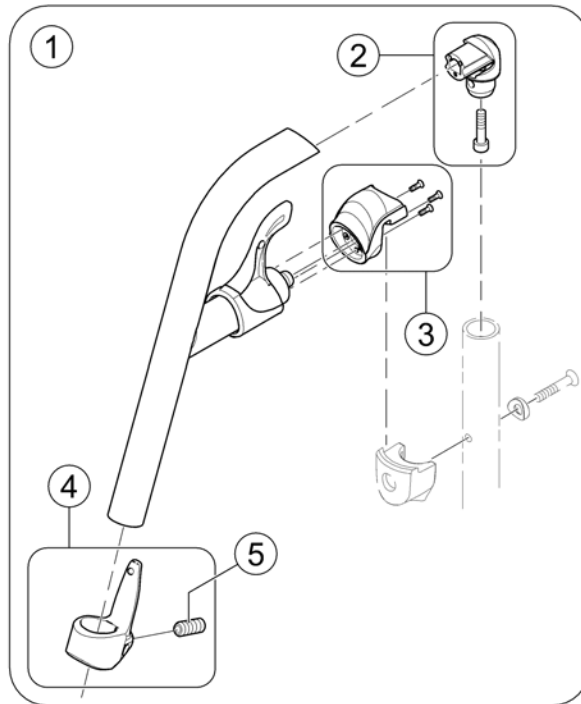

Pos.	Part number	Description	Valid from → until	Qty
1	SP1574349-9005	Hanger Standard 80° cpl. - black		1
2	SP1588727	Kit, Pivot Std. 80° Hanger		1
3	SP1588731	Nose locking Kit		1
4	SP1588730	Calf Strap Holder, cpl.		1
5	1424571	Springscrew		1


Footplate, Standard (01.06.2015 =>)

158854.iss

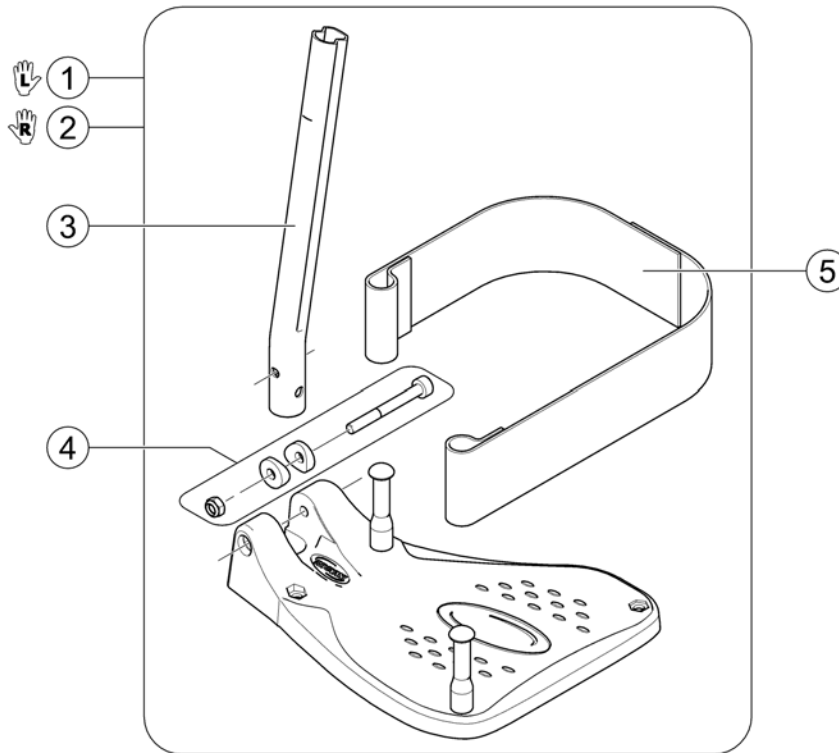


Pos.	Part number	Description	Valid from → until	Qty
1	SP1537944	Footrest Tube with Footplate for Hanger, cpl.43-45,5cm LH		1
2	SP1537942	Footrest Tube with Footplate for Hanger, cpl.43-45,5cm RH		1
3	1434561-0010	Footrest Tube, Short		1
4	1430433	Screw Kit, Footrest Fixed		1
5	SP1442467	Heel Strap cpl.		1

Legrest, manually height adjustable (01.06.2015 =>)

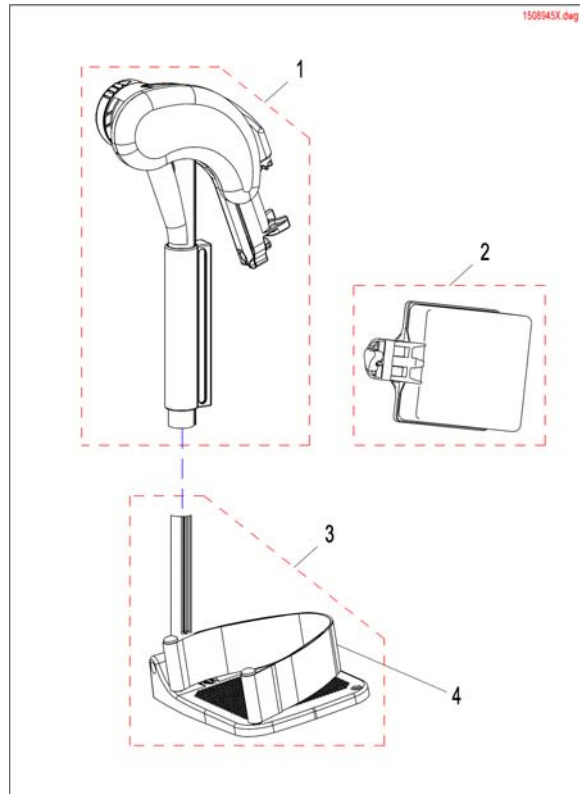

Pos.	Part number	Description	Valid from → until	Qty
1	1513006	HANGER ALU2008, AA, RIGHT, CPL		1
1	1513007	HANGER ALU2008, AA, LEFT, CPL		1
2	1516314	CALF PAD 15 (165x170x80), CPL		1
2	1518532	Calfpad, 19", Dartex black, cpl.		1
3	SP1442853	Lower Legrest RH With Angle Adjustable Footplate, With Heel Strap. Width		1
3	SP1442854	Lower Legrest LH With Angle Adjustable Footplate, With Heel Strap. Width		1
4	1430402	HEELSTRAP COMPLETE		1


Armrest

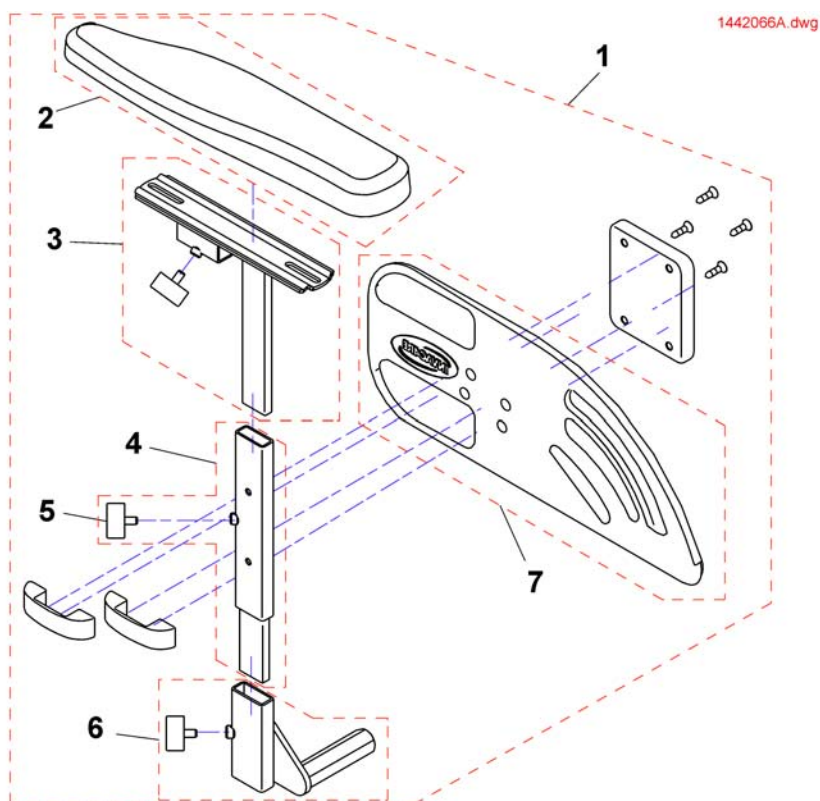

Pos.	Part number	Description	Valid from → until	Qty
a	1450296	Fixing Kit Side Part Plate		1
1	SP1442016	Side Part LH cpl.		1
1	SP1442017	Side Part RH cpl.		1
2	1442618	Armrest Pad With Assembly Kit - LH cpl.		1
2	1442619	Armrest Pad With Assembly Kit - RH cpl.		1
3	1442020-9005	Arm Pad Bracket LH		1
3	1442021-9005	Arm Pad Bracket RH		1
4	1442012-9005	Side Part Adjustment RH		1
4	1442015-9005	Side Part Adjustment LH		1
5	L10024	Wing knobs M6x16		1
6	1442024-9005	Armrest Support Tube LH		1
6	1442025-9005	Armrest Support Tube RH		1
7	1433291	Skirt-Guard (single part) - LH		1
7	1433292	Skirt-Guard (single part) - RH		1


Frontwheel

1530/21A.dwg

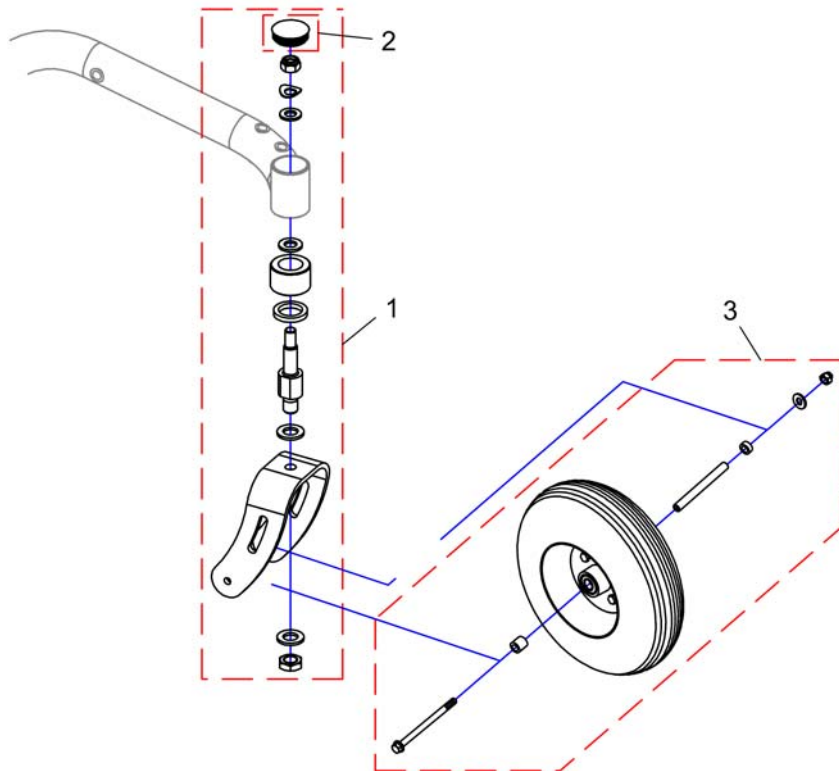


Pos.	Part number	Description	Valid from → until	Qty
-	SP1530722	Bearing-Kit (2 ballbearings & mounting parts, not shown)		1
1	SP1530720	Frontfork, cpl.		1
2	1421687	SLIDER SL 42, black		1
3	SP1544096	Frontwheel Ø220mm, puncture proof, cpl.		1

Drive Wheel

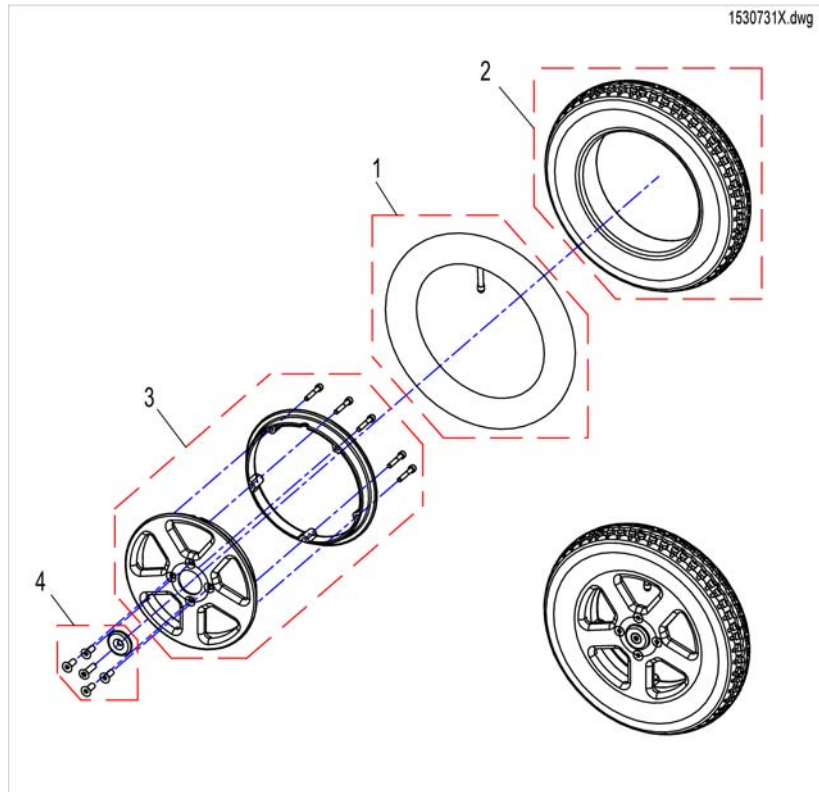

Pos.	Part number	Description	Valid from → until	Qty
1	M12016	Tube 3.00-8" with valve TR 87-90/90°		1
2	SP1530766	Tire		1
3	1442602	Wheel Rim Kit black, cpl.		1
4	1440294	Wheel Fixing Kit Wheel 12-1/2		1


Drive Wheel

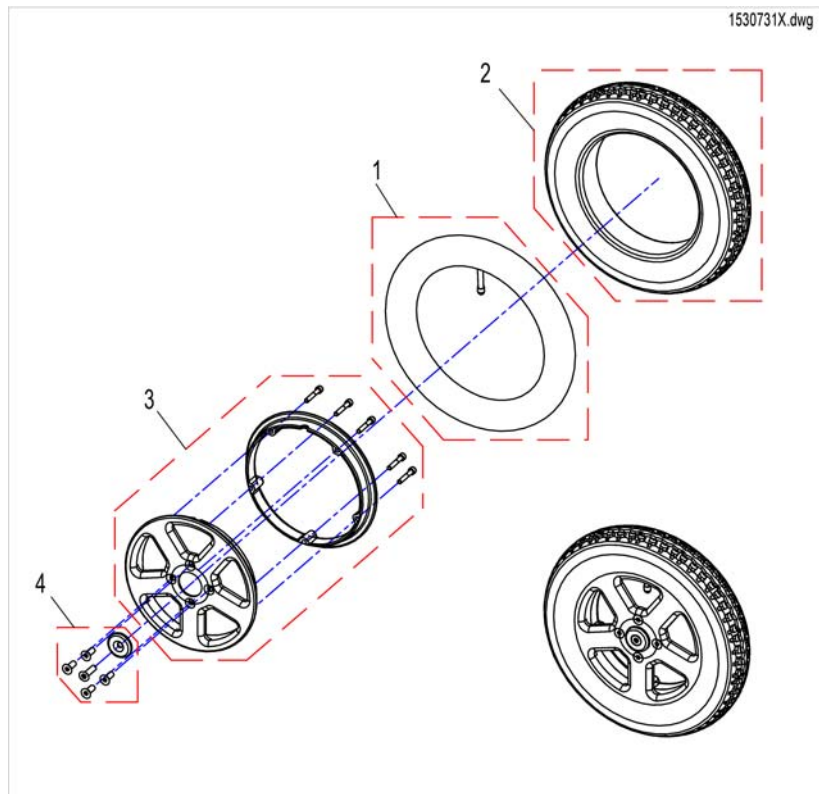

Pos.	Part number	Description	Valid from → until	Qty
1	M12016	Tube 3.00-8" with valve TR 87-90/90°	→ 2015-05-31	1
2	SP1530766	Tire	→ 2015-05-31	1
3	SP1141342	Tire 14 x 3.00" puncture proof	2015-06-01 →	1
3	1442602	Wheel Rim Kit black, cpl.		1
4	1440294	Wheel Fixing Kit Wheel 12-1/2		1


Motor & Decoupling 6/9 km/h

15095705TR iso

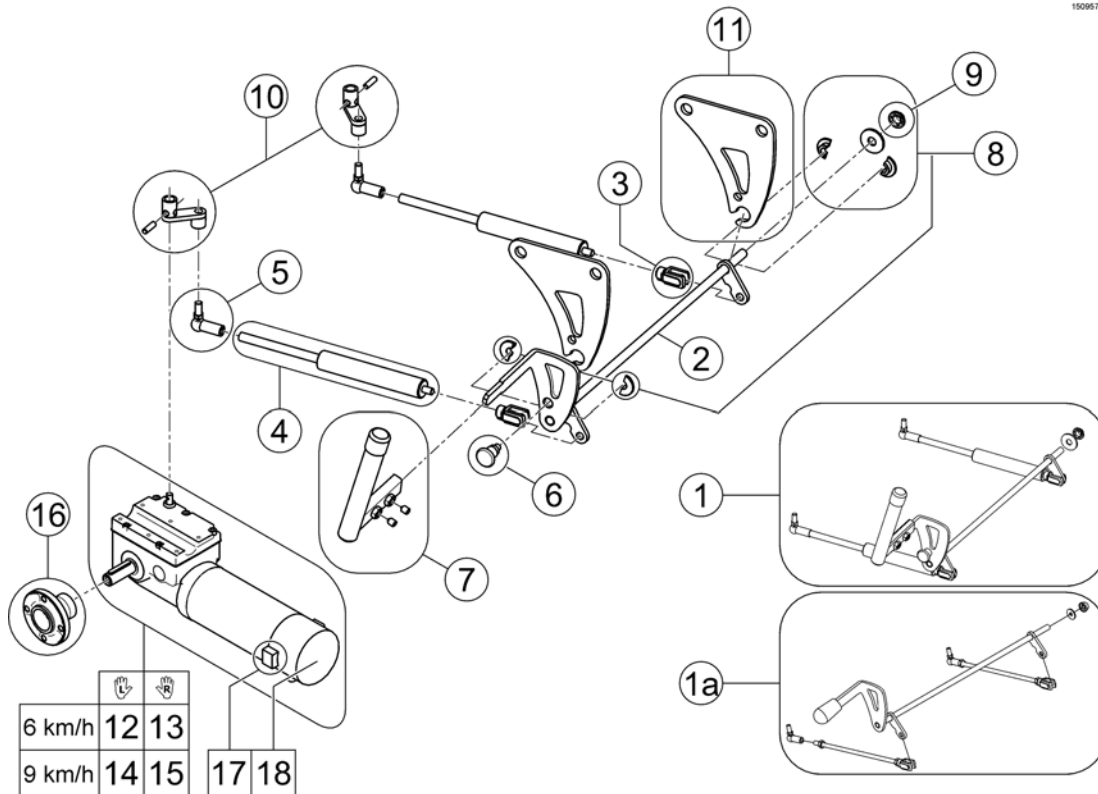

Pos.	Part number	Description	Valid from → until	Qty
1	SP1509570	Decoupling, complete <i>replaces SP1440712</i>	2008-07-16 →	1
1a	-	No longer available - please refer to Replacement Info for which part is to be used instead. <i>replaced by SP1509570</i>	→ 2008-07-15	0
2	1440643-9005	Disengager		1
3	1430139	Knuckle Joint		1
4	1509571	Gas spring		1
5	1430138	Angle Joint M6 R		1
6	5006742	Pull-Latch, small		1
7	SP1509652	Disengager lever, complete		1
8	1442690	Mounting-Kit Disengager		1
9	1442019	Starlock Cap 8 mm		1
10	1485273	Lever Arm Set incl. Assembly		1
11	1440701-0003	Holding Plate For Disengager		1
12	SP1436633	Motor 6km/h LH (Hua-Feng)	2008-07-16 →	1
13	SP1436632	Motor 6km/h RH (Hua-Feng)	2008-07-16 →	1
14	SP1528305	Motor 9km/h LH (Hua-Feng)		1
15	SP1528304	Motor 9km/h RH (Hua-Feng)		1
16	1509053-0002	Wheel Hub, 4-hole		1
17	1441655	Motor Brush Set (Hua-Feng)		1
18	P09199	Brake Kit ATG 106W (Hua-Feng)		1



1500784.indd

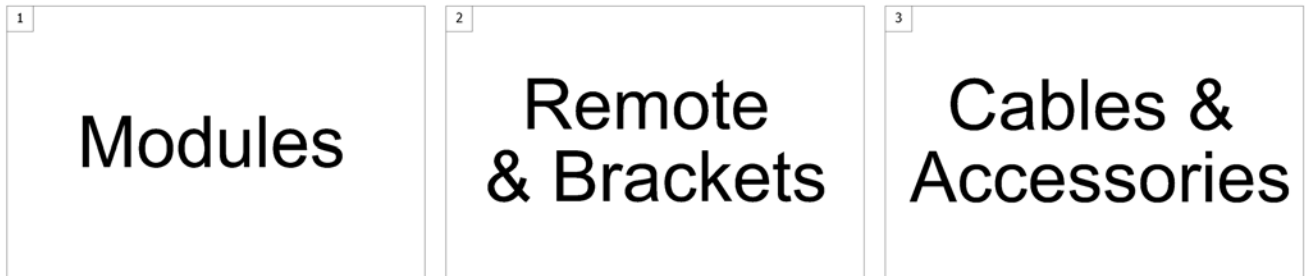


Pos.	Part number	Description	Valid from → until	Qty
1	-	Powermodules		1
1	-	Powermodul		1
2	-	Remotes & Brackets		1
3	-	Cables & Accessories		1

Powermodules

1537235X.jpg

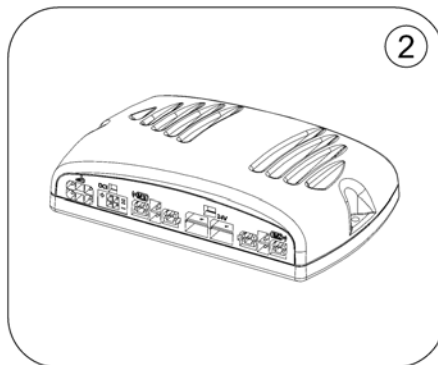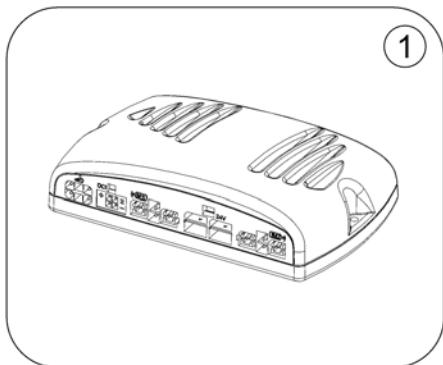

Pos.	Part number	Description	Valid from → until	Qty
1	SP1537235	Powermodule, Shark, DK-PMB50	→ 2013-12-31	1
2	SP1577053	Powermodule, Shark, 75A, 2 Actuators, DK-PMB28 <i>Latest Drive Profile has to be loaded! - Replaces DK-PMB31!</i>	2014-01-01 →	1

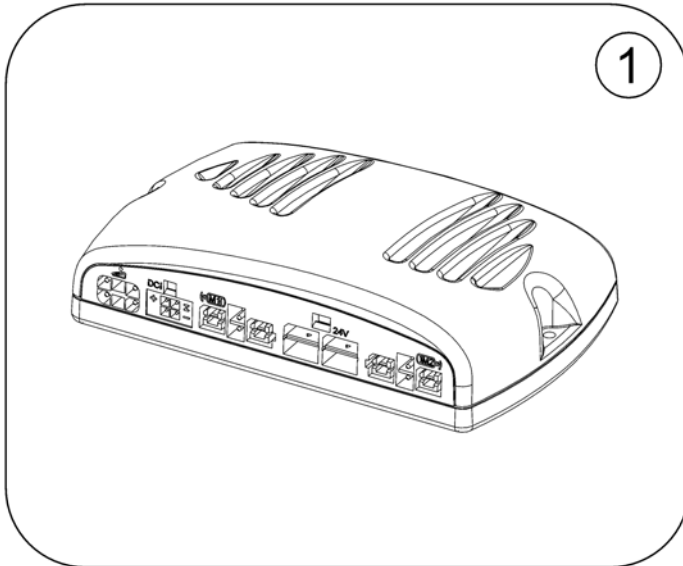


Pos.	Part number	Description	Valid from → until	Qty
1	SP1537235	Powermodule, Shark, DK-PMB50		1

Remotes & Brackets

Pos.	Part number	Description	Valid from → until	Qty
1	-	Remotes		1
1	-	Remotes		1
2	-	Remote Brackets		1
3	-	Lightboard		1
3	-	Lightboard		1

Remotes



1590135.iso

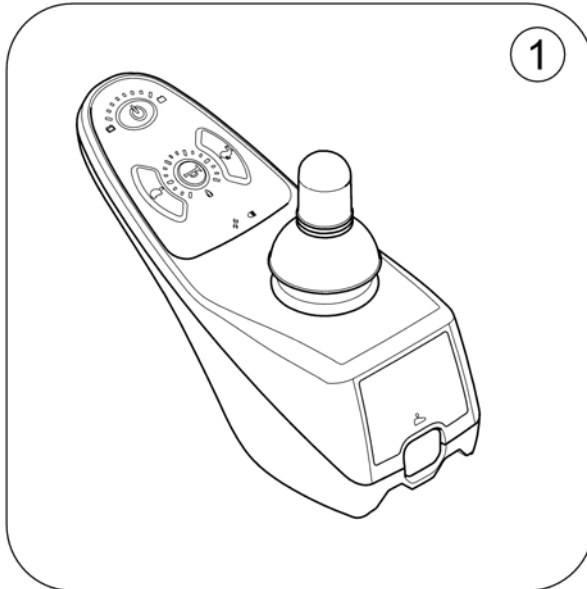

Pos.	Part number	Description	Valid from → until	Qty
-	-	A-Series 1136742 no longer available.	→ 2010-08-19	0
1	1492567	Remote DK-REM D01, Shark2 (Drive only)	2010-08-20 →	1

Remotes


1579646 iso

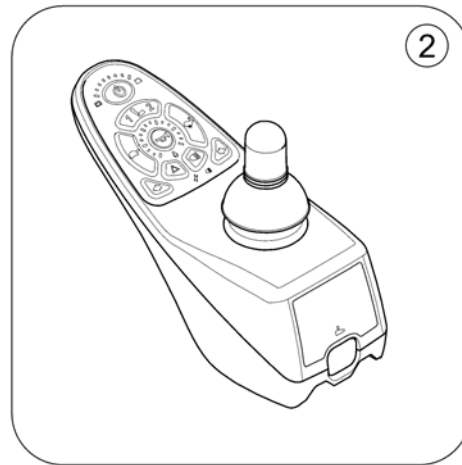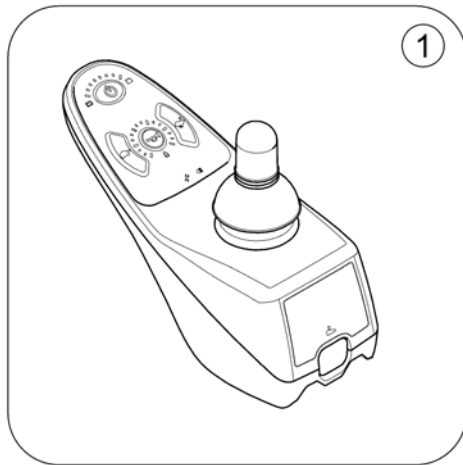

Pos.	Part number	Description	Valid from → until	Qty
-	-	A-Series 1136742 no longer available.	→ 2010-08-19	0
1	1492567	Remote DK-REM D01, Shark2 (Drive only)	2010-08-20 → 2013-12-31	1
2	1484904	Remote DK-REM D21, Shark2 (Drive, Adjustment, Light) <i>Replacement for DK-REMD31!</i>	2014-01-01 →	1

Remote Brackets


1421795X.000

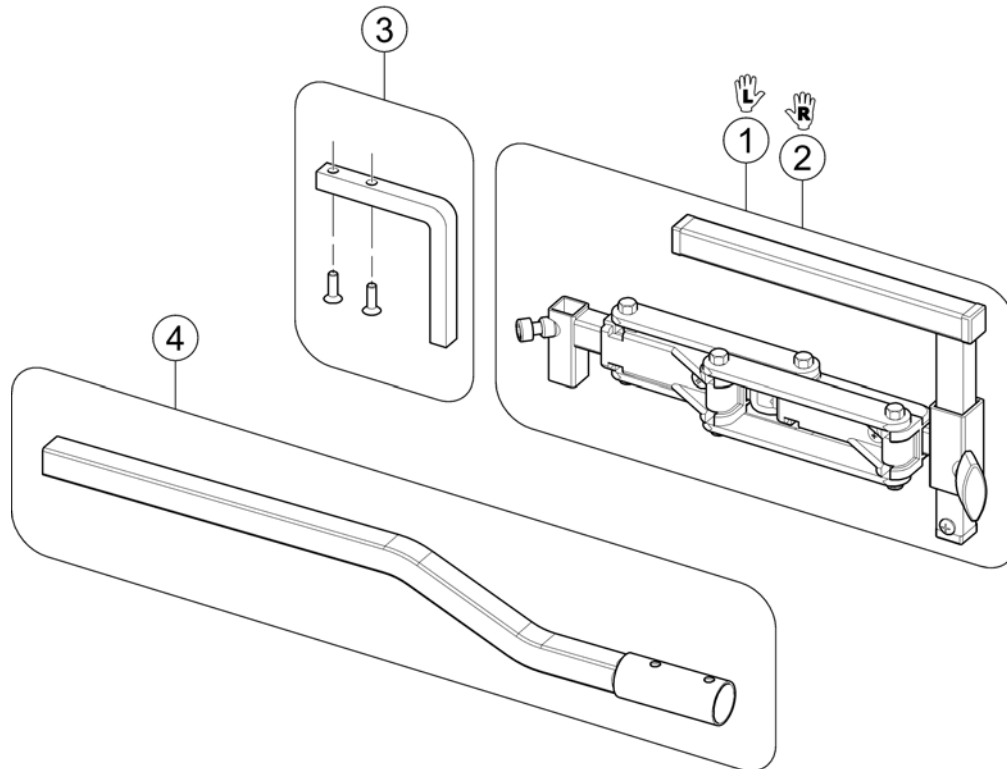


Pos.	Part number	Description	Valid from → until	Qty
1	SP1421796	Swing-Out Height-Adjustable Remote Holder LH		1
2	SP1421795	Swing-Out Height-Adjustable Remote Holder RH		1
3	SP1436543	Joystick Box Support Swing Away REM 24SD & Shark, With Screws		1
4	1433384-0216	Remote Bracket Standard		1

Lightboard

1531199X.dwg

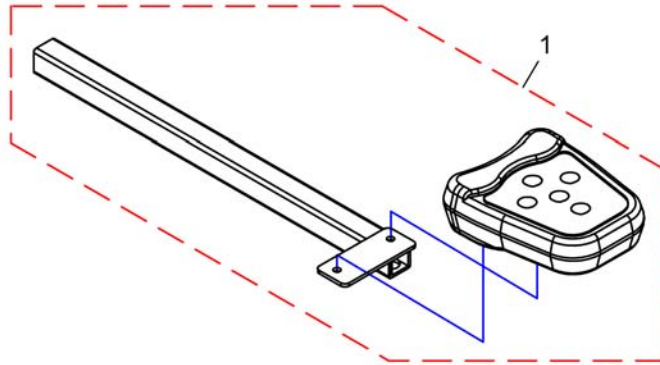

Pos.	Part number	Description	Valid from → until	Qty
1	SP1531199	Lightboard		1

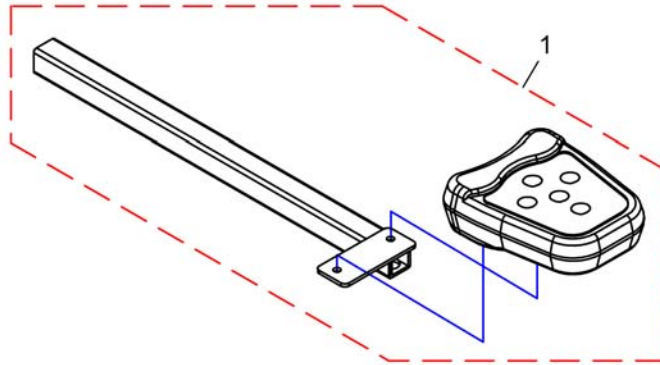


Pos.	Part number	Description	Valid from → until	Qty
1	SP1531199	Lightboard	→ 2013-12-31	1

Cables & Accessories

Pos.	Part number	Description	Valid from → until	Qty
-		Bus-Cables Shark		1
-		Programming Kits		1


SHARKBUSKABEL.00

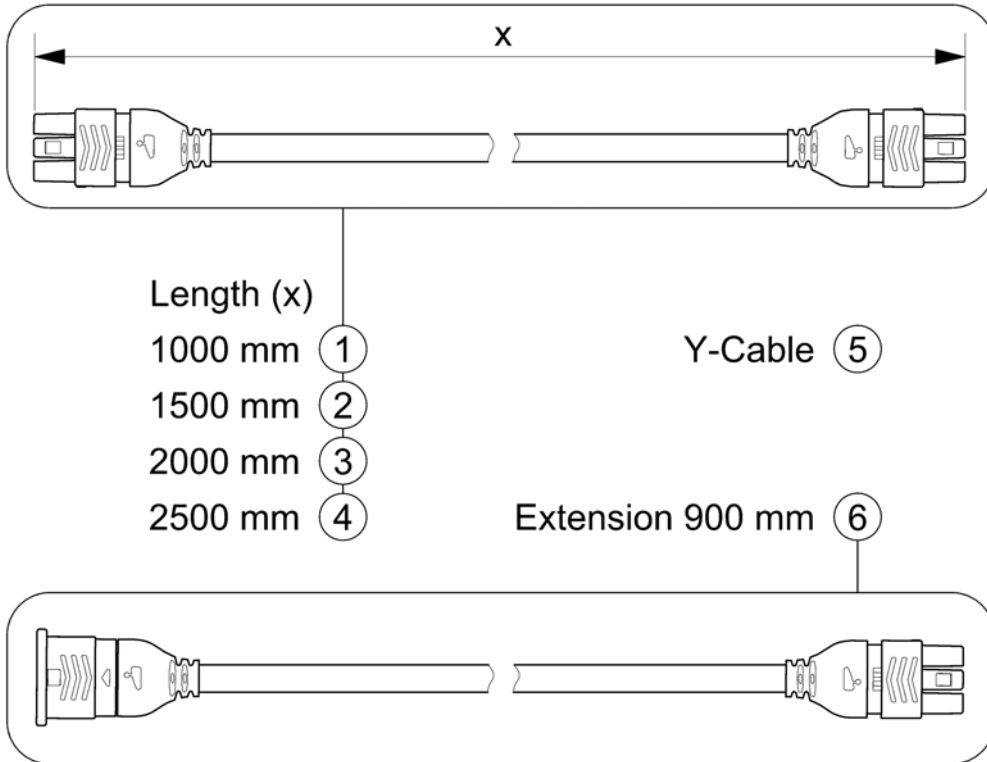

Pos.	Part number	Description	Valid from → until	Qty
1	1448922	Bus Cable Shark 1000mm		1
2	1450313	Buscable Shark 1500mm		1
3	1484911	Buscable Shark 2000mm		1
4	SP1510073	Buscable Shark 2500mm		1
5	1450314	Y-Cable		1
6	1116404	Extension Cable Shark 900mm		1

Programming Kits


1584805 iso

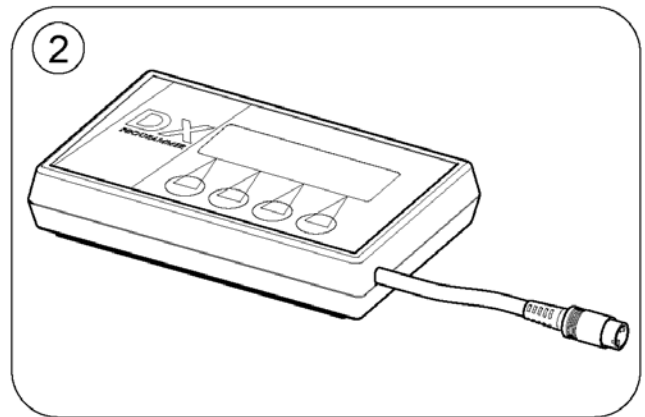

Pos.	Part number	Description	Valid from → until	Qty
1	1449280	USW wizard dongle kit, EDL-USW, cpl.		1
2	6001450	Hand Programmer, Instrucions In French, German, Spanish		1
	SP1441551	Programming Adapter Shark DK-ADAPT (necessary to program Shark)		1

Cables


1531069X.000

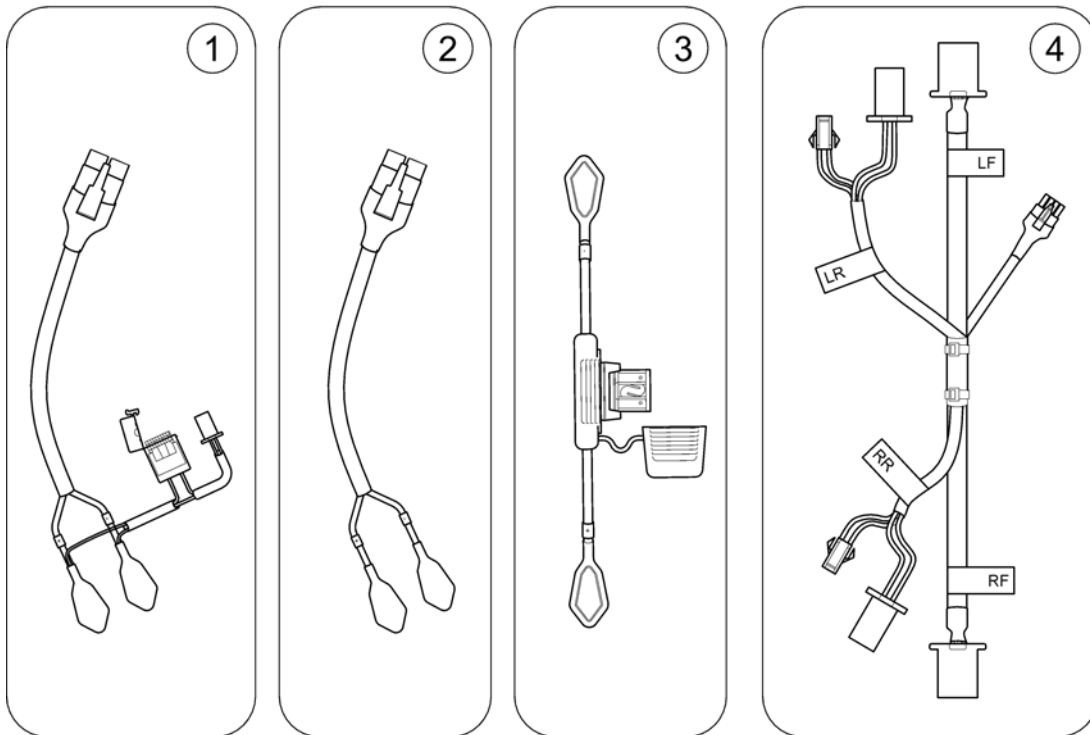

Pos.	Part number	Description	Valid from → until	Qty
1	SP1531069	Battery Input Wire with 5A Fuse		1
2	SP1579641	Battery Input Wire (German homologation)		1
3	SP1531070	Battery Series Wire with 60A Fuse		1
4	SP1579642	Wire Harness, Light (German homologation)		1

Charger

1577962.iso

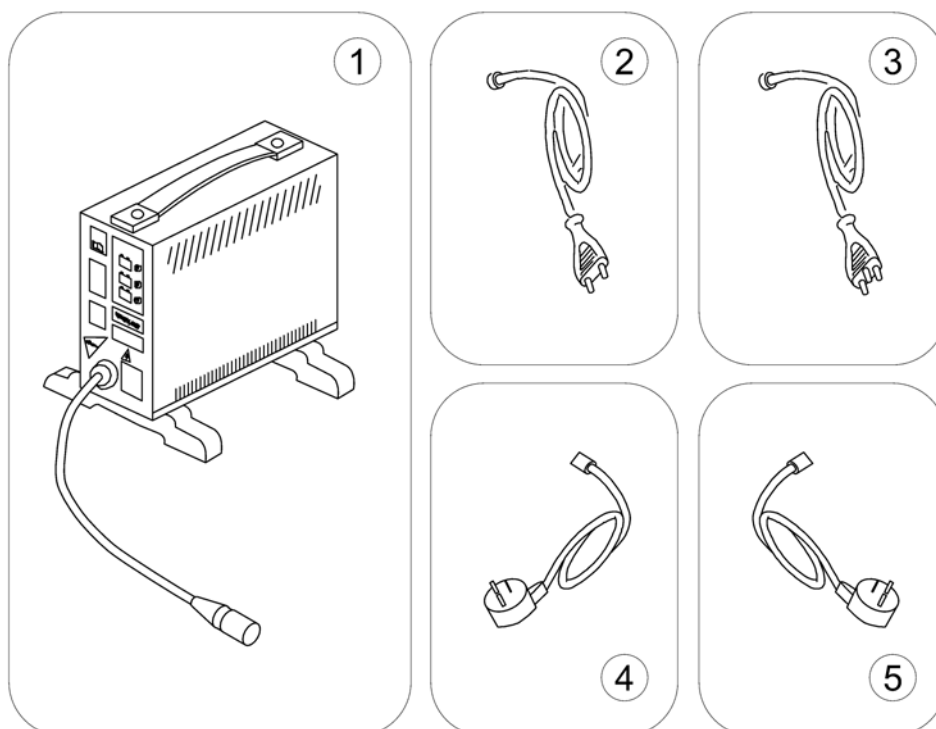

Pos.	Part number	Description	Valid from → until	Qty
1	SPP51981	Charger 8A without Power Cord		1
2	P21079	Power Cord Continental (Schuko GCB-0401)		1
3	1443841	Power Cord Swiss (black 6A/250V)		1
4	P21080	Power Cord UK (GCB-0403)		1
5	SP1553849	Power Cord Israel (IEC 320 C13 to SI-32)		1


Batteries

	Capacity [Ah]															
	1,2	12	30	31	32	36	38	40	40	50	50	50,9	60	73,6	75	97,6
Bora/SpectraXTR								8		10						
Dragon										10			13			
Dragon Vertic													13			
FDX													13			
G50													13	14		16
Kite													13	14		
Mirage				4				8								
Mistral3				4									12			
P9000XDT					5											
Pronto M41					5											
Pronto M61					5											
Spectra Blitz								8					12			
Spectra Comfort								7								
Spectra Plus								7								
Spectra Seat Riser								7								
Storm3										10			13	14		
Storm3 True Track	1													14		
Storm3 Vertic														14		
Storm4														14		
Storm4 Max														14		
Stream										10			13			
TDX-SP														14		
TDX-SP Narrow												12				
Tiger										10			13			
Topan													13			
Tornado								8								
Typhoon													13			
Typhoon GB	1												13			
Typhoon GB	1												13			
Zephyr	1												13			
Auriga			3					8		10						
Comet														14	15	
Leo						6			9							
Lynx		2														
Meteor														14		
Orion												11				
length [mm]	97	152	197	196	211	196	210	197	198	198	238	259	260	260	329	
width [mm]	49	99	132	132	132	130	175	168	165	166	165	140	169	171	168	171
height [mm]	51	102	180	183	183	179	175	175	171	176	170	235	178	210	223	238
weight [kg]	0,6	3,9	11,1	10,5	11	12	14,6	14,4	14,2	14	16	17,1	19,3	24,3	26	32,5



Pos.	Part number	Description	Valid from → until	Qty
i	-	Change batteries only pairwise and from the same type!		1
1	1440386	Battery, 12V/1,2Ah C20, GEL		1
2	SP1506493	Battery 12V/12Ah AGM		1
3	P50021	Battery, 12V/30Ah C20, GEL		1
4	P50029	Battery, 12V/31Ah C20, GEL		1
5	SP1521459	Battery, 12V/34Ah C20, AGM		1
6	1505498	Battery, 12V/36Ah C20, AGM		1
7	P50028	Battery, 12V/38Ah C20, GEL (UK & DK only)		1
8	PG50988	Battery, 12V/40Ah C20, GEL		1
9	SP1550692	Battery, 12V/40Ah C20, AGM		1
10	SP1522107	Battery, 12V/50Ah C20, AGM		1
11	SP1521695	Battery, 12V/50Ah C20 AGM		1
12	PG50989	Battery, 12V/50,9Ah C20, GEL		1
13	1432944	Battery, 12V/60Ah C20, GEL		1
14	P50016	Battery, 12V/73,6 Ah C20, Gel		1
15	SP1522026	Battery, 12V/75Ah C20, AGM		1
16	1478675	Battery, 12V/97,6Ah C20, GEL		1


Retaining Belts

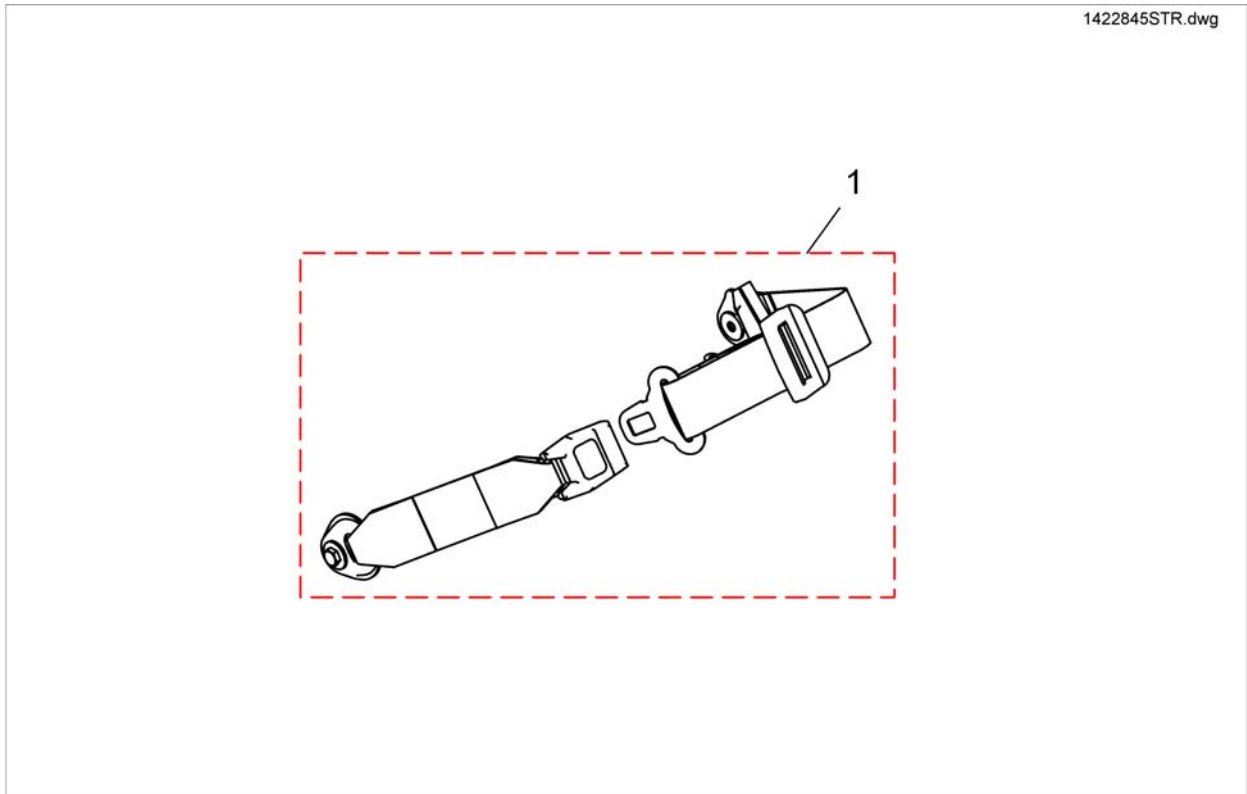

Pos.	Part number	Description	Valid from → until	Qty
1	1422845	Lap Belt, cpl.		1

Lightsystem


15306/0X.dwg

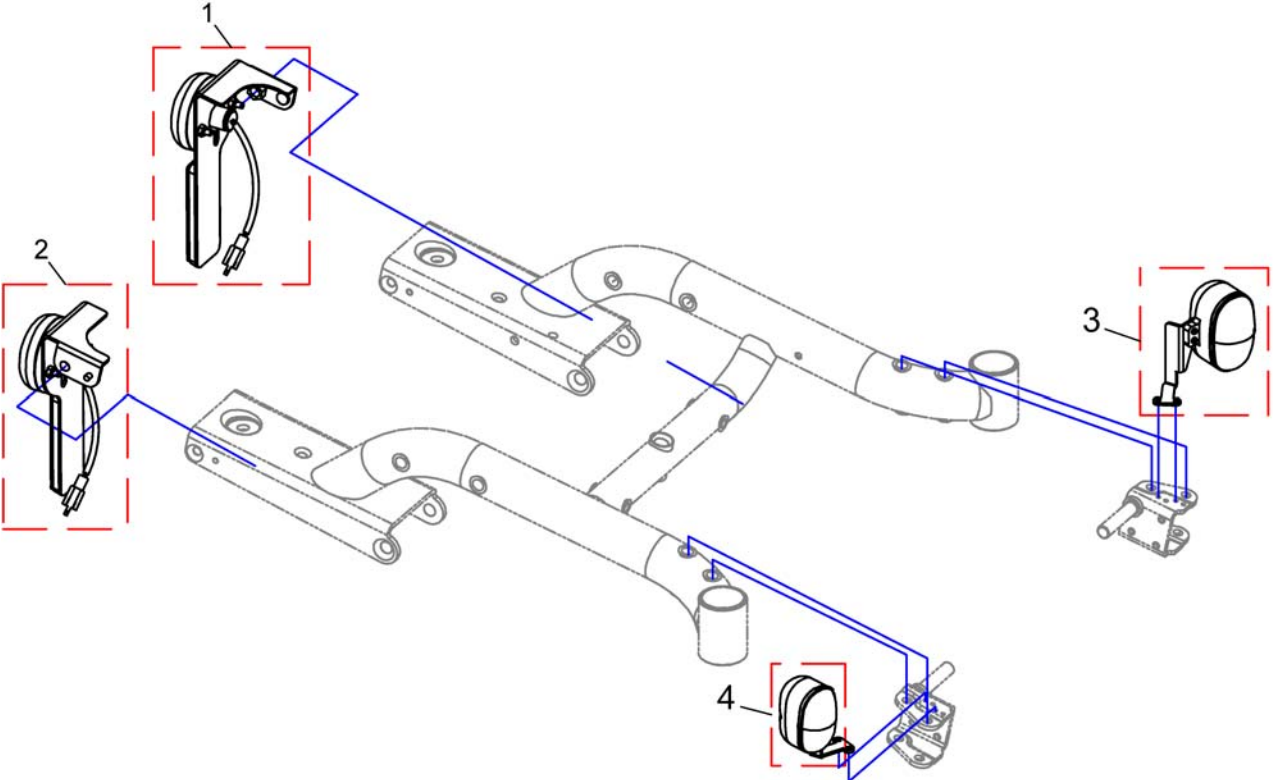

Pos.	Part number	Description	Valid from → until	Qty
-	SP1530210	Retrofitting-Kit Light, cpl. with Lightboard		1
1	SP1530676	Rearlight LH		1
2	SP1530678	Rearlight RH		1
3	SP1530697	Frontlight LH		1
4	SP1530698	Frontlight RH		1

Lightsystem


15/1669-00

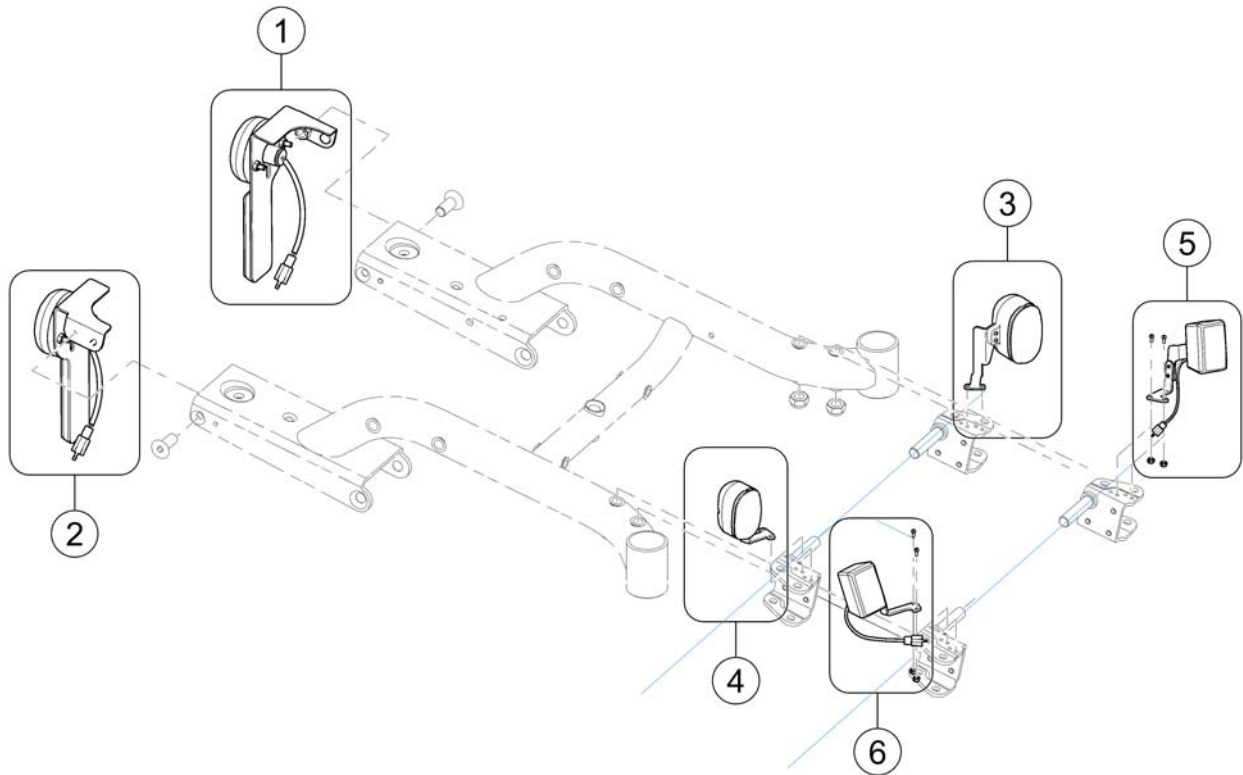

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530676	Rearlight LH		1
2	SP1530678	Rearlight RH		1
3	SP1530697	Frontlight LH	→ 2013-12-31	1
4	SP1530698	Frontlight RH	→ 2013-12-31	1
5	SP1579639	Headlamp Assy LH (German homologation)	2014-01-01 →	1
6	SP1579640	Headlamp Assy RH (German homologation)	2014-01-01 →	1
-	SP1530210	Retrofitting-Kit Light, cpl. with Lightboard	→ 2013-12-31	1

Rear View Mirror


158884.svg

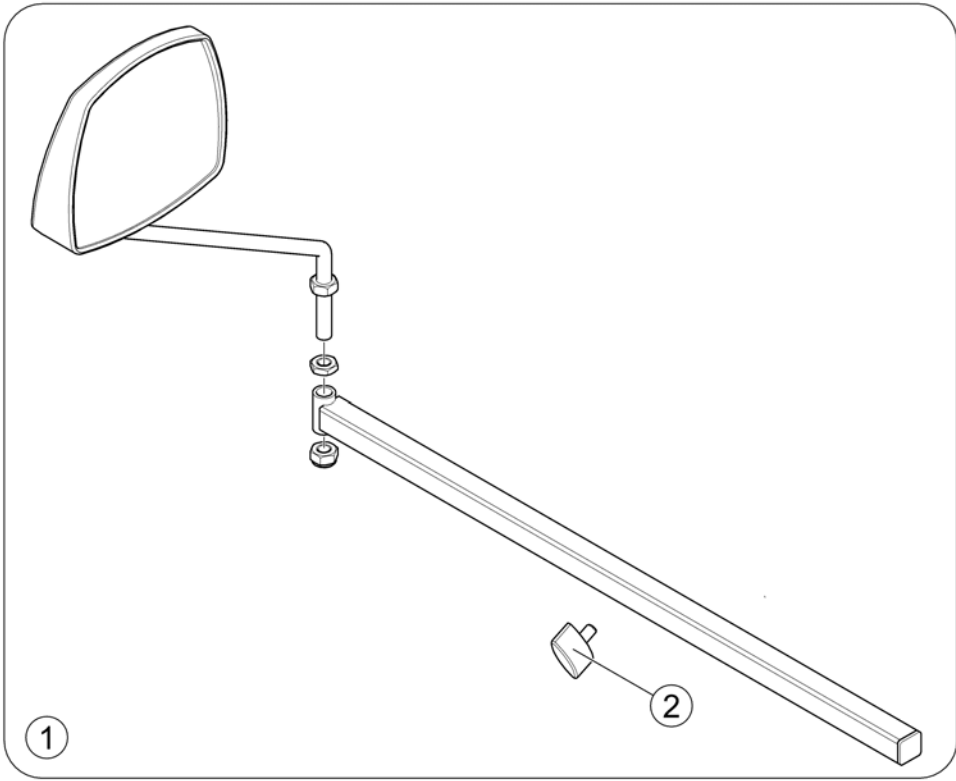

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422881	Rear Mirror Complete With Support		1
2	L31001	Wing Scew M6X10		1

Crash-Kit



1536575X.dwg

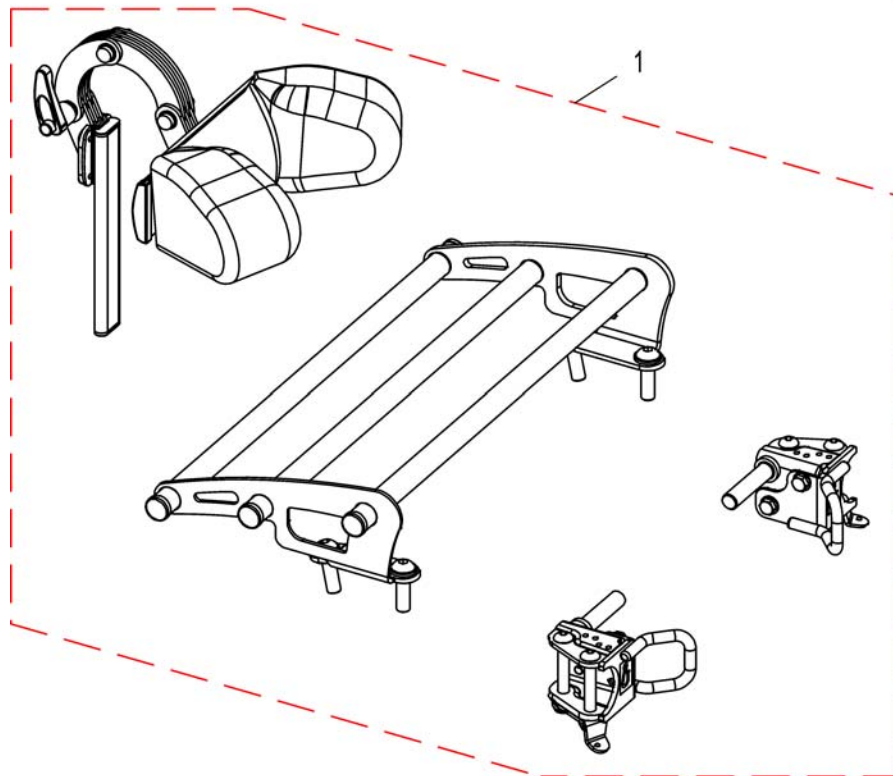

Pos.	Part number	Description	Valid from → until	Qty
1	SP1536575	Crash-Kit		1

Headrest


1509190.000

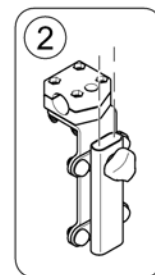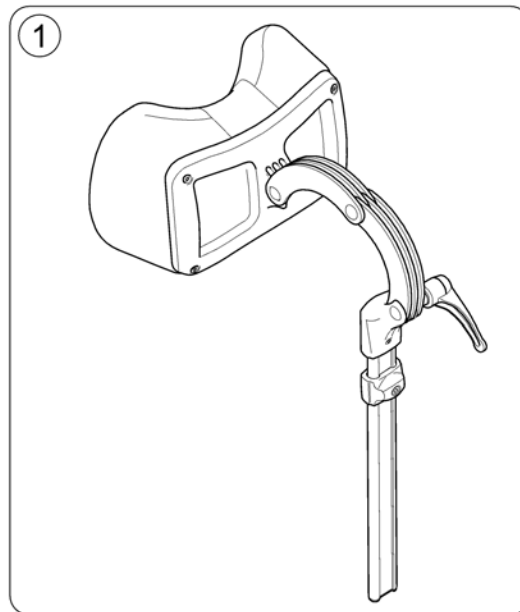

Pos.	Part number	Description	Valid from → until	Qty
1	1508848	NECKREST		1
2	SP1422914	Head-/Neckrest Bracket, cpl.		1

Caneholder

1588833 rev

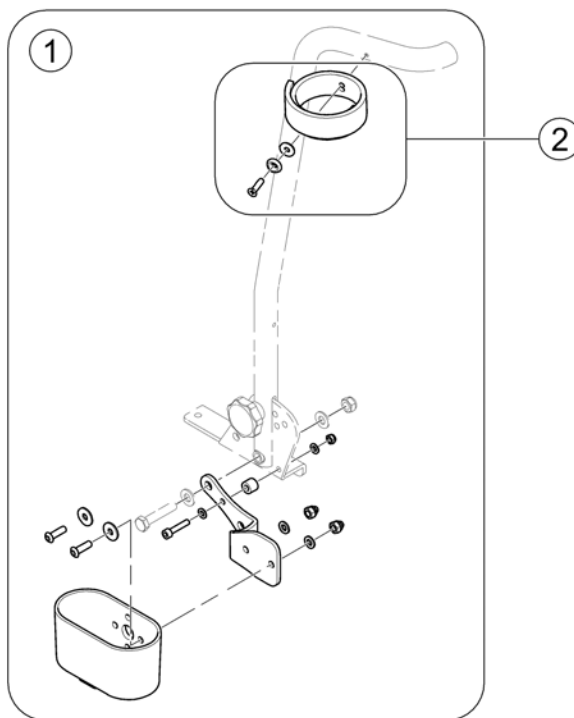

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422844	Caneholder Std. cpl.		1
2	SP1420493	Belt for caneholder		1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk