

# *Spare Parts Catalogue*

---

Version: 2015/09/01a EN



*Invacare*<sup>®</sup> **TYPHOON II**



**Yes, you can.®**

## **General Information**

*For any order of spare parts the following information is required:*

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

*Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.*

*The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.*



Yes, you can.<sup>®</sup>

## TABLE OF CONTENTS

---

### Information & User Manuals

User Manuals Typhoon II	7
User Manuals REM550	8
User Manuals REM34 & REM41 Tray/Centre Control	9
User Manuals IDC Dual Control	10
User Manuals ACS Handprogrammer	11
User Manual Proximity Head Remote DSL-105	12

### Chassis

Chassis	13
4-Point Tie Down	14
Cover	15
Walkingbeam	16
walking beam front with castors	17
walking beam back with castors	18
anti tipp system	19

### Seat

Tilt adjustment manual w/o Lifter	20
Tilt adjustment electr. w/o Lifter	21
Seat Riser with Tilt actuator	22
Actuator for Seat Riser	23
Bellow Typhoon II	24
Seat Frame Std.	25
Seat Plates	26
Seat Frame Junior	27
Seat Cushion for Standard Seat	28
Seat Cushion for Contour Seat	29
Seat Cushion for Flex Seat	30
Recaro Seat	32
Change list Recaro 24V	33
Seat Cushion for Junior Seat	34
Seat Cushions Optimist Seat	35

### Backrest

Backrest Standard Seat, settable	36
Backrest Recliner	37
Backrest Flex Seat	38
Backrest adjustment Flex/Contour electric	39
Backrest Cushion for Contour Seat	41
Backrest Cushion for Flex Seat	42
Backrest Cushion for Junior Seat	43
Optimist Seats	44

### Legrests / Hangers

Central Legrest, manual	45
Central Legrest, electr.	46
Legrest fixation	47
Hanger Vari F	48



Yes, you can.®

## TABLE OF CONTENTS

---

Hanger Vari A (04/2009 =>)	49
Hangers ADM LH/RH manual	50
Hanger Vari ADE/ADP electric (07/2005 =>)	51

### Footplates / Support tubes

Footplate	52
Footplate with support tube fix RH/LH	53
Footplate with support tube angle adjust. RH/LH	54
Support Tube with Footplate for central Legrest	55
Footrests Vari, Flip-Up (01.08.10 =>)	56
Footrests Vari, Flip-Up & angle adjustable (01.08.10 =>)	57

### Armrests

Fixed Armrest, removable	58
Armrests, cpl.	59
Sliding Armrest	61
Skirtguard for following armrest	62

### Castors

walking beam front with castors	17
walking beam back with castors	18

### Drive Wheels

Rear Wheels True Track	63
------------------------	----

### Propulsion

Powered declutching	64
GB-motor with manual declutching	65

### Electronic

Electronic	66
ACS Electronic Modules	67
ACS Remote & Brackets	68
REM 24 SD with brackets	69
Remote REM 24SD	70
Remote Bracket Standard	71
Remote Bracket Swing away & height adjust.	72
Remote Bracket Attendant for Standard-/Contour-/Flex-Seat	73
Remote Bracket Attendant for Recaro-Seat	74
Remote REM A & B with Brackets	75
Remote Bracket Standard	71
Remote Bracket Swing away & height adjust.	76
Remote Bracket Attendant for Standard-/Contour-/Flex-Seat	73
Remote Bracket Attendant for Recaro-Seat	74
Remote REM 550 with Brackets	77
Remote Bracket Standard	71
Remote Bracket Swing away & height adjust.	72
Remote Bracket Attendant for Standard-/Contour-/Flex-Seat	73
Remote Bracket Attendant for Recaro-Seat	74



Yes, you can.<sup>®</sup>

## TABLE OF CONTENTS

---

REM 34 with brackets .....	78
Remote Bracket Standard .....	79
Remote Bracket Swing away & height adjust. ....	80
G90A with brackets .....	81
Remote Bracket Standard .....	82
Remote Bracket Swing away & height adjust. ....	83
Remote Bracket Attendant for Standard-/Contour-/Flex-Seat .....	84
Remote Bracket Attendant for Recaro-Seat .....	85
G91S with brackets .....	86
Remote Bracket Standard .....	87
Chin Control with brackets .....	88
Small Chincontrol .....	89
Framework for chincontrol .....	90
Dual Control with brackets .....	91
Remote Bracket Dual Control for Standard-/Contour-/Flex-Seat .....	92
Remote Bracket Dual Control for Recaro Seat .....	93
Intuitive Dual Control with brackets .....	94
ACS Cables & Accessories .....	95
Bus-Cables ACS .....	96
T-Adapter .....	97
T-Adapter - Modulite Seat .....	98
Programming Kits .....	99
Magnetkey for Remotes .....	100

### Cable, Charger & Batteries

Batteries .....	101
Chargers .....	102
Battery support .....	103

### Safety Options

Front Light .....	104
Rear Light .....	105
Rear Light Protector Kit .....	106
Reflector .....	107

### Seat Accessories

Backrest Extension Flex .....	108
Lateral Support (Dartex) .....	109
Lateral Support - Junior Seat .....	110
Abduction Wedge .....	111
Headrest & Brackets .....	112
Neckrest & Brackets .....	113
Push Handles .....	114
Knee Support .....	115

### Accessories

Docking-Station .....	116
Rearview Mirror .....	117
Trays .....	118
other Options .....	119



## User Manuals Typhoon II

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
DA	1512526	User Manual Typhoon II Danish		1
DE	1511911	User Manual Typhoon II German		1
EN	1521645	User Manual Typhoon II English		1
NO	1512531	User Manual Typhoon II Norwegian		1
SV	1512525	User Manual Typhoon II Swedish		1

## User Manuals REM550

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DA 1528088	User Manual REM550 Danish		1
	DE 1528089	User Manual REM550 German		1
	EN 1528090	User Manual REM550 English		1
	ES 1528091	User Manual REM550 Spanish		1
	FI 1528092	User Manual REM550 Finsk		1
	FR 1528093	User Manual REM550 French		1
	IT 1528094	User Manual REM550 Italiano		1
	NL 1528095	User Manual REM550 Dutch		1
	NO 1528096	User Manual REM550 Norsk		1
	PT 1528097	User Manual REM550 Portuguese		1
	SV 1528098	User Manual REM550 Swedish		1
	HR SP1533279	User Manual REM550 Croatian		1
	CS SP1533280	User Manual REM551 Czech		1
	ET SP1533281	User Manual REM552 Estonian		1
	HU SP1533282	User Manual REM553 Hungarian		1
	LT SP1533283	User Manual REM554 Lithuanian		1
	PL SP1533284	User Manual REM555 Polish		1
	SL SP1533285	User Manual REM556 Slovenian		1



## User Manuals REM34 & REM41 Tray/Centre Control

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
DA	SP1538022	User Manual REM41 Tray Control Danish		1
EN	SP1538023	User Manual REM41 Tray Control English		1
ES	SP1538024	User Manual REM41 Tray Control Spanish		1
FR	SP1538025	User Manual REM41 Tray Control French		1
IT	SP1538026	User Manual REM41 Tray Control Italian		1
NL	SP1538027	User Manual REM41 Tray Control Dutch		1
NO	SP1538028	User Manual REM41 Tray Control Norwegian		1
PT	SP1538029	User Manual REM41 Tray Control Portuguese		1
SV	SP1538030	User Manual REM41 Tray Control Swedish		1
CS	SP1538031	User Manual REM41 Tray Control Czech		1
SL	SP1538032	User Manual REM41 Tray Control Slovenian		1
DE	SP1541402	User Manual REM41 Tray Control German		1

## User Manuals IDC Dual Control

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DE 1521480	User Manual Dual-Control German		1
	EN 1521481	User Manual Dual-Control English		1
	FR 1521482	User Manual Dual-Control French		1
	NL 1521483	User Manual Dual-Control Dutch		1
	NO 1521484	User Manual Dual-Control Norwegian		1
	SV 1521485	User Manual Dual-Control Swedish		1
	DA 1521486	User Manual Dual-Control Danish		1
	IT 1522252	User Manual Dual-Control Italian		1
	PT 1522254	User Manual Dual-Control Portuguese		1
	ES 1522255	User Manual Dual-Control Spanish		1

## User Manuals ACS Handprogrammer

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
DE	1419316	User Manual Handheld Programmer German		1
EN	1419317	User Manual Handheld Programmer English		1


## User Manual Proximity Head Remote DSL-105

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		


Pos.	Part number	Description	Valid from → until	Qty
EN	SP1555731	User Manual Proximity Head Remote DSL-105 English		1
DE	SP1555732	User Manual Proximity Head Remote DSL-105 German		1
FR	SP1555733	User Manual Proximity Head Remote DSL-105 French		1
DA	SP1555839	User Manual Proximity Head Remote DSL-105 Danish		1
ES	SP1555840	User Manual Proximity Head Remote DSL-105 Spanish		1
FI	SP1555841	User Manual Proximity Head Remote DSL-105 Finnish		1
IT	SP1555842	User Manual Proximity Head Remote DSL-105 Italian		1
NL	SP1555843	User Manual Proximity Head Remote DSL-105 Dutch		1
NO	SP1555844	User Manual Proximity Head Remote DSL-105 Norwegian		1
PT	SP1555845	User Manual Proximity Head Remote DSL-105 Portuguese		1
SV	SP1555846	User Manual Proximity Head Remote DSL-105 Swedish		1

# Chassis

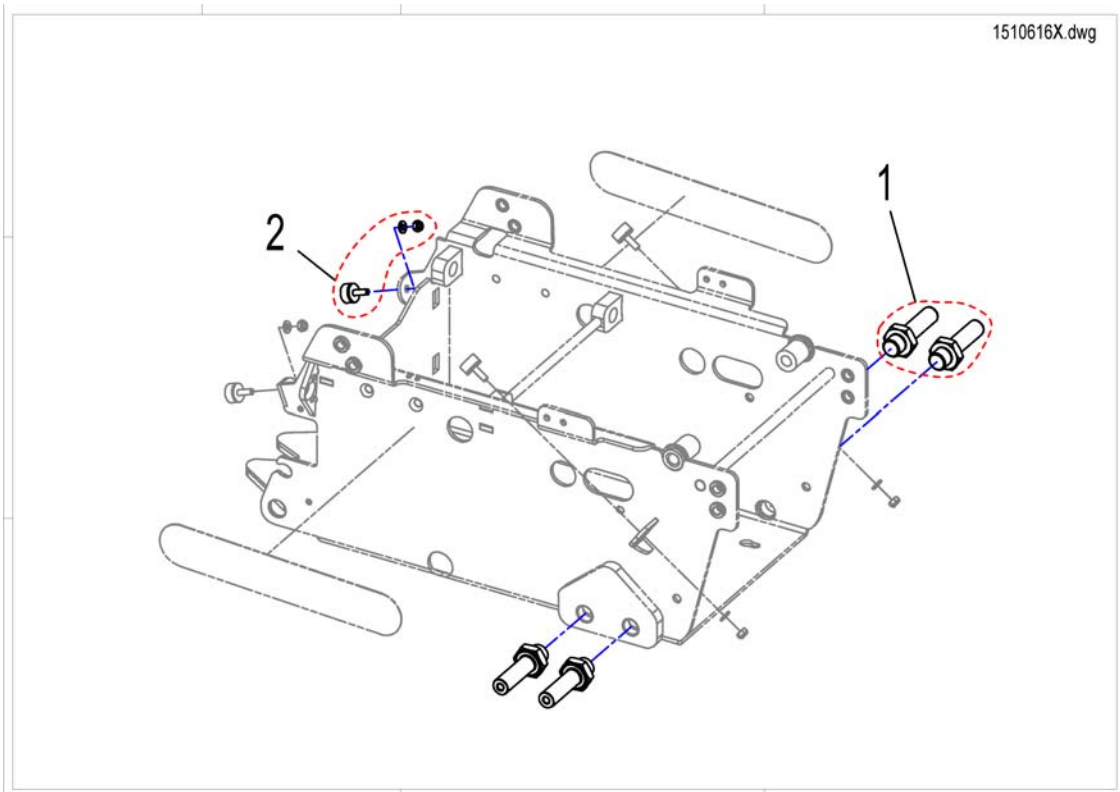


Pos.	Part number	Description	Valid from → until	Qty
1	SP1505168	Main Support Bolt		1
2	SP1440366	rubber buffer		1

## 4-Point Tie Down

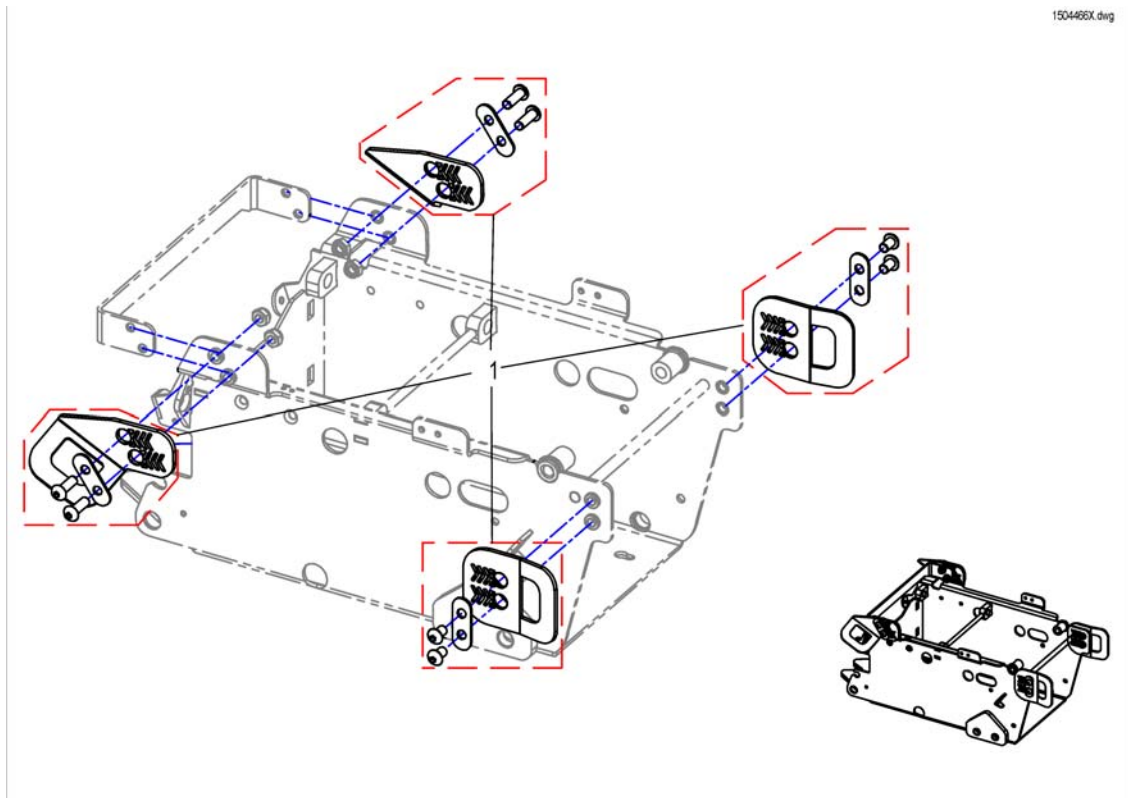


Pos.	Part number	Description	Valid from → until	Qty
1	SP1523486	Tie-Down-Retrofitting-Kit		1

# Cover

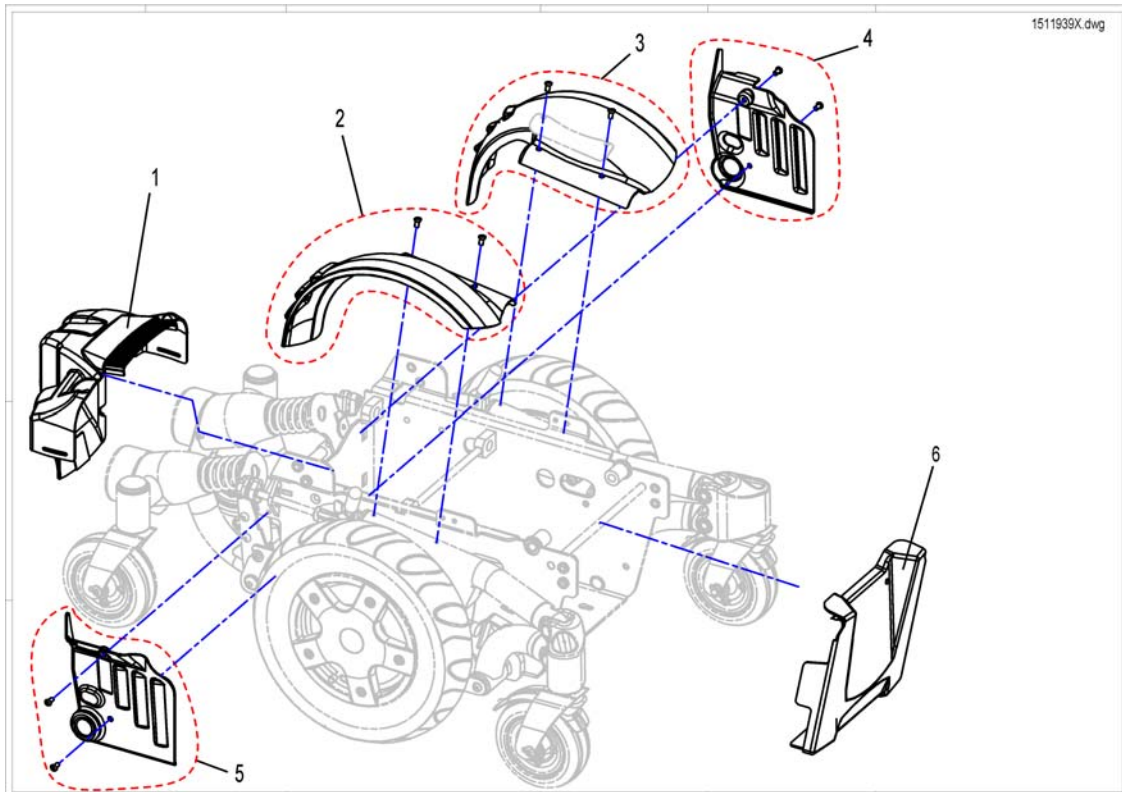

Pos.	Part number	Description	Valid from → until	Qty
1	SP1511371	Back Cover		1
2	SP1510472	Fender RH		1
3	SP1510471	Fender LH		1
4	SP1510835	Side cover LH		1
5	SP1510836	Side cover RH		1
6	SP1488532	Front Cover		1


# Walkingbeam

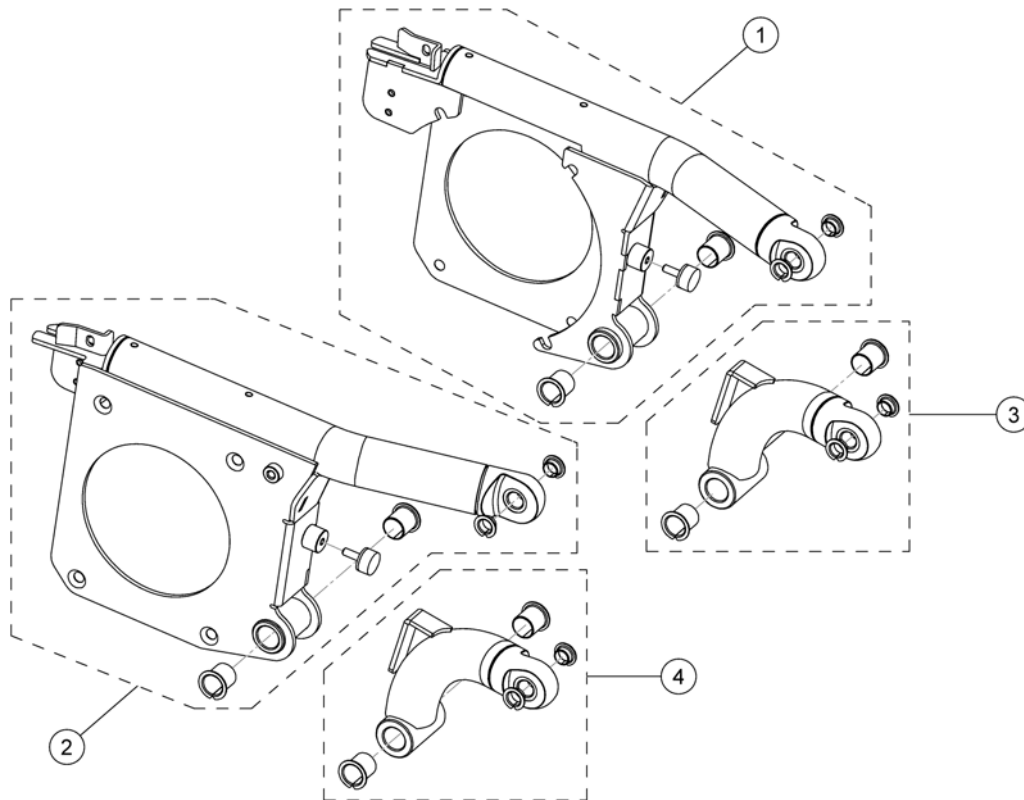

1505800X.iso

Pos.	Part number	Description	Valid from → until	Qty
1	SP1533968	Walkingbeam top, LH, blue (incl. sticker)		1
1	SP1533969	Walkingbeam top, LH, black (incl. sticker)		1
1	SP1533970	Walkingbeam top, LH, red (incl. sticker)		1
2	SP1533960	Walkingbeam top, RH, blue (incl. sticker)		1
2	SP1533961	Walkingbeam top, RH, black (incl. sticker)		1
2	SP1533962	Walkingbeam top, RH, red (incl. sticker)		1
3	SP1533965	Walkingbeam down, LH, blue		1
3	SP1533966	Walkingbeam down, LH, black		1
3	SP1533967	Walkingbeam down, LH, red		1
4	SP1533943	Walkingbeam down, RH, blue		1
4	SP1533944	Walkingbeam down, RH, black		1
4	SP1533959	Walkingbeam down, RH, red		1




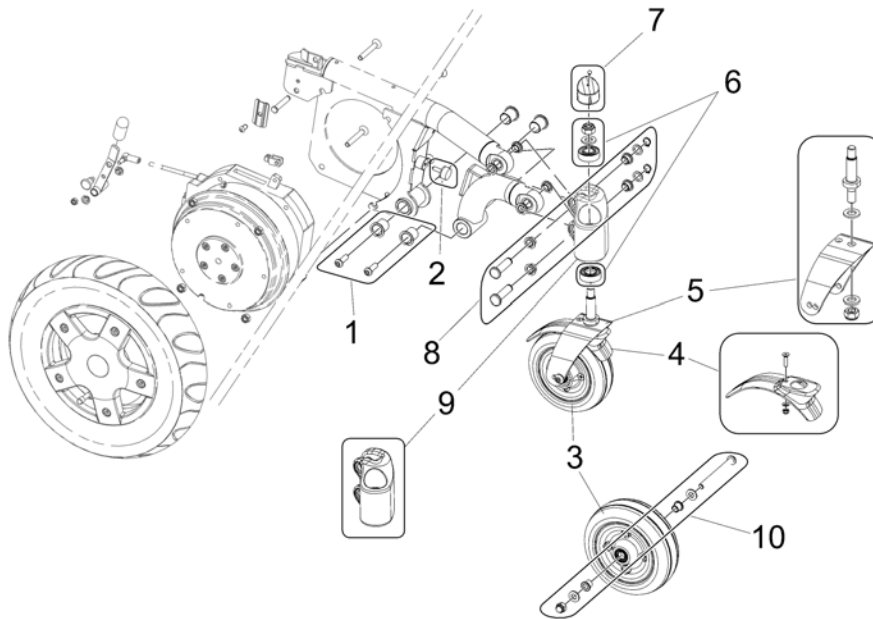

Pos.	Part number	Description	Valid from → until	Qty
1	SP1532584	Collar Bushing		2
2	SP1440366	rubber buffer		3
3	SP1511369	6x2" Wheel with bearings, black hub		4
4	SP1440527	Fender		5
5	SP1433754	Fork for Antitippers		6
6	SP1419254	Bearing kit cpl.		7
7	SP1436347	Cap		8
8	SP1524654	Bushes-Kit		1
9	SP1505181	Headtube LH		1
9	SP1505182	Headtube RH		1
10	1440564	SWIVEL WHEEL ASSEMBLY KIT		1

## walking beam back with castors


1506375X (6)

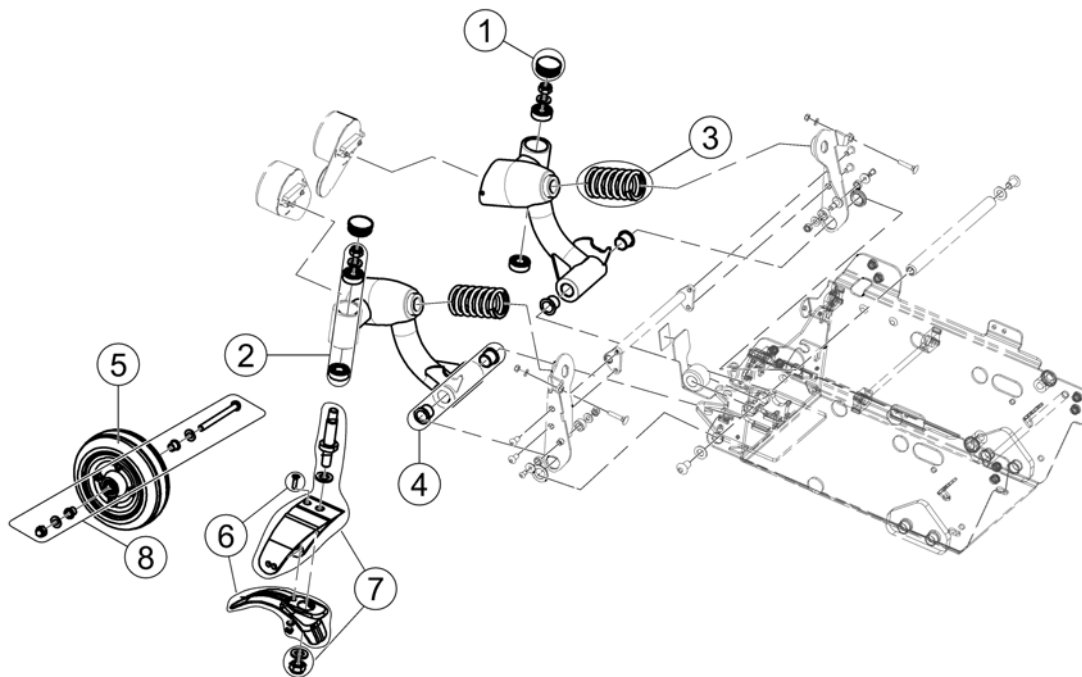

Pos.	Part number	Description	Valid from → until	Qty
1	SPL01043	Slider		1
2	SP1419254	Bearing kit cpl.		1
3	SP1511370	Pressure Spring d=6,0 mm (=> 100kg)		1
3	SP1504753	Pressure Spring d=6,5 mm (100kg =>)		1
4	SP1487723	Slide Bearing		1
5	SP1511369	6x2" Wheel with bearings, black hub		1
6	SP1440527	Fender		1
7	SP1433754	Fork for Antitippers		1
8	1440564	SWIVEL WHEEL ASSEMBLY KIT		1


# anti tipp system

1543130X.iso

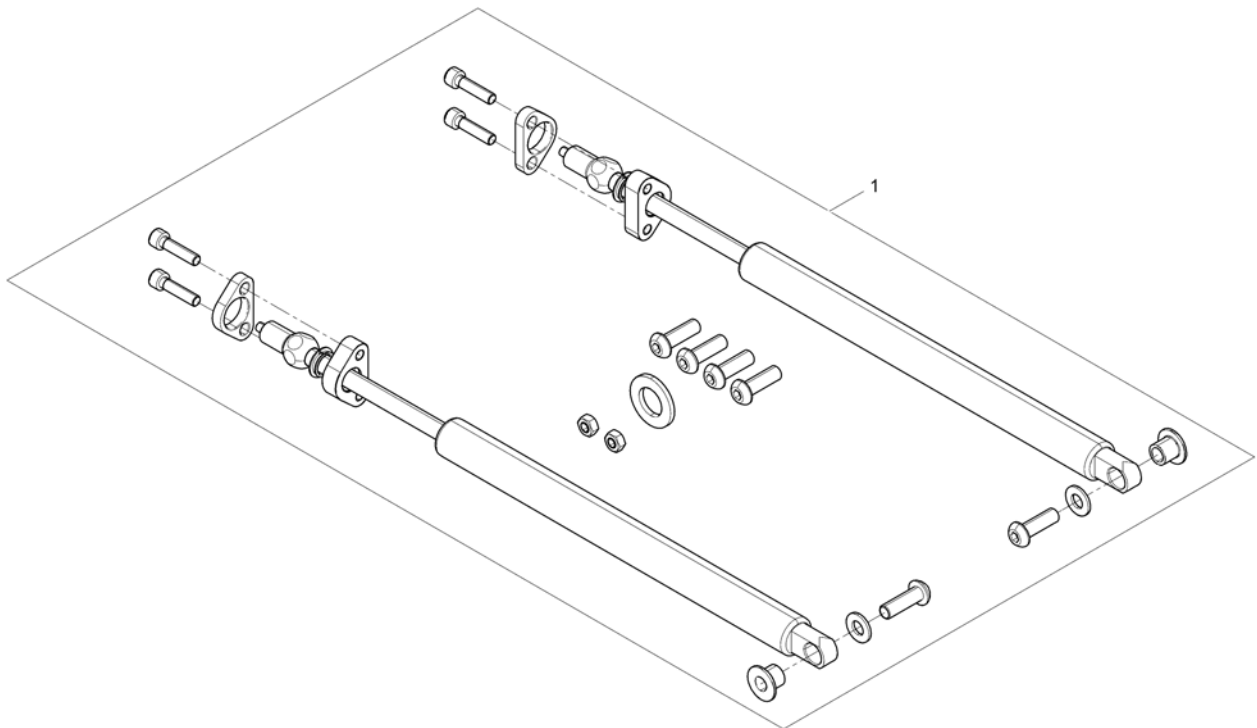


Pos.	Part number	Description	Valid from → until	Qty
1	SP1543130	Anti Tipp System		1

## Tilt adjustment manual w/o Lifter

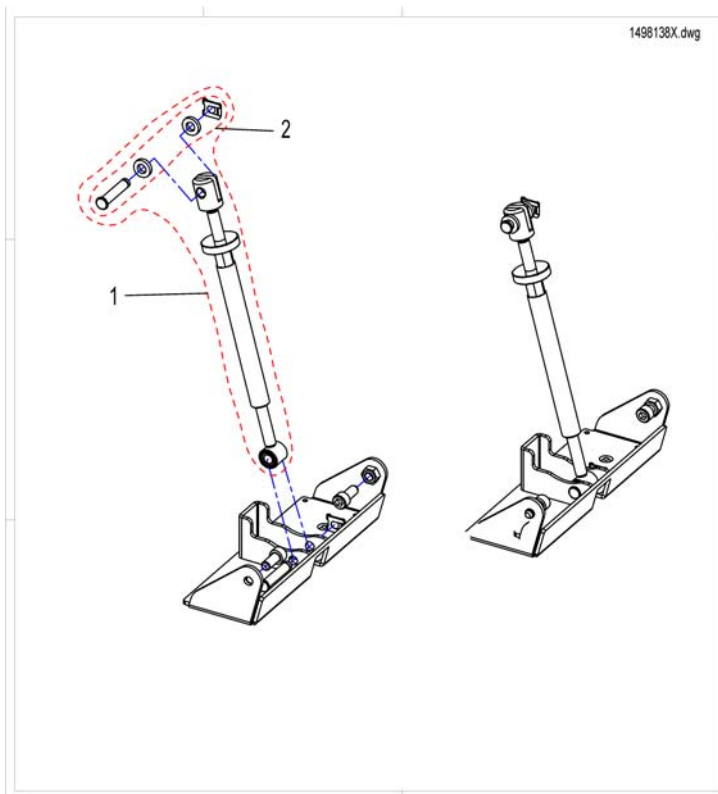


Pos.	Part number	Description	Valid from → until	Qty
1	SP1420004	Spindle for Tilt, cpl.		1
2	SP1419943	Fixation kit		1

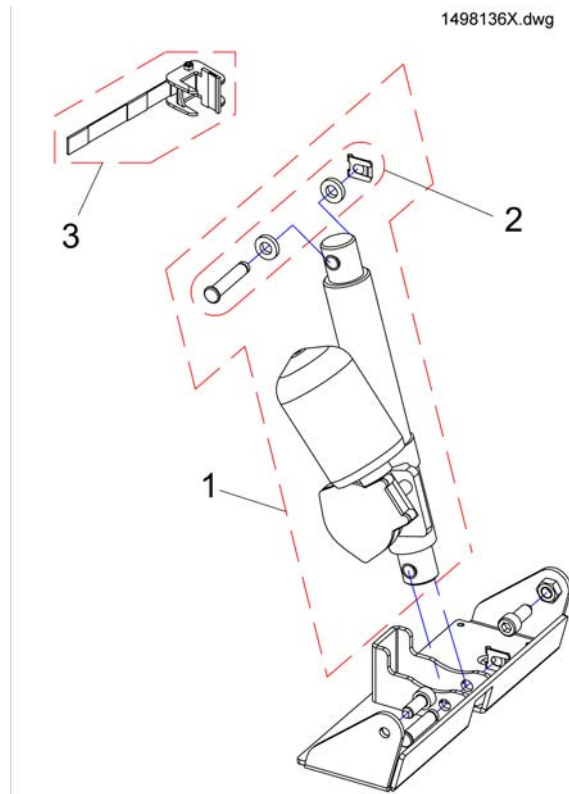

Pos.	Part number	Description	Valid from → until	Qty
1	SP1434913	actuator cpl.		1
2	SP1419943	Fixation kit		1
3	SP1498403	Lock, cpl.		1


# Seat Riser with Tilt actuator

1440589X.00

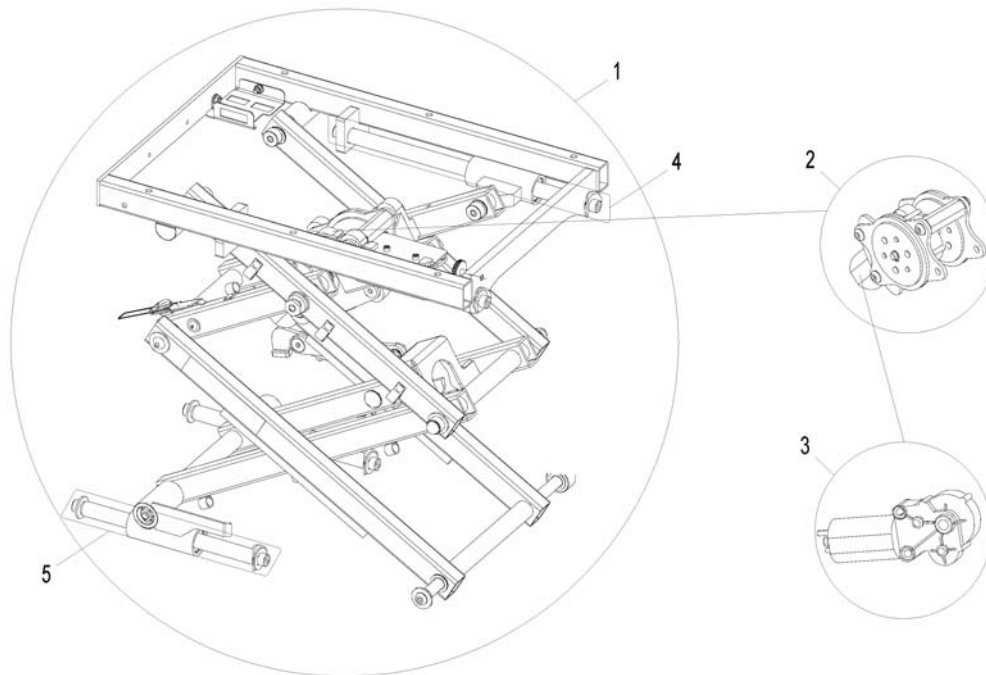


Pos.	Part number	Description	Valid from → until	Qty
1	SP1498319	Lifter unit without liftingmotor, complete		1
2	1442740	Adjusting unit for Tilt, assembled		1
3	SP1420475	Motor		1
4	SP1522409	Upper Bearing-Kit		1
5	SP1522419	Lower Bearing-Kit		1

# Actuator for Seat Riser

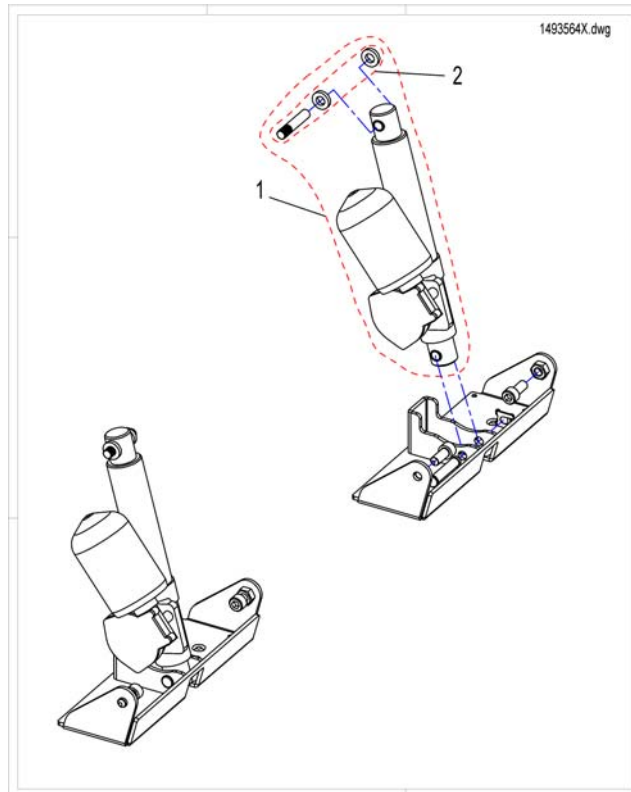

Pos.	Part number	Description	Valid from → until	Qty
1	SP1493564	Elevating motor cpl. 120/270mm		1
1	SP1498137	Elevating motor cpl. 120/300mm		1
2	SP1440503	Fixation kit		1


# Bellow Typhoon II

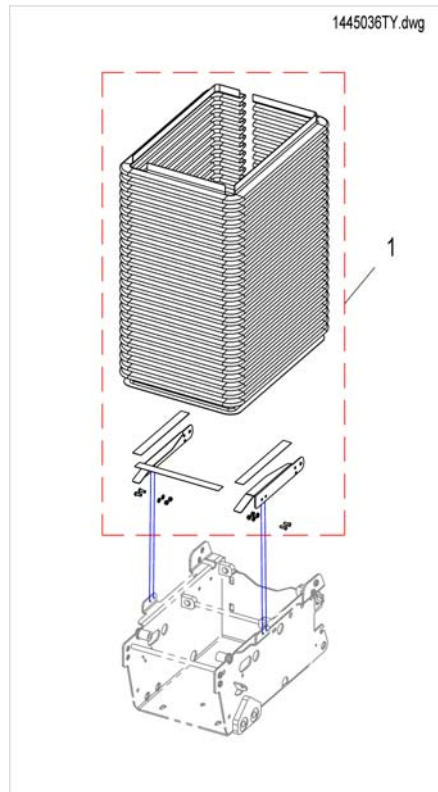

Pos.	Part number	Description	Valid from → until	Qty
-		In need, please contat Customer Service!		0




## Seat Frame Std.

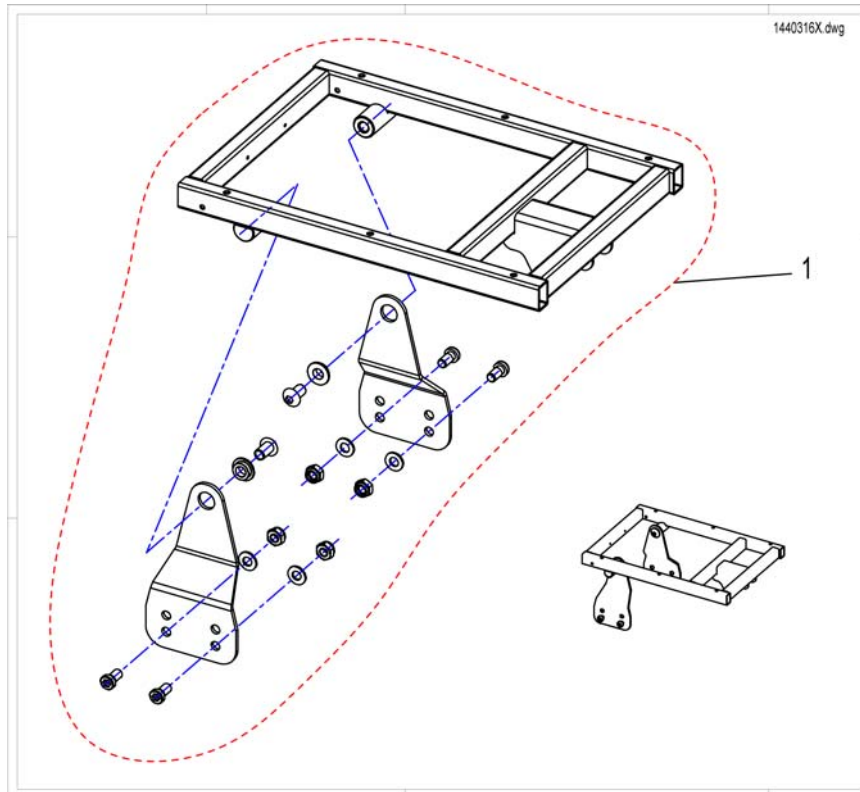


Pos.	Part number	Description	Valid from → until	Qty
1	SP1440316	Seat Support Standard cpl.		1

## Seat Plates

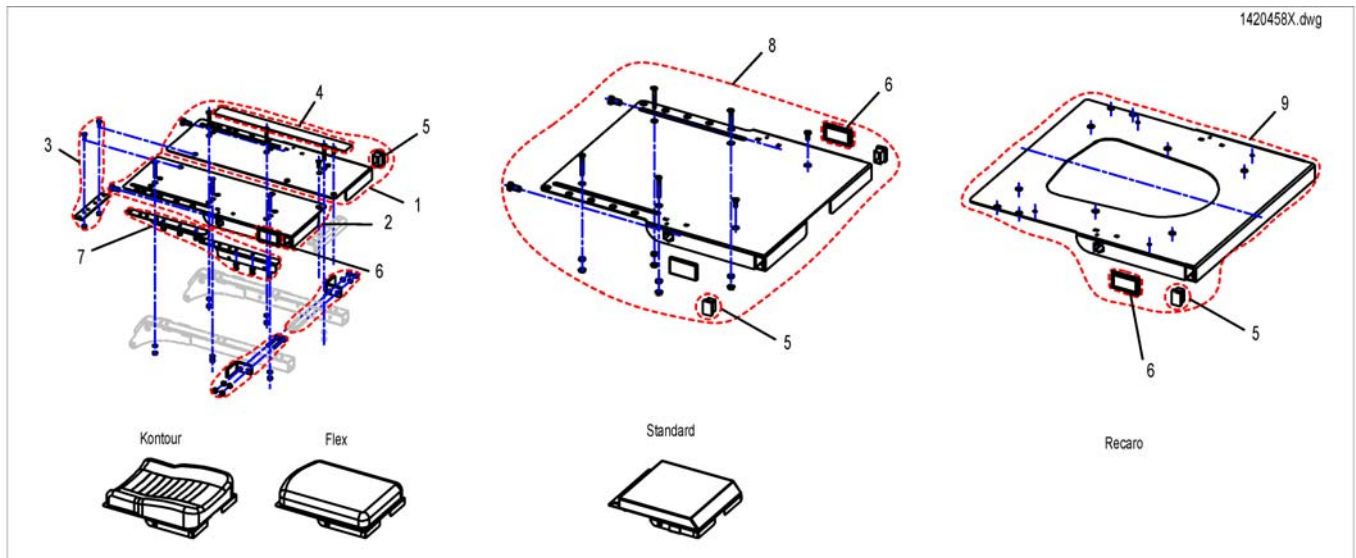

Pos.	Part number	Description	Valid from → until	Qty
1	SP1420458	Seat Plate LH		1
2	SP1420457	Seat Plate RH		1
3	SP1420496	Joining Plate		1
4	SPU21087	Velcro		1
5	SPL01060	Darning		1
6	SPP31298	Reflector, yellow		1
7	1510637	Stiffing Kit		1
8	SP1440623	Seat plate, cpl.		1


# Seat Frame Junior

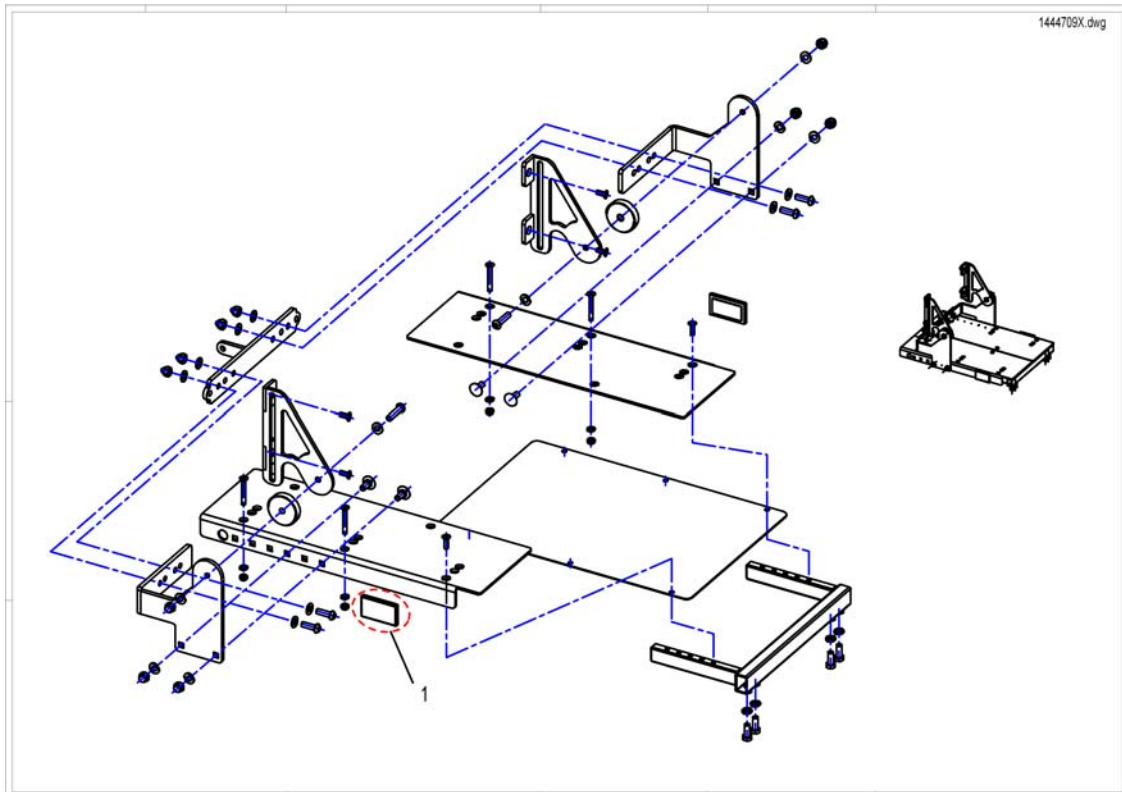

Pos.	Part number	Description	Valid from → until	Qty
1	SPP31298	Reflector, yellow		1

# Seat Cushion for Standard Seat

1571799.00

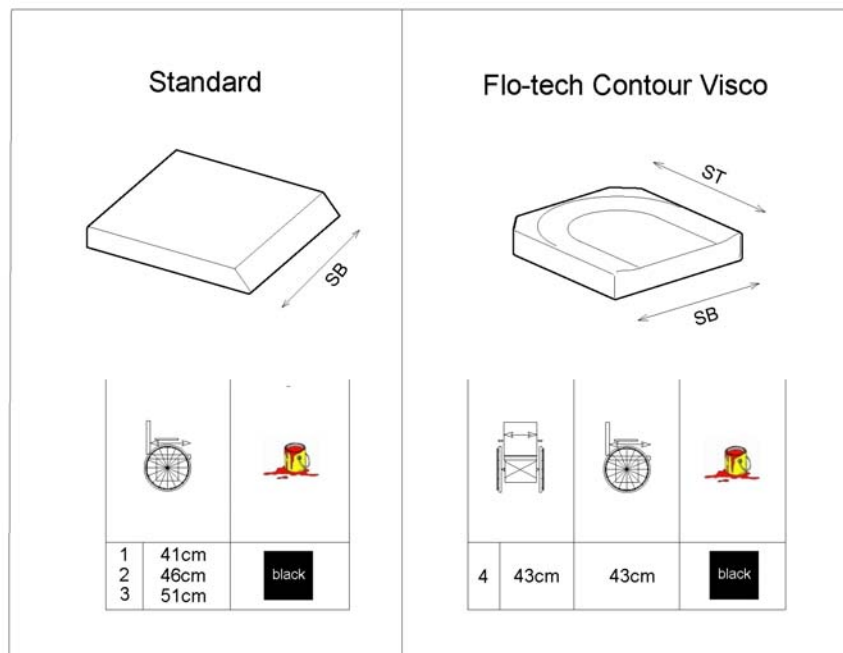


Pos.	Part number	Description	Valid from → until	Qty
1	1420460	Seat Cushion Standard SD 410mm <i>[1336+0326]</i>		1
2	1420461	Seat Cushion Standard SD 460mm <i>[1336+0327]</i>		1
3	1420462	Seat Cushion Standard SD 510mm <i>[1336+0330]</i>		1
4	1501113	Seat Cushion Flo-tech Contour Visco SW 43cm x SD 43cm <i>[1901]</i>		1

Contour



Flo-tech Contour Visco

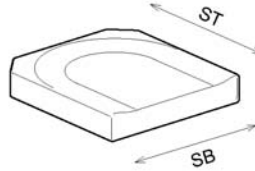


1	41cm	black
2	46cm	

3	43cm	43cm	black
4	45,5cm	53cm	
5	48cm	40,5cm	
6	48cm	43cm	

Pos.	Part number	Description	Valid from → until	Qty
1	1422072	Cushion Contour SW43cm, black [1337]		1
2	1422074	Cushion Contour SW48cm, black [1335]		1
3	SP1501113	Seat Cushion Flo-tech Contur Visco 43x43cm [1901]		1
4	SP1500993	Seat Cushion Flo-tech Contur Visco 45.5x53cm [1902]		1
5	SP1501116	Seat Cushion Flo-tech Contur Visco 48x40.5cm [1903]		1
6	SP1501117	Seat Cushion Flo-tech Contur Visco 48x43cm [1904]		1

# Seat Cushion for Flex Seat

1573087 fig

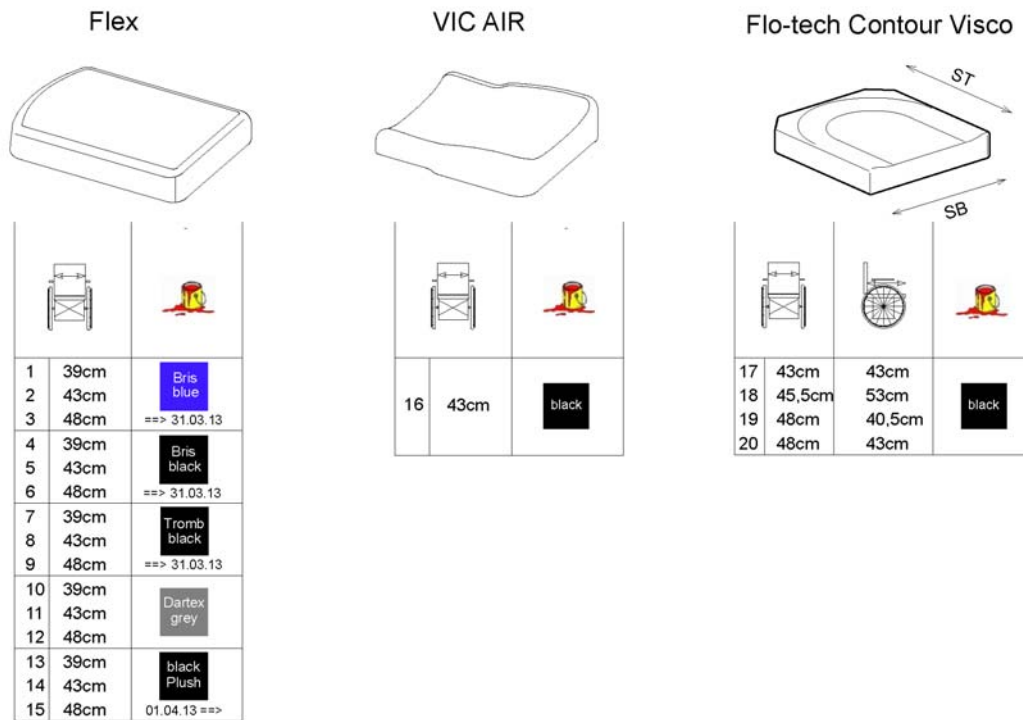


Pos.	Part number	Description	Valid from → until	Qty
1	1422529	Cushion Flex SW39cm, blue [1461]	→ 2013-03-31	1
2	1422530	Cushion Flex SW43cm, blue [1462]	→ 2013-03-31	1
3	1422532	Cushion Flex SW48cm, blue [1463]	→ 2013-03-31	1
4	1436561	Cushion Flex SW39cm, black [1461]	→ 2013-03-31	1
5	-	No longer available - please choose Black Plush [1462]	→ 2013-03-31	0
6	-	No longer available - please choose Black Plush [1463]	→ 2013-03-31	0
7	5252582-TR98	Seat cushion TROMB ZON 98 SW36 [1461]	→ 2013-03-31	1
8	5252583-TR98	Seat cushion TROMB ZON 98 SW41 [1462]	→ 2013-03-31	1
9	5252584-TR98	Seat cushion TROMB ZON 98 SW46 [1463]	→ 2013-03-31	1
10	5252582-TR23	SEAT CUSHION SB 39; DARTEX [1461]		1
11	5252583-TR23	CUSHION TROMB 410X530, SEAT, TR23 [1462]		1
12	5252584-TR23	CUSHION TROMB 460X530, SEAT, TR23 [1463]		1
13	SP1436561-TR35	Seat cushion SW39cm, black Plush [1461]	2013-04-01 →	1
14	SP1436562-TR35	Seat cushion SW43cm, black Plush [1462]	2013-04-01 →	1
15	SP1436563-TR35	Seat cushion SW48cm, black Plush [1463]	2013-04-01 →	1
16	1422531	Cushion Flex SW43cm, VIC AIR [1305]		1
17	SP1501113	Seat Cushion Flo-tech Contur Visco 43x43cm [1901]		1
18	SP1500993	Seat Cushion Flo-tech Contur Visco 45.5x53cm [1902]		1

Pos.	Part number	Description	Valid from → until	Qty
19	SP1501116	Seat Cushion Flo-tech Contur Visco 48x40.5cm [1903]		1
20	SP1501117	Seat Cushion Flo-tech Contur Visco 48x43cm [1904]		1

# Recaro Seat

1422546.dwg

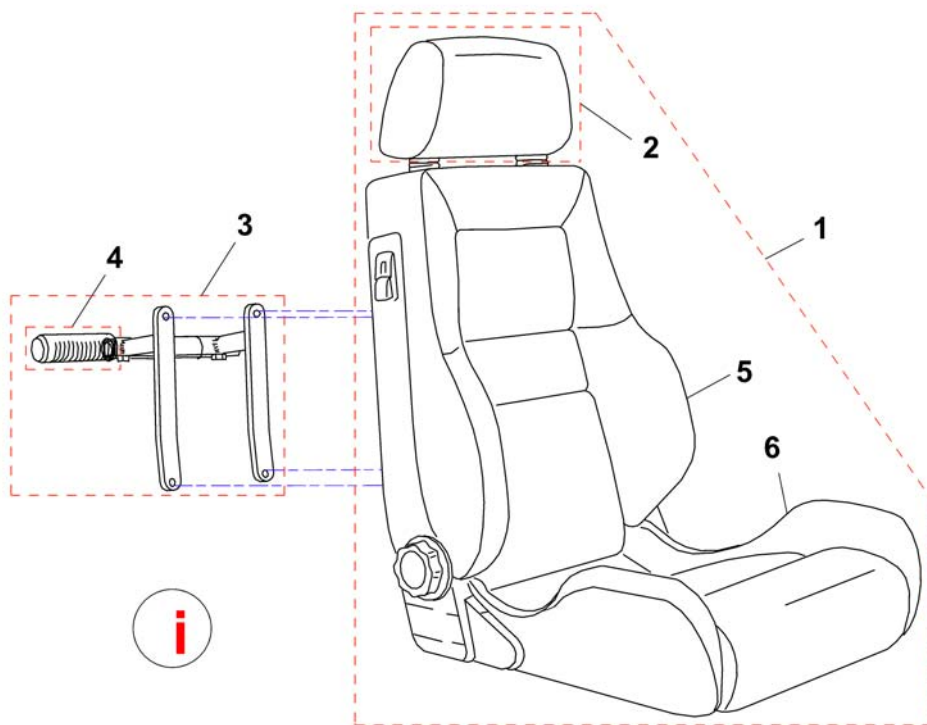

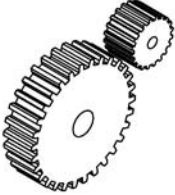



Pos.	Part number	Description	Valid from → until	Qty
i	-	Recaro Mobility 24V is no longer available. Please pay attention to the change list!		1
i	-	Change list Recaro 24V		1
	F367002	SAFETY BELT WITH VELCRO FASTNER F. RECARO		1
	Z163001A	SAFETY BELT WITH SNAP LOCK AND VELCRO FASTNER FOR RECARO		1
	SP1423173	RECARO SEAT MOUNTING KIT		1
1	SP1552597	Recaroseat Mobilty, 12V, cpl.		1
1	SP1422546	Recaro Seat N-Joy cpl.		1
2	N89999	HEADREST - SPORT NARDO		1
3	SP1423177	HOLDER F. PUSH HANDLE RECARO CPL.		1
4	1435421	Handle plastic black Ø22 "Invacare"		1
5	SP1552596	Backrest, Recaro Mobility, 12v, cpl. <i>Recaro Mobility 24V is no longer available. Please pay attention to the compatibility chart!</i>		1
5	SP1552781	Backrest Recaro Mobility, mechanic		1
6	SP1419207	Seatpart Recaro Mobility		1




# Change list Recaro 24V

1420465X.iso

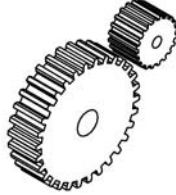
	 <b>24V</b>		 <b>12V</b>	 <b>Wizard</b>
<b>Storm4</b>	<b>O</b>	<b>O</b>	<b>P</b>	<b>P</b>
<b>G50</b>	<b>O</b>	<b>O</b>	<b>P</b>	<b>special</b>
<b>FDX</b>	<b>O</b>	<b>O</b>	<b>P</b>	<b>special</b>
<b>Typhoon</b>	<b>O</b>	<b>P</b>	<b>O</b>	<b>O</b>
<b>Typhoon II</b>	<b>O</b>	<b>P</b>	<b>O</b>	<b>O</b>
<b>Zephyr</b>	<b>O</b>	<b>P</b>	<b>O</b>	<b>O</b>
<b>Tornado</b>	<b>O</b>	<b>P</b>	<b>O</b>	<b>O</b>
<b>Storm3</b>	<b>O</b>	<b>P</b>	<b>O</b>	<b>O</b>

Pos.	Part number	Description	Valid from → until	Qty
-	-	For further information please contact your customer service.		1

## Seat Cushion for Junior Seat

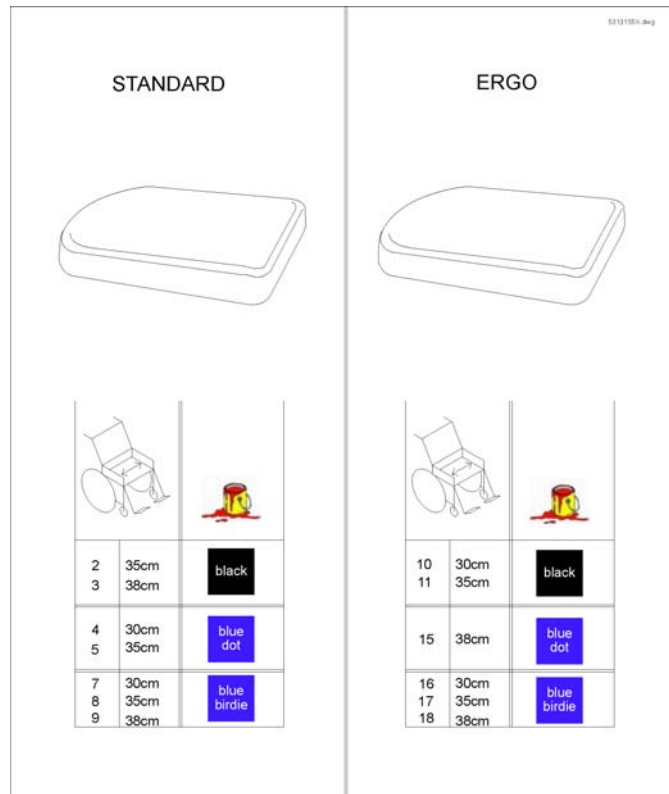


Pos.	Part number	Description	Valid from → until	Qty
2	5313181	Seat Cushion STD SW35cm, Vinyl Black		1
3	5313207	Seat Cushion STD SW38cm, Vinyl Black		1
4	5313152	Seat Cushion STD SW30cm, Blue Birdie		1
5	5313178	Seat Cushion STD SW35cm, Blue Birdie		1
7	5313154	Seat Cushion STD SW30cm, Blue Dot		1
8	5313180	Seat Cushion STD SW35cm, Blue Dot		1
10	5313159	Seat Cushion Ergo SW30cm, Vinyl Black		1
11	5313185	Seat Cushion Ergo SW35cm, Vinyl Black		1
15	5313208	Seat Cushion Ergo SW38cm, Blue Birdie		1
16	5313158	Seat Cushion Ergo SW30cm, Blue Dot		1
17	5313184	Seat Cushion Ergo SW35cm, Blue Dot		1
18	5313210	Seat Cushion Ergo SW38cm, Blue Dot		1

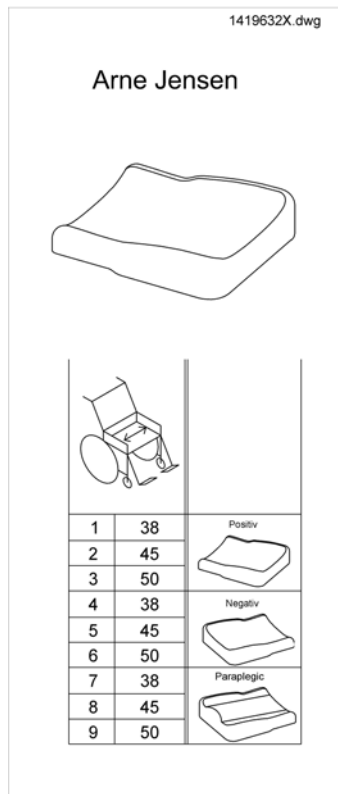

Pos.	Part number	Description	Valid from → until	Qty
1	1419632	Seat Cushion positive, SW 38cm, black		1
2	5313953	Seat Cushion positive, SW 45cm, black		1
3	5314038	Seat Cushion positive, SW 50cm, black		1
4	1419633	Seat Cushion negative, SW 38cm, black		1
5	5313958	Seat Cushion negative, SW 45cm, black		1
6	5314041	Seat Cushion negative, SW 50cm, black		1
7	1419634	Seat Cushion paraplegic, SW 38cm, black		1
8	5313962	Seat Cushion paraplegic, SW 45cm, black		1
9	5314044	Seat Cushion paraplegic, SW 50cm, black		1

## Backrest Standard Seat, settable


1422061X.iso

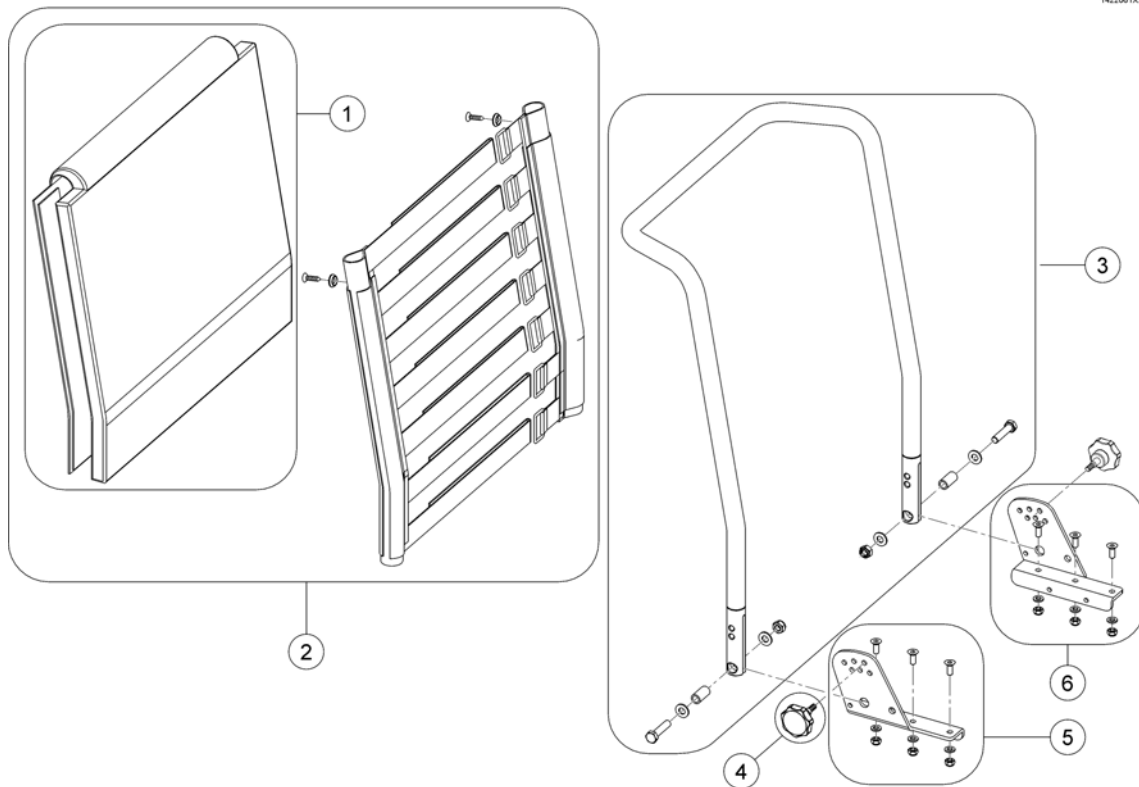

Pos.	Part number	Description	Valid from → until	Qty
1	1420445	Backrest upholstery, 480mm		1
1	1420452	Backrest upholstery, 540mm		1
2	1422062	Sling Back with Backrest upholstery 480 mm		1
2	1422063	Sling Back with Backrest upholstery 540mm		1
3	SP1419996	Backrest Frame 48cm height		1
3	SP1420465	Backrest Frame 54cm height		1
4	SP1420444	Star Grip M8		1
5	SP1541567	Backrest bracket RH		1
6	SP1541568	Backrest bracket LH		1


# Backrest Recliner

1443270X.060

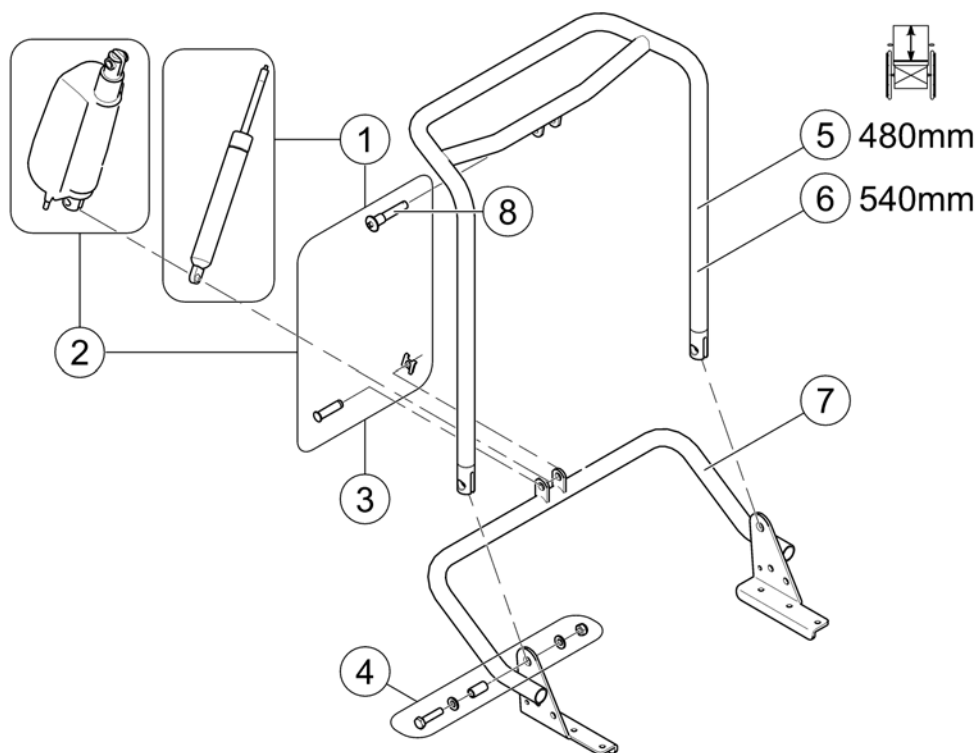

Pos.	Part number	Description	Valid from → until	Qty
-	SP1562940	Adaptercable Shark to reverse actuator		1
1	SPN368007	Gas Pressure Spring cpl.		1
2	SP1446520	Actuator ACS2 / Shark		1
3	SP1583897	Fixation Kit		1
4	SP1579749	Fixation Kit, Standard Backrest		2
5	1442864-9005	Backrest frame for 480 mm height (push handle), adjustable		1
6	1442866-9005	Backrest frame for 540 mm height (push handle), adjustable		1
7	1442845-9005	Lower Bracket for backrest frame		1
8	F368016.99	Locking Bolt - Quick Release		1


## Backrest Flex Seat

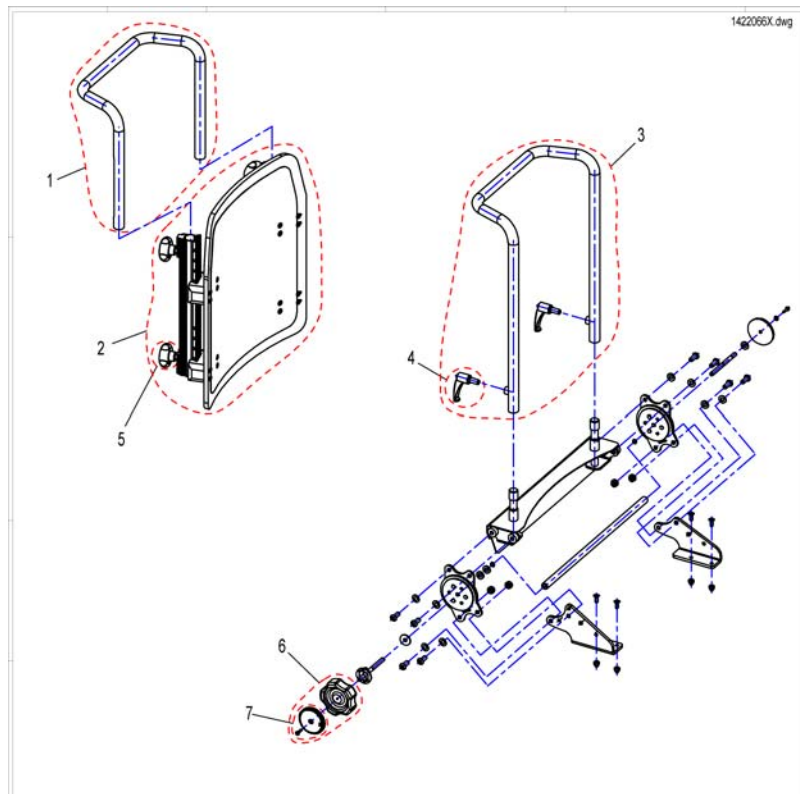

Pos.	Part number	Description	Valid from → until	Qty
1	SP1425840	Push Handle Flex SW 39cm		1
1	SP1425839	Push Handle Flex SW 43/48cm		1
2	SP1422126	Backrest Flex SW 39cm, without Upholstery, cpl.		1
2	SP1422125	Backrest Flex SW 48cm, without Upholstery, cpl.		1
2	SP1422124	Backrest Flex SW 43cm, without Upholstery, cpl.		1
3	SP1422070	Backframe Contur, Seatwidth 43/48cm, cpl.		1
4	L31996	Lever M10x20		1
5	SP5253088	Pressure Screw		1
6	SP1420483	Hand Wheel, cpl.		1
7	SP1421696	Reflector cpl.		1

# Backrest adjustment Flex/Contour electric

1440456X.dwg

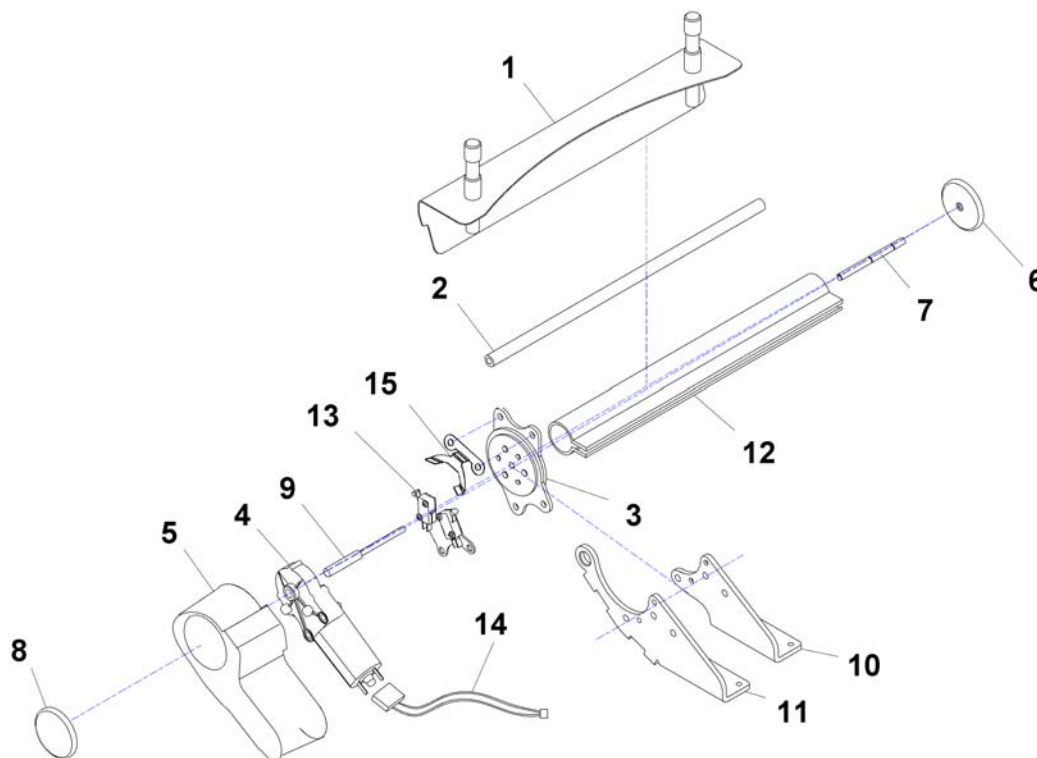

Pos.	Part number	Description	Valid from → until	Qty
	SP1440407	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 39CM, CPL.		1
	SP1440405	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 43CM, CPL.		1
	SP1440406	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 48CM, CPL.		1
	SP1440410	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 39CM, FOR SLIDING ARMRESTS, CPL.		1
	SP1440408	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 43CM, FOR SLIDING ARMRESTS, CPL.		1
	SP1440409	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 48CM, FOR SLIDING ARMRESTS, CPL.		1
	SP1423210	BACKREST ADJUSTMENT ASSEMBLY KIT		1
	SP1529386	No description available.		1
1	1422119-9005	BACKREST BRACKET 390 MM		1
1	1419992-9005	BACKREST BRACKET 430 MM		1
1	1422112-9005	BACKREST BRACKET 480 MM		1
2	1420489	CONNECTING TUBE 380 MM		1
2	1420490	CONNECTING TUBE 430 MM		1
2	1420491	CONNECTING TUBE 480 MM		1
3	SP1423211	SEAT BRACE ELECTRIC BACKREST, 1 PAIR		1
4	1420475	GEAR MOTOR GMPG		1
5	1420998	MOTOR CAP		1
5	1428057	MOTOR CAP FOR ELECTRIC BACKREST SLIDING ARMREST		1
6	1421696	PERFORATED REFLECTOR, YELLOW		1
7	1420488-0002	SUPPORT SHAFT		1
8	1420482	REFLECTOR		1
9	1420487-0002	MOTOR SHAFT		1
10	1420477-9005	BRACE BRACKET MANUAL/ELECTRICAL LH		1
10	1421708-9005	BRACE BRACKET ELECTRICAL, RH		1
11	1429563-9005	FITTING ATTACHMENT RH F. SLIDING ARMREST		1
11	1429564-9005	FITTING ATTACHMENT LH F. SLIDING ARMREST		1
12	1442577	PROTECTIVE COVERING 380MM		1
12	1442578	PROTECTIVE COVERING 430MM		1
12	1442579	PROTECTIVE COVERING 480MM		1

## Backrest adjustment Flex/Contour electric


Pos.	Part number	Description	Valid from → until	Qty
13	SP1440711	TWO MICRO SWITCHES WITH ASSEMBLY SHEET METAL, PRE-ASSEMBLED		1
14	SP1440726	ACTUATORCABLE RECLINER ACS2 / SHARK		1
15	1440456-0003	SWITCHING OFF SHEET METAL BACK		1



# Backrest Cushion for Contour Seat

1573703.iso

Contour

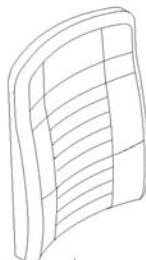

1	43cm	
2	48cm	

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422076	Back Cushion Contour, SW 43cm, black		1
2	SP1422078	Back Cushion Contour, SW 48cm, black		1
-	SP1423216	Mounting Hardware for Back Cushion		1

# Backrest Cushion for Flex Seat

1572719 no

Flex



1	39cm	blue
2	43cm	
3	48cm	==> 31.03.13
4	39cm	black
5	43cm	
6	48cm	==> 31.03.13
7	39cm	Dartex grey
8	43cm	
9	48cm	
10	39cm	black Plush
11	43cm	
12	48cm	01.04.13 ==>


VIC AIR



13	43cm	black

Pos.	Part number	Description	Valid from → until	Qty
1	1422534	Back Cushion SW39cm, blue	→ 2013-03-31	1
2	1422535	Back Cushion SW43cm, blue	→ 2013-03-31	1
3	1422536	Back Cushion SW48cm, blue	→ 2013-03-31	1
4	1436564	Back Cushion SW39cm, black	→ 2013-03-31	1
5	-	No longer available - please choose Black Plush	→ 2013-03-31	0
6	-	No longer available - please choose Black Plush	→ 2013-03-31	0
7	5252506-TR23	Back Cushion SW39cm, Dartex		1
8	5252507-TR23	Back Cushion SW43cm, Dartex		1
9	5252508-TR23	Back Cushion SW48cm, Dartex		1
10	SP1436564-TR35	Back Cushion SW39cm, black Plush	2013-04-01 →	1
11	SP1436565-TR35	Back Cushion SW43cm, black Plush	2013-04-01 →	1
12	SP1436566-TR35	Back Cushion SW48cm, black Plush	2013-04-01 →	1
13	1422537	Back Cushion SW43cm, VIC AIR		1

## Backrest Cushion for Junior Seat

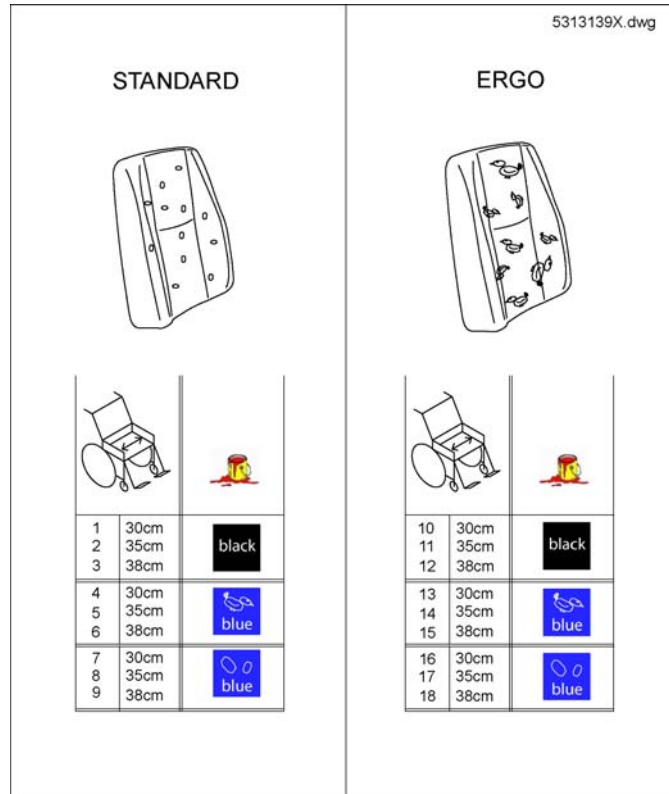

Pos.	Part number	Description	Valid from → until	Qty
1	5313142	Back Cushion STD SW30cm, Vinyl Black		1
2	5313168	Back Cushion STD SW35cm, Vinyl Black		1
3	5313194	Back Cushion STD SW38cm, Vinyl Black		1
4	5313139	Back Cushion STD SW30cm, Blue Birdie		1
5	5313165	Back Cushion STD SW35cm, Blue Birdie		1
6	5313191	Back Cushion STD SW38cm, Blue Birdie		1
7	5313141	Back Cushion STD SW30cm, Blue Dot		1
8	5313167	Back Cushion STD SW35cm, Blue Dot		1
9	5313193	Back Cushion STD SW38cm, Blue Dot		1
10	5313146	Back Cushion Ergo SW30cm, Vinyl Black		1
11	5313172	Back Cushion Ergo SW35cm, Vinyl Black		1
12	5313198	Back Cushion Ergo SW38cm, Vinyl Black		1
14	5313169	Back Cushion Ergo SW35cm, Blue Birdie		1
15	5313195	Back Cushion Ergo SW38cm, Blue Birdie		1
16	5313145	Back Cushion Ergo SW30cm, Blue Dot		1
17	5313171	Back Cushion Ergo SW35cm, Blue Dot		1
18	5313197	Back Cushion Ergo SW38cm, Blue Dot		1


# Optimist Seats

OPTIMIST 1		OPTIMIST 2		OPTIMIST 3		OPTIMIST 5					
1	38cm	46cm	5	38cm	46cm	11	45cm	60cm	13	45cm	70cm
2	45cm	46cm	6	38cm	46cm	12	50cm	70cm	14	50cm	70cm
3	45cm	60cm	7	45cm	46cm						
4	50cm	60cm	8	45cm	60cm						
			9	45cm	60cm						
			10	50cm	60cm						

Arne\_Jensen\_Sitze.dwg


Pos.	Part number	Description	Valid from → until	Qty
1	1419630	Backrest cushion Optimist Typ1, SW38cm		1
2	5313888	Backrest cushion Optimist Typ1, SW45cm		1
3	5313889	Backrest cushion Optimist Typ1, SW45cm		1
4	1419625	Backrest cushion Optimist Typ1, SW50cm		1
5	1419631	Backrest cushion Optimist Typ2, SW38cm		1
6	5313867	Backrest cushion Optimist Typ2, SW38cm, el. Strap		1
7	5313882	Backrest cushion Optimist Typ2, SW45cm		1
8	5313883	Backrest cushion Optimist Typ2, SW45cm		1
9	5313895	Backrest cushion Optimist Typ2, SW45cm, el. Strap		1
10	1419628	Backrest cushion Optimist Typ2, SW50cm		1
11	5313878	Backrest cushion Optimist Typ3, SW45cm		1
12	5313877	Backrest cushion Optimist Typ3, SW50cm		1
13	5313872	Backrest cushion Optimist Typ5, SW45cm		1
14	5313871	Backrest cushion Optimist Typ5, SW50cm		1

## Central Legrest, manual

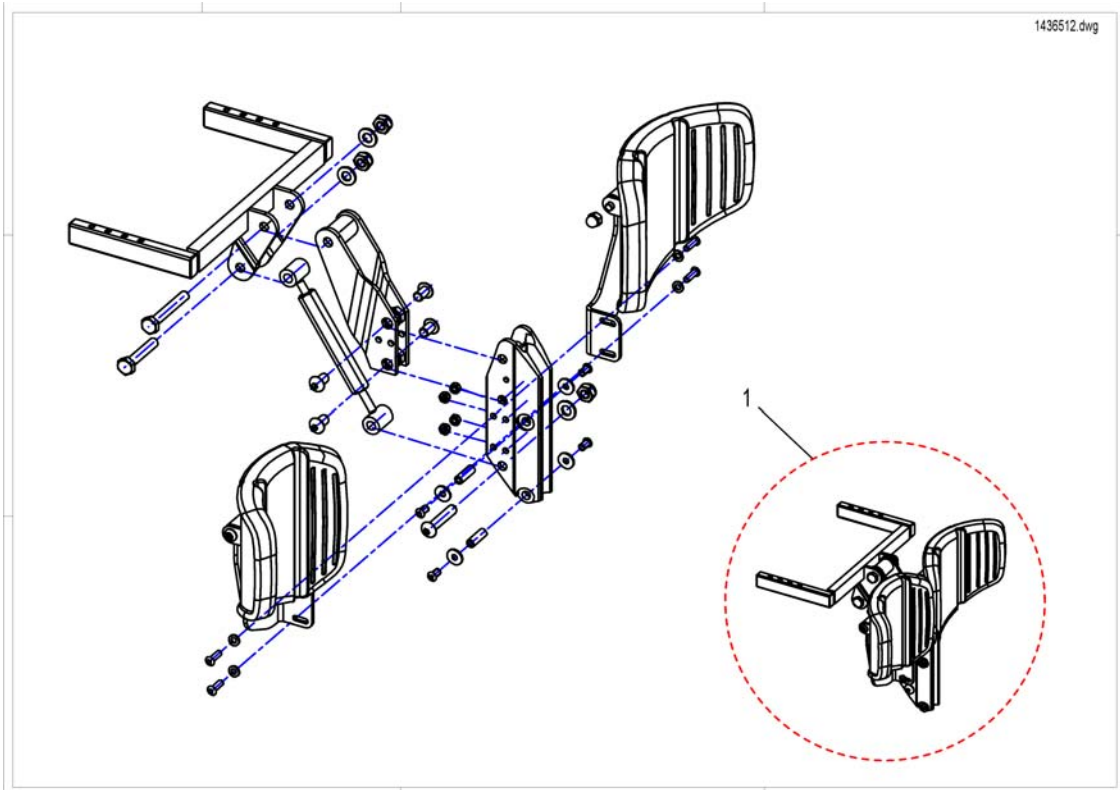


Pos.	Part number	Description	Valid from → until	Qty
1	SP1436512	Centre Hanger, manual, cpl.		1

## Central Legrest, electr.

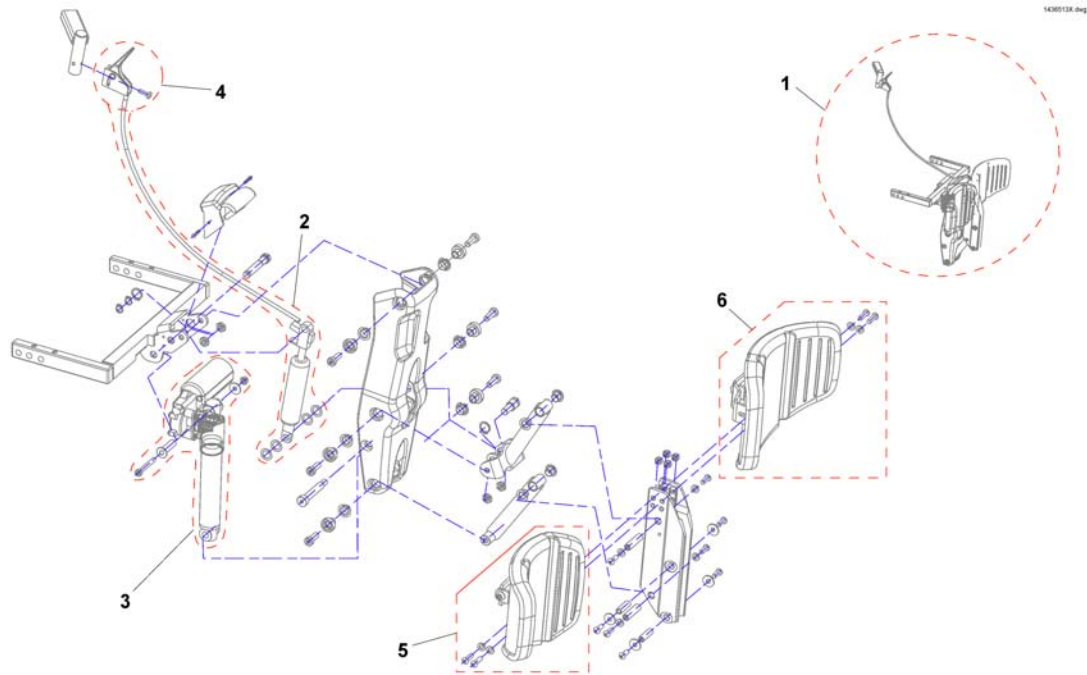

Pos.	Part number	Description	Valid from → until	Qty
1	SP1436513	Centre Hanger, electr. 100°, cpl.		1
2	SP1440593	Gas Pressure Spring cpl.		1
3	SP1440592	Actuator cpl.		1
4	SP1441590	Bowden Cable with Lever		1
5	1440594	Calf Plate with holder RH		1
6	1436076	Calf Plate with holder LH		1


## Legrest fixation

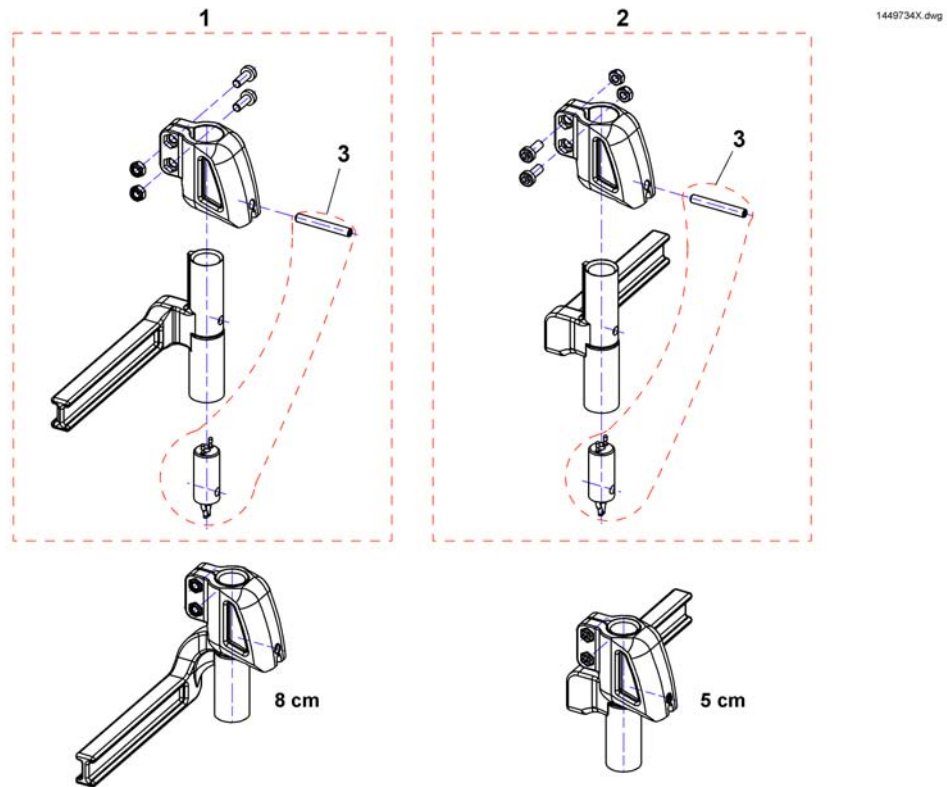

Pos.	Part number	Description	Valid from → until	Qty
1	SP1449733	Legrest Support Vari series, man. 5cm - LH		1
1	SP1449737	Footrest/Legrest receptacle, manuell, LH, 8cm		1
1	SP1449735	Footrest/Legrest receptable, powered, LH, 5cm		1
1	SP1449739	Footrest/Legrest receptacle, powered, LH, 8cm		1
2	SP1449732	Legrest Support Vari series, man. 5cm - RH		1
2	SP1449736	Footrest/Legrest receptacle, manuell, RH, 8cm		1
2	SP1449734	Footrest/Legrest receptable, powered, RH, 5cm		1
2	SP1449738	Footrest/Legrest receptacle, powered, RH, 8cm		1
3	SP1450228	Power Supply Cable Legrest (990mm)		1

# Hanger Vari F


144934 / X.dwg

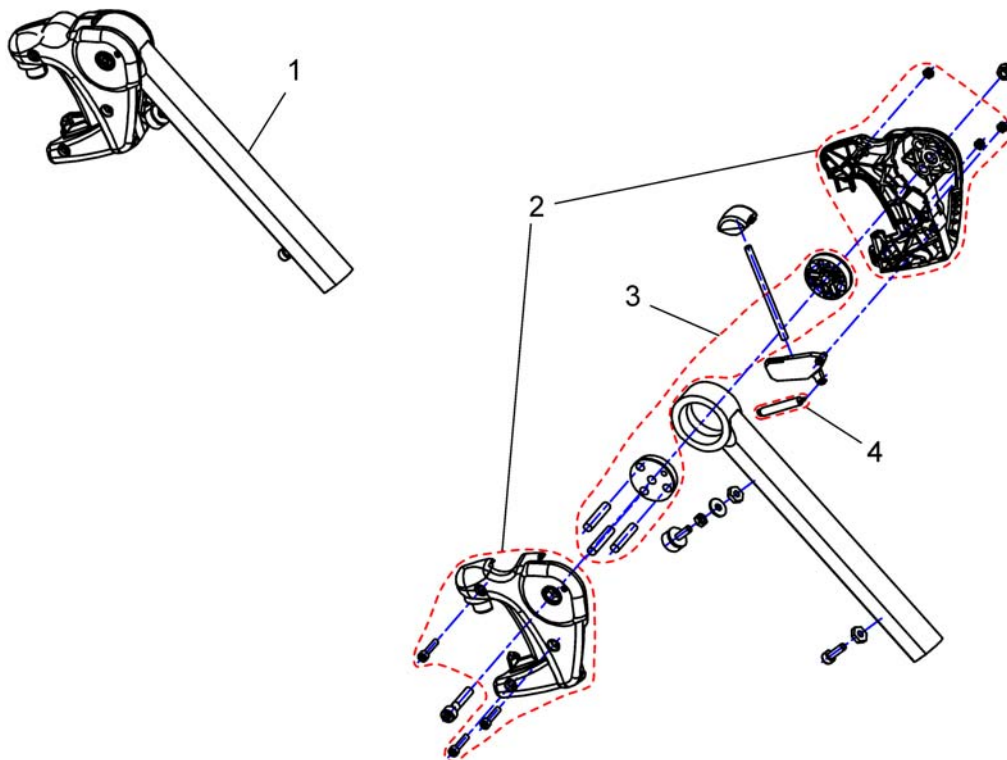

Pos.	Part number	Description	Valid from → until	Qty
1	SP1449347	Hanger Vari F LH cpl.		1
1	SP1449346	Hanger Vari F RH cpl.		1
2	SP1434955	Cover Legrest cpl.		1
3	1434956	Mechanic		1
4	SPZ173178	Spring		1




## Hanger Vari A (04/2009 =>)

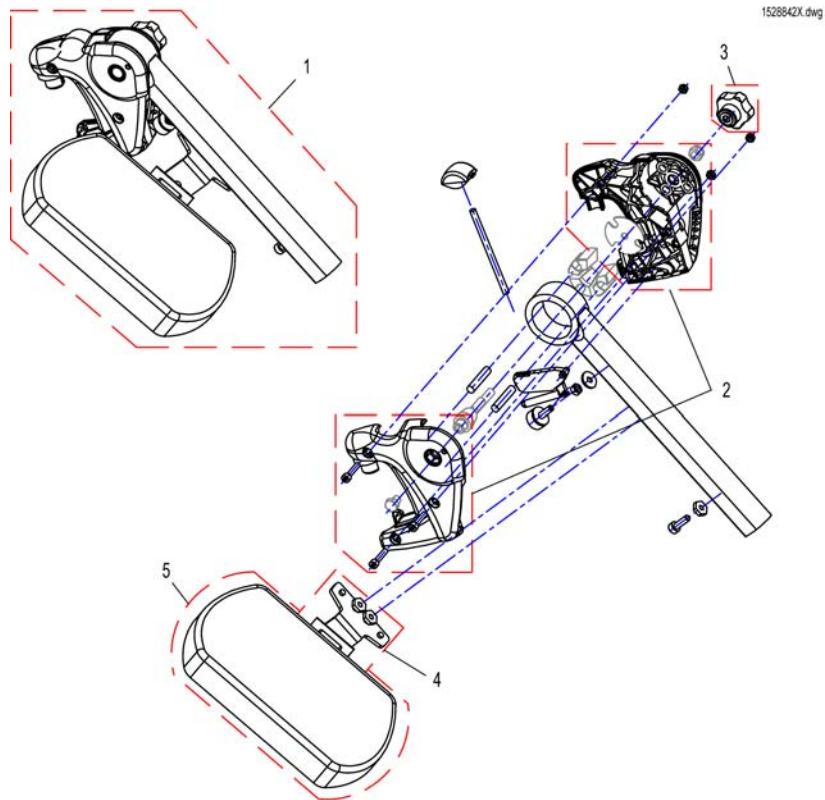

Pos.	Part number	Description	Valid from → until	Qty
-	SP1527820	Rework-Kit Vari A Legrest Version 1 => Version 2 <i>Vari A Version 1 =&gt; Version 2 Upgrade Kit</i>		1
1	SP1527498	Hanger Vari A LH, cpl.		1
1	SP1527492	Hanger Vari A RH, cpl.		1
2	SP1434955	Cover Legrest cpl.		1
3	L34011	Handknob M8 Tristarfemale		1
4	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
4	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
5	6301333	Legrest calf pad		1

# Hangers ADM LH/RH manual


1449409X.dwg

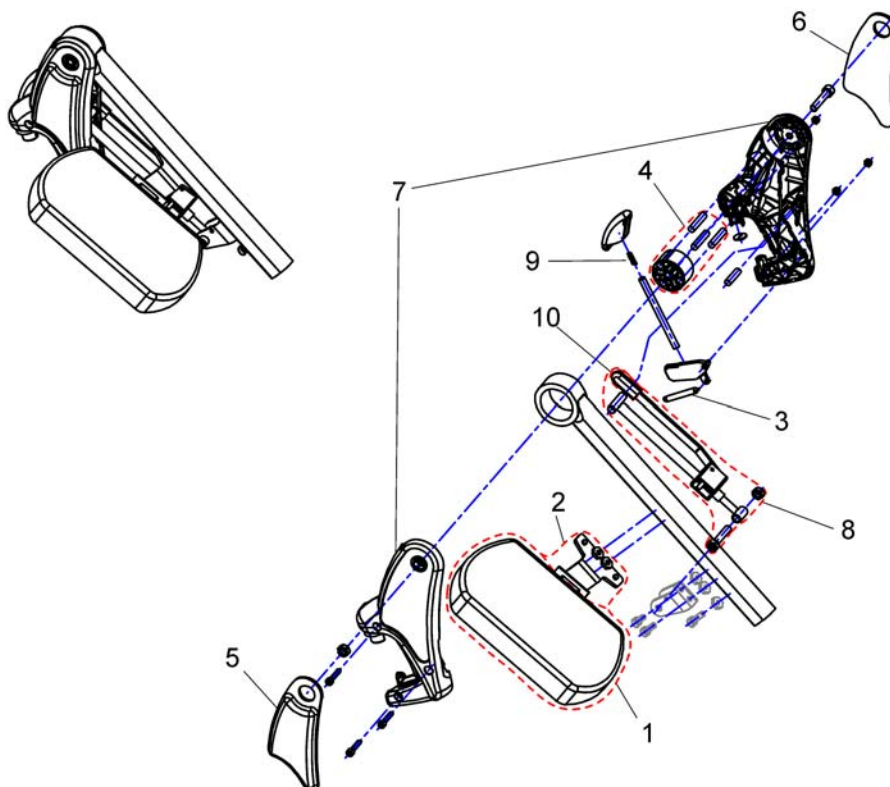

Pos.	Part number	Description	Valid from → until	Qty
1	6301333	Legrest calf pad		1
2	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
2	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
3	SPZ173178	Spring		1
4	SPZ174134	Floating Cylinder With Pin		1
5	SP1450410	Knee Upholstery LH		1
6	SP1450448	Knee Upholstery RH		1
7	SP1449293	Ad-Cover cpl.		1
8	SPZ174130	ADM Adjustment mechanism cpl.		1
9	SPN12196	Spring		1
10	SP1006490	Cap for tripping lever		1
11	SP1449410	Hanger ADM RH cpl.		1
11	SP1449409	Hanger ADM LH cpl.		1


# Hanger Vari ADE/ADP electric (07/2005 =>)

1449407X.dwg

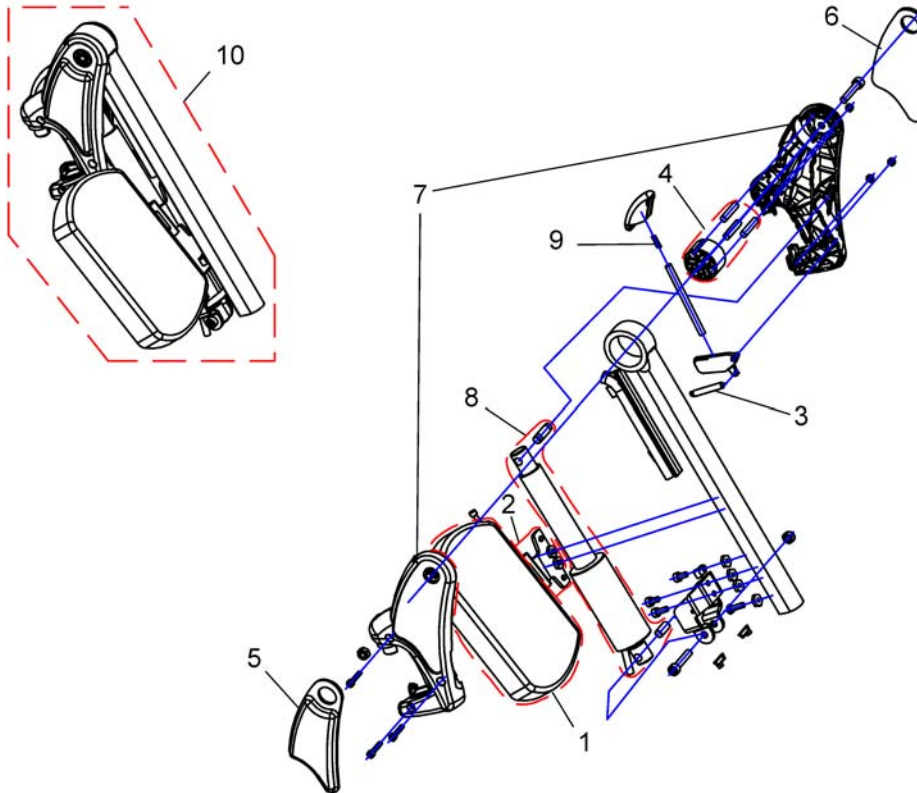


Pos.	Part number	Description	Valid from → until	Qty
1	6301333	Legrest calf pad		1
2	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
2	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
3	SPZ173178	Spring		1
4	SPZ174134	Floating Cylinder With Pin		1
5	SP1450448	Knee Upholstery RH		1
6	SP1450410	Knee Upholstery LH		1
7	SP1449293	Ad-Cover cpl.		1
8	SPZ151006	Actuator with circuit board, preassembled		1
9	SPN12196	Spring		1
10	SP1449408	Hanger ADP RH cpl.		1
10	SP1449407	Hanger ADP LH cpl.		1

# Footplate

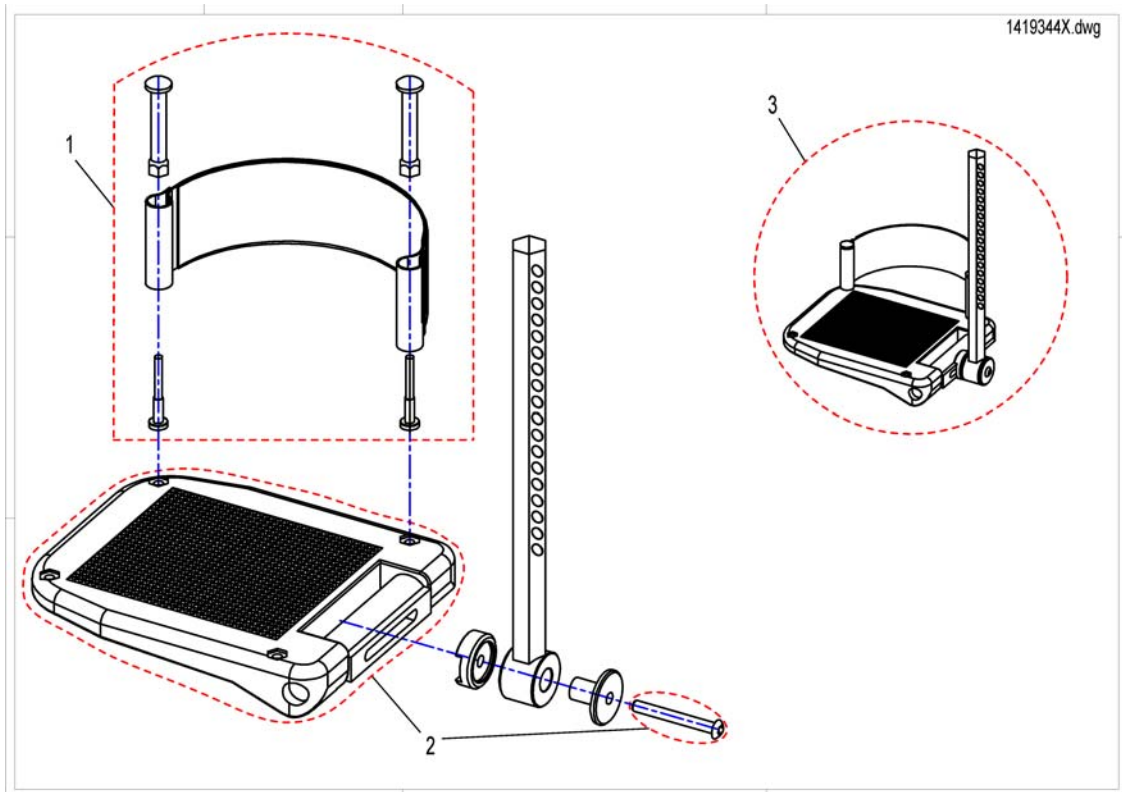


Pos.	Part number	Description	Valid from → until	Qty
1	SP5321579	Heel Strap		1
2	SP5106376	Footplate		1
3	SP1419344	Footplate LH Anlge And Depth Adjustable, With Support Tube cpl.		1
3	SP1419345	Footplate RH Anlge And Depth Adjustable, With Support Tube cpl.		1

## Footplate with support tube fix RH/LH

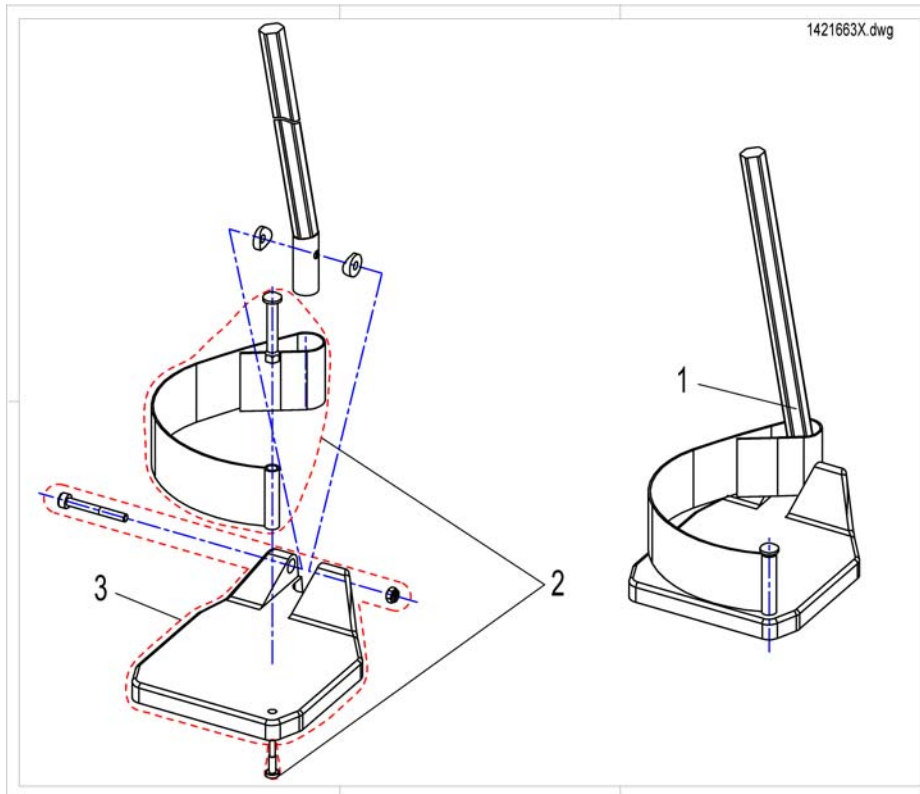

Pos.	Part number	Description	Valid from → until	Qty
1	SP1421662	Footplate RH fix, with Support Tube cpl.		1
1	SP1421663	Footplate LH fix, with Support Tube cpl.		1
2	SPB273011	Heel Strap		1
3	SPZ173217	Footplate LH		1
3	SPZ173218	Footplate RH		1


## Footplate with support tube angle adjust. RH/LH

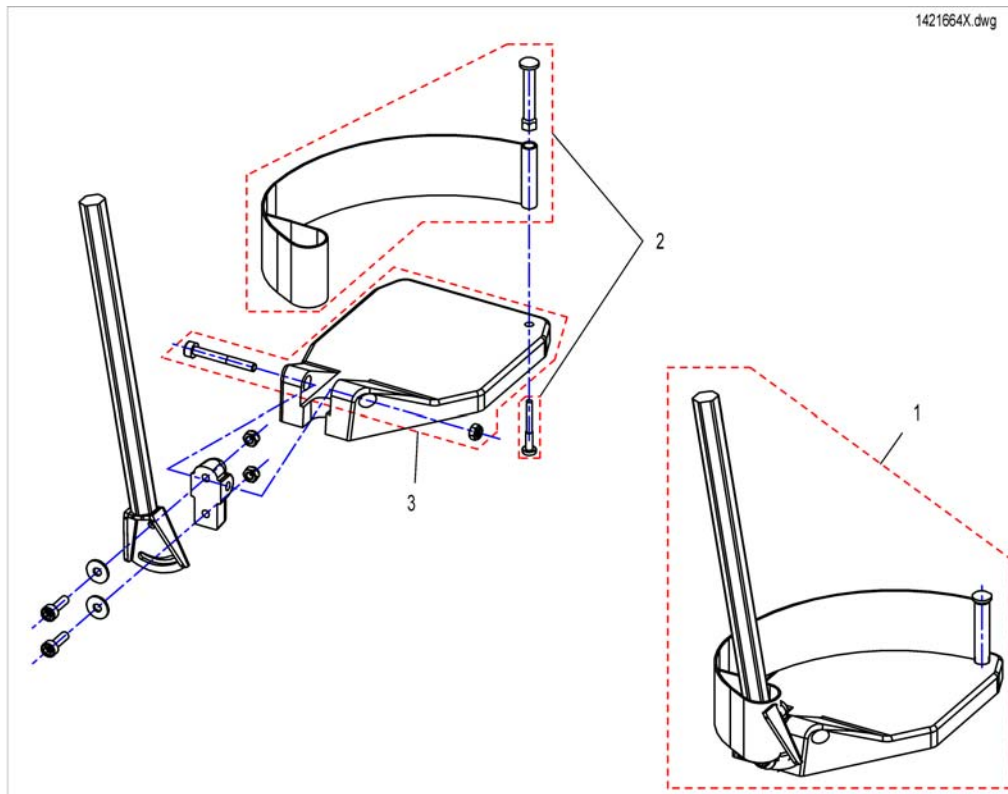

Pos.	Part number	Description	Valid from → until	Qty
1	SP1421664	Footplate RH anlge adjustable, with Support Tube cpl.		1
1	SP1421665	Footplate LH anlge adjustable, with Support Tube cpl.		1
2	SPB273011	Heel Strap		1
3	SPZ173217	Footplate LH		1
3	SPZ173218	Footplate RH		1

## Support Tube with Footplate for central Legrest


1440290X.dwg

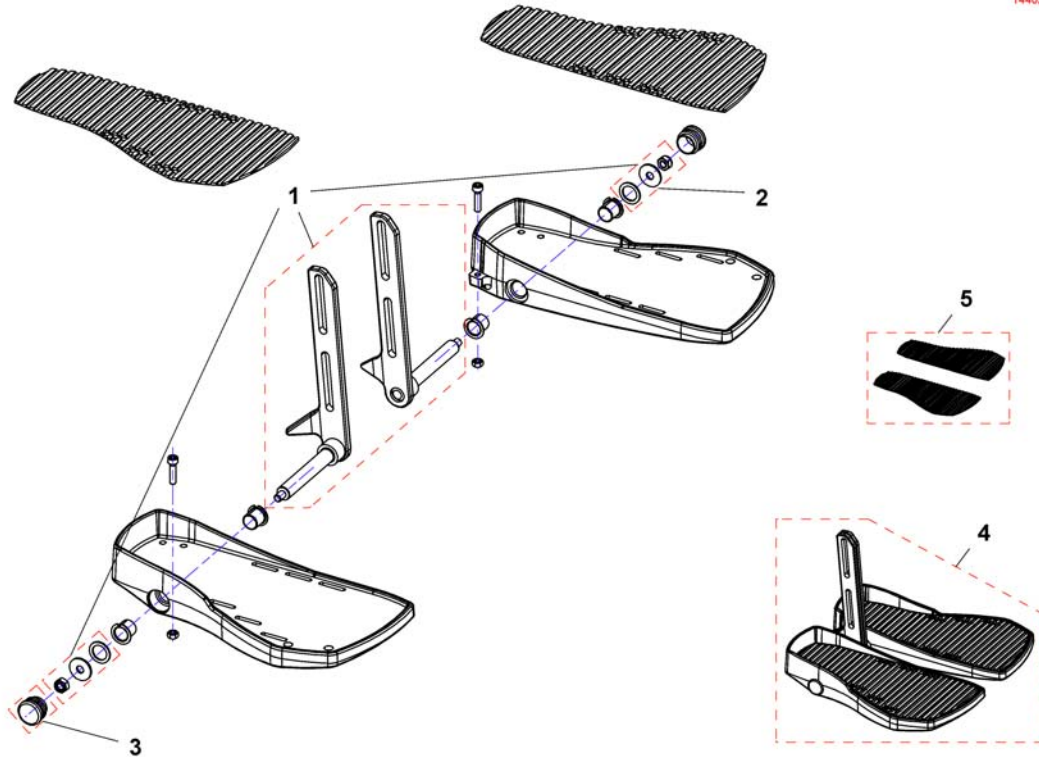

Pos.	Part number	Description	Valid from → until	Qty
1	SP1440442	Central Lower Legrest, small		1
1	SP1440443	Central Lower Legrest, medium		1
1	SP1440444	Central Lower Legrest, large		1
2	SPF00002	Fixation Kit		1
3	SP1440517	Darning		1
4	SP1440290	Central Lower Legrest, small (32-38cm), cpl.		1
4	SP1440291	Central Lower Legrest, medium (38-44cm), cpl.		1
4	SP1440292	Central Lower Legrest, large (44-50cm), cpl.		1
5	SP1546720	Footboardrubber		1

# Footrests Vari, Flip-Up (01.08.10 =>)


1529644X.jpg

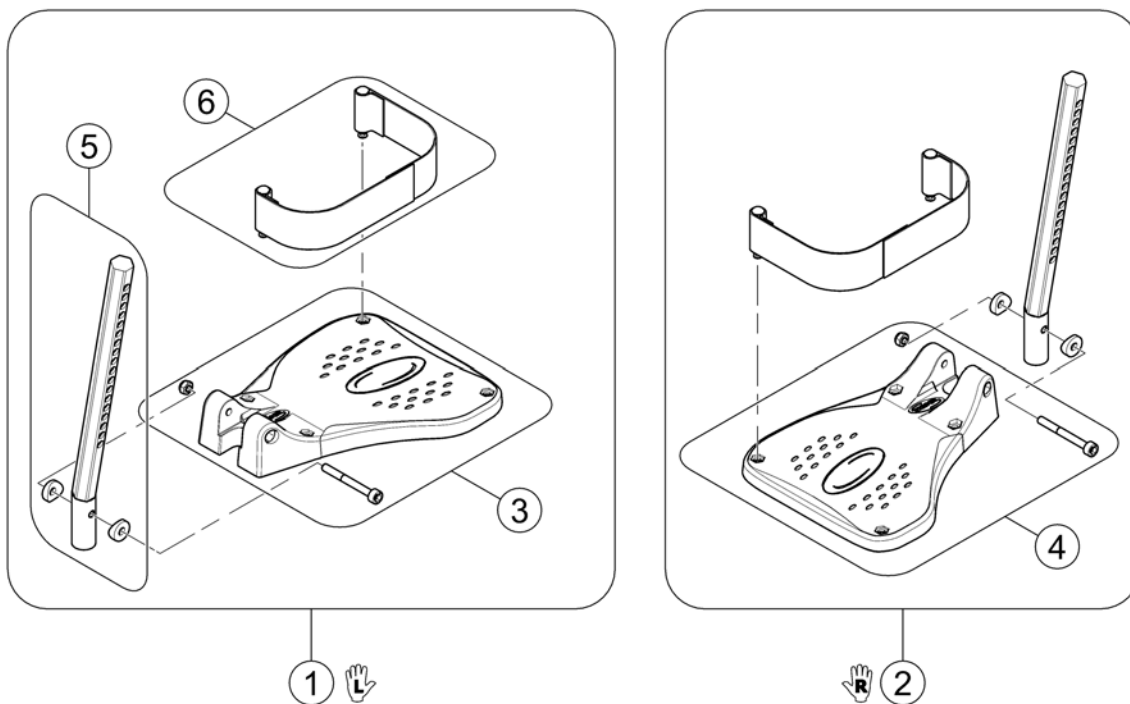

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541575	Footplate flip up, LH, cpl.		1
2	SP1541574	Footplate flip up, RH, cpl.		1
3	SP1529645	FOOTPLATE LH		1
4	SP1529644	FOOTPLATE RH		1
5	SP1543045	Plug-in tube LH/RH		1
6	SP1442467	Heel Strap cpl.		1




## Footrests Vari, Flip-Up & angle adjustable (01.08.10 =>)

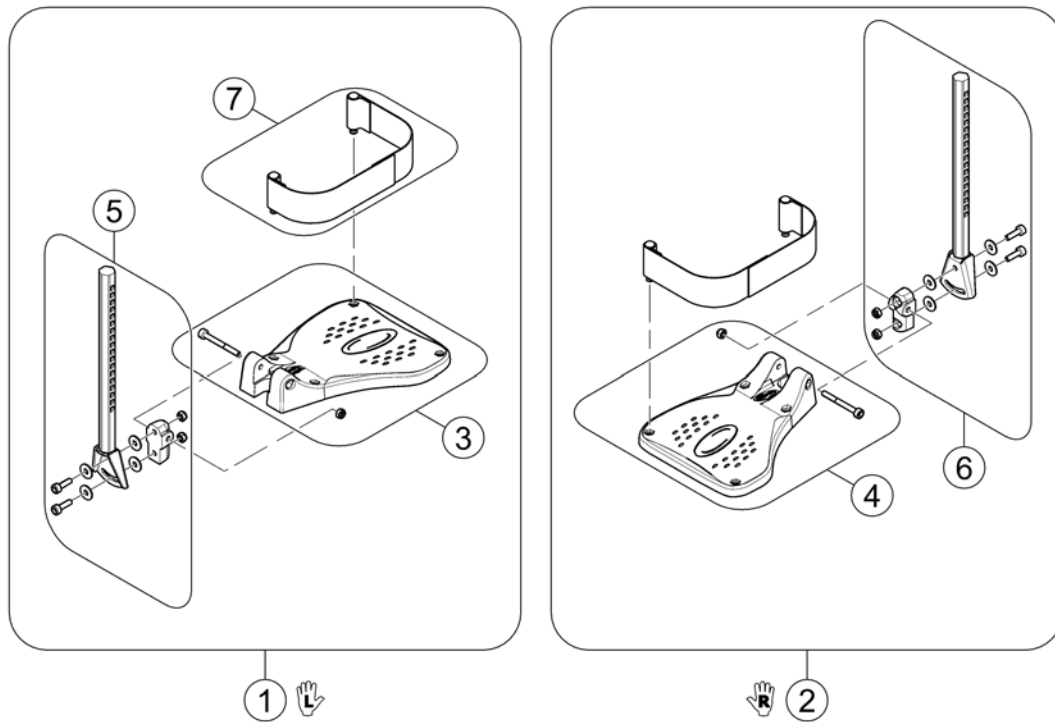

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541580	Footplate flip up, angle adjustable, LH, cpl.		1
2	SP1541579	Footplate flip up, angle adjustable, RH, cpl.		1
3	SP1529645	FOOTPLATE LH		1
4	SP1529644	FOOTPLATE RH		1
5	SP1423204	FOOT PLATE TUBE, ANGLE ADJUSTMENT LH CPL.		1
6	SP1423203	FOOT PLATE TUBE, ANGLE ADJUSTMENT RH CPL.		1
7	SP1442467	Heel Strap cpl.		1

## Fixed Armrest, removable

1421692X.dwg

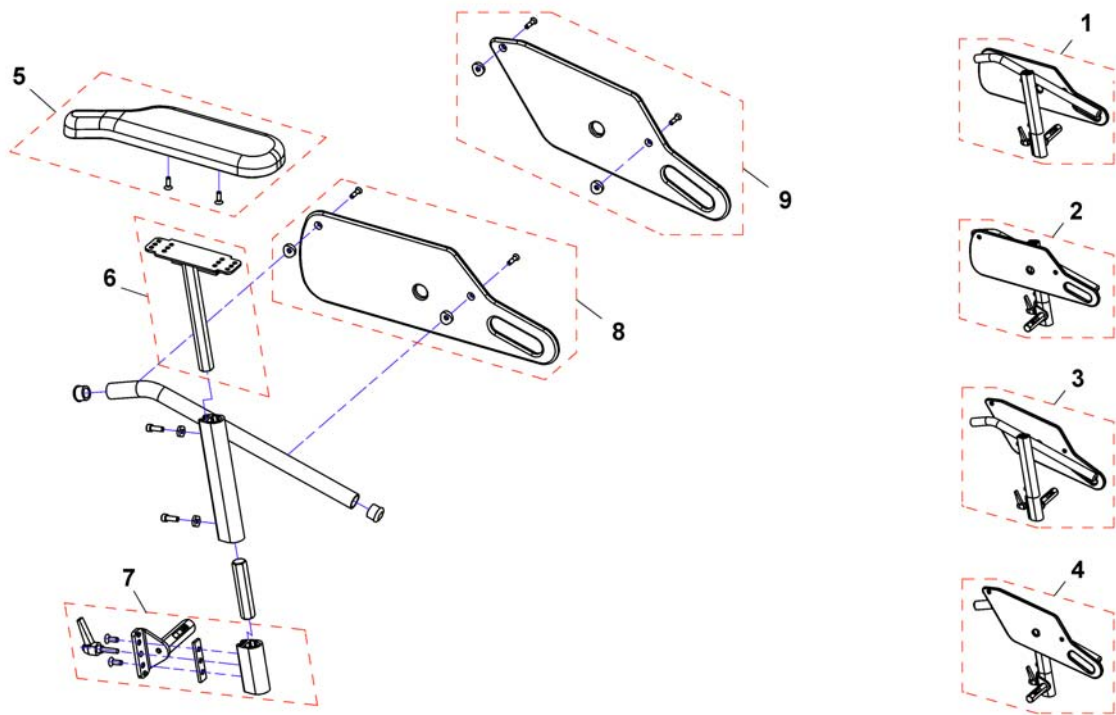

Pos.	Part number	Description	Valid from → until	Qty
1	SP1421692	Armrest, height max. 340mm, RH, Seatdepth 460mm & 510mm		1
1	SP1421693	Armrest, height max. 380mm, RH, Seatdepth 460mm & 510mm		1
2	SP1421694	Armrest, height max. 340mm, LH, Seatdepth 460mm & 510mm		1
2	SP1421695	Armrest, height max. 380mm, LH, Seatdepth 460mm & 510mm		1
3	SP1436533	Armrest, height max. 340mm, RH, Seatdepth 410mm		1
3	SP1436535	Armrest, height max. 380mm, RH, Seatdepth 410mm		1
4	SP1436534	Armrest, height max. 340mm, LH, Seatdepth 410mm		1
4	SP1436536	Armrest, height max. 380mm, LH, Seatdepth 410mm		1
5	SP1449043	Ampad, Standard (325mm), RH		1
5	SP1443004	Ampad, Standard (325mm), LH		1
5	SP1528845	Ampad, Extended (375mm), RH		1
5	SP1528846	Ampad, Extended (375mm), LH		1
6	SP1510856	Ampad holder RH <i>For chairs bevor 01/01/09 also order a new armpad.</i>		1
6	SP1510857	Ampad holder LH <i>For chairs bevor 01/01/09 also order a new armpad.</i>		1
7	SP1423223	Side part receptatcle RH, cpl.		1
7	SP1423225	Side part receptatcle LH, cpl.		1
8	SP1420471	Skirtguard, Seatdepth 460 & 510mm		1
9	SP1428078	Skirtguard, seat depth 410mm		1

# Armrests, cpl.

1421693X.dwg

Seat height 45cm/47cm Seatraiser - seatheight 47cm - 77cm					
Standard, Contour, Flex			01/01/09 ==>		
Armrest height 340mm		Armrest height 380mm			
		RH	LH	RH	LH
> 39cm	> 41cm	1	2	3	4
39cm	> 41cm	5	6	7	8
> 39cm	<= 41cm	9	10	11	12
39cm	<= 41cm	13	14	15	16
Recaro					
n.A.		n.A.		01/01/09 ==>	
electric			manuell		
3		4		17 18	
child seat					
RH		LH			
19		20			
armrest for Optimist Seat			armcushion for Optimist Seat		
cushion	RH	LH	cushion	RH	LH
30cm	21	22	30cm	27	28
35cm	23	24	35cm	29	30
40cm	25	26	40cm	31	32


Seatraiser 40cm - 70cm					
Standard, Contour, Flex			01/01/09 ==>		
Armrest height 340mm		Armrest height 380mm			
		RH	LH	RH	LH
> 39cm	> 41cm	33	34	35	36
39cm	> 41cm	37	38	39	40
> 39cm	<= 41cm	41	42	43	44
39cm	<= 41cm	45	46	47	48
Recaro					
n.A.		n.A.		01/01/09 ==>	
electric			manuell		
35		36		49 50	
child seat					
RH		LH			
51		52			
armrest for Optimist Seat			armcushion for Optimist Seat		
cushion	RH	LH	cushion	RH	LH
30cm	53	54	30cm	59	60
35cm	55	56	35cm	61	62
40cm	57	58	40cm	63	64

Pos.	Part number	Description	Valid from → until	Qty
1	SP1421692	Armrest, height max. 340mm, RH, Seatdepth 460mm & 510mm		1
2	SP1421694	Armrest, height max. 340mm, LH, Seatdepth 460mm & 510mm		1
3	SP1421693	Armrest, height max. 380mm, RH, Seatdepth 460mm & 510mm		1
4	SP1421695	Armrest, height max. 380mm, LH, Seatdepth 460mm & 510mm		1
5	SP1422542	Armrest, 340mm, SW 39cm, RH		1
6	SP1422543	Armrest, 340mm, SW 39cm, LH		1
7	SP1422548	Armrest, 380mm, SW 39cm, RH		1
8	SP1422549	Armrest, 380mm, SW 39cm, LH		1
9	SP1428096	Armrest, 340mm, SD 41cm, RH		1
10	SP1428097	Armrest, 340mm, SD 41cm, LH		1
11	SP1428098	Armrest, 380mm, SD 41cm, RH		1
12	SP1428099	Armrest, 380mm, SD 41cm, LH		1
13	SP1428100	Armrest, 340mm, SW 39cm, SD 41cm, RH		1
14	SP1428101	Armrest, 340mm, SW 39cm, SD 41cm, LH		1
15	SP1428102	Armrest, 380mm, SW 39cm, SD 41cm, RH		1
16	SP1428103	Armrest, 380mm, SW 39cm, SD 41cm, LH		1
17	SP1425845	Armrest cpl. 38cm RH Recaro man. Back		1
18	SP1425846	Armrest cpl. 38cm LH Recaro man. Back		1
19	5313557	Armrest Junior Seat RH		1
20	5313558	Armrest Junior Seat LH		1
21	1419657	Armrest Optimist 30cm RH		1
22	1419658	Armrest Optimist 30cm LH		1
23	1419659	Armrest Optimist 35cm RH		1
24	1419660	Armrest Optimist 35cm LH		1
25	1419661	Armrest Optimist 40cm RH		1
26	1419662	Armrest Optimist 40cm LH		1
27	1419647	Armrest Pad Optimist 30cm RH		1
28	1419648	Armrest Pad Optimist 30cm LH		1
29	5314011	Armrest Pad Optimist 35cm RH		1
30	5314010S	Armrest Pad Optimist 35cm LH		1
31	5314011L	Armrest Pad Optimist 40cm RH		1

**Armrests, cpl.**

Pos.	Part number	Description	Valid from → until	Qty
32	5314010	Armrest Pad Optimist 40cm LH		1
33	SP1436521	Armrest cpl. 34cm RH		1
34	SP1436522	Armrest cpl. 34cm LH		1
35	SP1436523	Armrest cpl. 38cm RH		1
36	SP1436524	Armrest cpl. 38cm LH		1
37	SP1436525	Armrest cpl. 34cm SW39cm RH		1
38	SP1436526	Armrest cpl. 34cm SW39cm LH		1
39	SP1436527	Armrest cpl. 38cm SW39cm RH		1
40	SP1436528	Armrest cpl. 38cm SW39cm LH		1
41	SP1436529	Armrest cpl. 34cm SD 41cm RH		1
42	SP1436530	Armrest cpl. 34cm SD 41cm LH		1
43	SP1436531	Armrest cpl. 38cm SD 41cm RH		1
44	SP1436532	Armrest cpl. 38cm SD 41cm LH		1
45	SP1436533	Armrest, height max. 340mm, RH, Seatdepth 410mm		1
46	SP1436534	Armrest, height max. 340mm, LH, Seatdepth 410mm		1
47	SP1436535	Armrest, height max. 380mm, RH, Seatdepth 410mm		1
48	SP1436536	Armrest, height max. 380mm, LH, Seatdepth 410mm		1
49	SP1436537	Armrest cpl. 38cm RH Recaro		1
50	SP1436538	Armrest cpl. 38cm LH Recaro		1
51	5313557	Armrest Junior Seat RH		1
52	5313558	Armrest Junior Seat LH		1
53	1419657	Armrest Optimist 30cm RH		1
55	1419659	Armrest Optimist 35cm RH		1
56	1419660	Armrest Optimist 35cm LH		1
57	1419661	Armrest Optimist 40cm RH		1
58	1419662	Armrest Optimist 40cm LH		1
59	1419647	Armrest Pad Optimist 30cm RH		1
60	1419648	Armrest Pad Optimist 30cm LH		1
61	5314011	Armrest Pad Optimist 35cm RH		1
62	5314010S	Armrest Pad Optimist 35cm LH		1
63	5314011L	Armrest Pad Optimist 40cm RH		1
64	5314010	Armrest Pad Optimist 40cm LH		1


## Sliding Armrest

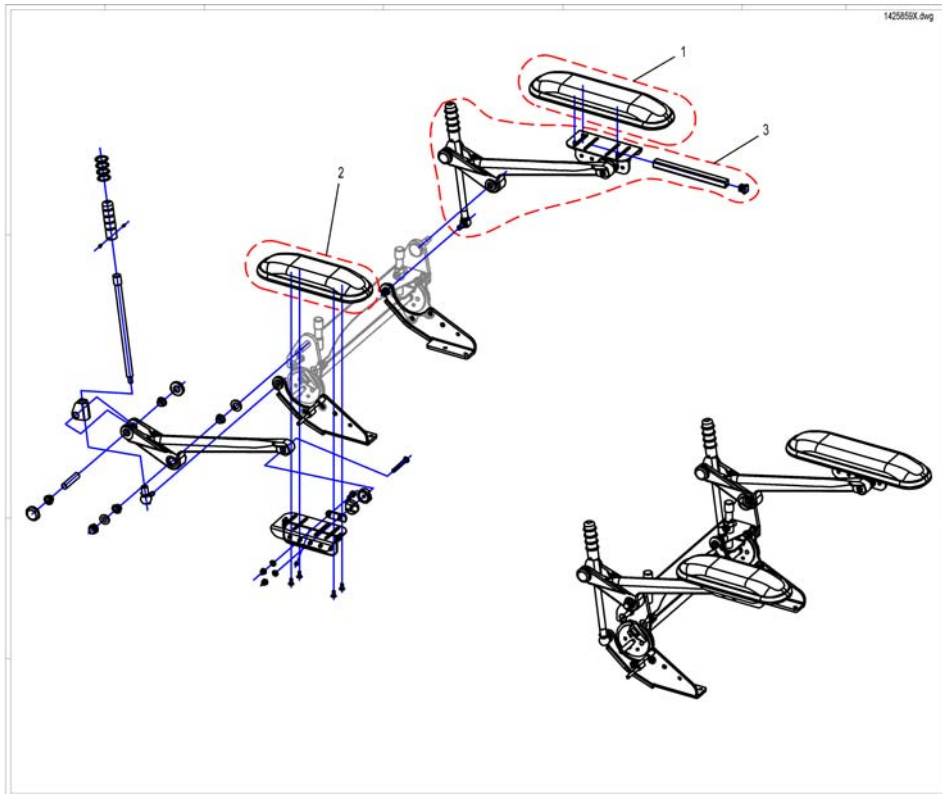

Pos.	Part number	Description	Valid from → until	Qty
1	SP1513484	Armrest long		1
2	SP1513485	Armrest short		1
3	SP1425844	ALU-Armrest		1

## Skirtguard for following armrest


1425860X.dwg

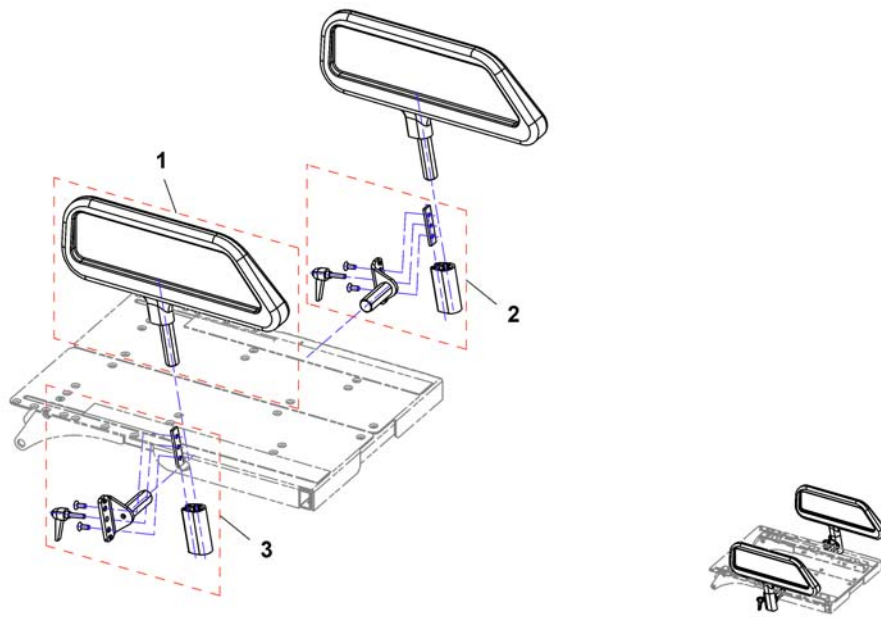

Pos.	Part number	Description	Valid from → until	Qty
1	SP1426303	Skirtguard		1
2	SP1442549	Bracket for Armrest / Skirtguard LH		1
3	SP1442550	Bracket for Armrest / Skirtguard RH		1

## Rear Wheels True Track


1433537.dwg

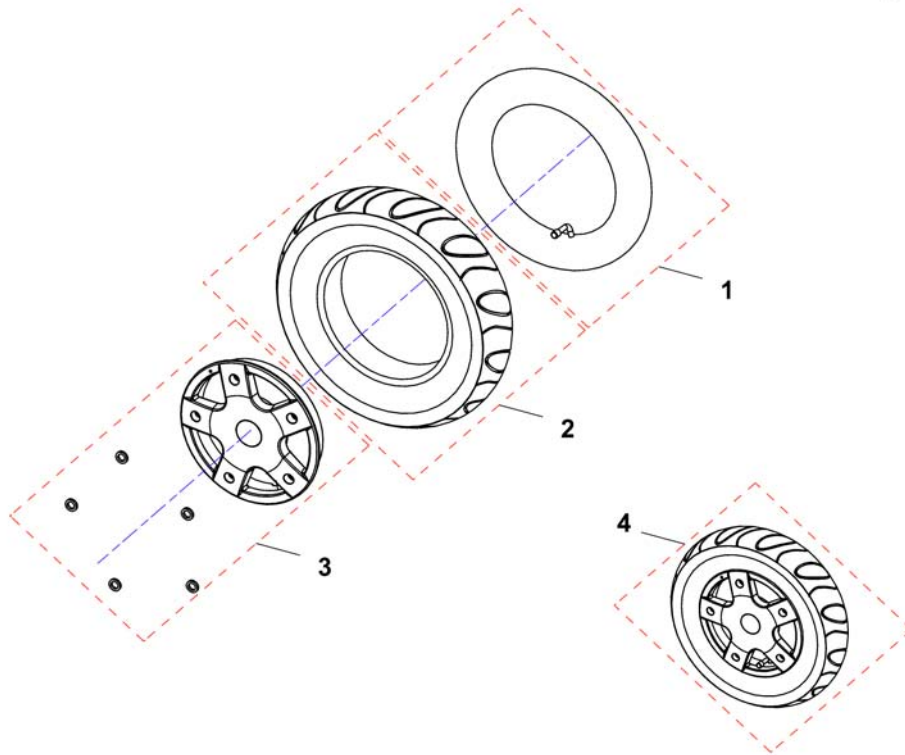

Pos.	Part number	Description	Valid from → until	Qty
1	M12016	Tube 3.00-8" with valve TR 87-90/90°		1
1	1484644	Tube 3.00-8" with valve TR 87-90/90°, puncture protect		1
2	1415173	Tyre 3.00-8", grey		1
2	1430215	Tyre 3.00-8", grey, puncture proof		1
2	1419257	Tyre 3.00-8", black		1
2	1430217	Tyre 3.00-8", black, puncture-proof		1
-	M11968	Tyre 3.00-8", grey, Trelleborg		1
3	SP1429347	Rim, cpl.		1
4	SP1433537	Drive wheel, pneumatic, grey, cpl.		1
4	SP1433541	Drive wheel, puncture protected, grey, cpl.		1
4	SP1445027	Drive wheel, puncture proof, grey, cpl.		1
4	SP1433538	Drive wheel, pneumatic, black, cpl.		1
4	SP1433542	Drive wheel, puncture protected, black, cpl.		1
4	SP1446516	Drive wheel, puncture proof, black, cpl.		1
4	SP1513430	Drive wheel, pneumatic, grey, Heavy Duty, cpl.		1
-	SP1433543	Drive wheel (True Track), Trelleborg, pneumatic, cpl.		1
-	SP1433544	Drive wheel (True Track), Trelleborg, puncture protected, cpl.		1

## Powered declutching

1440344X.dwg

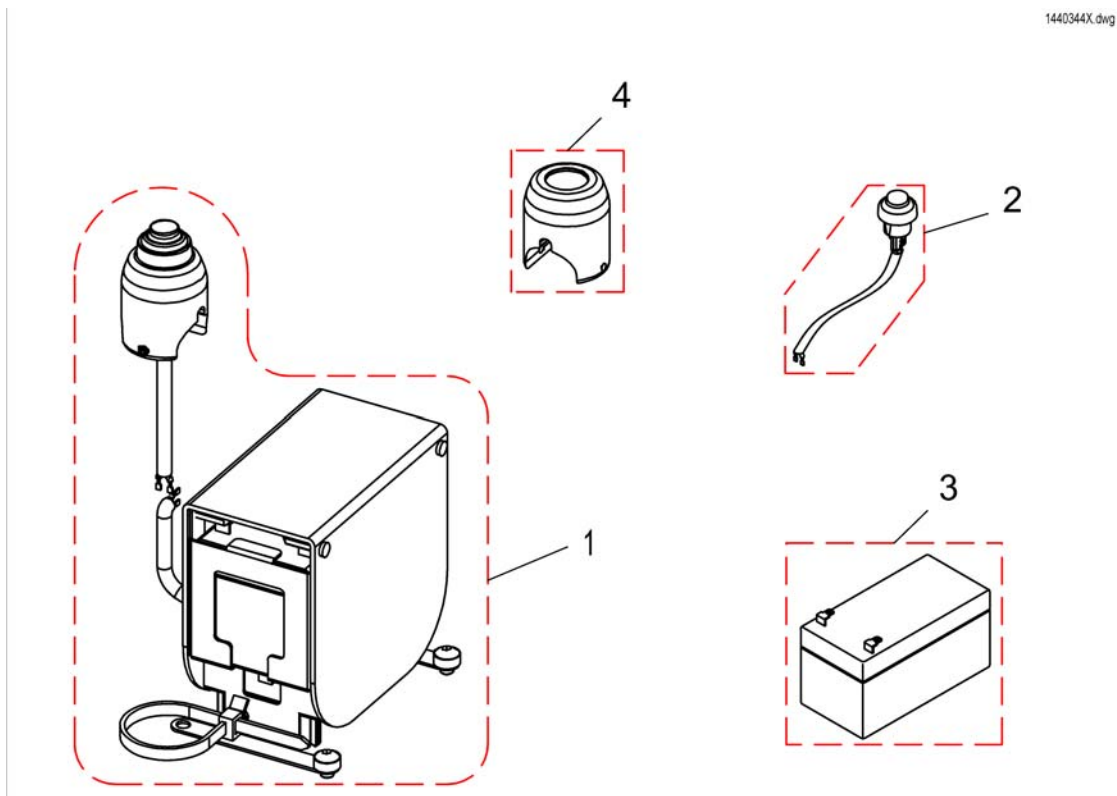


Pos.	Part number	Description	Valid from → until	Qty
1	SP1440384	electr. Declutching		1
2	SP1440729	pushbutton with cable		1
3	SP1440386	Batterie		1
4	1440595-0004	Switch Box		1



# GB-motor with manual declutching




1505734X.dwg

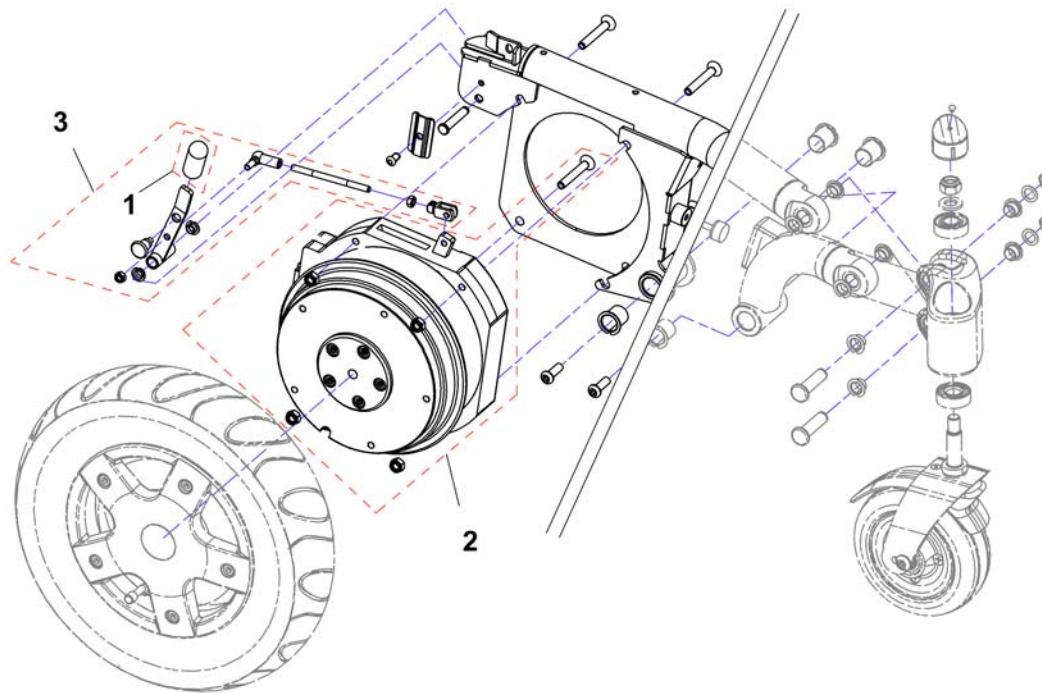

Pos.	Part number	Description	Valid from → until	Qty
-	SP1134214	Gasket Kit GM Motorplugs		1
1	SP5007791	Declutching Grip		1
2	SP1433563	Motor Gearless Brushless (True Track) LH		1
2	SP1433564	Motor Gearless Brushless (True Track) RH		1
3	SP1505176	Declutching cpl.		1


# Electronic

1500784.indd

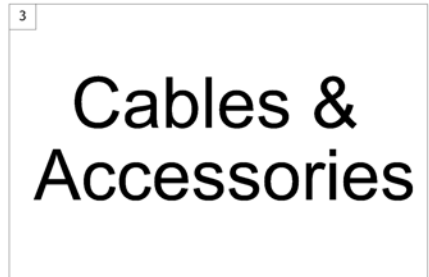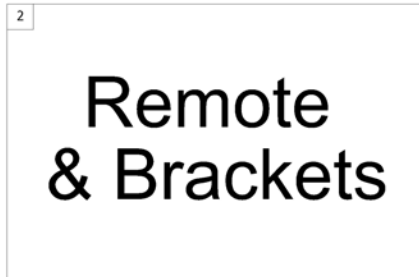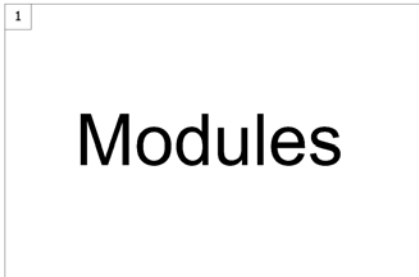


Pos.	Part number	Description	Valid from → until	Qty
1	-	ACS Electronic Modules		1
2	-	ACS Remote & Brackets		1
3	-	ACS Cables & Accessories		1

## ACS Electronic Modules

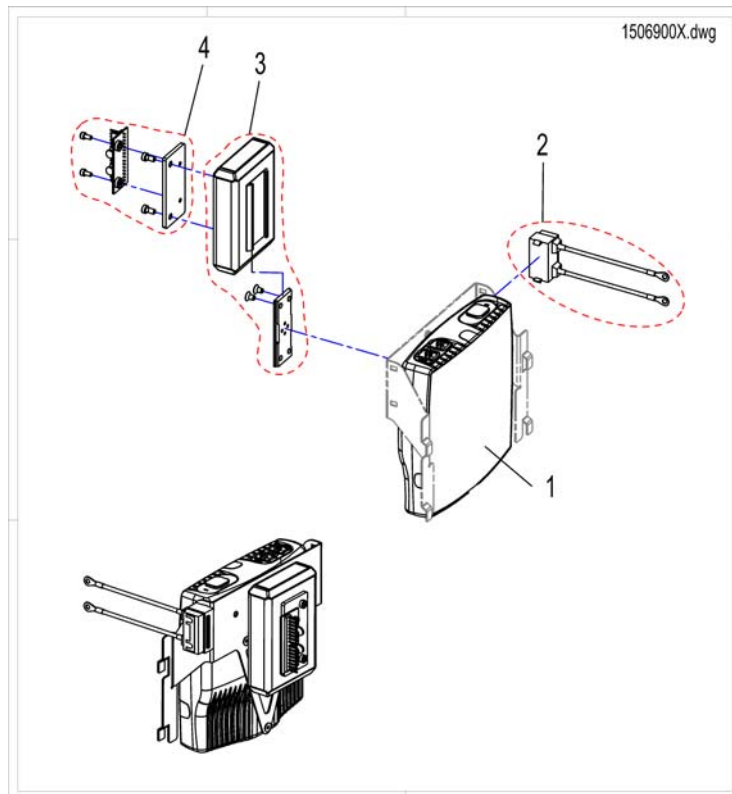


Pos.	Part number	Description	Valid from → until	Qty
1	SP1440306	Electronicholder with Power Modul GBM		1
2	SP1436212	Fuse Holder		1
3	SP1440307	CLAM		1
4	SP1498327	Light Board cpl.		1

## ACS Remote & Brackets

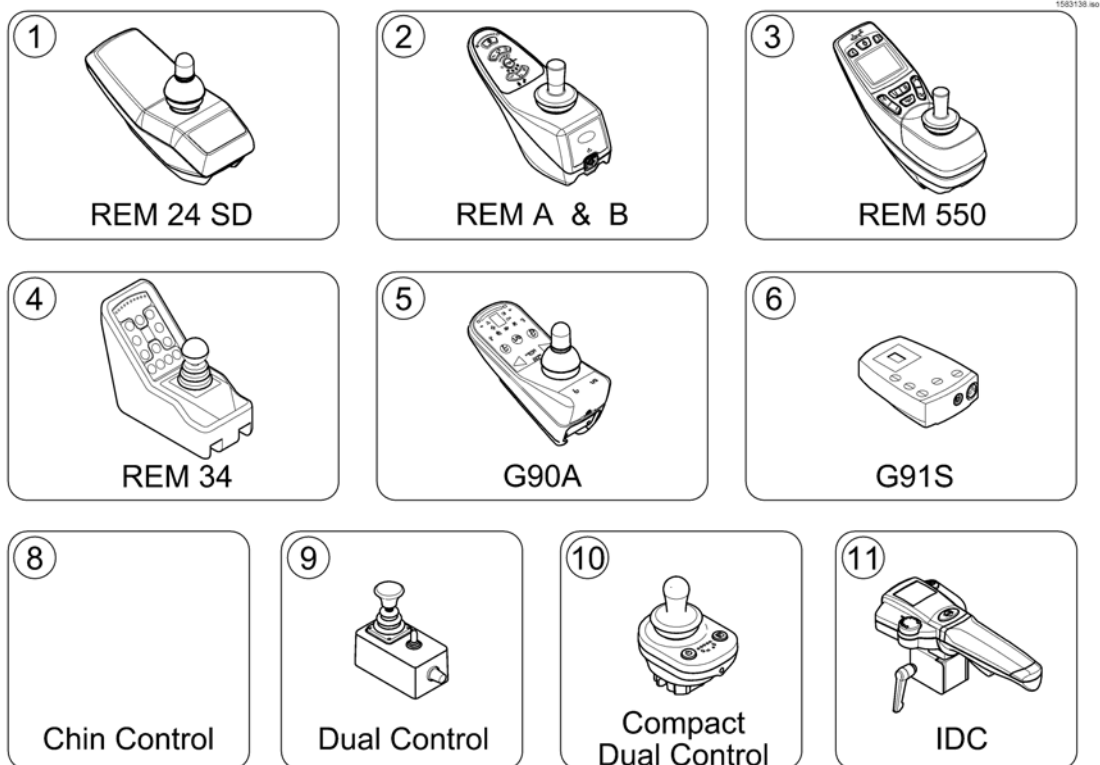

Pos.	Part number	Description	Valid from → until	Qty
1	-	REM 24 SD with brackets <i>[1601/1602]</i>		1
2	-	Remote REM A & B with Brackets <i>[1638/1639/1643/1644]</i>		1
3	-	Remote REM 550 with Brackets <i>[1617/1618]</i>		1
4	-	REM 34 with brackets <i>[1611]</i>		1
5	-	G90A with brackets <i>[1614/1616]</i>		1
6	-	G91S with brackets <i>[1615]</i>		1
7	-	Chin Control with brackets <i>[1609/1610]</i>		1
8	-	Dual Control with brackets <i>[1291/1292]</i>		1
9	-	Intuitive Dual Control with brackets <i>[1645]</i>		1


## REM 24 SD with brackets

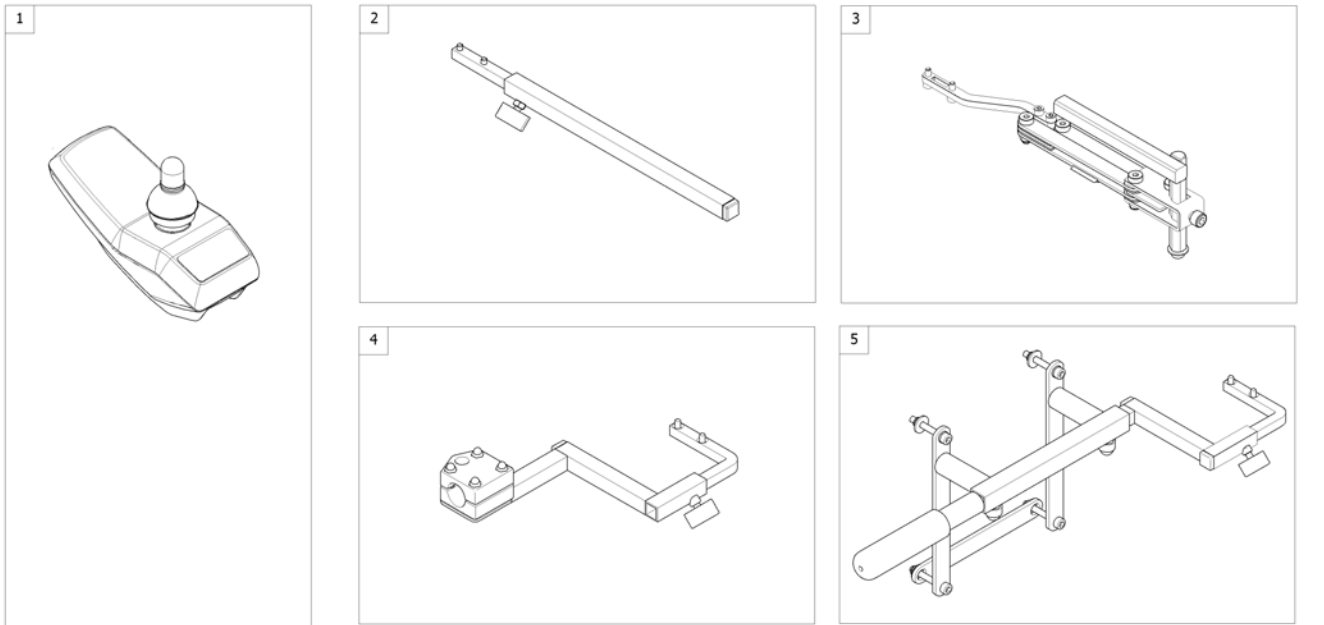

Pos.	Part number	Description	Valid from → until	Qty
1	-	Remote REM 24SD <i>[1601/1602]</i>		0
2	-	Remote Bracket Standard <i>[1601+1207/1208]</i>		1
3	-	Remote Bracket Swing away & height adjust. <i>[1601+1707/1708]</i>	2008-07-01 →	1
4	-	Remote Bracket Attendant for Standard-/Contour-/Flex-Seat <i>[1602+1207/1208]</i>		1
5	-	Remote Bracket Attendant for Recaro-Seat <i>[1602+1207/1208]</i>		1

# Remote REM 24SD


142202 860

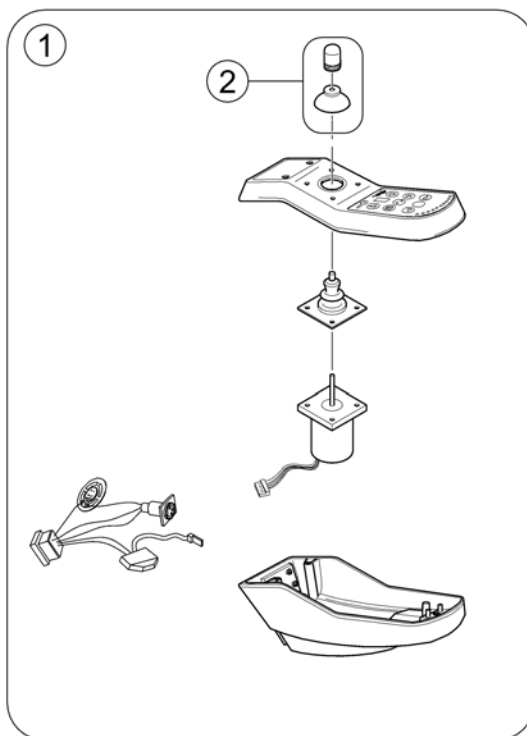

Pos.	Part number	Description	Valid from → until	Qty
1	1436541	Remote REM24SD		1
2	SPZ183229	Remote knob cpl.		1

# Remote Bracket Standard


1559851.iso

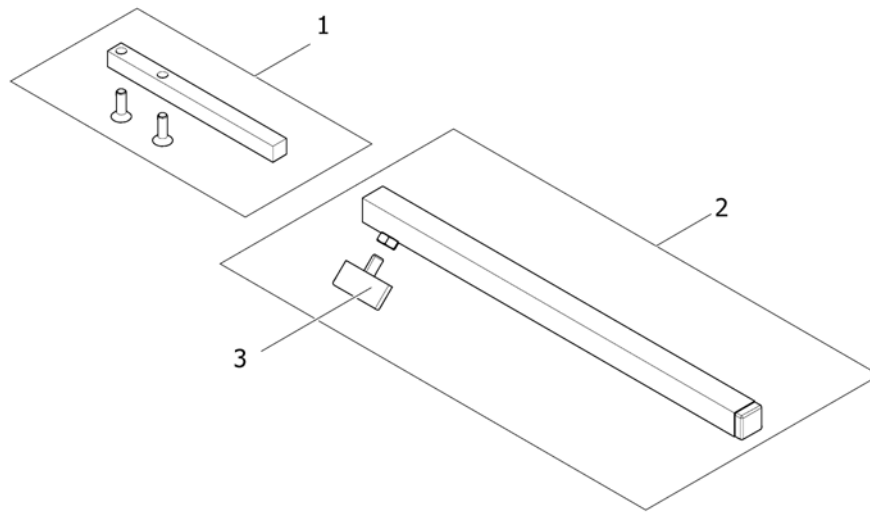

Pos.	Part number	Description	Valid from → until	Qty
1	SP1436542	Support, Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing Scew M6X10		1

## Remote Bracket Swing away & height adjust.

1559852.iso

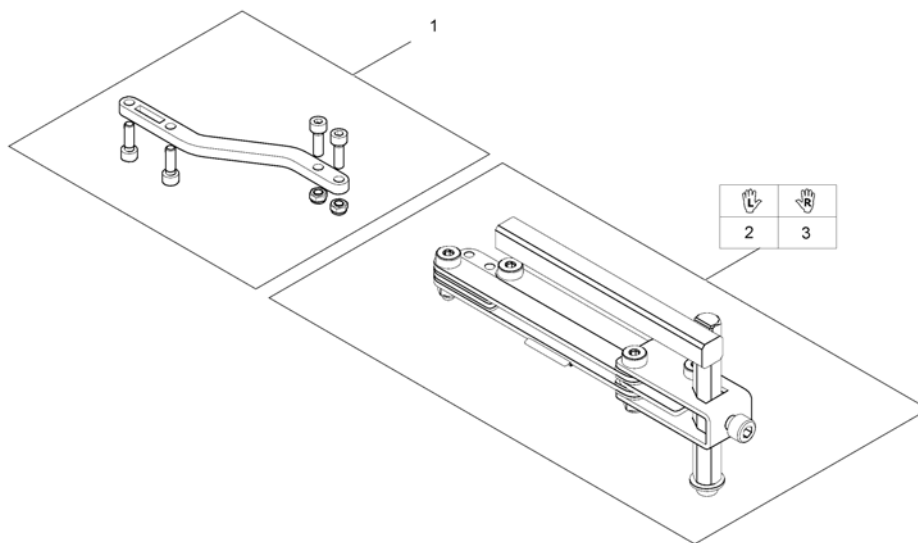


Pos.	Part number	Description	Valid from → until	Qty
1	SP1498222	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1



# Remote Bracket Attendant for Standard-/Contour-/Flex-Seat


1560837.00

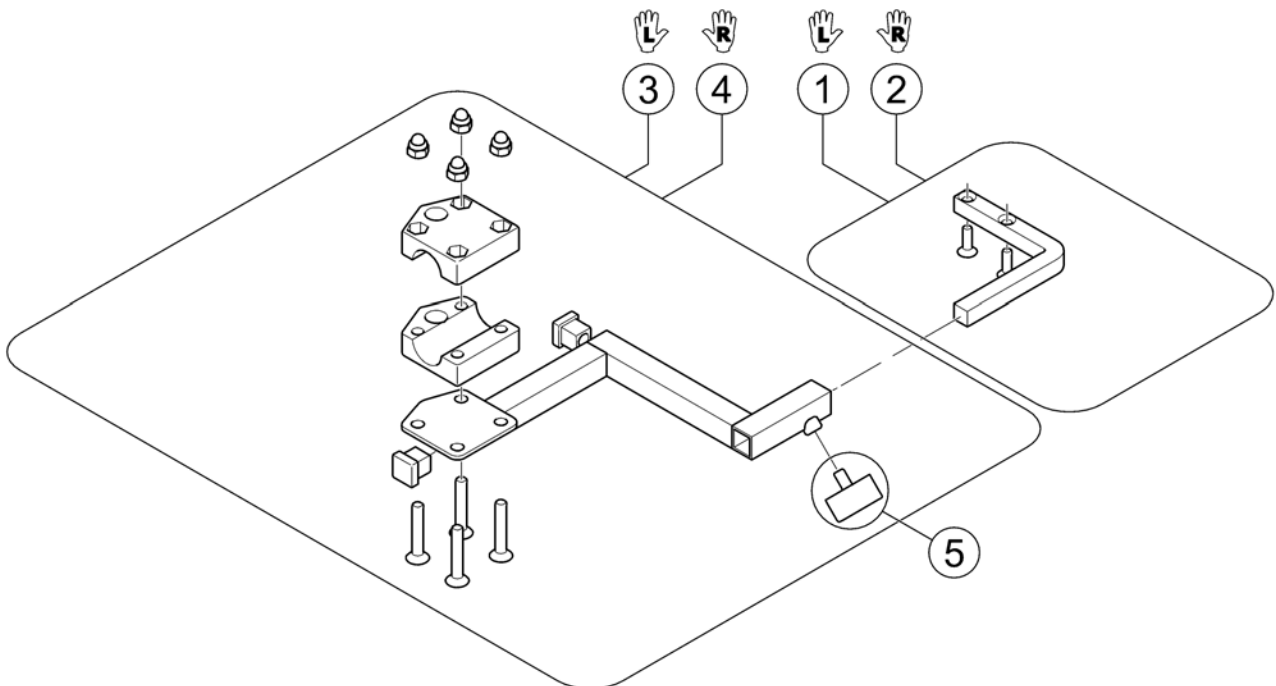

Pos.	Part number	Description	Valid from → until	Qty
1	1422806	Remote Support LH		1
2	1422810	Remote Support RH		1
3	SP1422937	Bracket for attendant control LH		1
4	SP1422936	Bracket for attendant control RH		1
5	L31001	Wing ScREW M6X10		1


# Remote Bracket Attendant for Recaro-Seat

1563480.iso

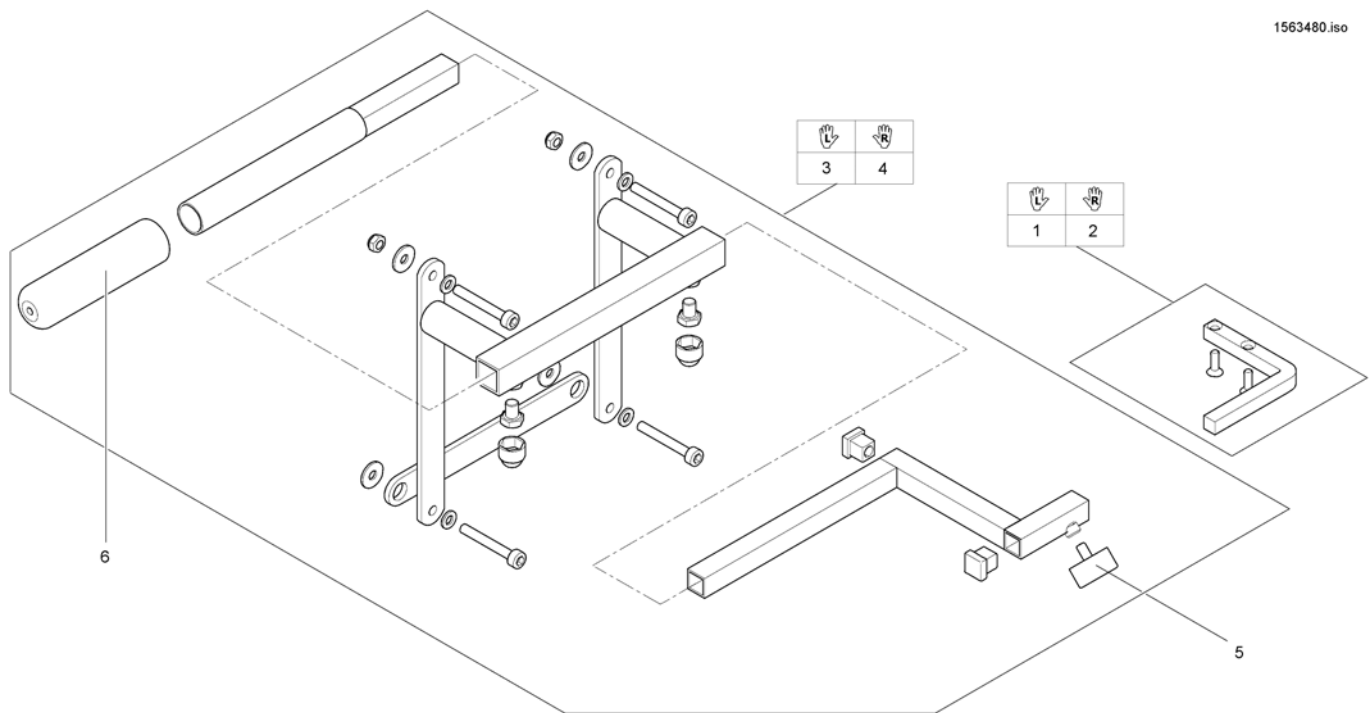

Pos.	Part number	Description	Valid from → until	Qty
1	1422806	Remote Support LH		1
2	1422810	Remote Support RH		1
3	SP1422814	Assebmly kit for dual control cpl., Recaro, LH		1
4	SP1422813	Assebmly kit for dual control cpl., Recaro, RH		1
5	L31001	Wing Scew M6X10		1
6	1435421	Handle plastic black Ø22 "Invacare"		1


1503649 no

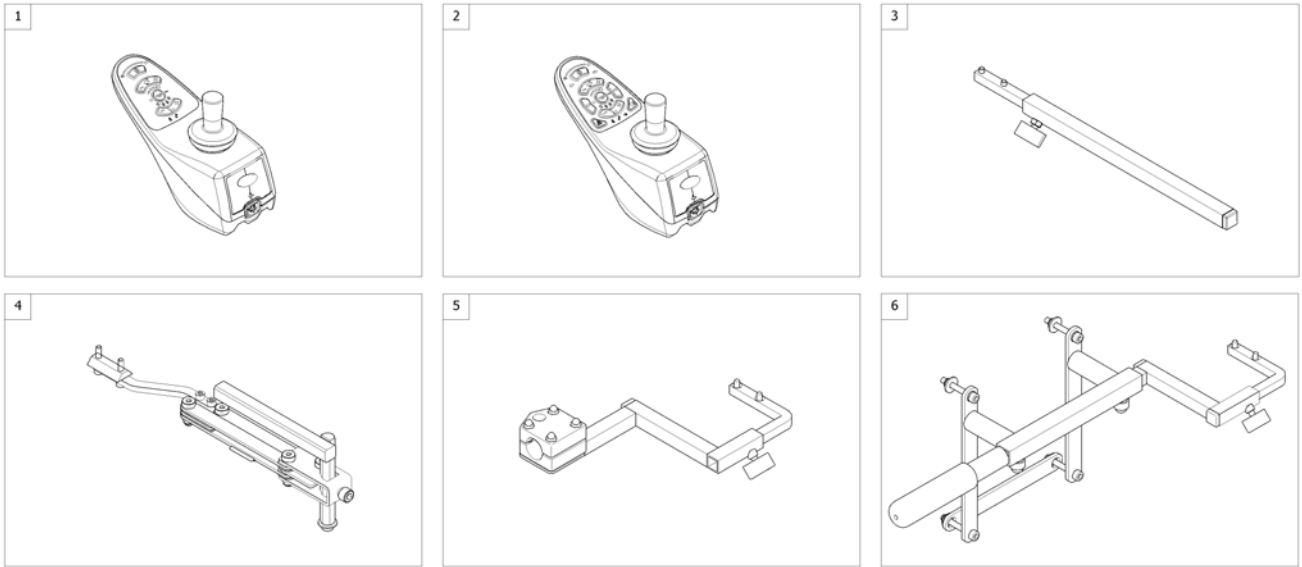


Pos.	Part number	Description	Valid from → until	Qty
1	SP1523440	Remote "REM A" <i>[1638/1643]</i>		1
2	SP1523445	Remote "REM B" <i>[1639/1644]</i>		1
3	-	Remote Bracket Standard <i>[1638/1639+1207/1208]</i>		1
4	-	Remote Bracket Swing away & height adjust. <i>[1638/1639+1707/1708]</i>		1
5	-	Remote Bracket Attendant for Standard-/Contour-/Flex-Seat <i>[1643/1644+1207/1208]</i>		1
6	-	Remote Bracket Attendant for Recaro-Seat <i>[1643/1644+1207+1208]</i>		1

## Remote Bracket Swing away & height adjust.

1559857.iso

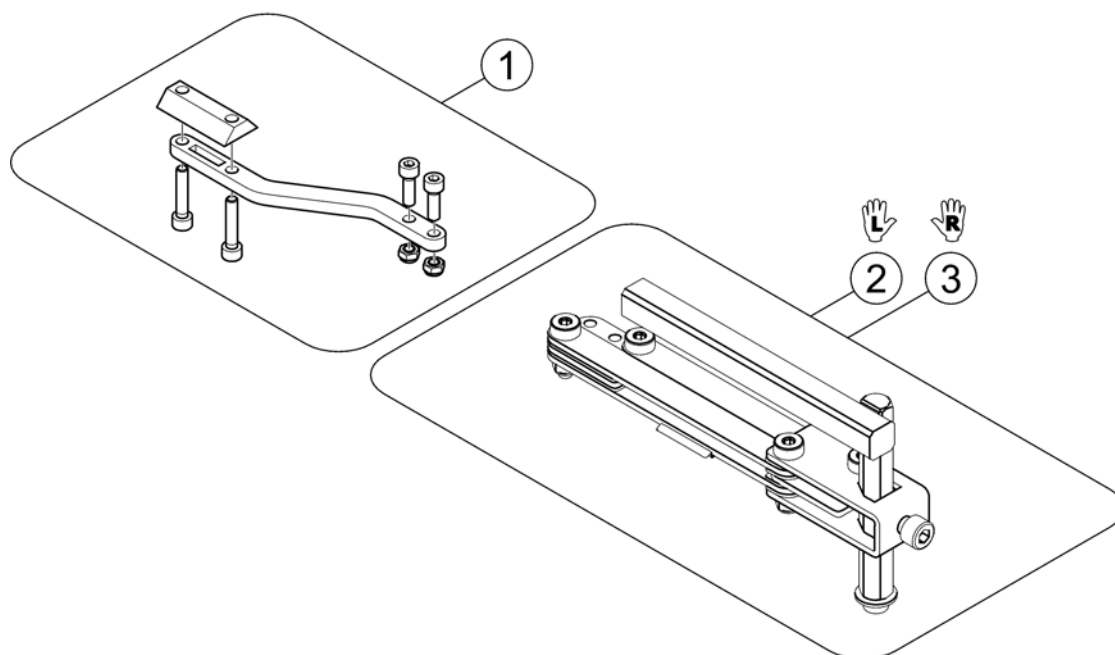



Pos.	Part number	Description	Valid from → until	Qty
1	SP1536180	Support for Remote Bracket, height adjustable & swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

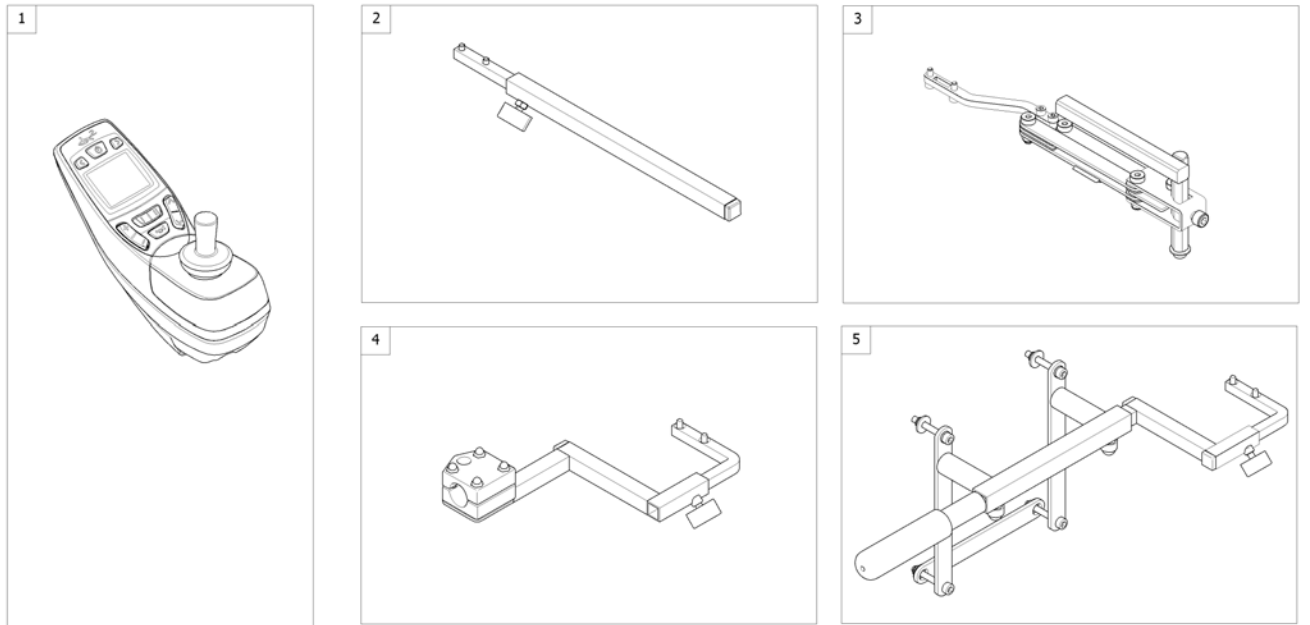

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523444	Remote REM550 <i>[1617/1618]</i>		1
2	-	Remote Bracket Standard <i>[1617+1207/1208]</i>		1
3	-	Remote Bracket Swing away & height adjust. <i>[1617+1707/1708]</i>		1
4	-	Remote Bracket Attendant for Standard-/Contour-/Flex-Seat <i>[1618/1207/1208]</i>		1
5	-	Remote Bracket Attendant for Recaro-Seat <i>[1618+1207/1208]</i>		1

## REM 34 with brackets


1560846 iso

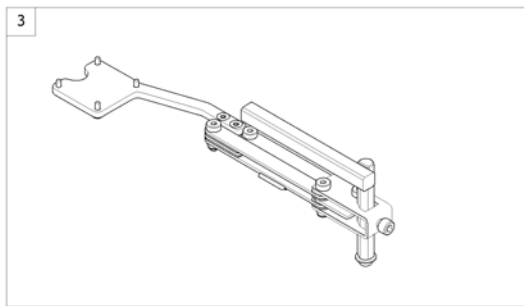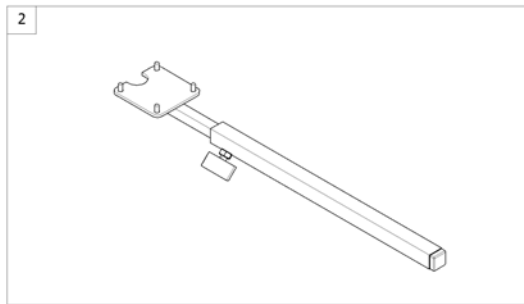

Pos.	Part number	Description	Valid from → until	Qty
1	SP1440677	Remote REM 34 [1611]		1
2	-	Remote Bracket Standard [1611+1207/1208]		1
3	-	Remote Bracket Swing away & height adjust. [1611+1707/1708]		1

# Remote Bracket Standard


1560844.iso

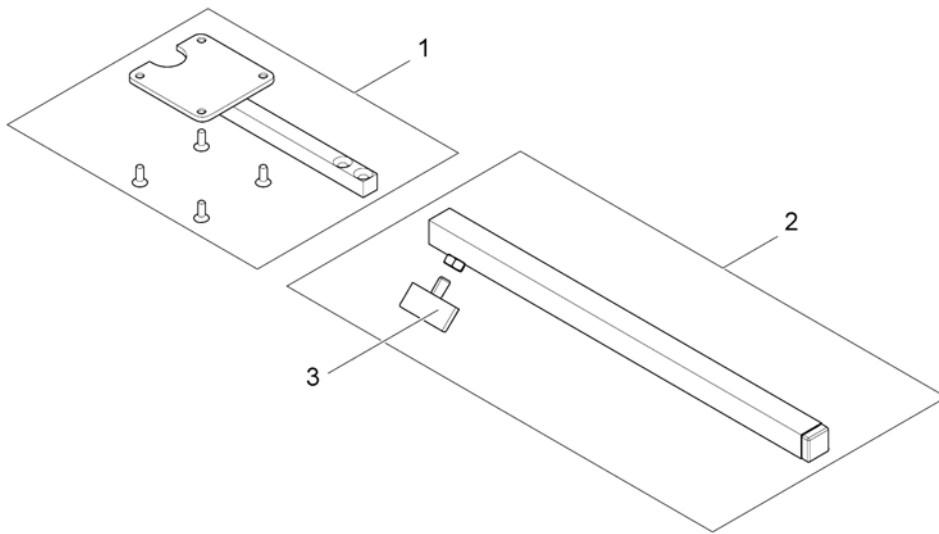

Pos.	Part number	Description	Valid from → until	Qty
1	SP1434921	Remote Support Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing ScREW M6X10		1

# Remote Bracket Swing away & height adjust.


1560845.iso

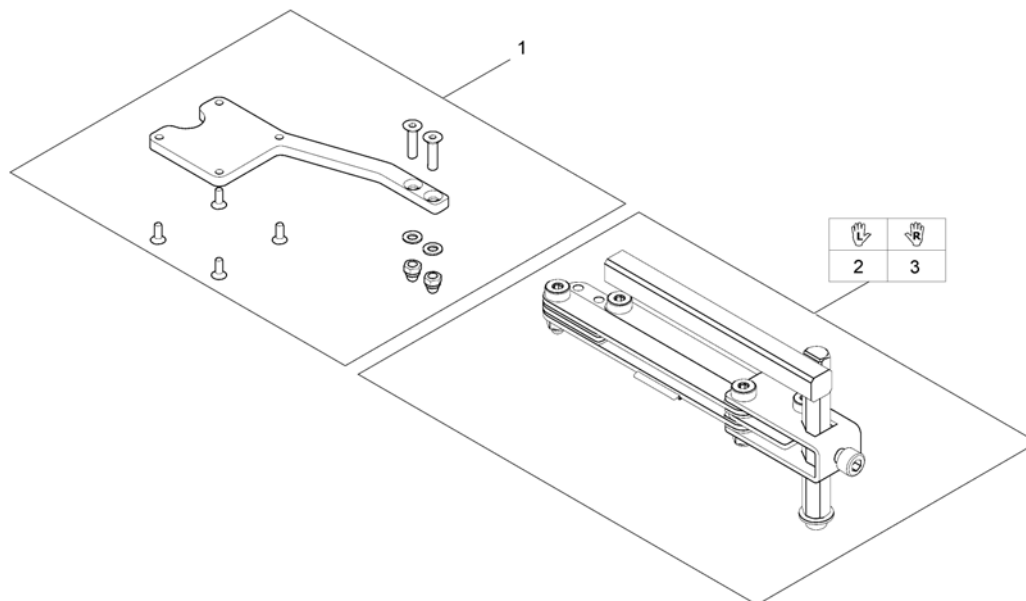

Pos.	Part number	Description	Valid from → until	Qty
1	SP1527125	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1




## G90A with brackets

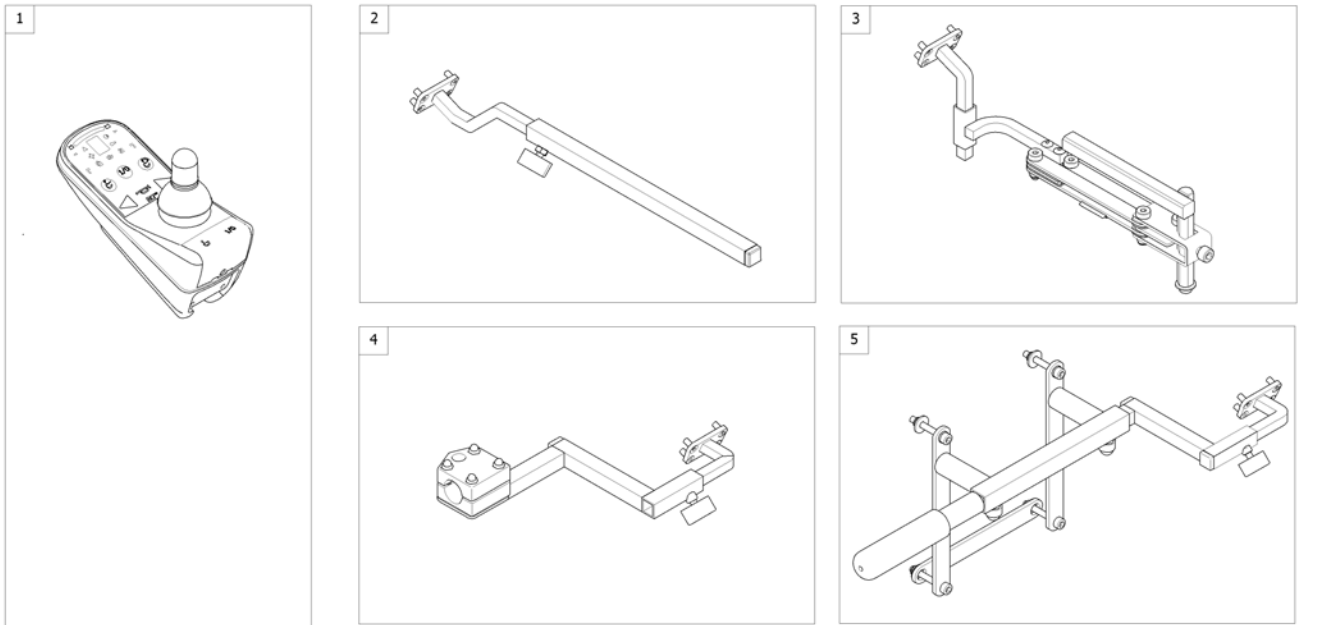

Pos.	Part number	Description	Valid from → until	Qty
1	SP1504720	Remote G90A <i>[1614/1616]</i>		1
2	-	Remote Bracket Standard <i>[1614+1207/1208]</i>		1
3	-	Remote Bracket Swing away & height adjust. <i>[1614+1707/1708]</i>		1
4	-	Remote Bracket Attendant for Standard-/Contour-/Flex-Seat <i>[1616+1207/1208]</i>		1
5	-	Remote Bracket Attendant for Recaro-Seat <i>[1616+1207/1208]</i>		1

# Remote Bracket Standard


1562939.iso

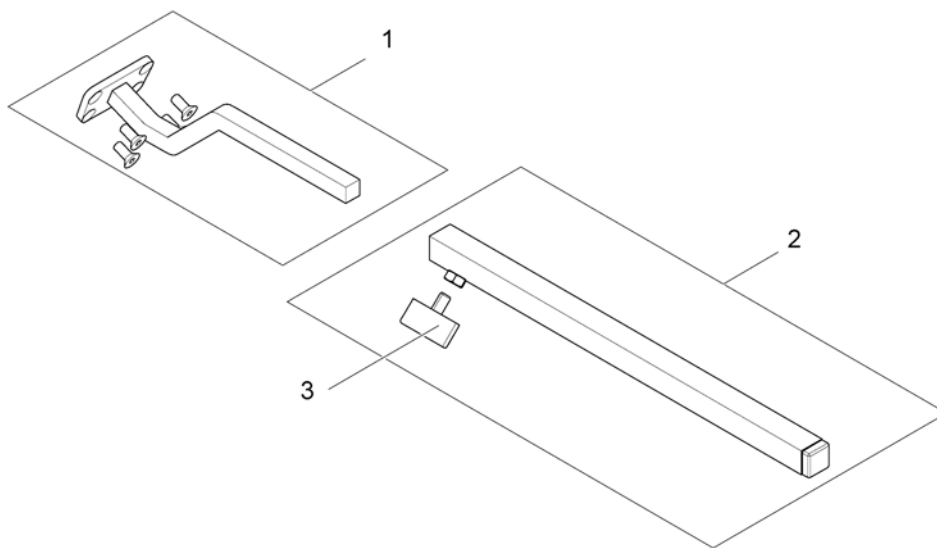

Pos.	Part number	Description	Valid from → until	Qty
1	SP1504822	Remote Support Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing ScREW M6X10		1

# Remote Bracket Swing away & height adjust.


1560310.iso

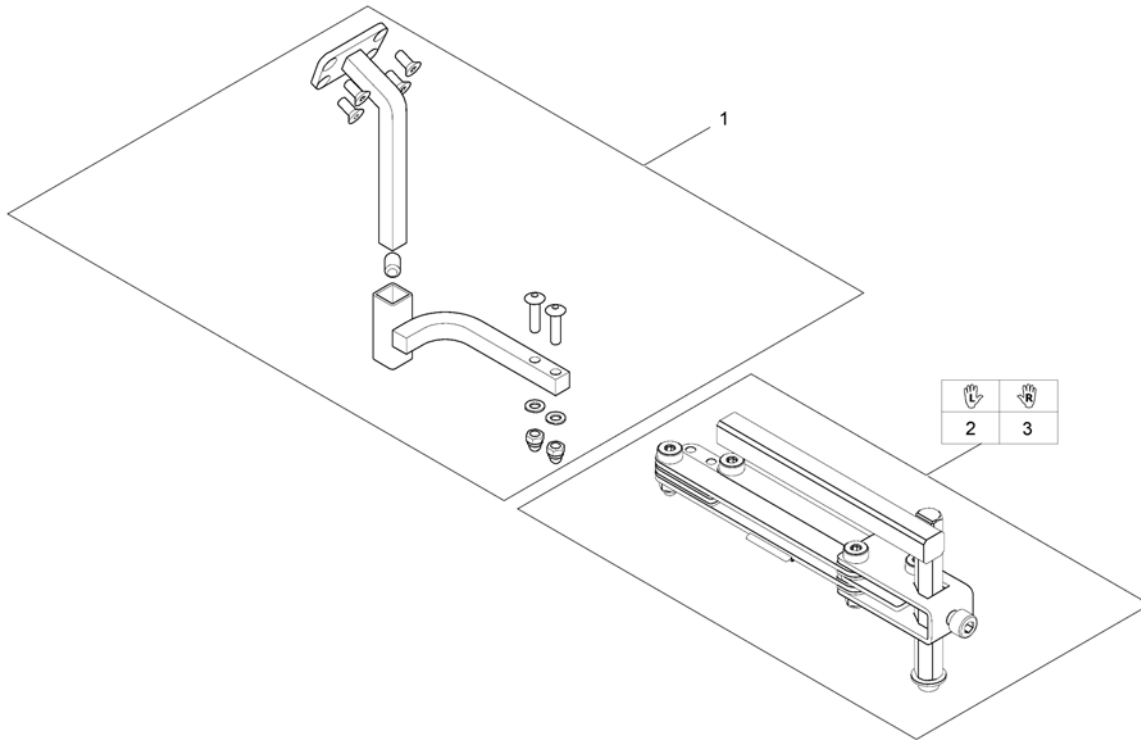

Pos.	Part number	Description	Valid from → until	Qty
1	SP1517352	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

# Remote Bracket Attendant for Standard-/Contour-/Flex-Seat


1560853.iso

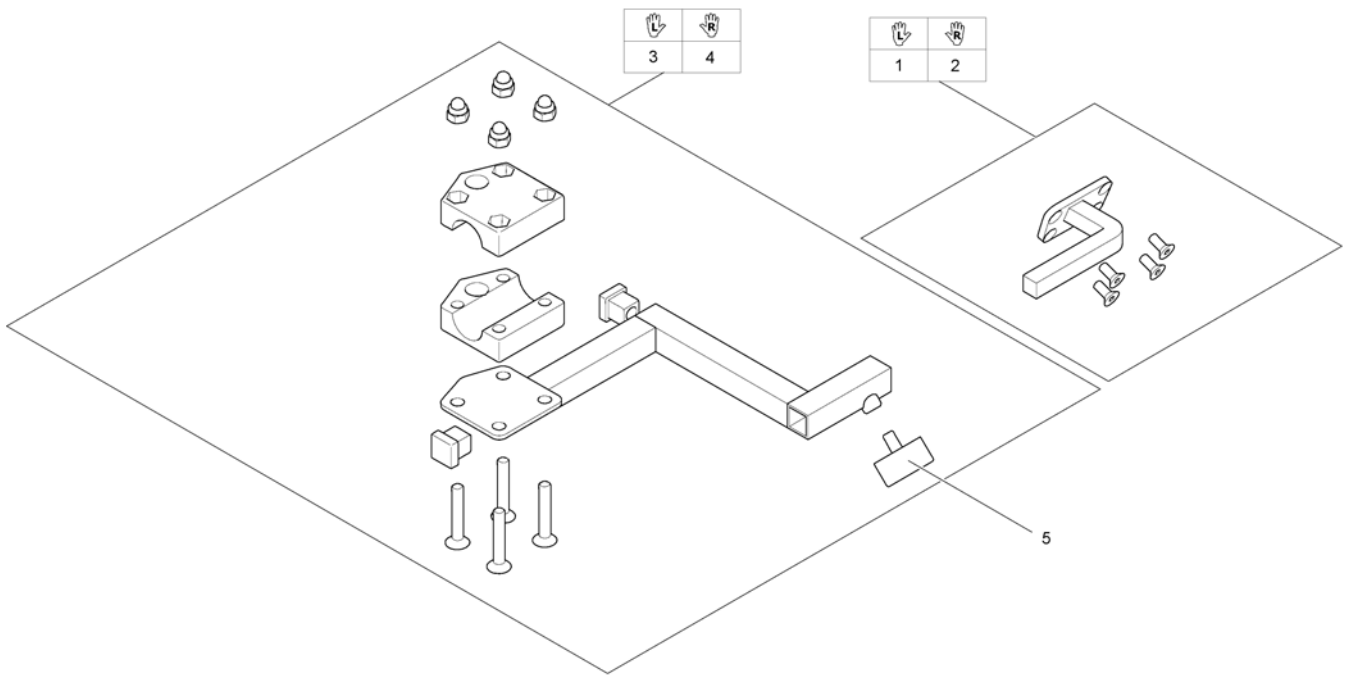

Pos.	Part number	Description	Valid from → until	Qty
1	SP1509435	Remote Support attendant control G90A, LH		1
2	SP1509439	Remote Support attendant control G90A, RH		1
3	SP1422937	Bracket for attendant control LH		1
4	SP1422936	Bracket for attendant control RH		1
5	L31001	Wing Screw M6X10		1

# Remote Bracket Attendant for Recaro-Seat



1563503.iso

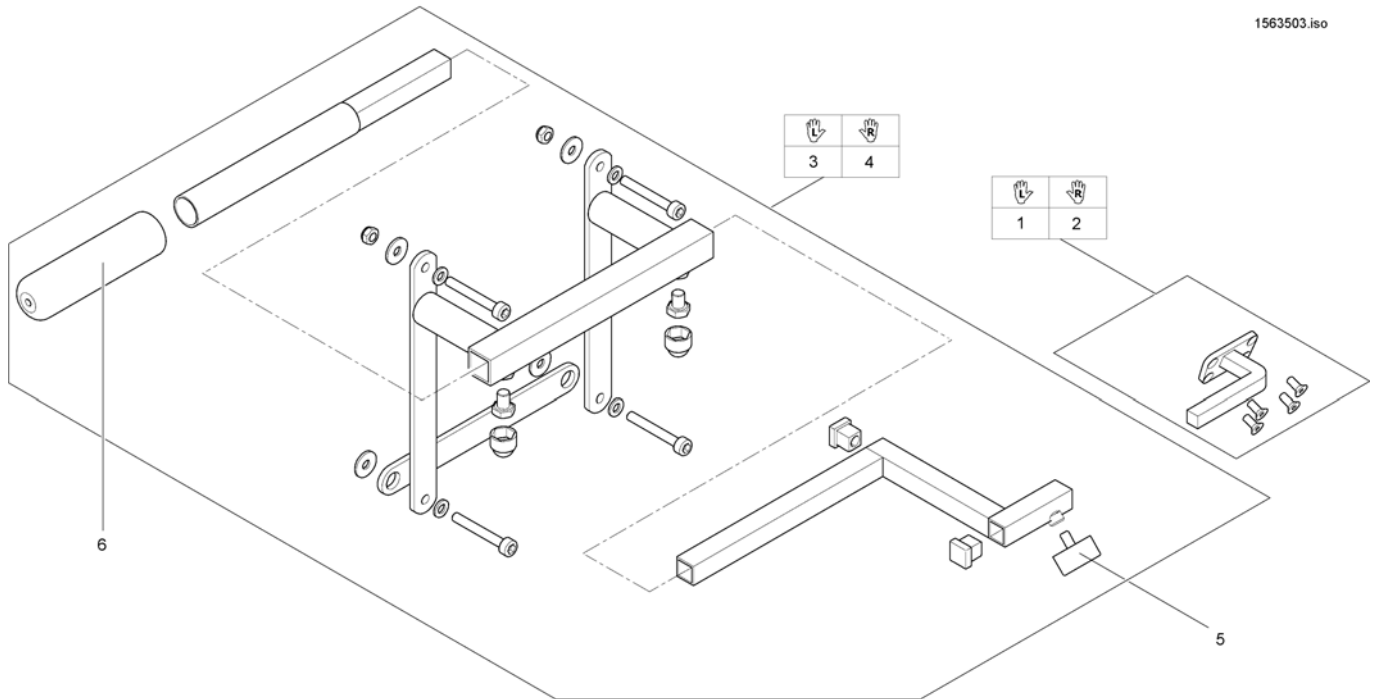

Pos.	Part number	Description	Valid from → until	Qty
1	SP1509435	Remote Support attendant control G90A, LH		1
2	SP1509439	Remote Support attendant control G90A, RH		1
3	SP1422814	Assebmly kit for dual control cpl., Recaro, LH		1
4	SP1422813	Assebmly kit for dual control cpl., Recaro, RH		1
5	L31001	Wing Scew M6X10		1
6	1435421	Handle plastic black Ø22 "Invacare"		1

## G91S with brackets


1560850 iso

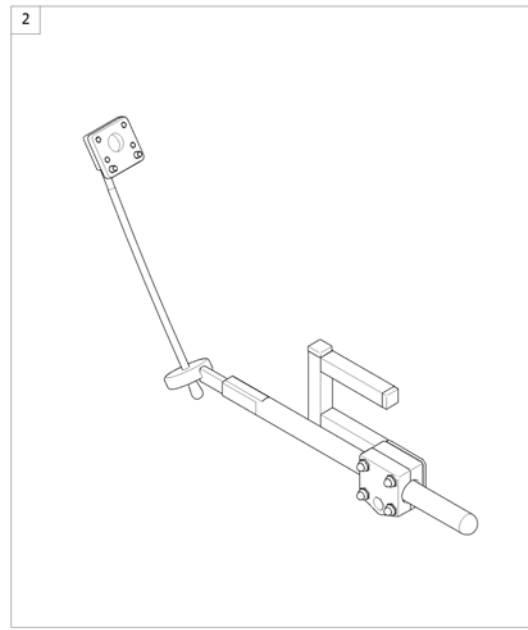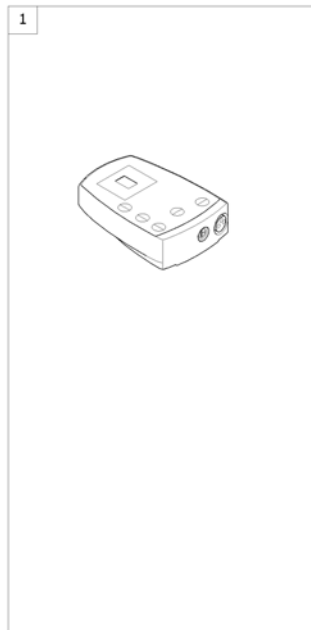

Pos.	Part number	Description	Valid from → until	Qty
1	SP1505187	Remote Control G91S [1615]		1
2	-	Remote Bracket Standard [1615+1207/1208]		1

# Remote Bracket Standard

1560654.iso

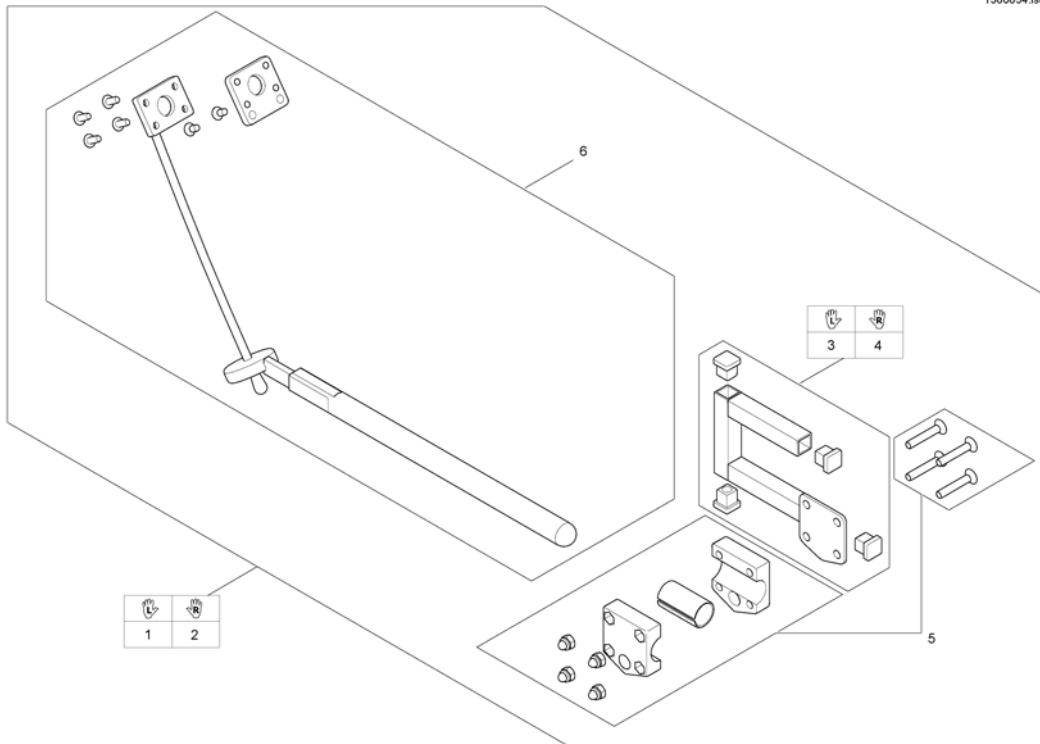


Pos.	Part number	Description	Valid from → until	Qty
1	SP1509718	Remote Bracket G91/G91S LH, cpl		1
2	SP1509690	Remote Bracket G91/G91S RH, cpl.		1
3	SP1509716	Adapter G91 LH		1
4	SP1509711	Adapter G91 RH		1
5	SP1527503	Gripper clamp cpl.		1
6	SP1527504	Swan neck remote bracket with adapter plate		1

## Chin Control with brackets

Pos.	Part number	Description	Valid from → until	Qty
-		Small Chincontrol [1609]		1
-		Framework for chincontrol [1609+1288/1289]		1




# Small Chincontrol

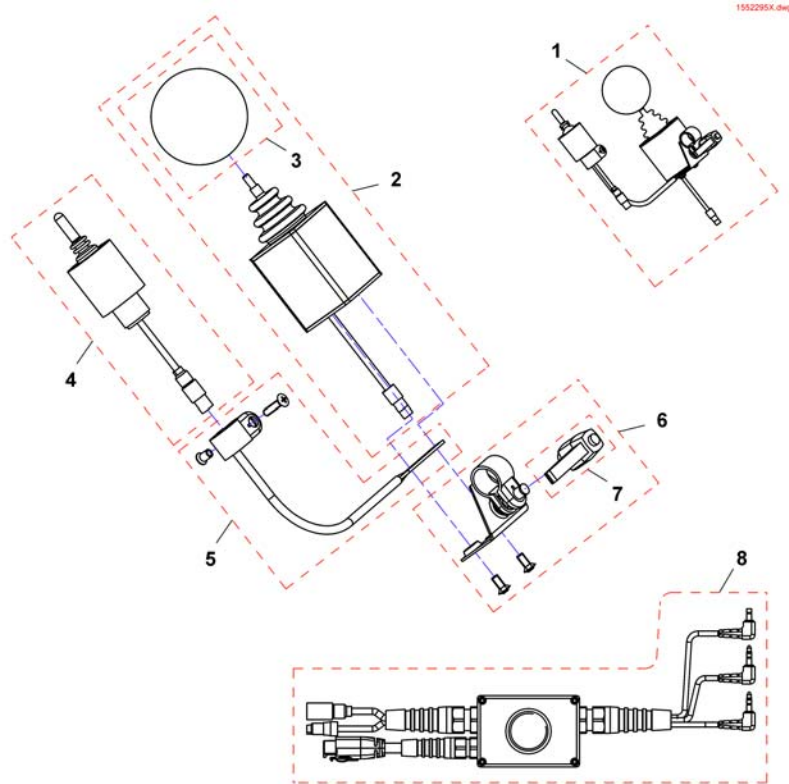

Pos.	Part number	Description	Valid from → until	Qty
1	1448705	Chin control REM24, cpl.		1
1	1421770	Chin control G80/G90, cpl.		1
1	SP1544255	Chin control G91, cpl.		1
2	Z183282	Joystick, kmpl.		1
3	Z183265	Foam control knob cpl.		1
4	1442943	Switchjoystick REM24, kmpl.		1
4	Z183283	Switchjoystick G80/G90, cpl.		1
4	SP1544250	Switchjoystick G91, kmpl.		1
5	SP1552294	Switchjoystick bracket		1
6	SP1552295	Joystick clamp		1
7	L32901	Clampinglever		1
8	SP1442944	Harness REM24		1
8	Z183275	Harness G80/G90		1
8	SP1541715	Harness G91		1


## Framework for chincontrol

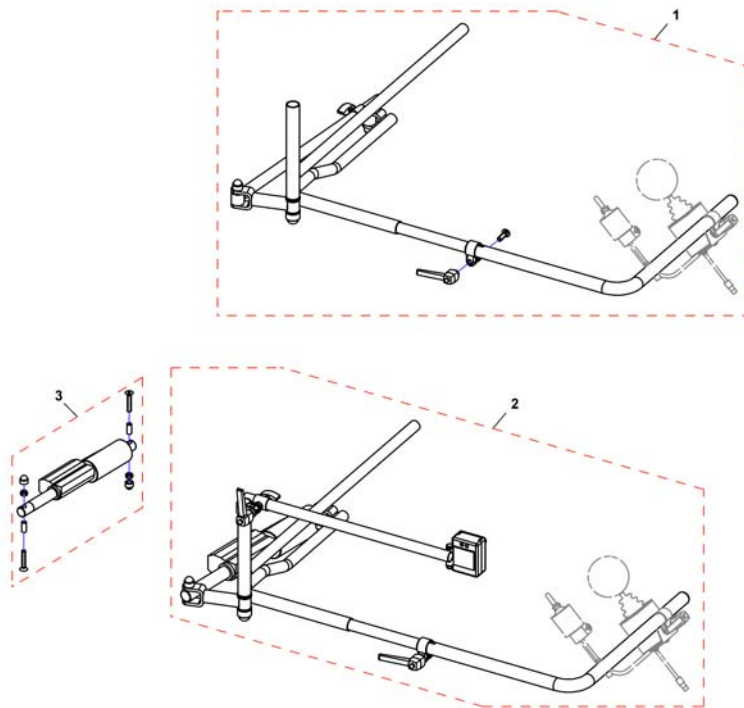

Pos.	Part number	Description	Valid from → until	Qty
-	Z167994	CHIN CONTROL SUPPORT . CPL. <i>Not shown / Nicht abgebildet / Non illustré</i>		1
1	SP1422943	Framework for chin control, manual, horizontal swing away, LH, cpl.		1
1	SP1422942	Framework for chin control, manual, horizontal swing away, RH, cpl.		1
2	SP1422945	Framework for chincontrol, powered, horizontal swing away, LH, cpl. <i>incl. Actuator</i>		1
2	SP1422944	Framework for chin control, powered, horizontal swing away, RH, cpl. <i>incl. Actuator</i>		1
-	SP1552335	Hardware, chin control, powered, horizontal swing away, LH <i>Not shown / Nicht abgebildet / Non illustré</i>		1
-	SP1552336	Hardware, chin control, powered, horizontal swing away, RH <i>Not shown / Nicht abgebildet / Non illustré</i>		1
-	SP1422946	Framework for chincontrol, powered, vertikal swing away <i>Not shown / Nicht abgebildet / Non illustré</i>		1
3	SP1552337	Actuator for horizontal swing away chin control, cpl.		1
3	SP1552418	Gestell für Kinnbedienung, elektrisch, vertikal abschwenkbar <i>Not shown / Nicht abgebildet / Non illustré</i>		1

## Dual Control with brackets


1563006 iso

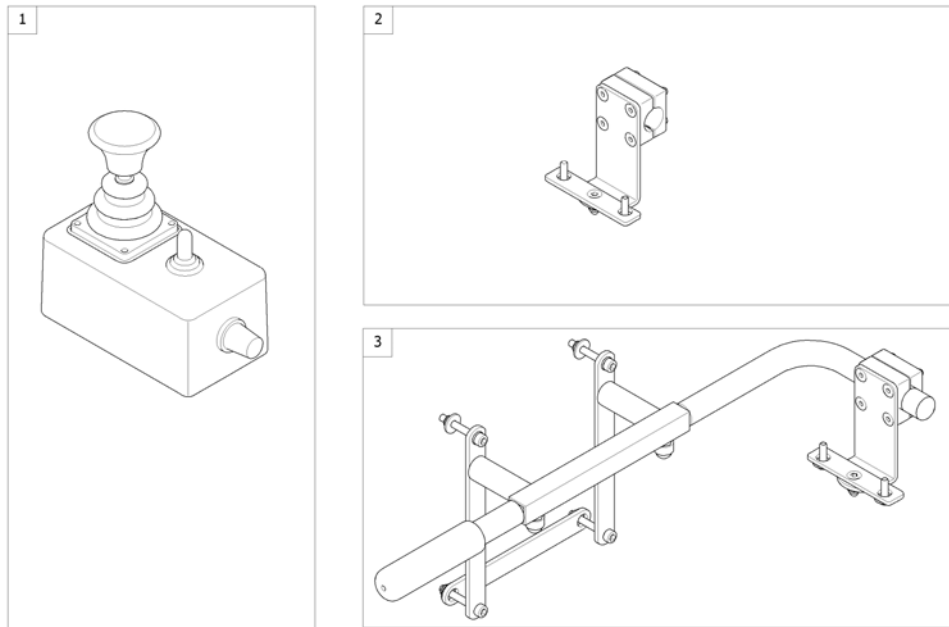

Pos.	Part number	Description	Valid from → until	Qty
1	6001425	ACS dual control [1291/1292]		1
2	-	Remote Bracket Dual Control for Standard-/Contour-/Flex-Seat [1291/1292]		1
3	-	Remote Bracket Dual Control for Recaro Seat [1291/1292]		1

# Remote Bracket Dual Control for Standard-/Contour-/Flex-Seat


1560885.iso

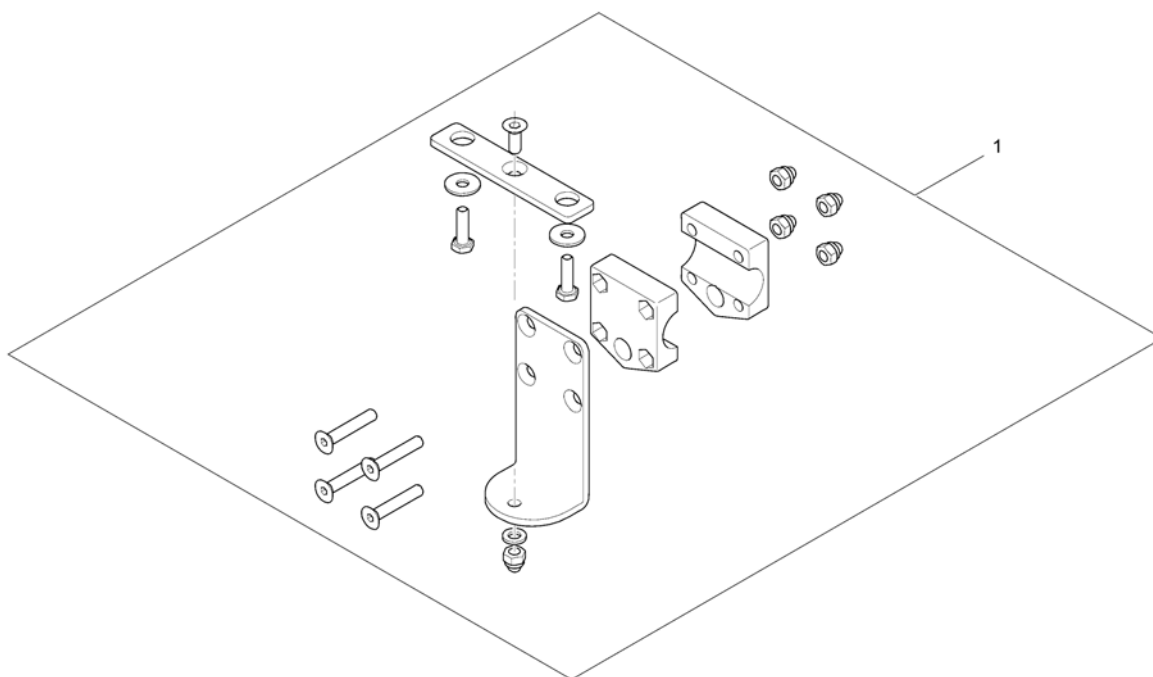

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422818	Remote Bracket Dual Control		1

# Remote Bracket Dual Control for Recaro Seat


1563504.iso

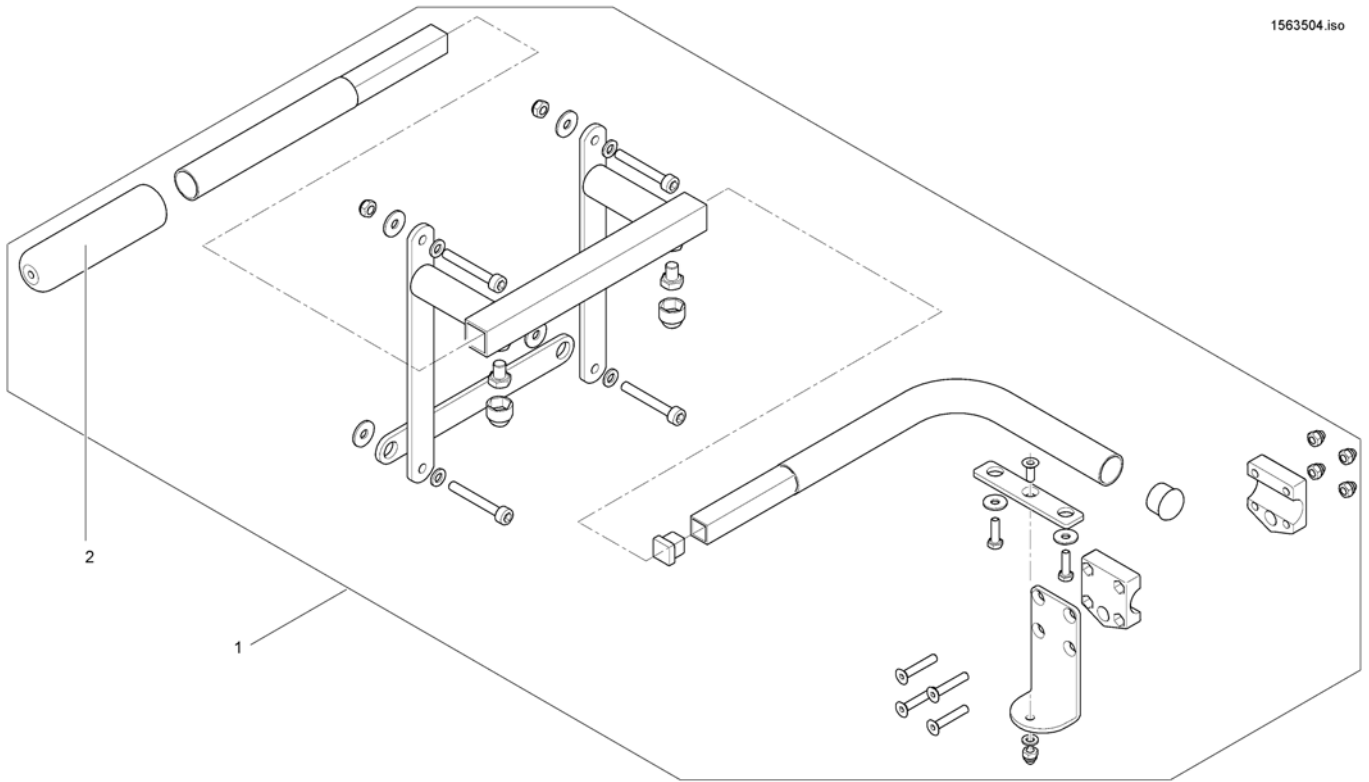

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422819	Assembly kit for dual control Recaro		1
2	1435421	Handle plastic black Ø22 "Invacare"		1

# Intuitive Dual Control with brackets

1509875X.iso

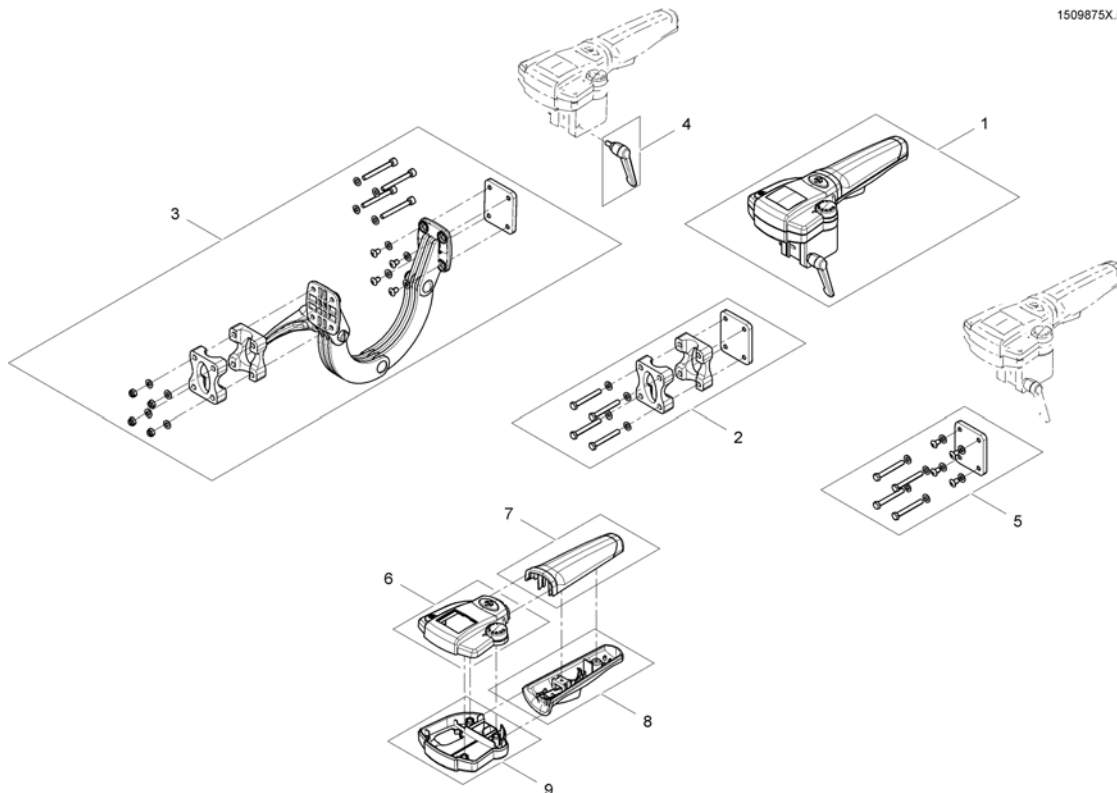


Pos.	Part number	Description	Valid from → until	Qty
1	SP1509875	Intuitiv Dual Control (IDC) <i>For chairs build before 01/02/10 please also order SP1537148</i>		1
2	SP1522310	Fixed bracket		1
3	SP1521488	Adjustable bracket		1
4	SP1510135	Clamp lever M5x10		1
5	SP1537148	Supportplate IDC for chairs before 01.02.10		1
6	SP1579573	Upper Housing		1
7	SP1511060	Handhold upper part		1
8	SP1511099	Handhold lower part		1
9	SP1510470	Housing lower part		1

**ACS Cables & Accessories**

<b>Pos.</b>	<b>Part number</b>	<b>Description</b>	<b>Valid from → until</b>	<b>Qty</b>
-		Bus-Cables ACS		1
-		T-Adapter		0
-		T-Adapter - Modulite Seat		1
-		Programming Kits		1
-		Magnetkey for Remotes		1

## Bus-Cables ACS

ACSBUSKABEL.iso

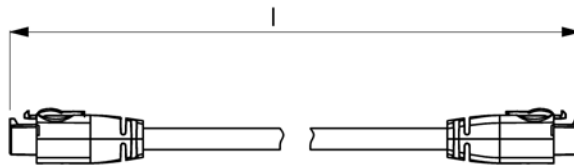


Position	length l [mm]
1	300
2	500
3	700
4	1100
5	1700
6	2000
7	2500
8	2700
9	1060

Pos.	Part number	Description	Valid from → until	Qty
1	GSM63003	Bus cable ACS, 300mm		1
2	SPGSM63005	Bus cable ACS, 500mm <i>Replace GSM63004</i>		1
3	GSM63007	Bus cable ACS 700mm		1
4	GSM63011	Bus cable ACS, 1100mm		1
5	1559862	Bus cable ACS, 1700mm <i>Replace GSM63016</i>		1
6	GSM63020	Bus cable ACS, 2000mm <i>Replace 1419386</i>		1
7	1440424	Bus cable ACS, 2500mm		1
8	SP1559873	Bus cable ACS, 2700mm <i>Replace GSM63026</i>		1
9	SP1550402	Extension cable ACS, 1060mm		1



# T-Adapter


1583783 iso

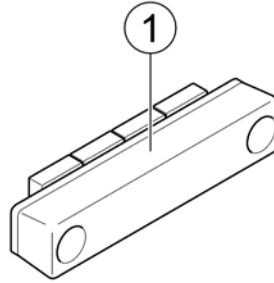

Pos.	Part number	Description	Valid from → until	Qty
1	P10072	ACS-Bus T-Adapter (Splitter)		1

# T-Adapter - Modulite Seat


1583360 iso

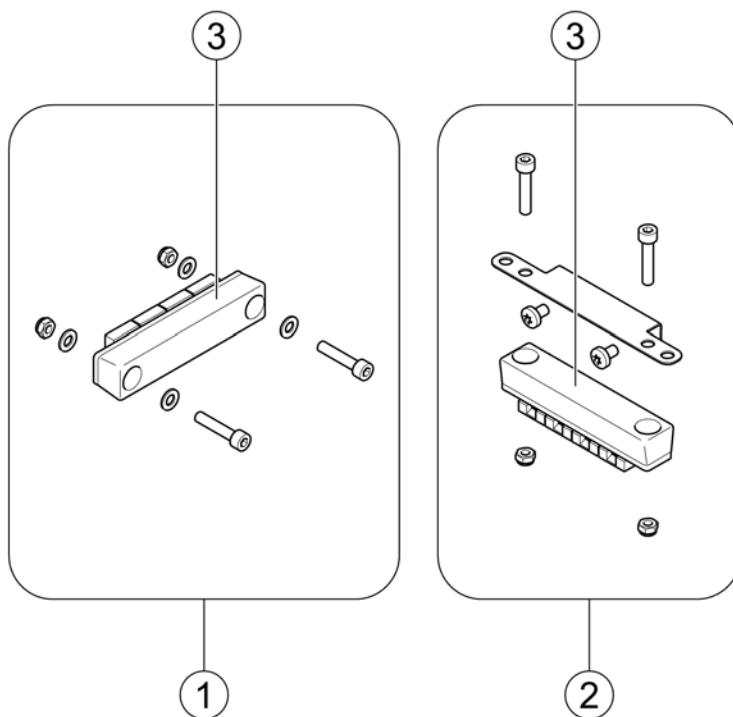

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571826	T-Adapter - Modulite Seat		1
2	SP1569958	T-Adapter - Modulite Seat with MBL-Modul		1
3	P10072	ACS-Bus T-Adapter (Splitter)		1

# Programming Kits


1584805 iso

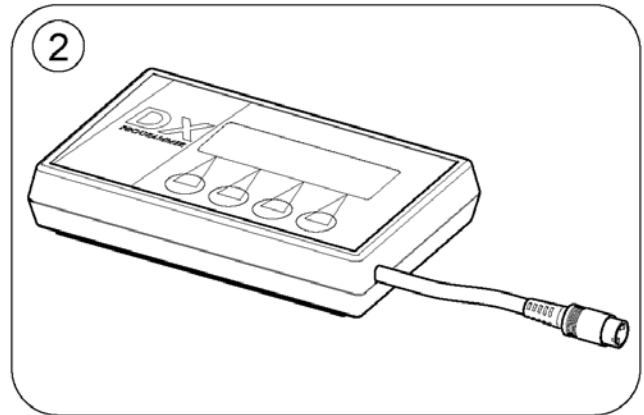

Pos.	Part number	Description	Valid from → until	Qty
1	1449280	USW wizard dongle kit, EDL-USW, cpl.		1
2	6001450	Hand Programmer, Instrucions In French, German, Spanish		1
	SP1441551	Programming Adapter Shark DK-ADAPT (necessary to program Shark)		1

# Magnetkey for Remotes

1564186.iso

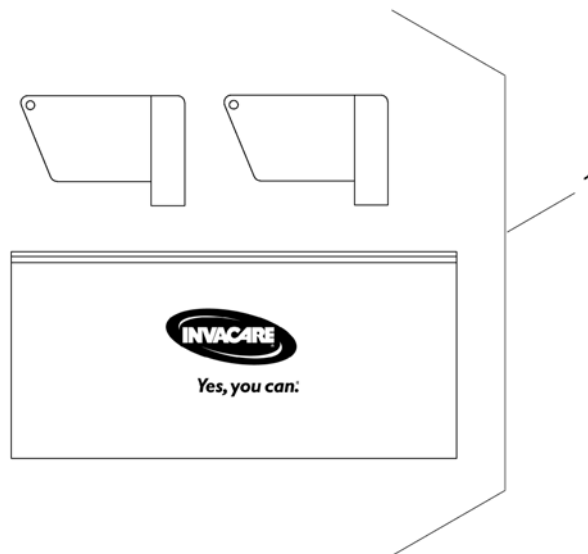

Pos.	Part number	Description	Valid from → until	Qty
1	SP1419695	Magnet key with bag		1

# Batteries


	Capacity [Ah]															
	1,2	12	30	31	32	36	38	40	40	50	50	50,9	60	73,6	75	97,6
Bora/SpectraXTR								8		10						
Dragon										10			13			
Dragon Vertic													13			
FDX													13			
G50													13	14		16
Kite													13	14		
Mirage				4				8								
Mistral3				4									12			
P9000XDT					5											
Pronto M41					5											
Pronto M61					5											
Spectra Blitz								8					12			
Spectra Comfort								7								
Spectra Plus								7								
Spectra Seat Riser								7								
Storm3										10			13	14		
Storm3 True Track	1													14		
Storm3 Vertic														14		
Storm4														14		
Storm4 Max														14		
Stream										10			13			
TDX-SP														14		
TDX-SP Narrow													12			
Tiger										10			13			
Topan													13			
Tornado								8								
Typhoon													13			
Typhoon GB	1												13			
Typhoon GB	1												13			
Zephyr	1												13			
Auriga			3					8		10						
Comet														14	15	
Leo						6			9							
Lynx		2														
Meteor														14		
Orion												11				
length [mm]	97	152	197	196	211	196	210	197	198	198	238	259	260	260	329	
width [mm]	49	99	132	132	132	130	175	168	165	166	165	140	169	171	168	171
height [mm]	51	102	180	183	183	179	175	175	171	176	170	235	178	210	223	238
weight [kg]	0,6	3,9	11,1	10,5	11	12	14,6	14,4	14,2	14	16	17,1	19,3	24,3	26	32,5



Pos.	Part number	Description	Valid from → until	Qty
i	-	Change batteries only pairwise and from the same type!		1
1	1440386	Battery, 12V/1,2Ah C20, GEL		1
2	SP1506493	Battery 12V/12Ah AGM		1
3	P50021	Battery, 12V/30Ah C20, GEL		1
4	P50029	Battery, 12V/31Ah C20, GEL		1
5	SP1521459	Battery, 12V/34Ah C20, AGM		1
6	1505498	Battery, 12V/36Ah C20, AGM		1
7	P50028	Battery, 12V/38Ah C20, GEL (UK & DK only)		1
8	PG50988	Battery, 12V/40Ah C20, GEL		1
9	SP1550692	Battery, 12V/40Ah C20, AGM		1
10	SP1522107	Battery, 12V/50Ah C20, AGM		1
11	SP1521695	Battery, 12V/50Ah C20 AGM		1
12	PG50989	Battery, 12V/50,9Ah C20, GEL		1
13	1432944	Battery, 12V/60Ah C20, GEL		1
14	P50016	Battery, 12V/73,6 Ah C20, Gel		1
15	SP1522026	Battery, 12V/75Ah C20, AGM		1
16	1478675	Battery, 12V/97,6Ah C20, GEL		1

# Chargers


1589799.iso

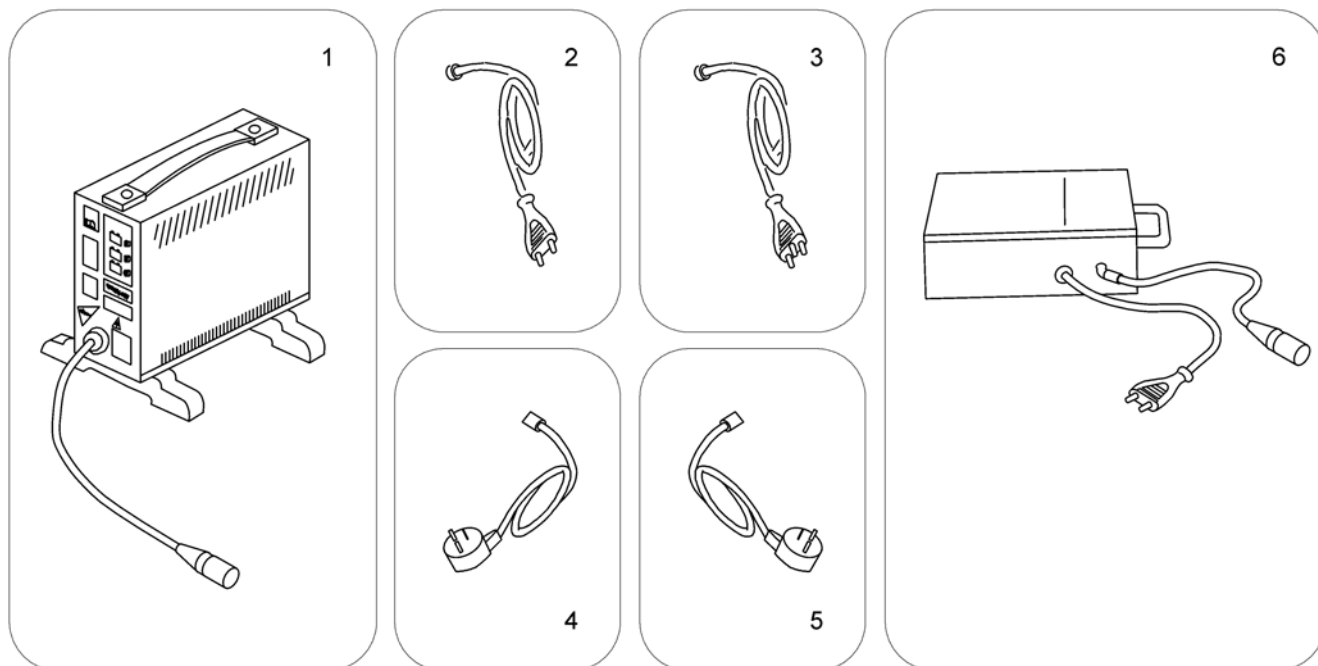

Pos.	Part number	Description	Valid from → until	Qty
1	SPP51981	Charger 8A without Power Cord <i>[1343]</i>		1
2	P21079	Power Cord Continental (Schuko GCB-0401)		1
3	P21080	Power Cord UK (GCB-0403)		1
4	1443841	Power Cord Swiss (black 6A/250V)		1
5	SP1553849	Power Cord Israel (IEC 320 C13 to SI-32)		1
6	5813987	Charger 8A (Protectionclass IP54) incl. Power Cord Continental (Sweden only) <i>[1388]</i>		1


# Battery support

1440013.png

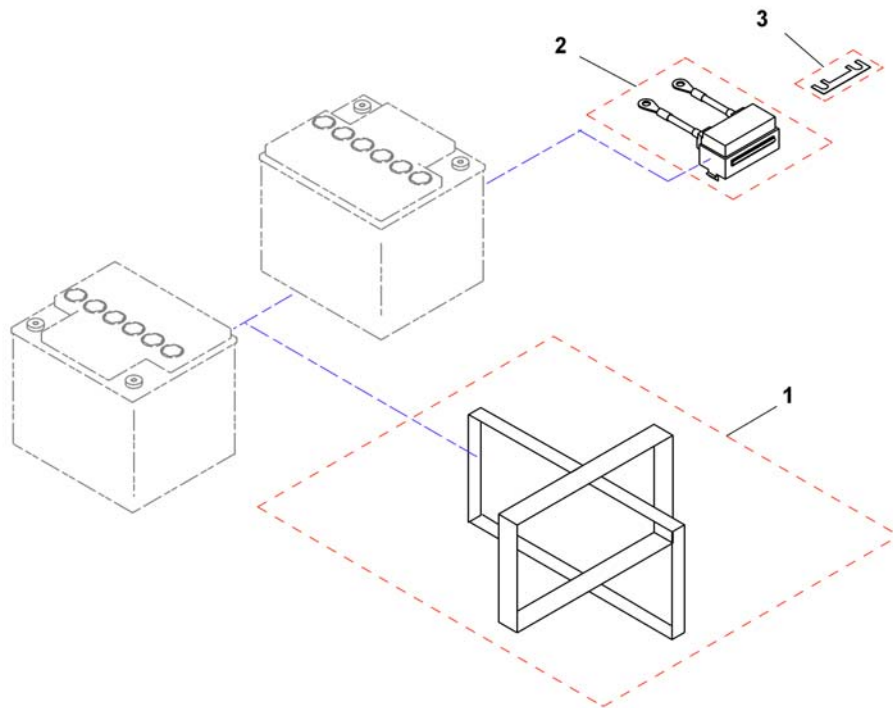

Pos.	Part number	Description	Valid from → until	Qty
	1436209	ACS Battery-cable 60Ah Standard		1
	1436211	Battery Cable for True Track		1
	1436212	Strip Fuse Holder 80A		1
	SP1442944	Harness REM24		1
	1498080	Main cable harness with lifter		1
	1498090	Main cable harness without Lifter		1
1	1440403	Battery Strap 60Ah		1
2	1436210	Fuse Bracket with Fuse 50A		1
3	1432561	Strip Fuse Holder with 80A Fuse		1
3	P12191	Fuse 50A		1


# Front Light

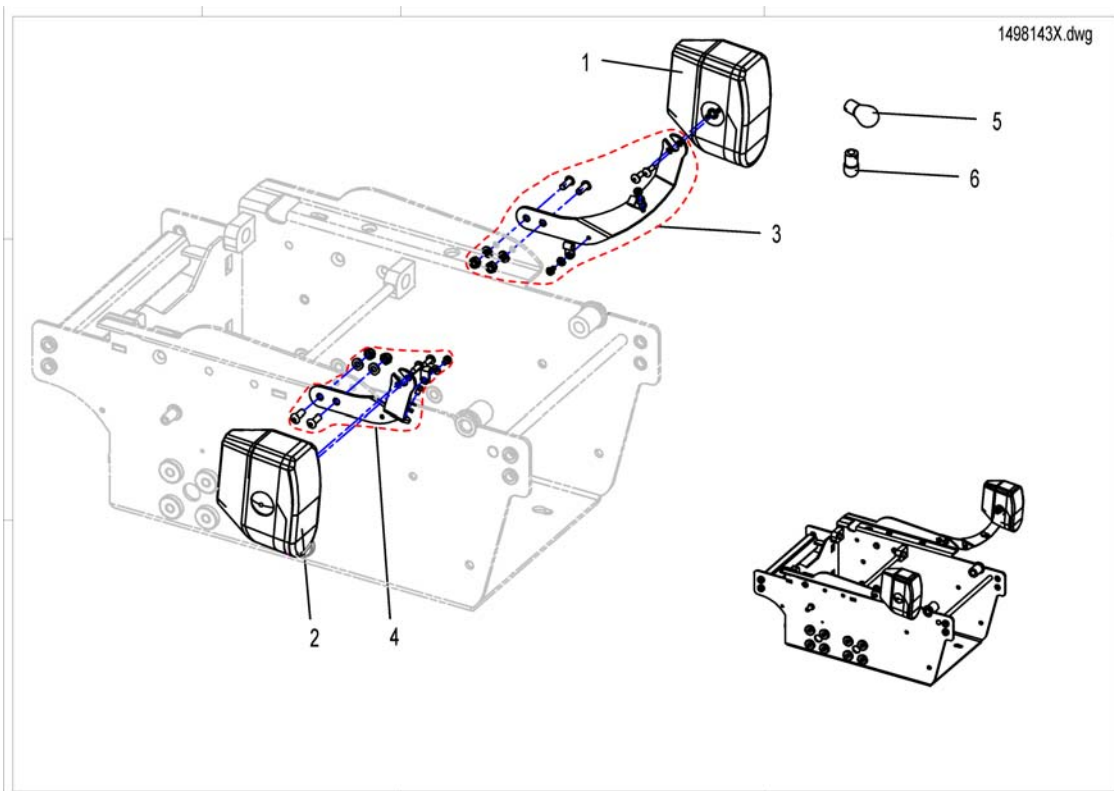

Pos.	Part number	Description	Valid from → until	Qty
1	SP1420413	Headlamp LH 24V cpl.		1
2	SP1420412	Headlamp RH 24V cpl.		1
3	SP1493974	Lamp Fixation LH		1
4	SP1493971	Lamp Fixation RH		1
5	SP1419271	Light Bulb 24V/21W, yellow (Turn Signal front/rear)		1
6	SPP31108	Light Bulb 24V/10W (BA15s), white (Front)		1




## Rear Light

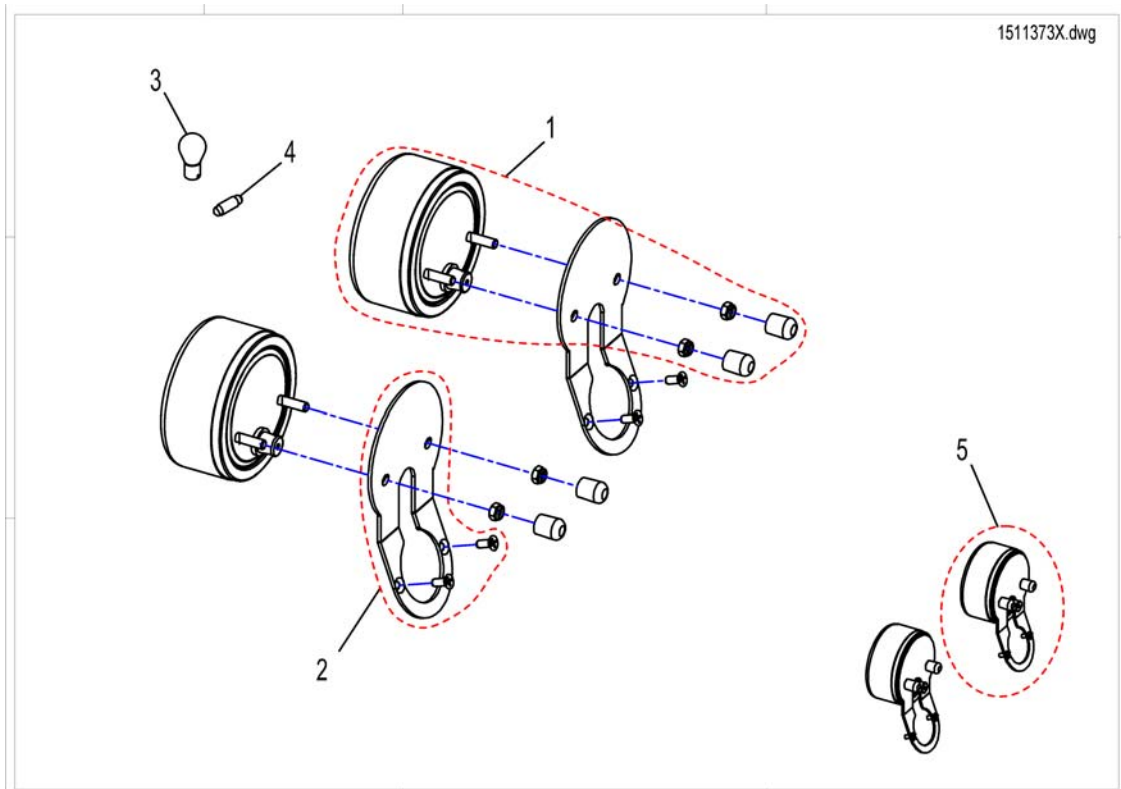

Pos.	Part number	Description	Valid from → until	Qty
!!!	-	Rear Light Protector Kit		0
1	SP1440450	Rear Light		1
2	SP1513249	Lamp Holder		1
3	SP1419271	Light Bulb 24V/21W, yellow (Turn Signal front/rear)		1
4	SPP31191	Incandescent Lamp 24V/5W white		1
5	SP1511373	Rear Light, cpl.		1


# Rear Light Protector Kit

1585420.00

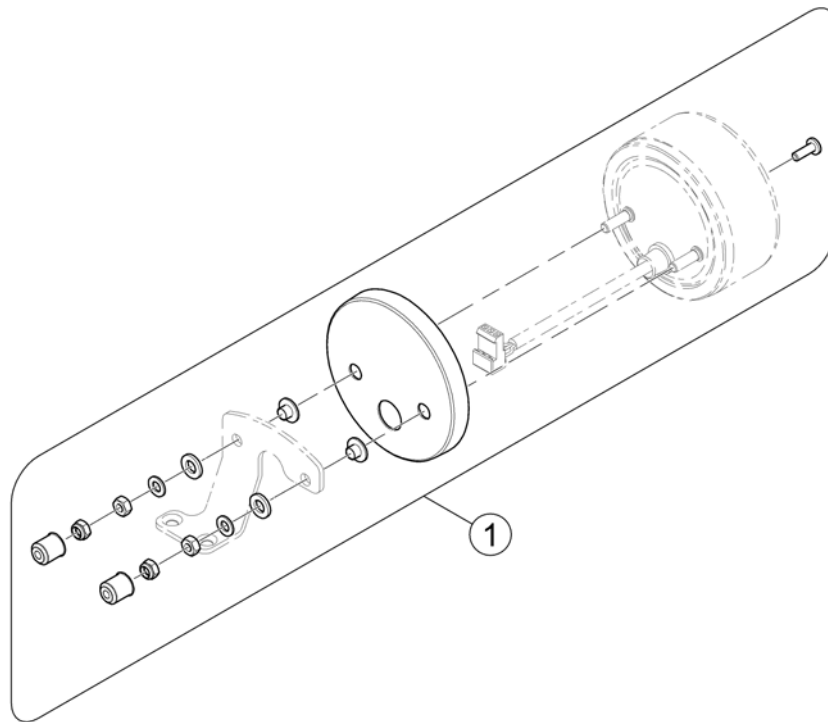

Pos.	Part number	Description	Valid from → until	Qty
1	SP1581827	Rear Light Protector Exchange Kit		1


# Reflector

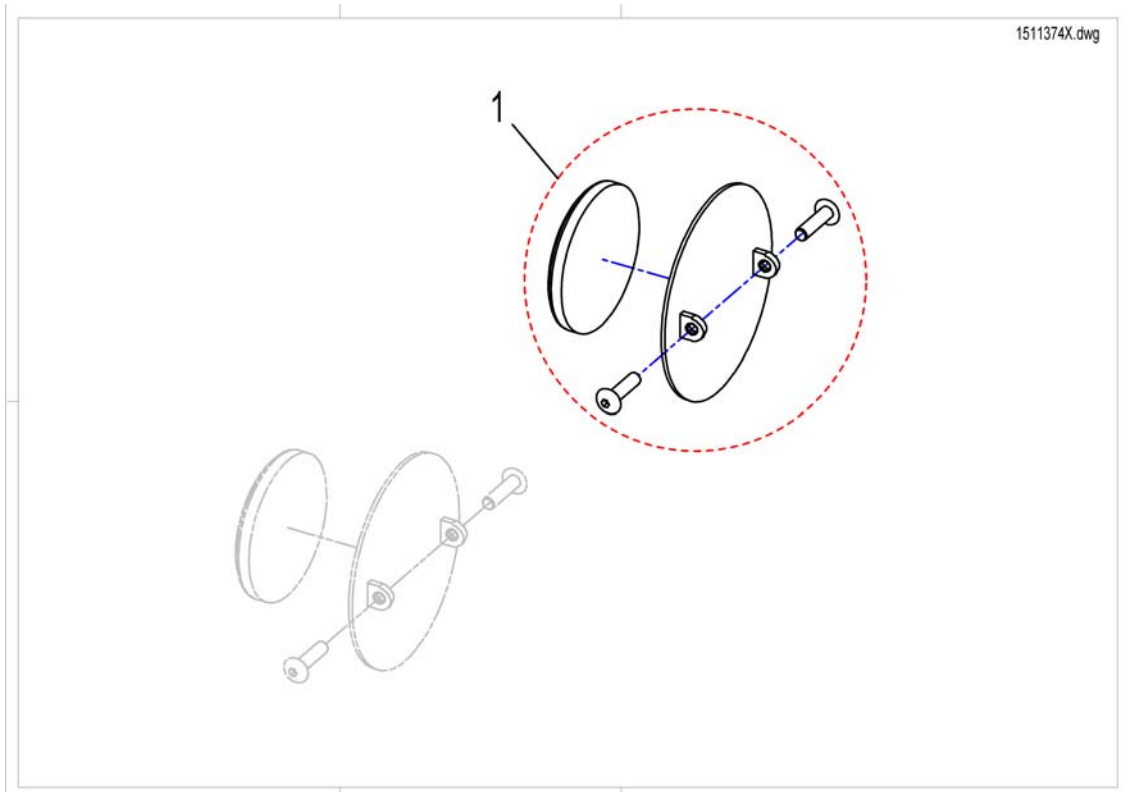

Pos.	Part number	Description	Valid from → until	Qty
1	SP1511374	Reflector cpl.		1

# Backrest Extension Flex

1573746.jpg

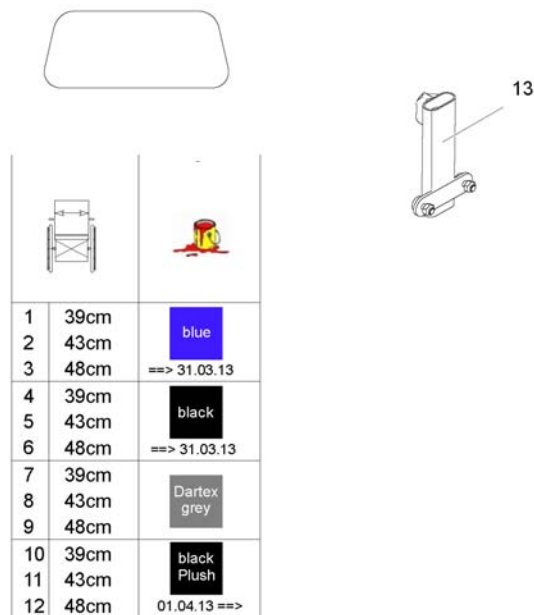


1	39cm	blue
2	43cm	
3	48cm	==> 31.03.13
4	39cm	black
5	43cm	
6	48cm	==> 31.03.13
7	39cm	Dartex grey
8	43cm	
9	48cm	
10	39cm	black Plush
11	43cm	
12	48cm	01.04.13 ==>

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422918	Backrest Extension for Flex Seat, SW39cm, blue	→ 2013-03-31	1
2	SP1422919	Backrest Extension for Flex Seat, SW43cm, blue	→ 2013-03-31	1
3	SP1422920	Backrest Extension for Flex Seat, SW48cm, blue	→ 2013-03-31	1
4	1436567	Backrest Extension for Flex Seat, SW39cm, black	→ 2013-03-31	1
5	1436568	Backrest Extension for Flex Seat, SW43cm, black	→ 2013-03-31	1
6	1440283	Backrest Extension for Flex Seat, SW48cm, black	→ 2013-03-31	1
7	5253035-TR23	Backrest Extension for Flex Seat, SW39cm, Dartex		1
8	5253036-TR23	Backrest Extension for Flex Seat, SW43cm, Dartex		1
9	5253037-TR23	Backrest Extension for Flex Seat, SW48cm, Dartex		1
10	SP1436567-TR35	Backrest Extension for Flex Seat, SW39cm, black Plush	2013-04-01 →	1
11	SP1436568-TR35	Backrest Extension for Flex Seat, SW43cm, black Plush	2013-04-01 →	1
12	SP1440283-TR35	Backrest Extension for Flex Seat, SW48cm, black Plush	2013-04-01 →	1
13	SP1422921	Fixation for Backrest Extension		1

## Lateral Support (Dartex)


1526377.iss

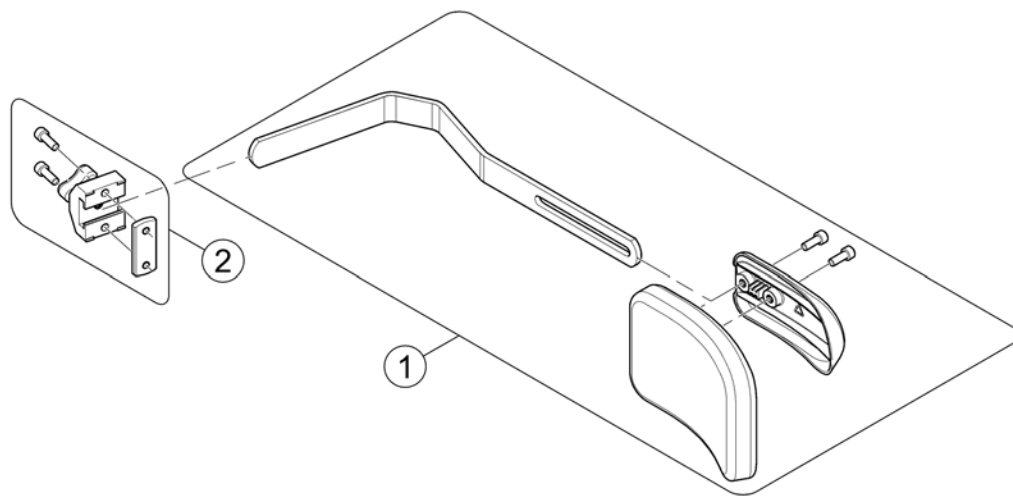

Pos.	Part number	Description	Valid from → until	Qty
1	SP1561841	Trunk support, cpl.		1
2	5253170	Lateral Support Bracket		1

# Lateral Support - Junior Seat


5313271X.dwg

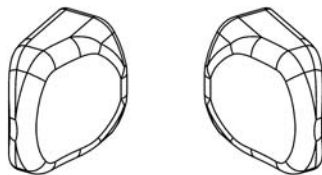

RH	LH	
1	2	blue dot
3	4	vinyl grey

Pos.	Part number	Description	Valid from → until	Qty
1	SP5313271	Trunk Support RH for Junior, blue Dot		1
2	SP5313275	Trunk Support LH for Junior, blue Dot		1
3	5313272	TRUNK SUPPORT VINYL BLACK RH CPL.		1
4	5313276	TRUNK SUPPORT VINYL BLACK LH CPL.		1


## Abduction Wedge

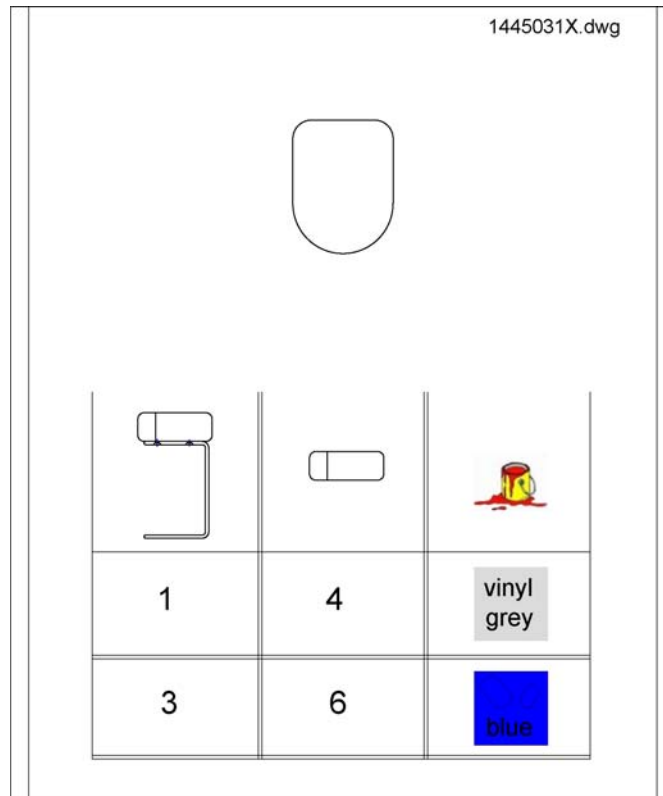

Pos.	Part number	Description	Valid from → until	Qty
1	SP1445033	Abduction Wedge Junior, grey Vinyl, with Bracket		1
3	SP1445032	Abduction Wedge Junior, Blue Dot, with Bracket		1
4	SP5313257	Abduction Wedge Junior, grey Vinyl, wo Bracket		1
6	SP5313256	Abduction Wedge Junior, Blue Dot, wo Bracket		1

# Headrest & Brackets

1422825X.iso

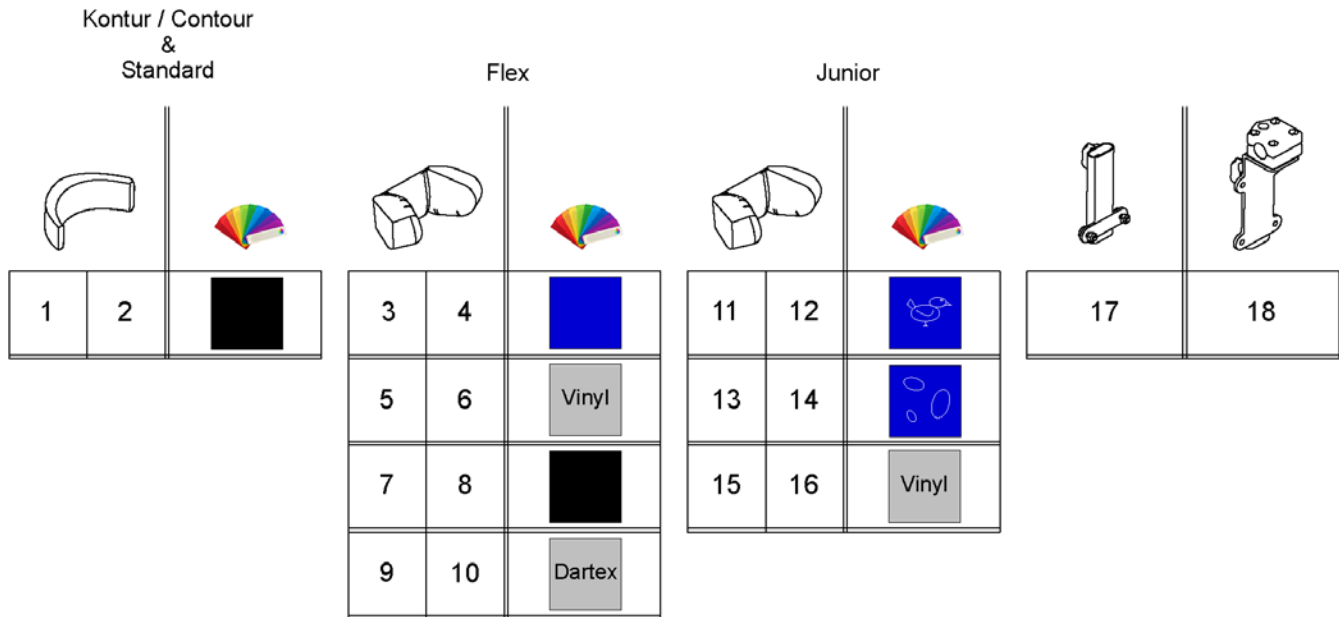


Pos.	Part number	Description	Valid from → until	Qty
1	SP1422925	Headrest, standard/contour seat, black, complete		1
2	SP1422865	Headrest cushion contour black, without bracket		1
3	-	No longer available.		0
4	-	No longer available.		0
5	-	No longer available - please choose Dartex grey. <i>SP1428092</i>		0
6	-	No longer available - please choose Dartex grey. <i>SP5252229</i>		0
7	SP1440285-TR35	Headrest Flex, Plush black, cpl.		1
8	SP1585039	Headrest Cushion Flex, Plush black (chenille TR35), w/o bracket		1
9	SP1428092	Headrest Flex, Dartex grey, complete		1
10	SP5252229	Headrest cushion Flex Dartex grey, without Bracket		1
11	-	No longer available.		0
12	-	No longer available.		0
13	SP5313307	Headrest cushion Junior, blue Dot, complete		1
14	SP5313261	Headrest cushion Junior, blue Dot, without bracket		1
15	SP5313308	Headrest Cushion Junior, Vinyl grey, complete		1
16	SP5313262	Headrest Cushion Junior, Vinyl grey, without bracket		1
17	SP1422547	Headrest Bracket, Contour-/Flex Seat, complete		1
18	SP1422914	Head-/Neckrest Bracket, cpl.		1




## Neckrest & Brackets

5313999X.dwg

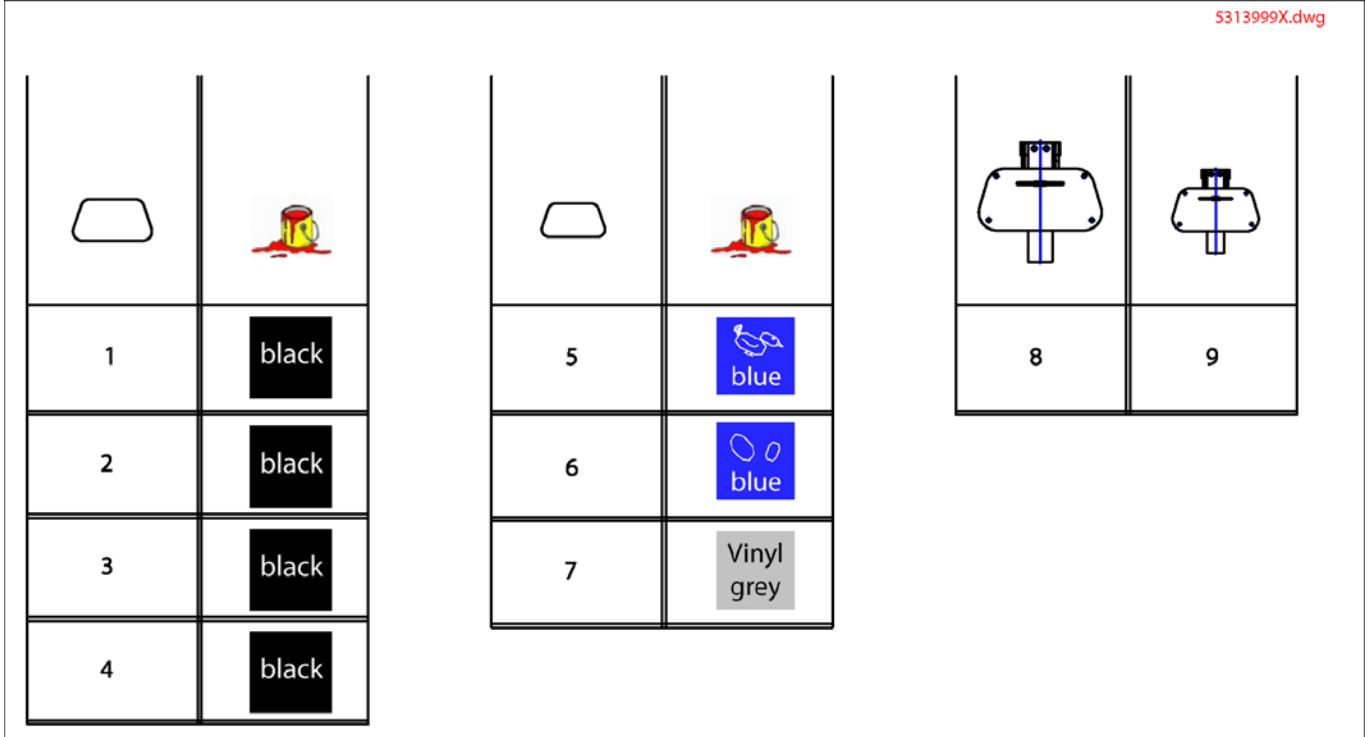


Pos.	Part number	Description	Valid from → until	Qty
1	SP5313999	Neckrest Cushion Std., black		1
2	SP1419643	Neckrest Cushion Typ4, black		1
3	SP1419640	Neckrest Cushion Std. small, black		1
4	SP1419646	Neckrest Cushion Typ4 small, black		1
5	SP5313309	Neckrest Cushion Junior, Blue Birdie		1
6	SP5313311	Neckrest Cushion Junior, Blue Dot		1
7	SP5313312	Neckrest Cushion Junior, grey Vinyl		1
8	SP1419636	Bracket for Neckrest		1
9	SP1419637	Neckrest Holder Typ3 short		1

## Push Handles

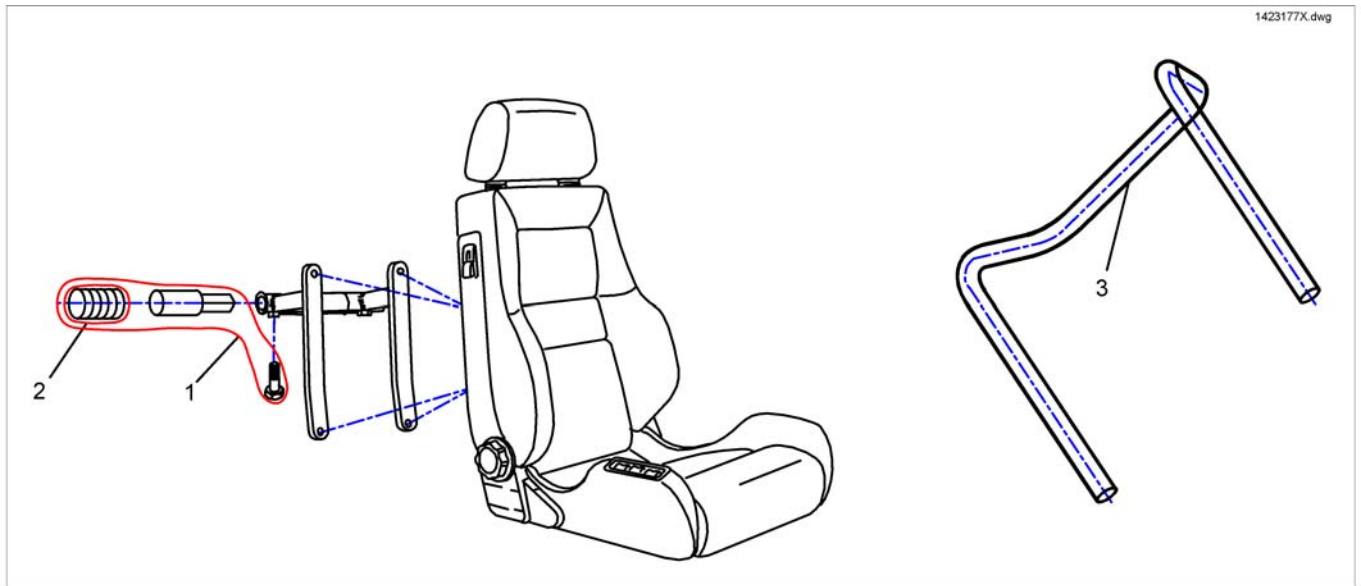


Pos.	Part number	Description	Valid from → until	Qty
1	SPL10009	Push Handle for Recaro		1
2	SPG16040A	plastic grip		1
3	SP1425840	Push Handle Flex SW 39cm		1
3	SP1425839	Push Handle Flex SW 43/48cm		1

# Knee Support

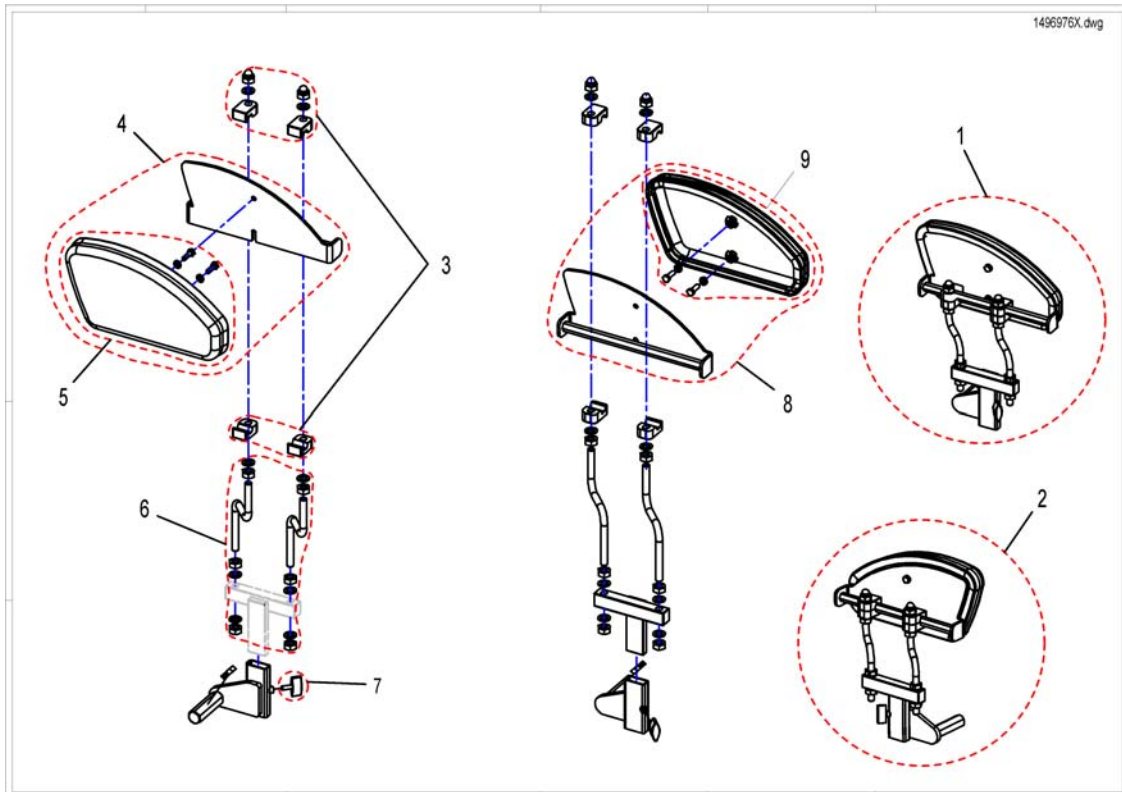


Pos.	Part number	Description	Valid from → until	Qty
1	SP1496375	Knee Support cpl. LH		1
2	SP1496976	Knee Support cpl. RH		1
3	SP1496330	Mounting Kit		1
4	SP1496374	Side upholstery RH with bracket		1
5	SP1450243	Side Upholstery RH		1
6	SP1496329	Swing wire		1
7	SPL10024	Wing Screw M6x16		1
8	SP1496977	Side upholstery LH with bracket		1
9	SP1450240	Side Upholstery LH		1

## Docking-Station

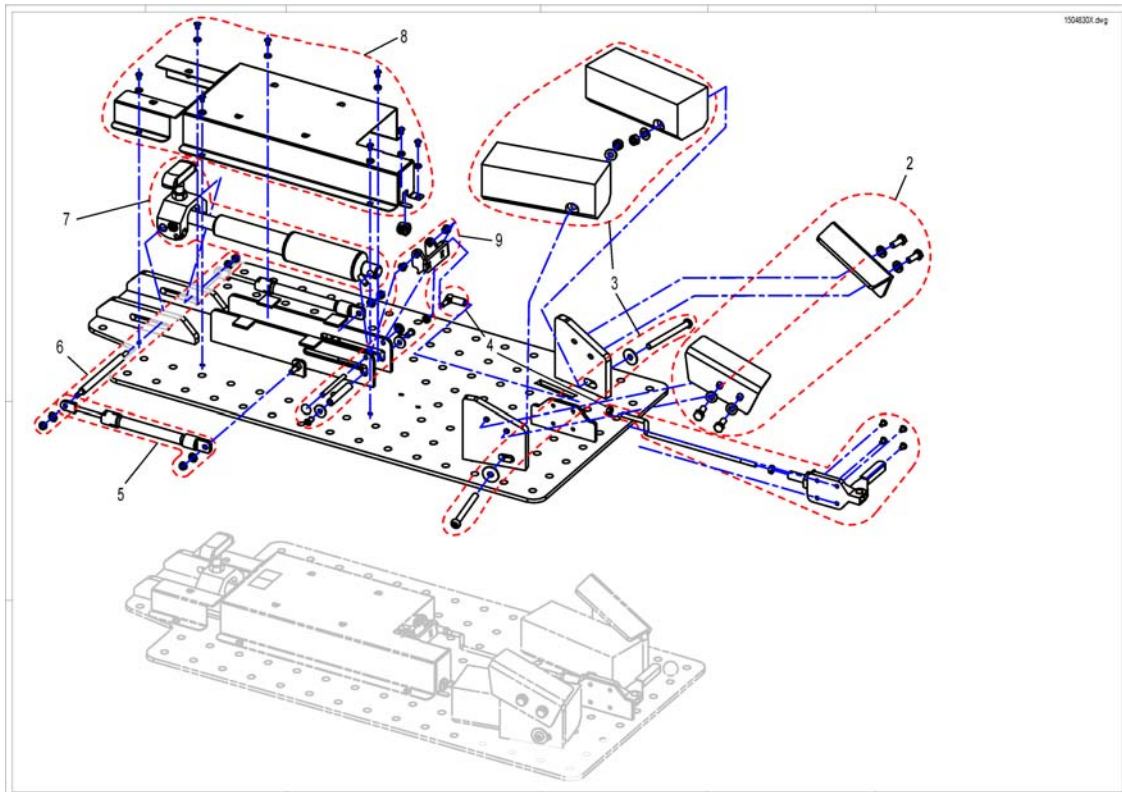

Pos.	Part number	Description	Valid from → until	Qty
2	SP1514109	Guidance Bracket		1
3	SP1509045	buffer stop		1
4	SP1510996	Driving Rod		1
5	SP1498420	pressure Spring cpl.		1
6	SP1495181	Stud		1
7	SP1498421	t-brace cpl.		1
8	SP1496135	Cover		1
9	SP1495247	Chain Guide		1
10	1509665	Chair-Part Dockingstation Zephyr/Typhoon2		1


## Rearview Mirror

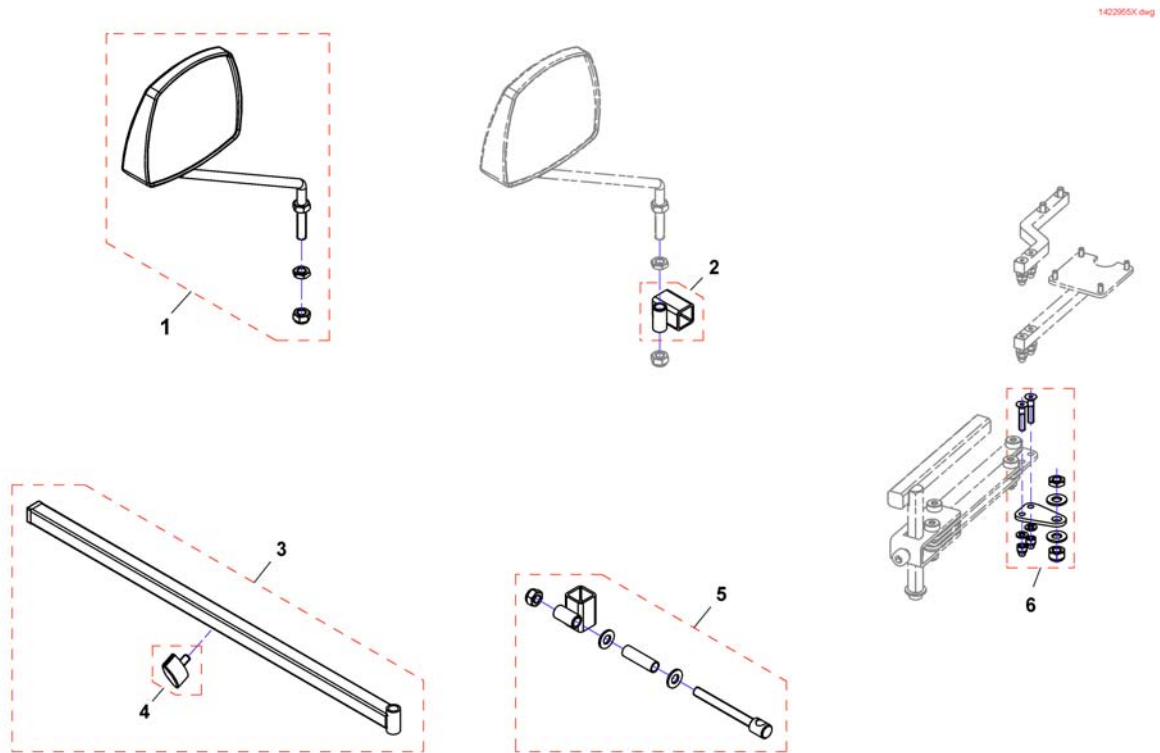

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422955	Rear Mirror		1
2	F276005.00	Mirror bracket		1
3	SP1423435	Rear Mirror Support Rail cpl.		1
4	L31001	Wing ScREW M6X10		1
5	SP1422957	Mirror bracket		1
6	SP1527151	Adapter to mount mirror and remote at the same side		1


# Trays

1535050X.dwg

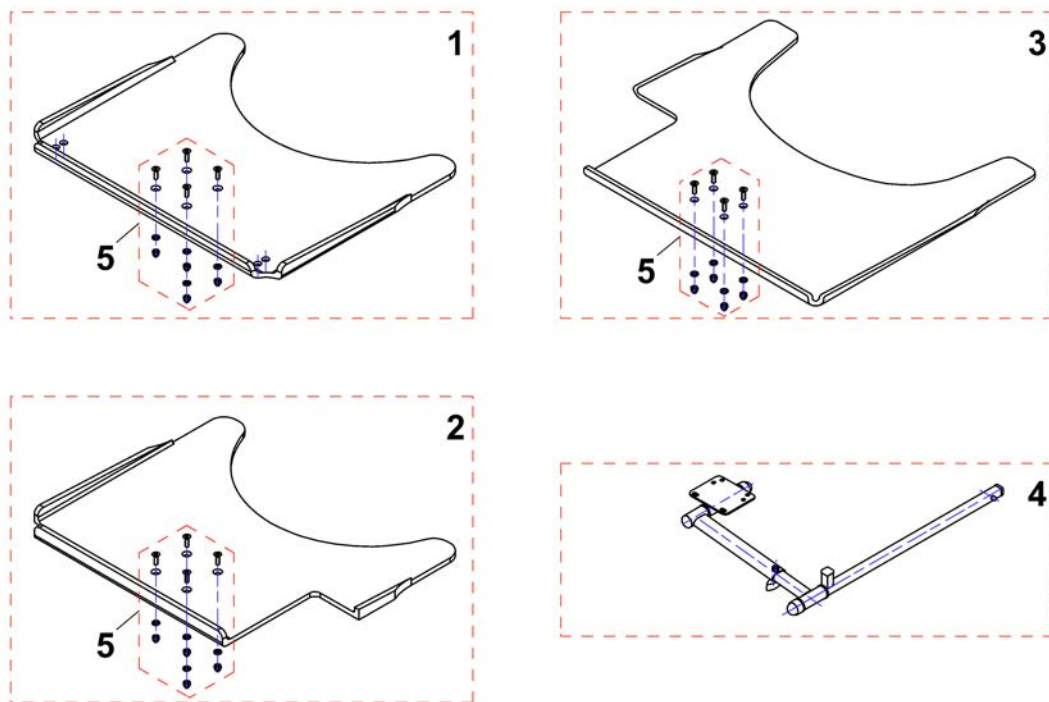

Pos.	Part number	Description	Valid from → until	Qty
1	SP1535050	Tray table, 540mm, no Cutout		1
1	SP1535010	Tray table, 620mm, no Cutout		1
1	SP1535011	Tray table, 660mm, no Cutout		1
2	SP1535009	Tray table, 540mm, Cutout LH		1
2	SP1534974	Tray table, 620mm, Cutout LH		1
2	SP1534975	Tray table, 660mm, Cutout LH		1
3	SP1535051	Tray table, 540mm, Cutout RH		1
3	SP1535012	Tray table, 620mm, Cutout RH		1
3	SP1535049	Tray table, 660mm, Cutout RH		1
4	SPZ177011	Tray Support Fixing Kit cpl.SW 380/505 mm		1
5	Z177025	Fastening-Kit Tray		1

## other Options

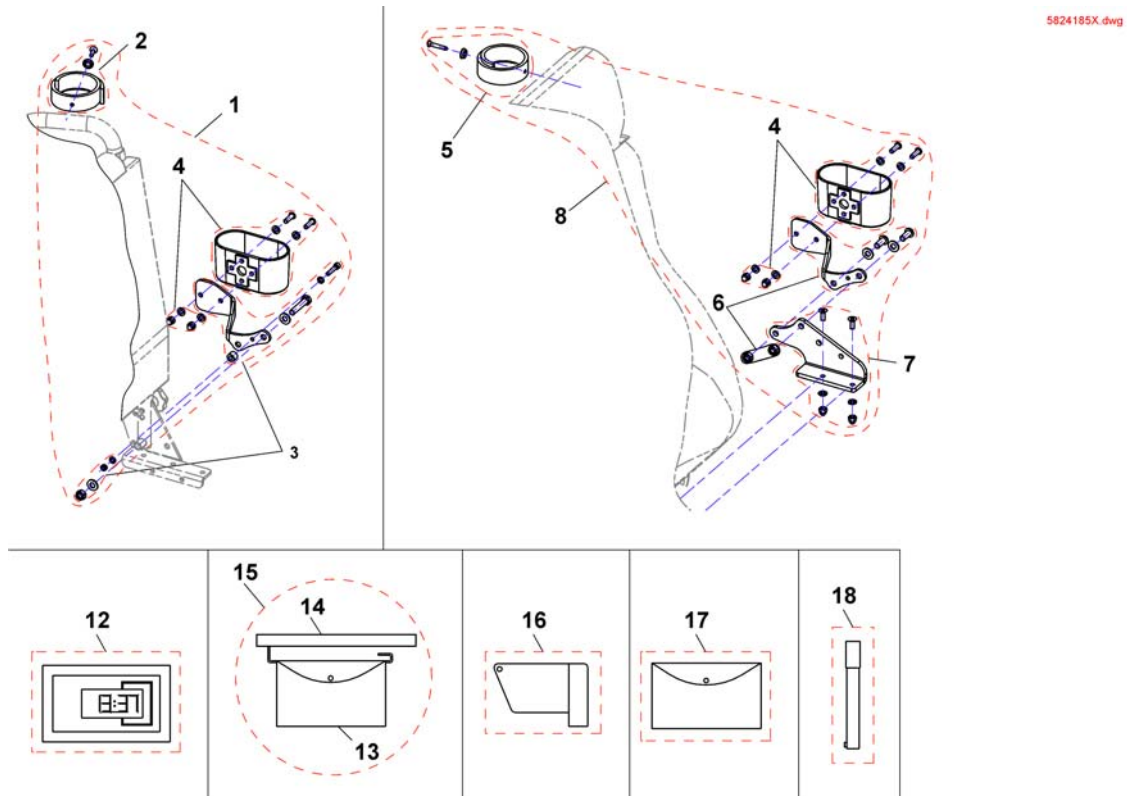

Pos.	Part number	Description	Valid from → until	Qty
1	SP1442901	Cane Holder for Standard-Seat, LH		1
1	SP1534877	Cane Holder for Standard Seat, cpl.		1
-	1449280	USW wizard dongle kit, EDL-USW, cpl.		1
2	SP5009551	Belt for Caneholder		1
3	SP5007160	Crutchholder cup		1
4	SP5824185	Cup for Caneholder		1
5	SP1420493	Belt for caneholder		1
7	SP1420477	bracket for caneholder Recaro		1
8	SP1442725	Cane Holder for Recaro Seat cpl.		1
12	SP1485700	Hour Counter for G-Motors		1
13	SP5314967	Bag for armrest		1
14	SP1430855	Holder for armrest bag		1
15	SP1430416	Bag for side panel, cpl. RH		1
16	SP1419695	Magnet key with bag		1
17	SPZ176015A	Nylon Bag		1
18	SPQ99991	Air Pump		1
19	SP1524591	Adapter Cable for Hour Counter		1



Yes, you can.®



---

**Belgium & Luxemburg:**

Invacare nv, Autobaan 22, B-8210 Loppem  
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11  
belgium@invacare.com / www.invacare.be

**Denmark:**

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby  
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01  
denmark@invacare.com / www.invacare.dk

**Deutschland:**

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny  
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66  
info@invacare-aquatec.com / www.invacare-aquatec.de

**European Distributor Organisation:**

Invacare, Kleiststraße 49, D-32457 Porta Westfalica  
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541  
edo@invacare.com / www.invacare.eu.com

**España:**

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)  
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20  
contactsp@invacare.com / www.invacare.es

**France:**

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes  
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24  
contactfr@invacare.com / www.invacare.fr

**Ireland:**

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland  
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085  
ireland@invacare.com / www.invacare.ie

**Italia:**

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)  
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34  
italia@invacare.com / www.invacare.it

**Nederland:**

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede  
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758  
nederland@invacare.com, csede@invacare.com / www.invacare.nl

**Norge:**

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo  
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01  
norway@invacare.com, island@invacare.com / www.invacare.no

**Österreich:**

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee  
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4  
info-austria@invacare.com / www.invacare.at

**Portugal:**

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio  
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39  
portugal@invacare.com / www.invacare.pt

**Sverige & Suomi:**

Invacare AB, Fagerstagatan 9, S-163 91 Spånga  
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08  
sweden@invacare.com, finland@invacare.com / www.invacare.se

**Switzerland:**

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil  
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81  
switzerland@invacare.com / www.invacare.ch

**United Kingdom:**

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ  
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201  
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220  
uk@invacare.com / www.invacare.co.uk