

# Spare Parts Catalogue

---

Version: PartList\_A10\_20131106a.xls EN





**Yes, you can.®**

## **General Information**

*For any order of spare parts the following information is required:*

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

*Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.*

*The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.*



Yes, you can.®

## TABLE OF CONTENTS

---


Power base .....	5
Mud Guard .....	6
Power base .....	7
Wheel ejector rear wheel .....	8
Damper .....	9
Anti tippers .....	10
Parking brake right .....	11
Parking brake left .....	12
Adapter for battery pack .....	13
Front wheels .....	14
Front wheel left Adventure A10 .....	15
Front wheel right Adventure A10 .....	16
Drive Wheel .....	17
Drive Wheel spare parts .....	18
Control unit .....	19
Control unit spare parts .....	20
Control unit accessories .....	21
Interface and charger .....	22
Interface fuses .....	23
Interface cables .....	24
Charger .....	25
Lights .....	26
External interface .....	27
Cable charging adapter Adventure A10 .....	28
Battery pack .....	29
Battery pack spare parts .....	30
Accessories .....	31
Standard seat .....	32
Function seat .....	33
Seat support functional seat .....	34
Seat support standard seat .....	35
Armrest .....	36
Footrest .....	37
Swivel arm accessories .....	38
Swivel arm spare parts .....	39
Lateral support .....	40
Calf support .....	41
Headrest .....	42
Push handels .....	43
Attendant control .....	44
Middle control .....	45
Tray table .....	46
Luggage bag .....	47
Curb climber .....	48
Mechanical seat angle tilt mechanism .....	49
Electrical seat angle tilt mechanism .....	50






Pos.	Part number	Description	Valid from → until	Qty
		Mud Guard		1
		Power base		1
		Wheel ejector rear wheel		1
		Damper		1
		Anti tippers		1
		Parking brake right		1
		Parking brake left		1
		Adapter for battery pack		1
		Front wheels		1

Alber Adventure |  
**Mud Guard**

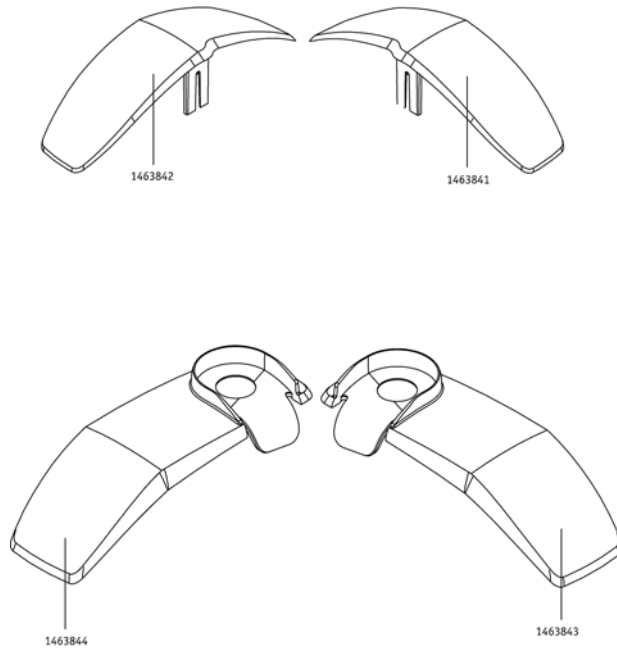


Pos.	Part number	Description	Valid from → until	Qty
	1463841	Mud guard rear right complete		1
	1463842	Mud guard rear left complete Adventure A10		1
	1463843	Mud guard front right complete		1
	1463844	Mud guard front left complete		1

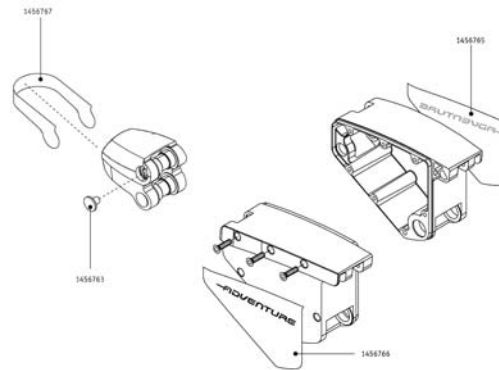

Pos.	Part number	Description	Valid from → until	Qty
	1456765	Sticker side piece right for Adventure A10		1
	1456766	Sticker side piece left for Adventure A10		1
	1456763	Cover plug for buckler		2
	1456767	Sticker rocker head		3

# Wheel ejector rear wheel

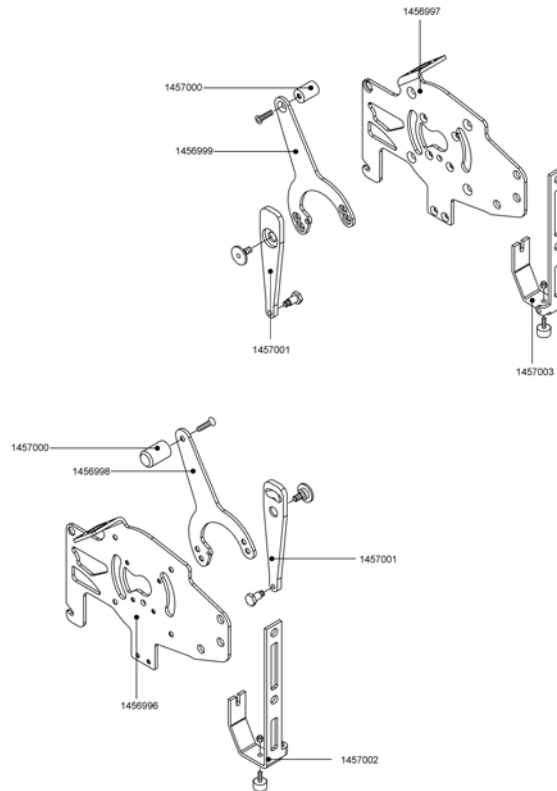


Pos.	Part number	Description	Valid from → until	Qty
	1456996	Adaptation plate for wheel ejector rear wheel left Adventure A10		1
	1456997	Adaptation plate for wheel ejector rear wheel right Adventure A10		1
	1456998	Actuating arm for wheel ejector rear wheel left Adventure A10		1
	1456999	Actuating arm for wheel ejector rear wheel right Adventure A10		1
	1457000	Actuating pin for wheel ejector rear wheel left and right Adventure A10		1
	1457001	Actuating arm for brake left and right		1
	1457002	Support plate for rear wheel left		1
	1457003	Support plate for rear wheel right		1






Pos.	Part number	Description	Valid from → until	Qty
	1457137	Spring template set Adventure A10		1

# Anti tippers

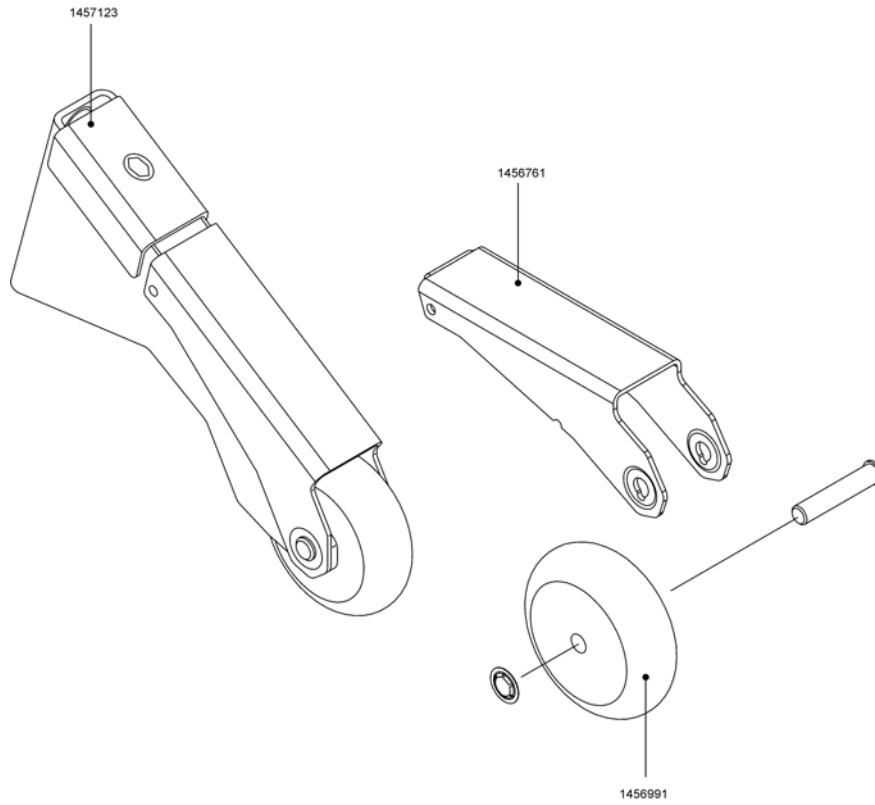


Pos.	Part number	Description	Valid from → until	Qty
	1456761	Jack-up device for anti-tippers Adventure A10		1
	1456991	Wheel for anti-tippers Adventure A10		1
	1457123	Anti-tippers Adventure A10 left and right		1

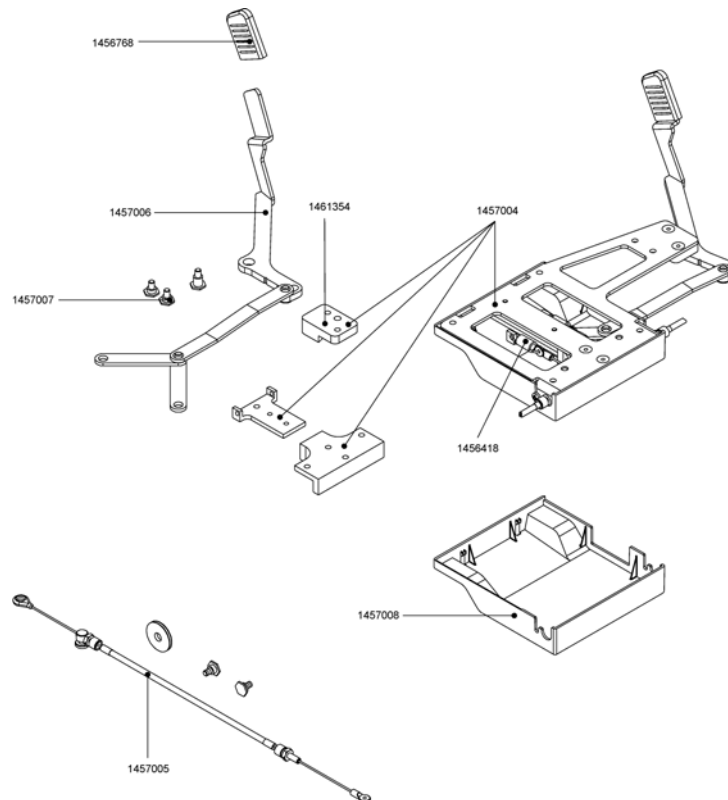**Parking brake right**


Pos.	Part number	Description	Valid from → until	Qty
	1456418	Tension spring brake Adventure A10		1
	1456768	Cap for brake lever for parking brake Adventure A10		1
	1457004	Base plate for parking brake right Adventure A10		1
	1457005	Bowden cable complete for parking brake Adventure A10		1
	1457006	Brake lever for parking brake right Adventure A10		1
	1457007	Bearing bolt set for brake lever right Adventure A10		1
	1457008	Cover for parking brake right Adventure A10		1
	1461354	Lever bearing right parking brake Adventure A10		1

# Parking brake left

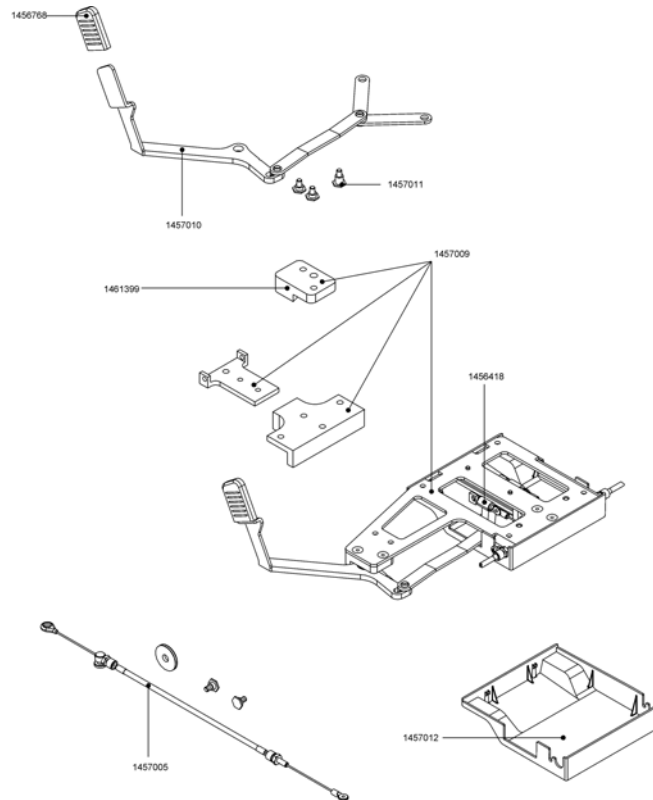


Pos.	Part number	Description	Valid from → until	Qty
	1456418	Tension spring brake Adventure A10		1
	1456768	Cap for brake lever for parking brake Adventure A10		1
	1457005	Bowden cable complete for parking brake Adventure A10		1
	1457009	Base plate for parking brake left Adventure A10		1
	1457010	Brake lever for parking brake left Adventure A10		1
	1457011	Bearing bolt set for brake lever left Adventure A10		1
	1457012	Cover for parking brake left Adventure A10		1
	1461399	Lever bearing left parking brake Adventure A10		1

# Adapter for battery pack

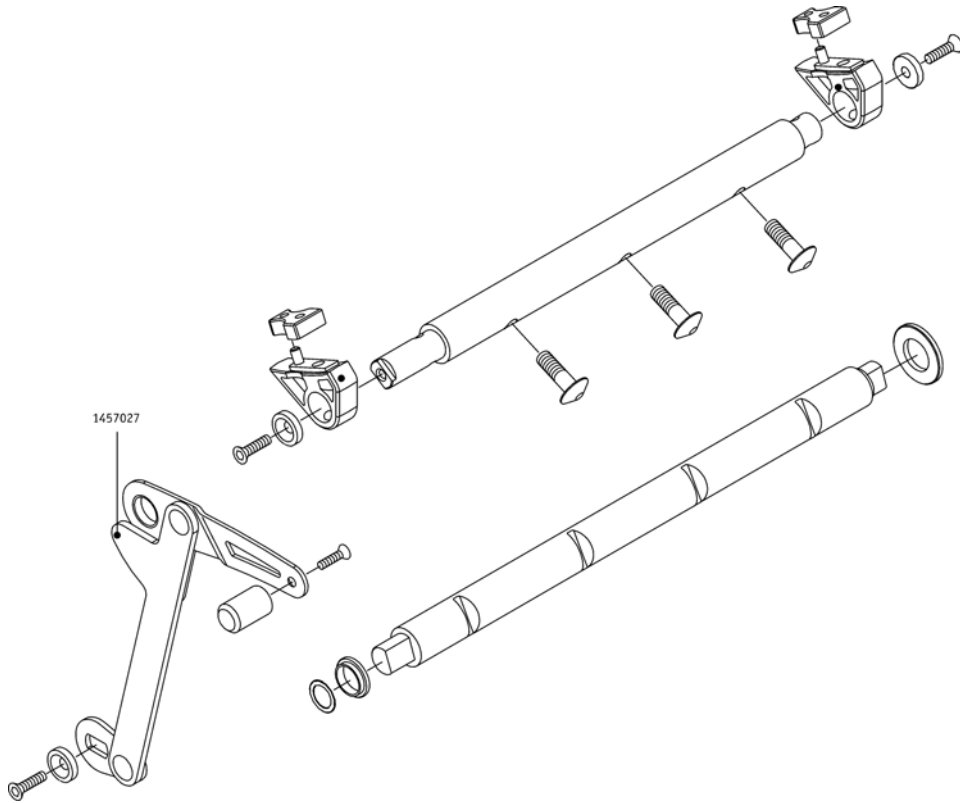


Pos.	Part number	Description	Valid from → until	Qty
	1457027	Locking lever for batteries		1

# Front wheels

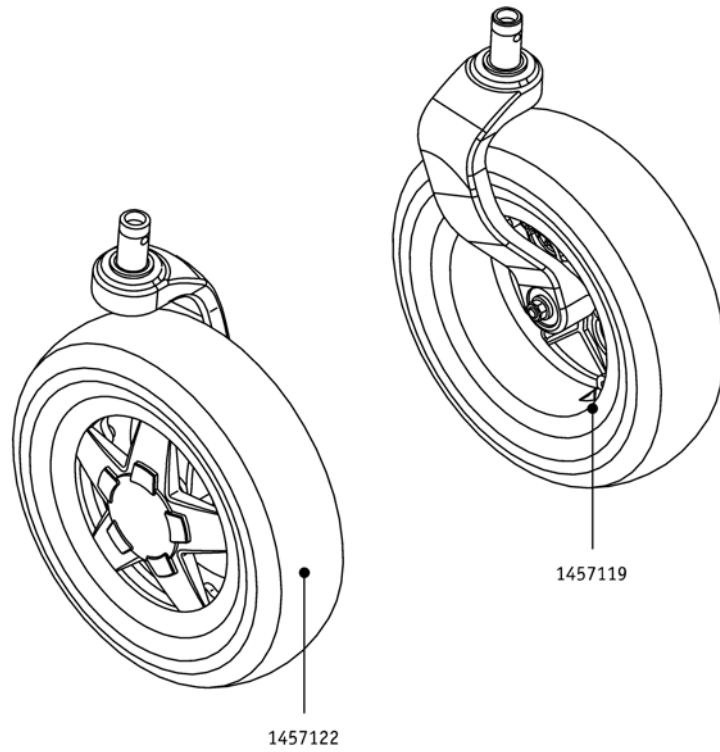


Pos.	Part number	Description	Valid from → until	Qty
		Front wheel left Adventure A10		1
		Front wheel right Adventure A10		1
	1457122	Front wheel left Adventure A10		1
	1457119	Front wheel right Adventure A10		1

# Front wheel left Adventure A10

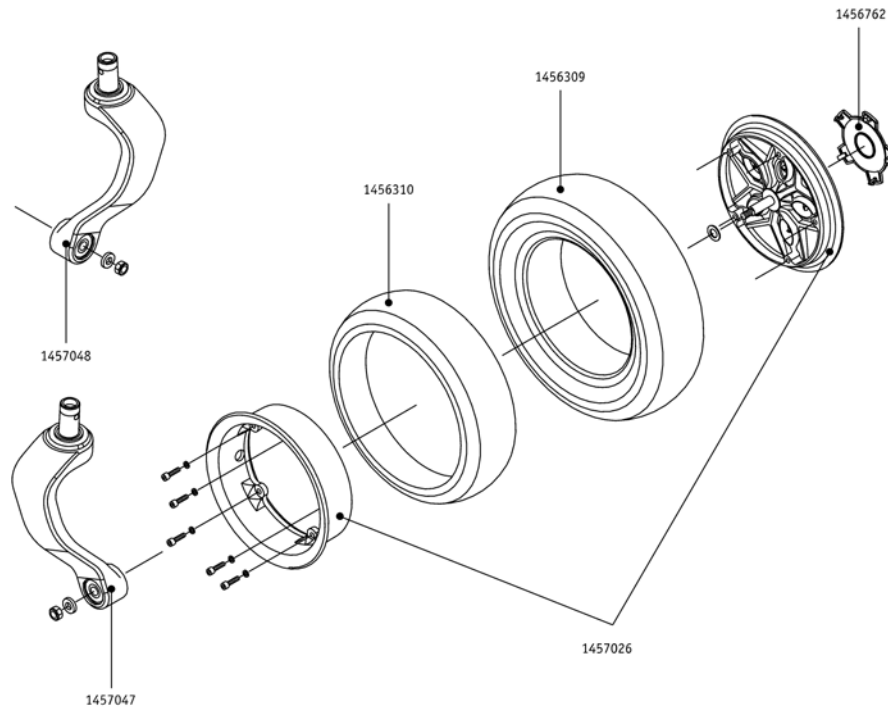

Pos.	Part number	Description	Valid from → until	Qty
	1457048	Fork for front wheel left Adventure A10		1
	1457047	Fork for front wheel right Adventure A10		1
	1457026	Rim front wheel Adventure A10 complete		1
	1456310	Front wheel inner tube Adventure A10 75/70-6		1
	1456309	Front wheel tyre Adventure A10 75/70-6		1
	1456762	Cover for front wheel Adventure A10		1

# Front wheel right Adventure A10

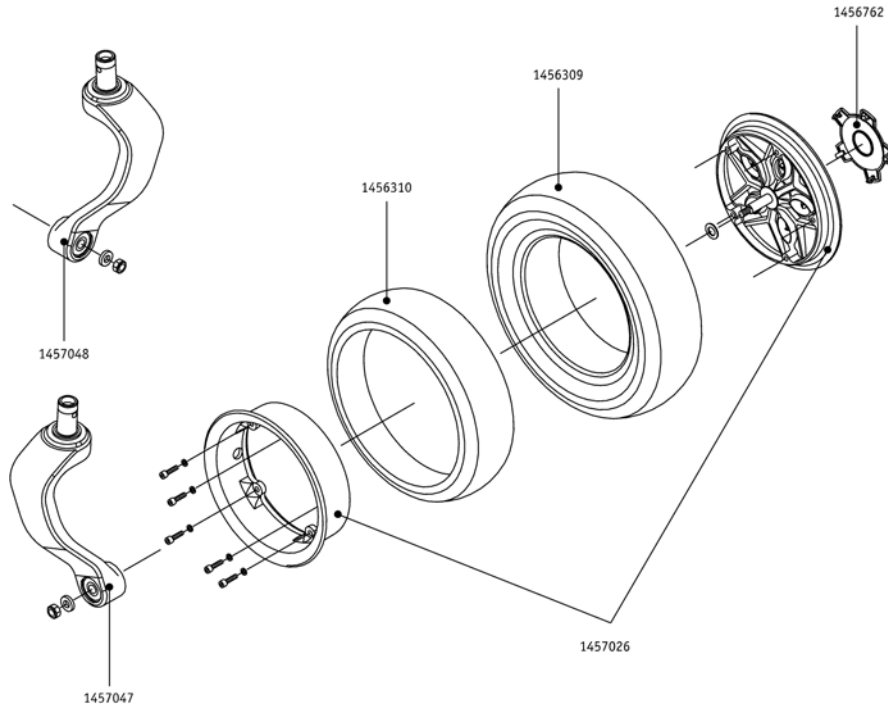


Pos.	Part number	Description	Valid from → until	Qty
	1457048	Fork for front wheel left Adventure A10		1
	1457047	Fork for front wheel right Adventure A10		1
	1457026	Rim front wheel Adventure A10 complete		1
	1456310	Front wheel inner tube Adventure A10 75/70-6		1
	1456309	Front wheel tyre Adventure A10 75/70-6		1
	1456762	Cover for front wheel Adventure A10		1





Pos.	Part number	Description	Valid from → until	Qty
		Drive Wheel spare parts		1
	1457062	Drive unit complete Adventure A10 6 km/h version		1
	1457063	Drive unit complete Adventure A10 10 km/h version		1
	1457064	Drive unit complete Adventure A10 12 km/h version		1

# Drive Wheel spare parts

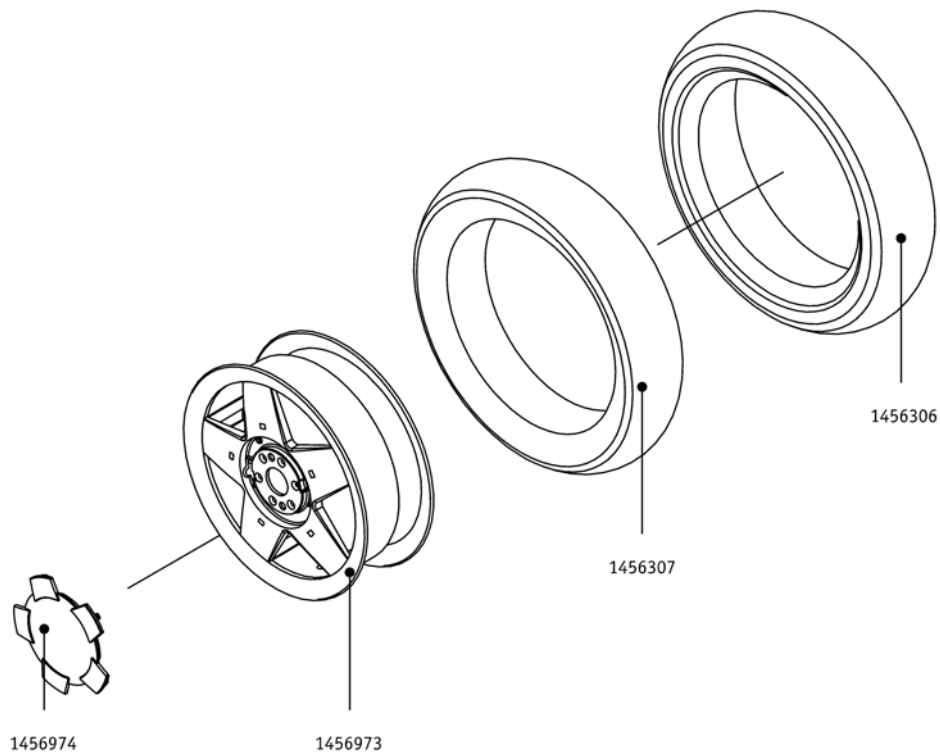


Pos.	Part number	Description	Valid from → until	Qty
	1456974	Cover for rear wheel Adventure A10		1
	1456973	Rear wheel rim complete Adventure A10		1
	1456307	Rear wheel inner tube Adventure A10 90/70-10		1
	1456306	Rear wheel tyre Adventure A10 90/70-10		1




Pos.	Part number	Description	Valid from → until	Qty
		Control unit spare parts		1
		Control unit accessories		1
	1457071	Control unit (with spiral cable) Adventure A10		1

# Control unit spare parts

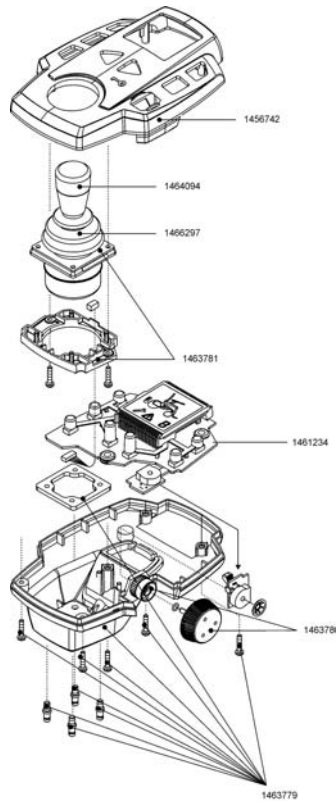

Pos.	Part number	Description	Valid from → until	Qty
	1456742	Housing control unit upper part Adventure A10		1
	1461234	Control unit electronics Adventure A10 complete		1
	1463779	Housing control unit lower part Adventure A10		1
	1463780	Adjusting wheel control unit Adventure A10		1
	1463781	Joystick for control unit Adventure A10		1
	1464094	Joystick knob conic		1
	1466297	Gaiter for joystick Adventure A10		1

## Control unit accessories

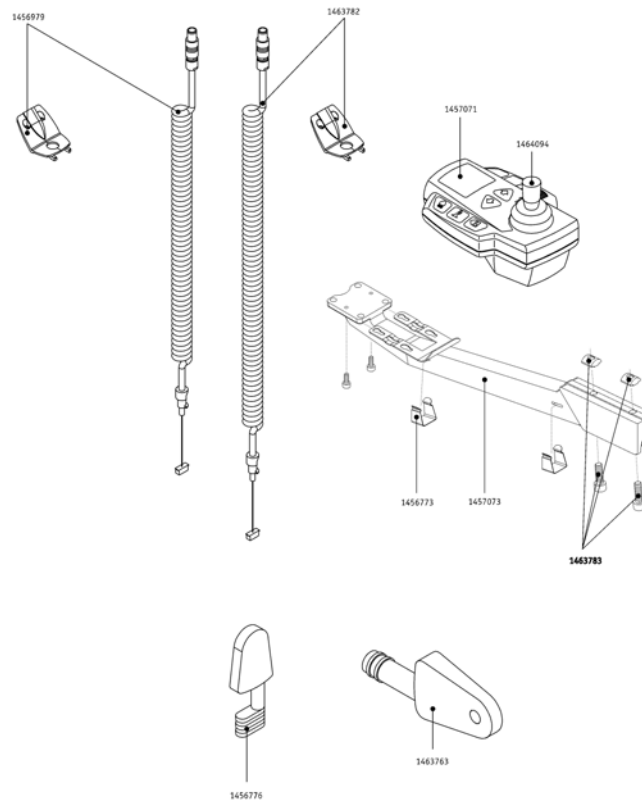


Pos.	Part number	Description	Valid from → until	Qty
	1456773	Control unit cable clamp for Adventure A10		1
	1456776	Immobiliser key		1
	1456979	Spiral cable and service window for control unit		1
	1457071	Control unit (with spiral cable) Adventure A10		1
	1457073	Bracket for control unit Adventure A10		1
	1463763	Programming key for Adventure A10		1
	1463782	Spiral cable (long version) and service window for control unit		1
	1463783	Sliding blocks and screws for control unit bracket Adventure A10		1
	1464094	Joystick knob conic		1

## Interface and charger




Pos.	Part number	Description	Valid from → until	Qty
		Interface fuses		1
		Interface cables		1
		Charger		1
		Lights		1
		External interface		1
		Cable charging adapter Adventure A10		1

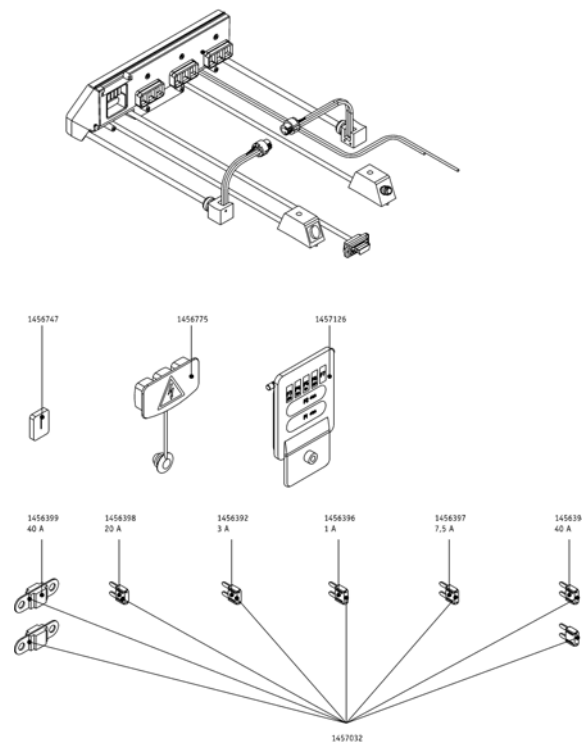


Pos.	Part number	Description	Valid from → until	Qty
	1456392	Fuse 3A flat		1
	1456394	Fuse Maxi fuse 40 A		1
	1456396	Fuse 1 A		1
	1456397	Fuse 7.5 A		1
	1456398	Fuse 20 A		1
	1456399	Fuse 40 A		1
	1456775	Contact protection electronic box Adventure		1
	1457032	Fuse set Adventure		1
	1457126	Fuse cover interface Adventure pre-assembled		1

# Interface cables

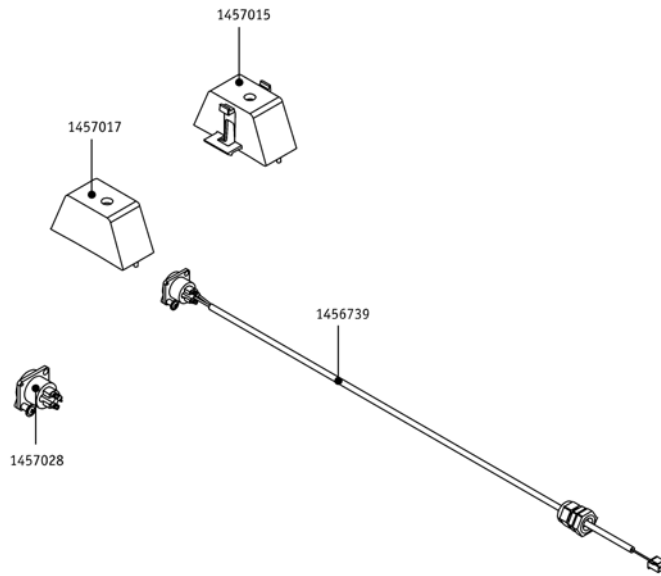


Pos.	Part number	Description	Valid from → until	Qty
	1456739	Repair set Adventure for connecting cable for charger socket		1
	1457015	Housing for control unit socket Adventure		1
	1457017	Housing for charger socket Adventure		1
	1457028	XLR socket for charger socket		1



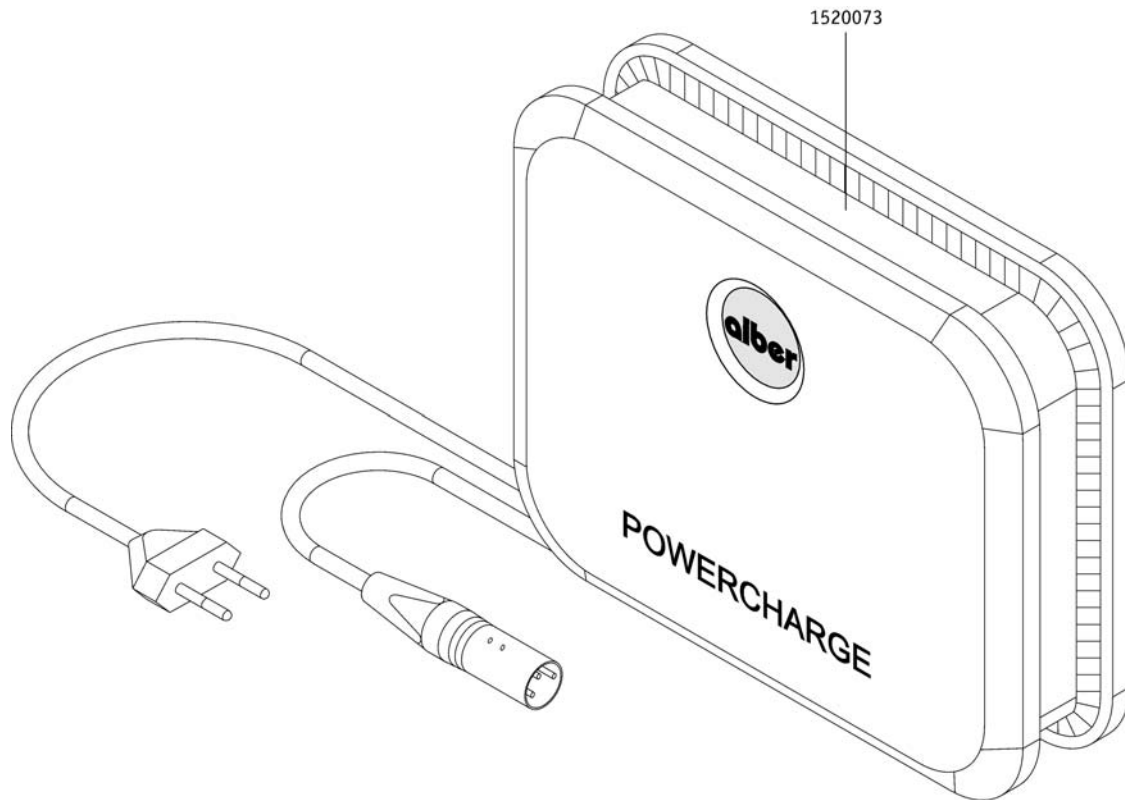


Pos.	Part number	Description	Valid from → until	Qty
	1520073	Charger A10 with main cable EU		1

# Lights

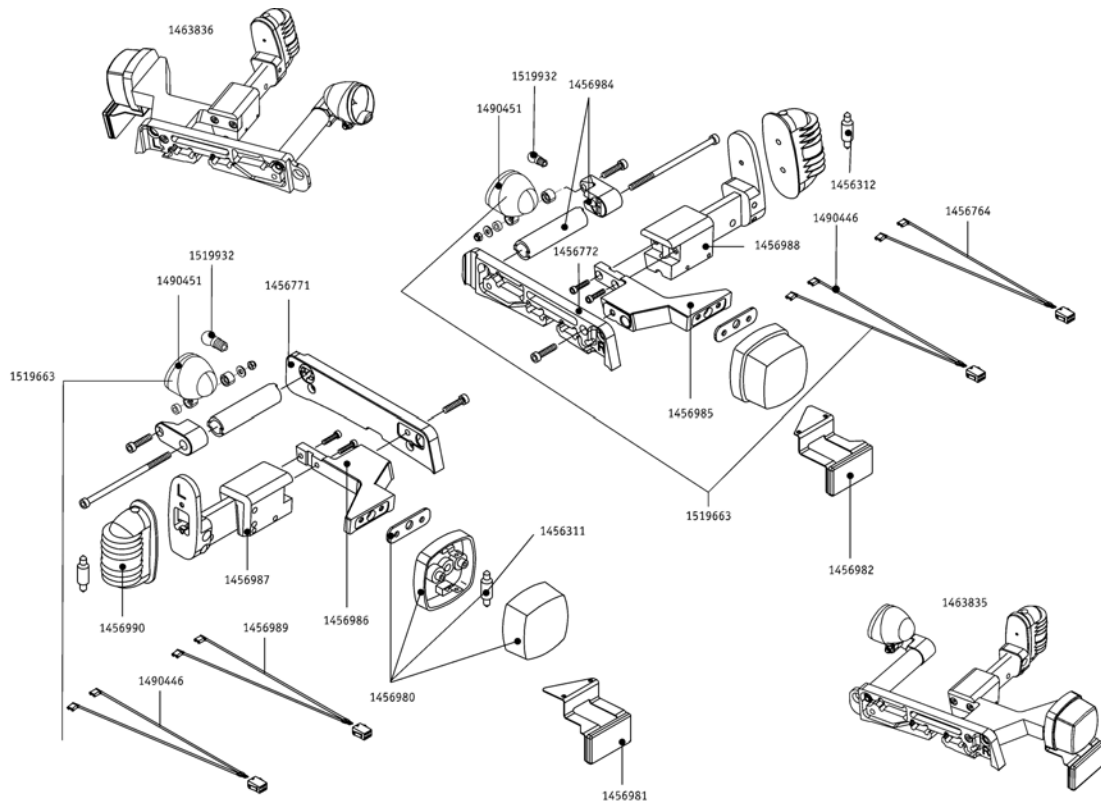


Pos.	Part number	Description	Valid from → until	Qty
	1456311	Rear light bulb Adventure A10		1
	1456312	Indicator light bulb Adventure A10		1
	1456764	Cable for tail-light Adventure A10 left and right		1
	1456771	Base plate Adventure A10 lighting left		1
	1456772	Base plate Adventure A10 lighting right		1
	1456980	Rear light left/right Adventure A10		1
	1456981	Reflector left Adventure A10		1
	1456982	Reflector right Adventure A10		1
	1456984	Adapter for headlamp Adventure A10 left and right		1
	1456985	Holder for tail-light right Adventure A10		1
	1456986	Holder for tail-light left Adventure A10		1
	1456987	Indicator light holder left Adventure A10		1
	1456988	Indicator light holder right Adventure A10		1
	1456989	Indicator cable Adventure A10		1
	1456990	Indicator left/right Adventure A10		1
	1490451	Headlight MICRO-FF 6V-2,4W		1
	1519663	Headlight replacement kit Micro FF Adventure A10		1
	1519932	Headlamp light bulb Micro FF Adventure A10		2

## External interface

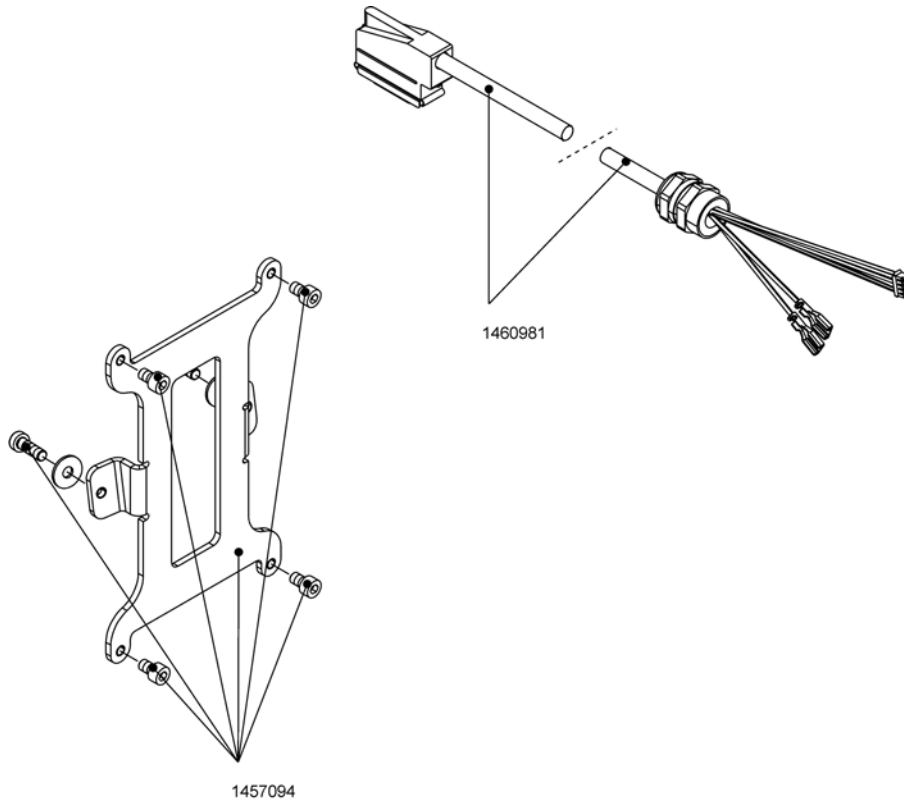

Pos.	Part number	Description	Valid from → until	Qty
	1457094	Fixation for external interface for Adventure A10		1
	1460981	Connecting cable to external interface Adventure A10		1

# Cable charging adapter Adventure A10

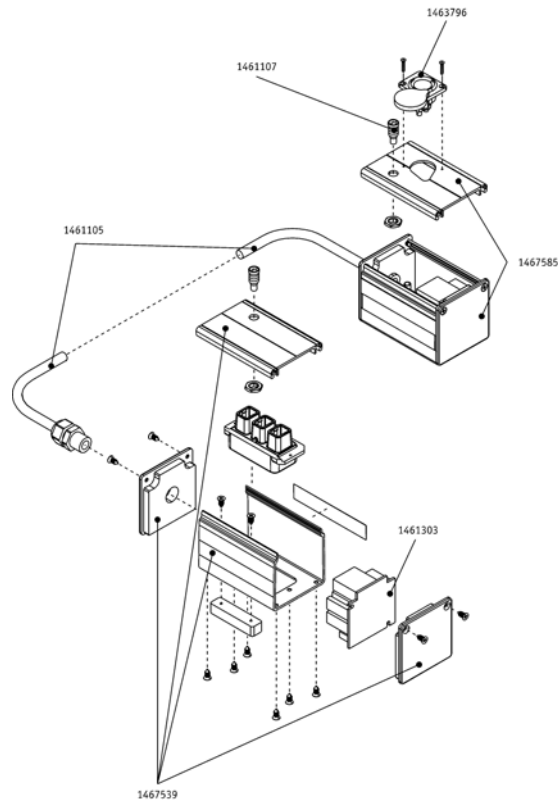


Pos.	Part number	Description	Valid from → until	Qty
	1461105	Cable charging adapter Adventure A10		1
	1461107	Cable confection LED for charging adapter Adventure A10		1
	1461303	Circuit board for charging adapter Adventure A10		1
	1463796	Charger connection for charging adapter Adventure A10		1
	1467539	Housing components for charging adapter Adventure A10		1
	1467585	Housing part with plug for off-board charger Adventure A10		1



Pos.	Part number	Description	Valid from → until	Qty
	1457070	Battery pack complete for Adventure A10		1
		Battery pack spare parts		1

# Battery pack spare parts

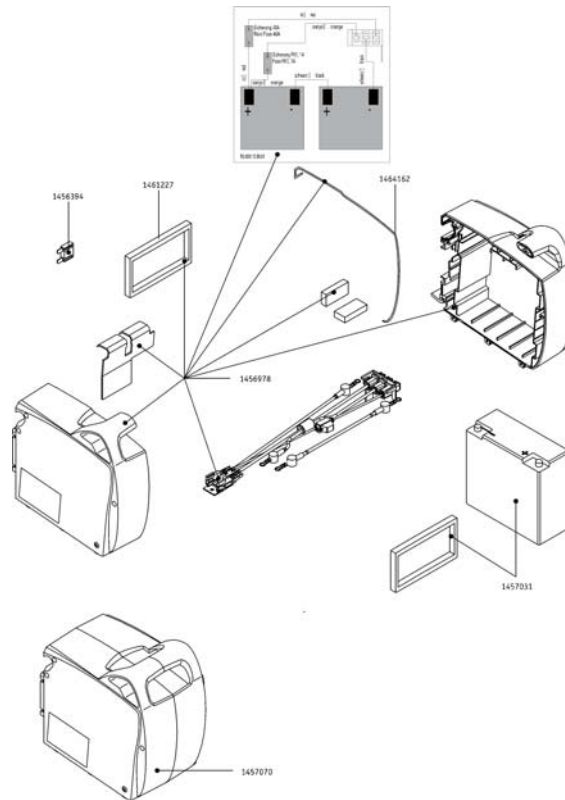


Pos.	Part number	Description	Valid from → until	Qty
	1456394	Fuse Maxi fuse 40 A		1
	1456978	Housing battery pack set for Adventure		1
	1457031	Battery lead-gel for Adventure A10		1
	1457070	Battery pack complete for Adventure A10		1
	1461227	Seal for front battery pack		1
	1464162	Sealing cord D 2,0mm 500mm long		1




Pos.	Part number	Description	Valid from → until	Qty
		Standard seat		1
		Function seat		1
		Seat support functional seat		1
		Seat support standard seat		1
		Armrest		1
		Footrest		1
		Swivel arm accessories		1
		Swivel arm spare parts		1
		Lateral support		1
		Calf support		1
		Headrest		1
		Push handels		1
		Attendant control		1
		Middle control		1
		Tray table		1
		Luggage bag		1
		Curb climber		1
		Mechanical seat angle tilt mechanism		1
		Electrical seat angle tilt mechanism		1

# Standard seat

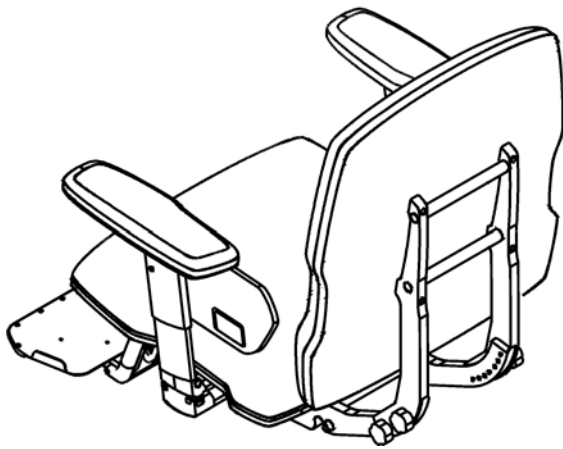


1457033

1457034

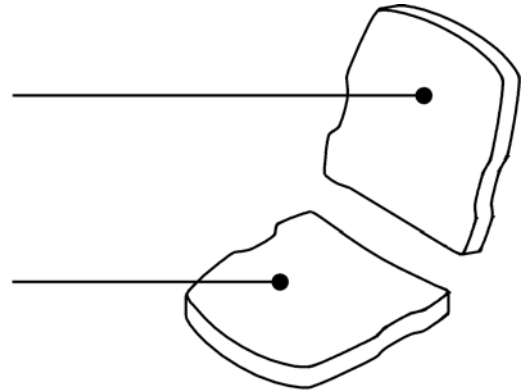


Pos.	Part number	Description	Valid from → until	Qty
	1457033	Backrest plate with cushion for standard seat Adventure A10		1
	1457034	Seat plate with cushion for standard seat Adventure A10		1



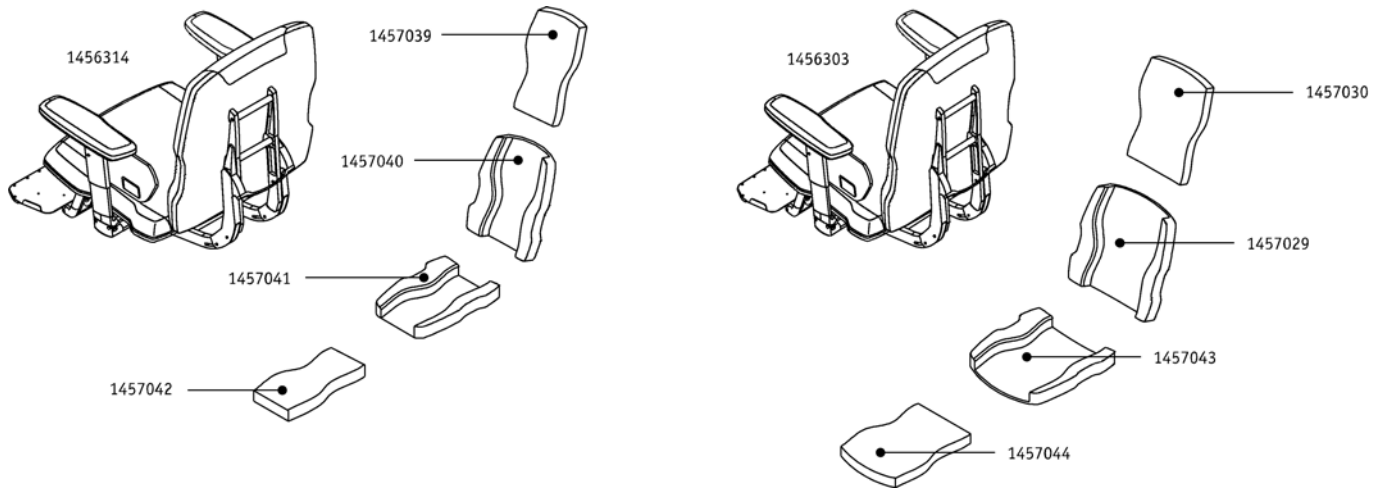


Pos.	Part number	Description	Valid from → until	Qty
	1456303	Function seat with contour cushion Adventure A10 44-52cm (wide)		1
	1456314	Function seat with contour cushion for Adventure A10 38-44cm (narrow version)		1
	1457029	Backrest plate wide for contour cushion for function seat Adventure A10		1
	1457030	Contour cushion for backrest plate for function seat (wide) Adventure A10		1
	1457039	Contour cushion for backrest plate for function seat (narrow) Adventure A10		1
	1457040	Backrest plate narrow for contour cushion for function seat Adventure A10		1
	1457041	Seat plate narrow for contour cushion for function seat Adventure A10		1
	1457042	Contour cushion for seat plate for function seat (narrow) Adventure A10		1
	1457043	Seat plate wide for contour cushion for function seat Adventure A10		1
	1457044	Contour cushion for seat plate for function seat (wide) Adventure A10		1

# Seat support functional seat

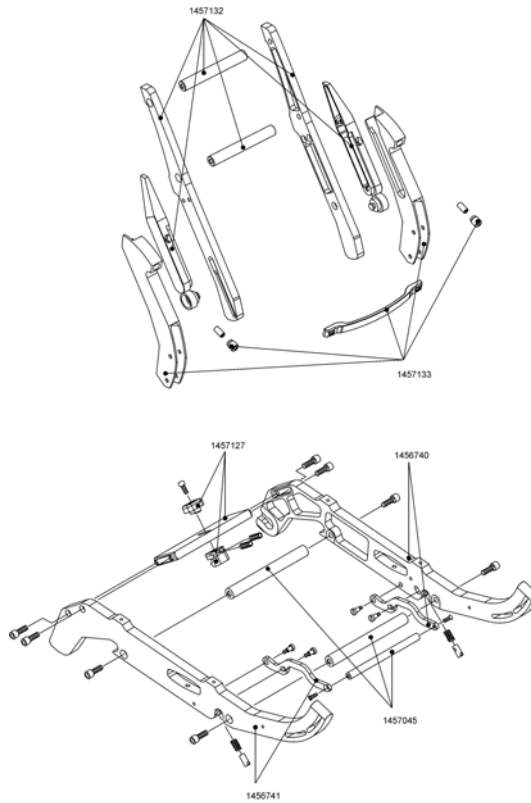


Pos.	Part number	Description	Valid from → until	Qty
	1456740	Seat support right for function seat Adventure A10		1
	1456741	Seat support left for function seat Adventure A10		1
	1457045	Assembly set for seat holders for function seat Adventure A10		1
	1457127	Limit stop for footrest for seats Adventure A10		1
	1457132	Support for backrest for function seat Adventure A10		1
	1457133	Holder for backrest for function seat Adventure A10		1

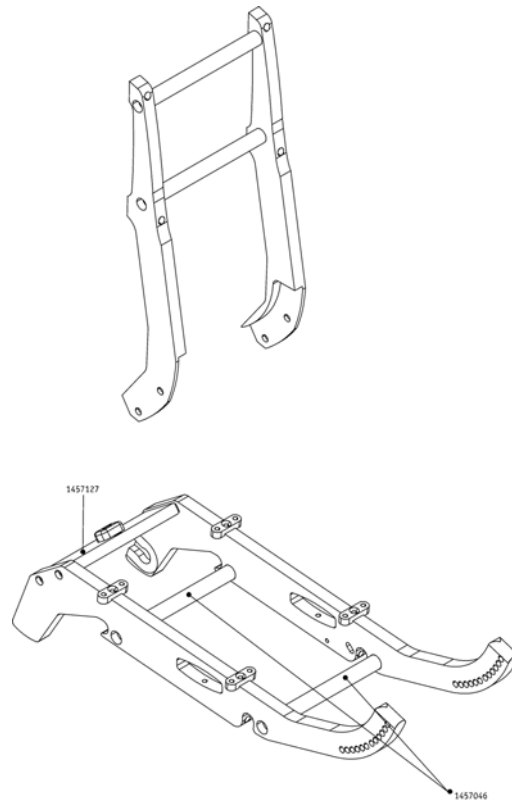
**Seat support standard seat**

Pos.	Part number	Description	Valid from → until	Qty
	1456736	Seat support right for standard seat Adventure A10		1
	1456737	Seat support left for standard seat Adventure A10		1
	1457046	Assembly set for seat holders for standard seat Adventure A10		1
	1457067	Support for backrest for standard seat Adventure A10		1
	1457127	Limit stop for footrest for seats Adventure A10		1
	1489297	Screw		1

# Armrest

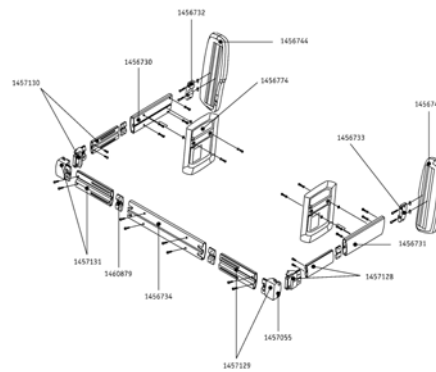


Pos.	Part number	Description	Valid from → until	Qty
	1456730	Holder for armrest right for seats Adventure A10		1
	1456731	Holder for armrest left for seats Adventure A10		1
	1456732	Fixation module right for armrest for seats Adventure A10		1
	1456733	Fixation module left for armrest for seats Adventure A10		1
	1456734	Basic support for armrest for seats Adventure A10		1
	1456743	Arm support left for seats Adventure A10		1
	1456744	Arm support right for seats Adventure A10		1
	1456774	Clothes protection left and right Adventure A10		1
	1457055	Lock-out release for armrest for function seat Adventure A10		1
	1457128	Vertical attachment for armrest left for function seat Adventure A10		1
	1457129	Horizontal attachment for armrest left for function seat Adventure A10		1
	1457130	Vertical attachment for armrest right for function seat Adventure A10		1
	1457131	Horizontal attachment for armrest right for function seat Adventure A10		1

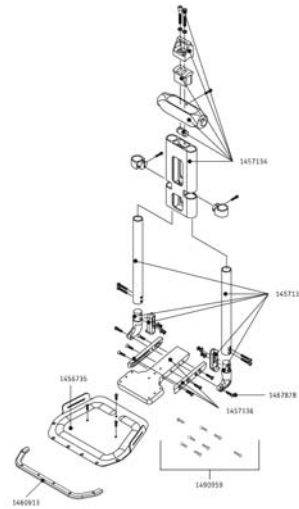


Pos.	Part number	Description	Valid from → until	Qty
	1456735	Footrest for function seat/standard seat Adventure A10		1
	1457068	Adaptation for legrest standard seat Adventure A10		1
	1457069	Lower leg tube for legrest standard seat Adventure A10		1
	1457134	Adaptation for legrest for function seat Adventure A10		1
	1457135	Lower leg tube for legrest function seat Adventure A10		1
	1457136	Adaptation set for footrest Adventure A10		1
	1460913	Protection footrest		1
	1467878	Flat headed screw DIN 923 - M 6 x 12		1
	1490959	Fixation set for footrest Adventure A10		1

## Swivel arm accessories

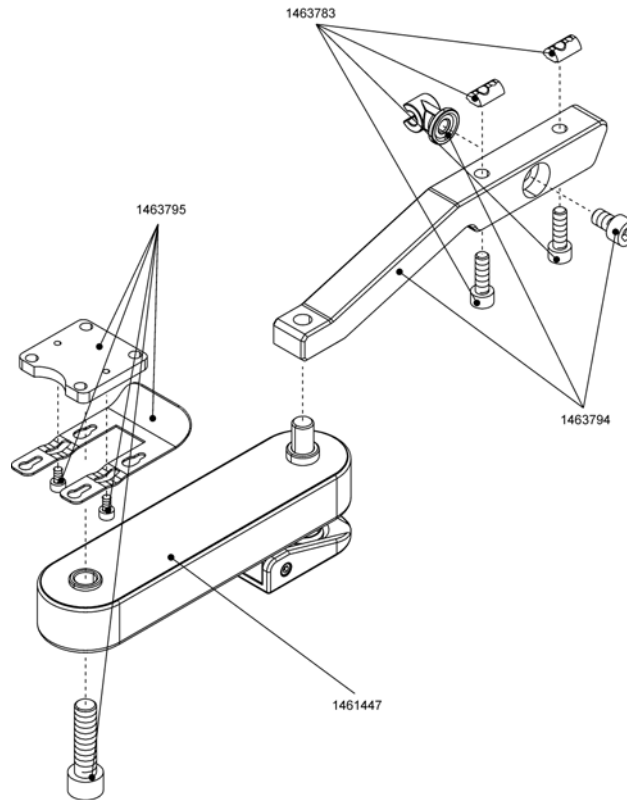


Pos.	Part number	Description	Valid from → until	Qty
	1461447	Swivel arm Adventure A10		1
	1463783	Sliding blocks and screws for control unit bracket Adventure A10		1
	1463794	Bracket for swivel arm Adventure A10		1
	1463795	Bracket for control unit for swivel Adventure A10		1

## Swivel arm spare parts

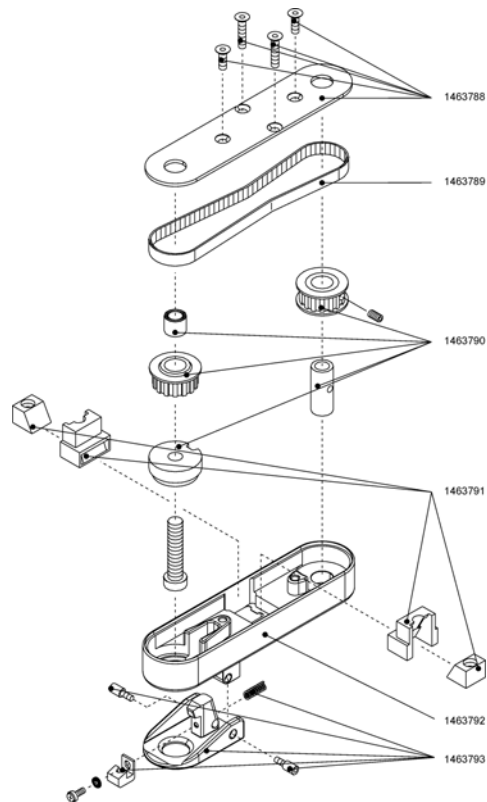


Pos.	Part number	Description	Valid from → until	Qty
	1463788	Covering plate for swivel arm Adventure A10		1
	1463789	Toothed belt for swivel arm Adventure A10		1
	1463790	Belt washer for swivel arm Adventure A10		1
	1463791	Clamping mechanism for swivel arm Adventure A10		1
	1463792	Housing for swivel arm Adventure A10		1
	1463793	Control plate for swivel arm Adventure A10		1

# Lateral support

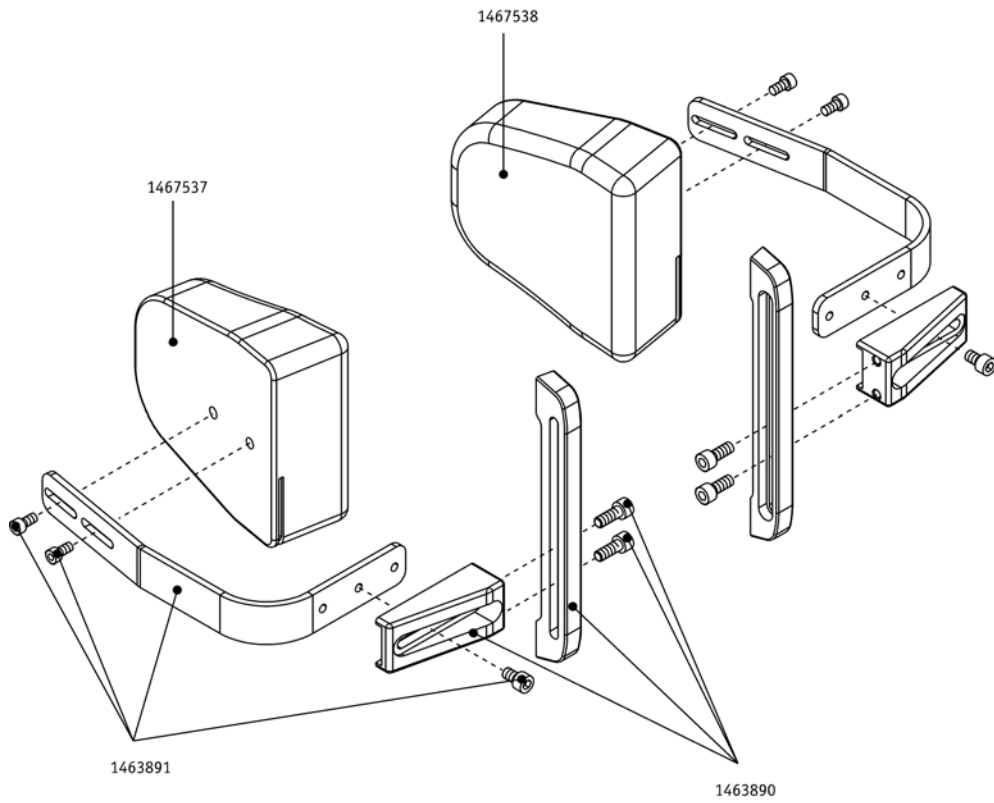


Pos.	Part number	Description	Valid from → until	Qty
	1463890	Adapter for lateral support Adventure A10		1
	1463891	Holder for lateral support Adventure A10		1
	1467537	Lateral support left Adventure A10		1
	1467538	Lateral support right Adventure A10		1



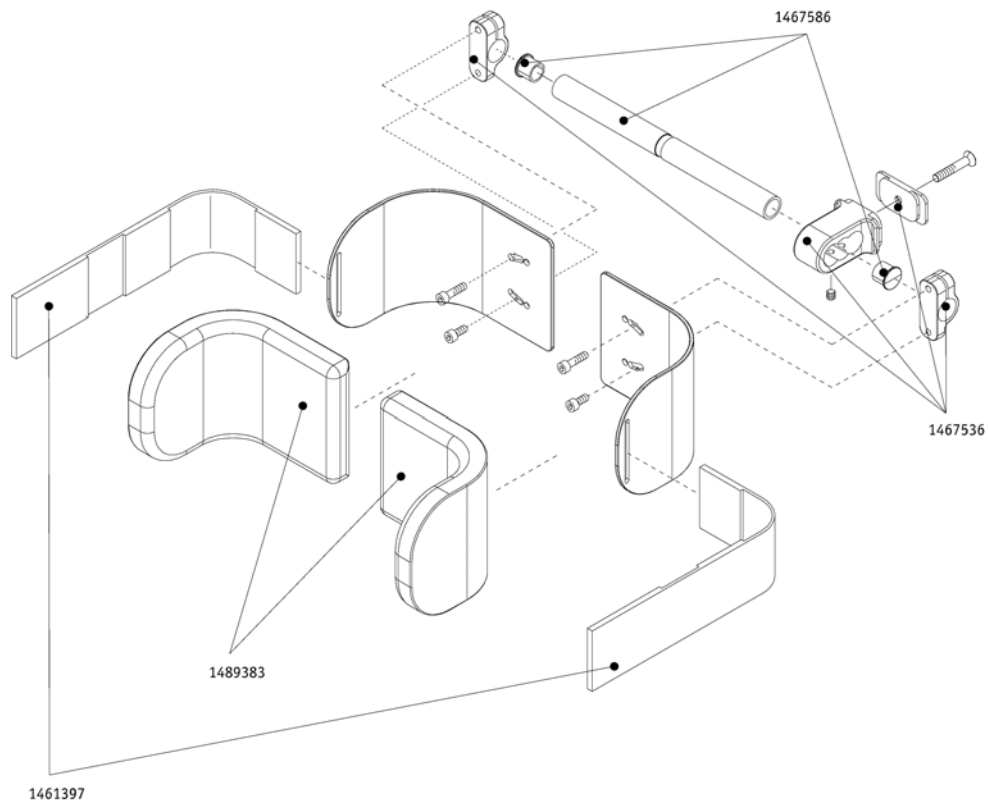


Pos.	Part number	Description	Valid from → until	Qty
	1461397	Foot belt Adventure A10		1
	1467536	Holder for calf support Adventure A10		1
	1467586	Tube for calf support Adventure A10		1
	1489383	Cushions for calf support Adventure A10		1

# Headrest

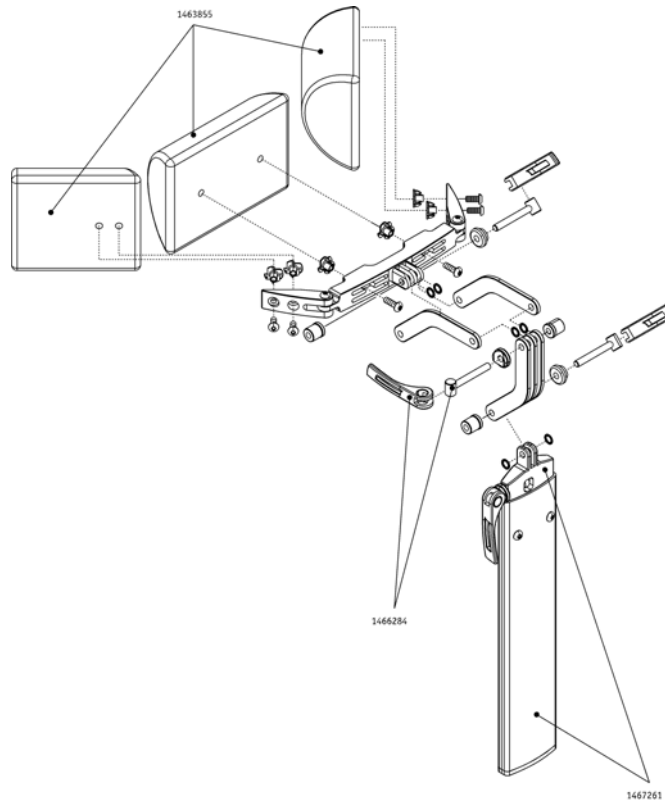


Pos.	Part number	Description	Valid from → until	Qty
	1463855	Pad set for headrest Adventure A10		1
	1466284	Bolt lever M 6 x 35 for headrest Adventure A10		1
	1467261	Profile for headrest Adventure A10		1

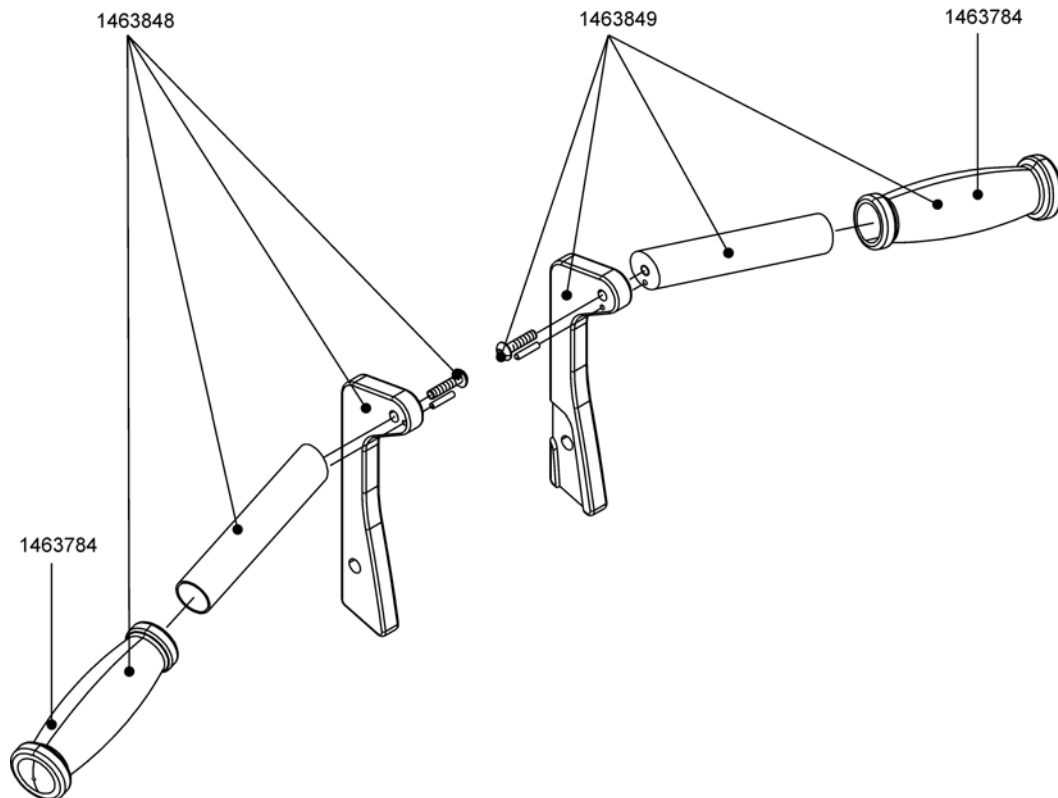


Pos.	Part number	Description	Valid from → until	Qty
	1463784	Handle cover for Adventure A10		1
	1463848	Push handle Adventure A10 left		1
	1463849	Push handle Adventure A10 right		1

# Attendant control

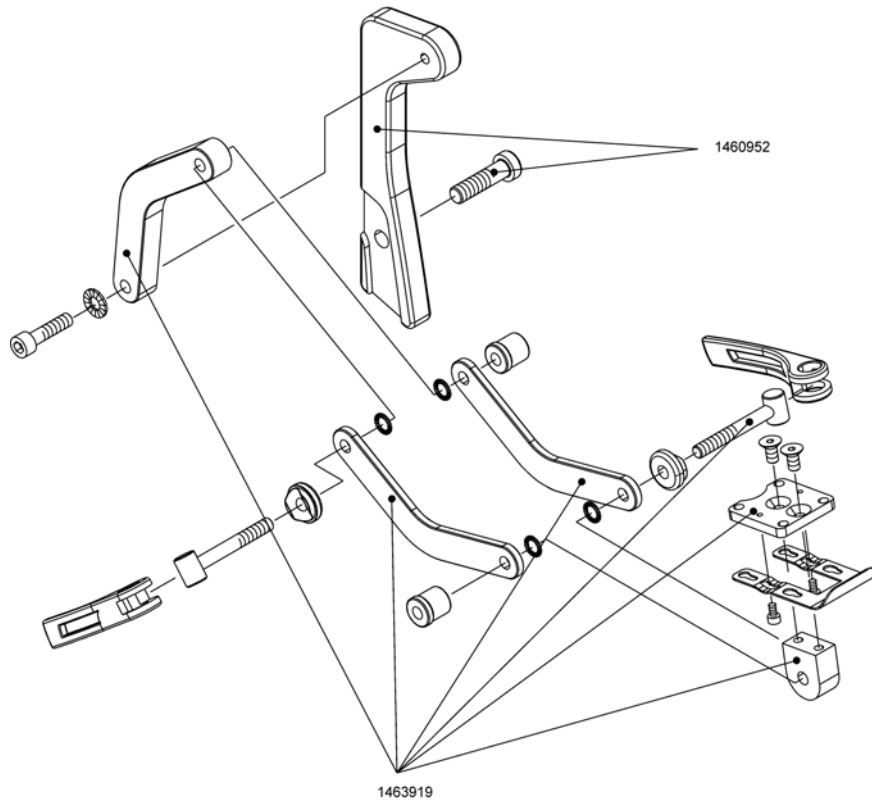


Pos.	Part number	Description	Valid from → until	Qty
	1460952	Bracket for attendant control Adventure A10		1
	1463919	Joint for attendant control Adventure A10		1

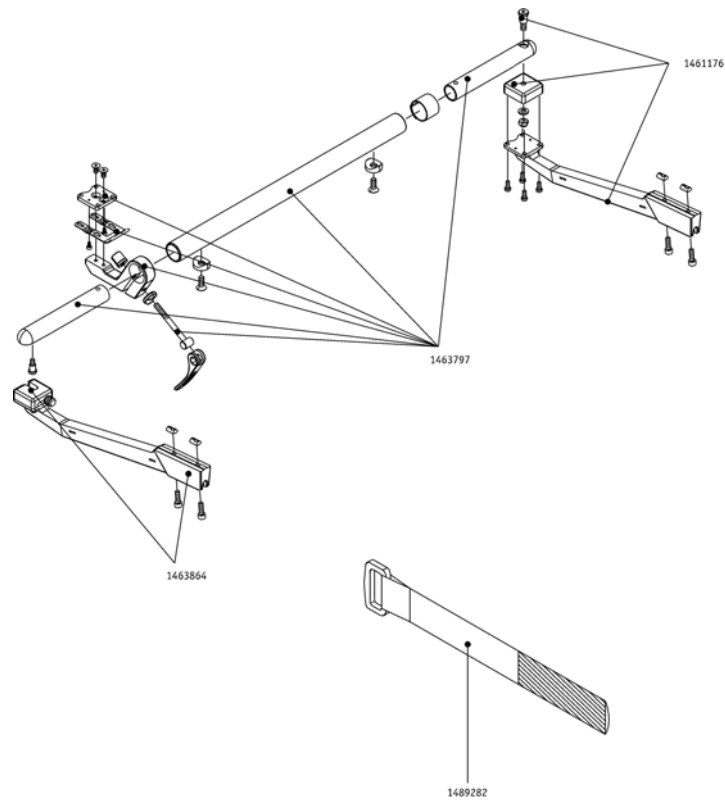


Pos.	Part number	Description	Valid from → until	Qty
	1461176	Bracket for middle control Adventure A10		1
	1463797	Adaptation tube for middle control unit for Adventure A10		1
	1463864	Bracket for middle control Adventure A10		1
	1489282	Strap 150 mm, Black		1

# Tray table

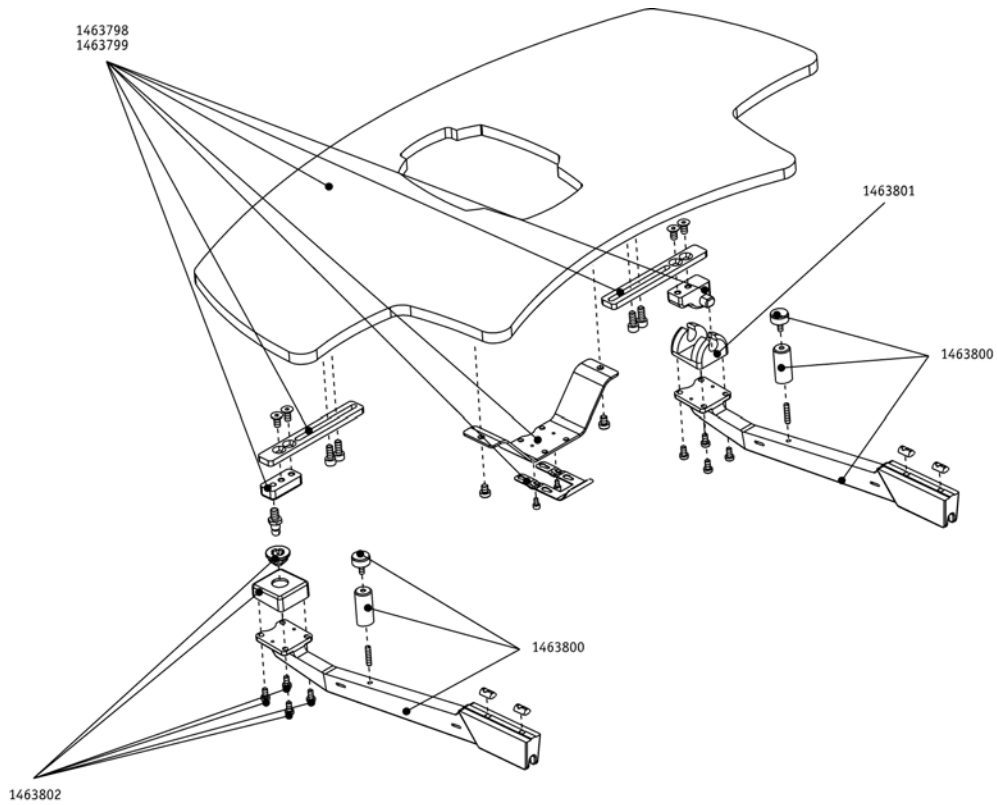


Pos.	Part number	Description	Valid from → until	Qty
	1463798	Plate for tray table small Adventure A10		1
	1463799	Plate for tray table wide Adventure A10		1
	1463800	Bracket for tray table Adventure A10		1
	1463801	Joint for tray table attachment Adventure A10		1
	1463802	Coupling for tray table Adventure A10		1

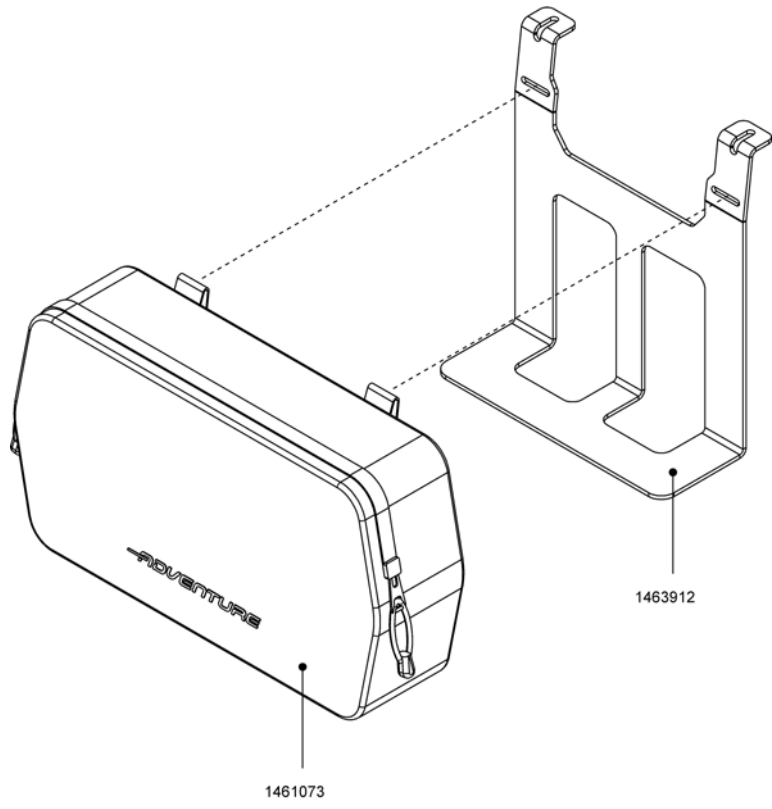

Pos.	Part number	Description	Valid from → until	Qty
	1461073	Luggage bag for Adventure A10		1
	1463912	Adaptation plate for luggage bag Adventure A10 incl. velcro strap		1


Alber Adventure |  
**Curb climber**

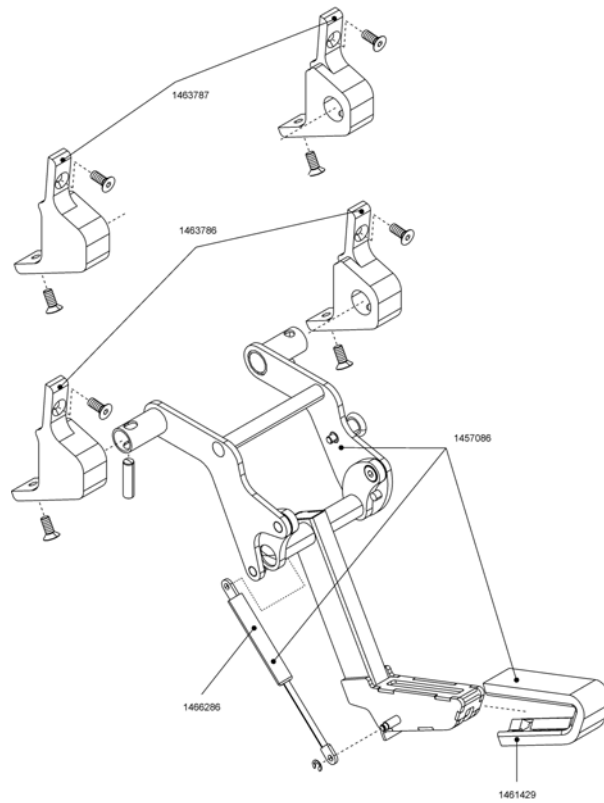

Pos.	Part number	Description	Valid from → until	Qty
	1457086	Curb climber Adventure A10 complete pre-assembled		1
	1461429	Profile rubber for curb climber Adventure A10		1
	1463786	Adapter set for curb climber for function seat Adventure A10		1
	1463787	Adapter set for curb climber for standard seat Adventure A10		1
	1466286	Gas spring for curb climber Adventure A10		1




## Mechanical seat angle tilt mechanism

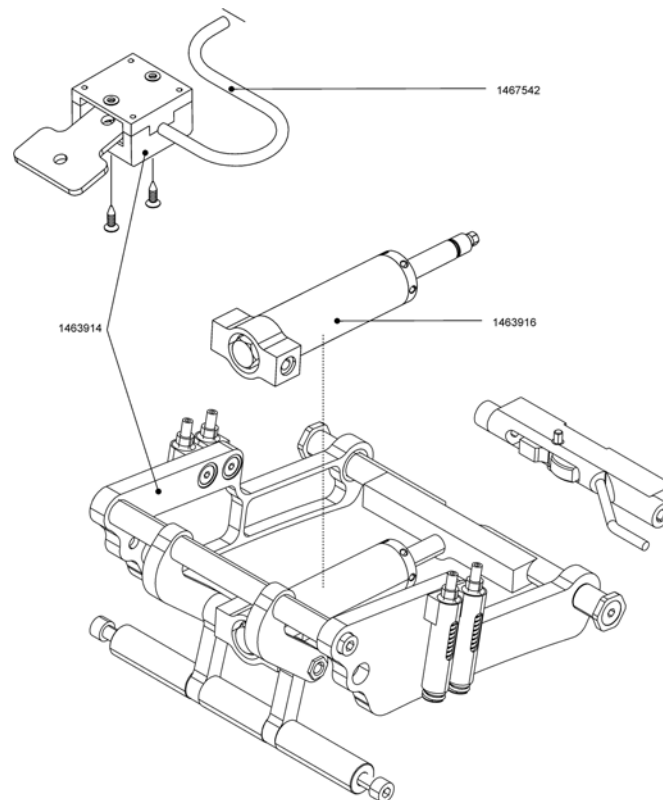

Pos.	Part number	Description	Valid from → until	Qty
	1463914	Mechanism for mechanical seat angle tilt Adventure A10		1
	1463916	Adjusting element for mechanical seat tilt Adventure A10		1
	1467542	Bowden cable for mechanical seat angle tilt Adventure A10		1

# Electrical seat angle tilt mechanism

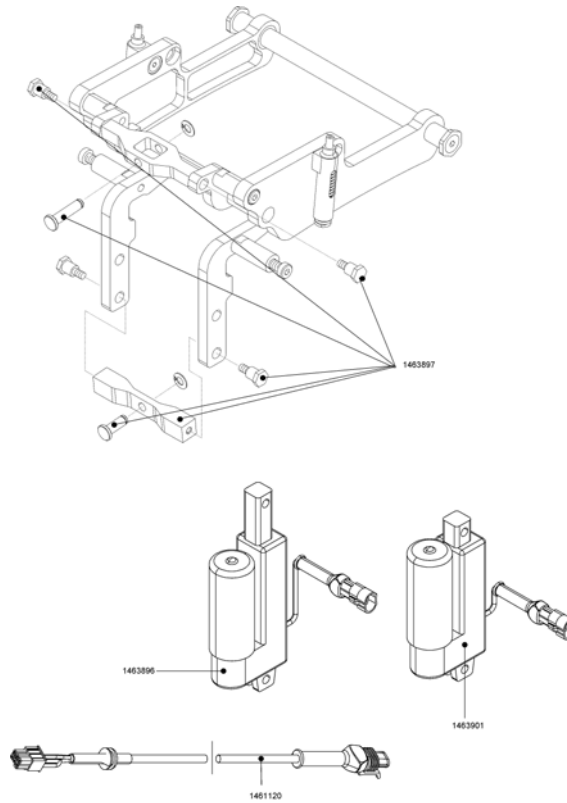

Pos.	Part number	Description	Valid from → until	Qty
	1461120	Cable confection electrical seat angle adjustment		1
	1463896	Drive unit complete for electrical seat angle tilt Adventure A10		1
	1463897	Electrical seat angle tilt mechanism		1
	1463901	Drive unit complete for electrical seat angle tilt Adventure A10 (CC-version)		1





Yes, you can.®



---

**Belgium & Luxemburg:**

Invacare nv, Autobaan 22, B-8210 Loppem  
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11  
belgium@invacare.com / www.invacare.be

**Denmark:**

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby  
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01  
denmark@invacare.com / www.invacare.dk

**Deutschland:**

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny  
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66  
info@invacare-aquatec.com / www.invacare-aquatec.de

**European Distributor Organisation:**

Invacare, Kleiststraße 49, D-32457 Porta Westfalica  
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541  
edo@invacare.com / www.invacare.eu.com

**España:**

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)  
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20  
contactsp@invacare.com / www.invacare.es

**France:**

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes  
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24  
contactfr@invacare.com / www.invacare.fr

**Ireland:**

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland  
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085  
ireland@invacare.com / www.invacare.ie

**Italia:**

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)  
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34  
italia@invacare.com / www.invacare.it

**Nederland:**

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede  
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758  
nederland@invacare.com, csede@invacare.com / www.invacare.nl

**Norge:**

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo  
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01  
norway@invacare.com, island@invacare.com / www.invacare.no

**Österreich:**

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee  
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4  
info-austria@invacare.com / www.invacare.at

**Portugal:**

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio  
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39  
portugal@invacare.com / www.invacare.pt

**Sverige & Suomi:**

Invacare AB, Fagerstagatan 9, S-163 91 Spånå  
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08  
sweden@invacare.com, finland@invacare.com / www.invacare.se

**Switzerland:**

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil  
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81  
switzerland@invacare.com / www.invacare.ch

**United Kingdom:**

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ  
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201  
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220  
uk@invacare.com / www.invacare.co.uk