

AQUATEC®

Ocean Ergo

ACCESSORIES AND ADJUSTMENTS



Yes, you can.®

Introduction

This useful guide explains how the Ocean Ergo Family can be customised to suit a wide range of physical conditions by making simple adjustments or by adding some easy to fit, off-the-shelf accessories.

Meet the Ocean Ergo Family...



Ocean 24'' Ergo XL ▲

Plus-size self-propelling model

Ocean Ergo XL ▲

Plus-size attendant propelled model

Ocean Ergo ▲

Attendant propelled model

Ocean 24'' Ergo ▲

Self-propelling model

Accessories

Seating and backrests ▼



Standard soft seat

- ▶ With key shaped cut out.



Soft seat insert

- ▶ Can be used with or without the soft seat when commode function is not required.



Small soft seat

- ▶ With small, oval opening for smaller and young adults.



Small soft seat insert

- ▶ Can be used with the small soft seat when the commode function is not required.



Ergonomic soft seat

- ▶ Extra soft, textured surface for pelvic stability.



Variable soft seat

- ▶ Hygiene cut out can be positioned in all 4 directions.



Universal soft seat

- ▶ Comfortable, closed seat surface.



XL back

- ▶ Increases the width between the armrests by 80mm. Can be retrofitted to standard Ocean Ergo models.



Solid plastic backrest

- ▶ Offers a stable upright position and is easy to clean.



Wider XL armrests

- ▶ Increases the distance between the armrests by 40 mm on each side.



Soft backrest cushion ▲

- ▶ For additional stability and comfort. Will also benefit smaller users and young adults.



Comfort backrest ▲

- ▶ For additional comfort and easy cleaning.

Positioning ▼



Lateral supports

- ▶ For safe positioning. Width and height adjustable – also for small users and young adults.



Ergonomic headrest holder

- ▶ Safe and comfortable with various adjustment positions (*Pad not included*).



Ergonomic headrest/neckrest

- ▶ Offers exceptional management of the head position (Neckrest shown).



Calf support with foot rest

- ▶ Padded, both height and angle adjustable with additional foot support.

Safety ▼



Safety bar

- ▶ For additional safety and reassurance.



Safety belts Ocean

- ▶ For safe positioning. Individually adjustable, high load capacity and easy to assemble. Machine washable.



Armrest lock

- ▶ Locks armrest into the lower or higher position.



Calf and amputee support

- ▶ Padded, both height and angle adjustable. For use with the standard plastic seat, standard soft seat and small soft seat. (Complete support shown).

Toileting ▼



Splash guard

- ▶ Clips on easily for use with the standard plastic seat, standard soft seat and small soft seat. May also help as a pommel.



Toilet pan

- ▶ Ergonomic handle allows carer to remove pan from holder easily
- ▶ Capacity scale inside the pan
- ▶ Lid holder when lid is not required.

Adjustments

All Ocean Ergo models can be adjusted to suit a range of user sizes.

Armrests

- ▶ Ergonomic, swing away armrests can be adjusted to different heights. Can be adjusted between 215 - 290mm above the seat plate.

Footrests


- ▶ Height adjustable, swing away and detachable footrests
- ▶ No tools required.

Seat Height

- ▶ Seat to ground height can be adjusted between 475 mm - 600 mm
- ▶ No tools required.

Footplates

- ▶ Angle adjustable, flip-up and textured footplates with heel loops
- ▶ No tools required.

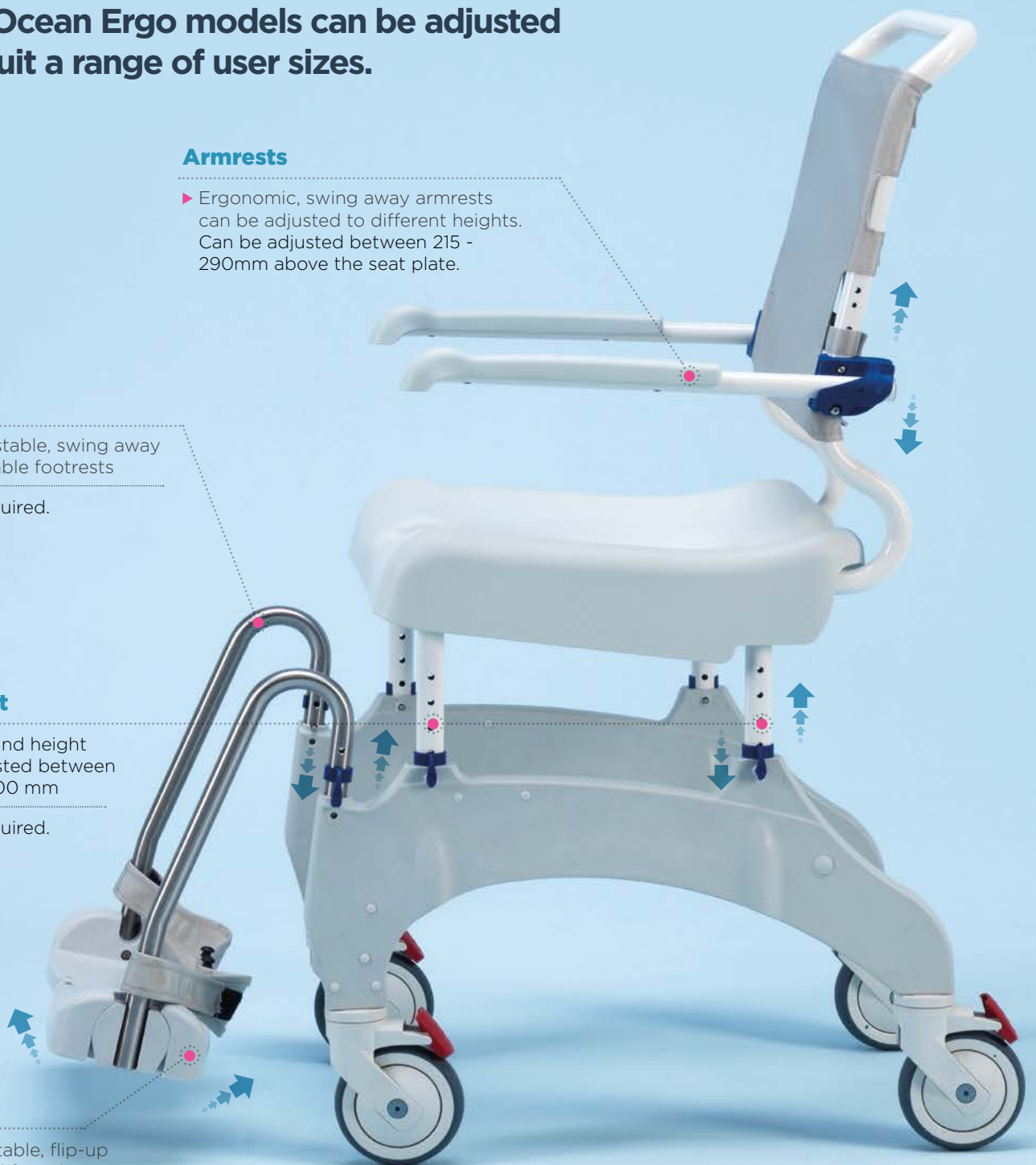

Compatibility

- ✓ - Compatible
- ✗ - Not compatible
- - Included

Many accessories are also compatible across the Ocean and new Ocean Ergo ranges.*

	Product code	Ocean Ergo	Ocean 24" Ergo	Ocean Ergo XL	Ocean 24" Ergo XL
Standard soft seat	AP1603142	✓	✓	✓	✓
Soft seat insert	AP1603143	✓	✓	✓	✓
Small soft seat	AP1603145	✓	✓	✓	✓
Small soft seat insert	AP1603152	✓	✓	✓	✓
Splash guard	AP1603155	✓	✓	✓	✓
Universal soft seat	AP1603154	✓	✓	✓	✓
Ergonomic special soft seat	1531259	✓	✓	✓	✓
Variable soft seat	1544469	✓	✓	✓	✓
Safety bar	10167-10	✓	✓	✗	✗
Safety bar XL	1470078	✗	✗	✓	✓
Chest belt	1470081	✓	✓	✓	✓
Pelvic belt	1470082	✓	✓	✓	✓
Armrest locking device	1470077	✓	✓	✓	✓
XL back	AP1604015	✓	✓	-	-
Fixed backrest	1575299	✓	✓	✗	✗
Soft backrest cushion	1558769	✓	✓	✓	✓
Wider XL armrests	1470086	✓	✓	✗	✗
24" wheels (set)	16379	✓	-	✗	-
Calf and amputee support right	1556893	✓	✓	✓	✓
Calf and amputee support left	1556894	✓	✓	✓	✓
Calf support with foot support right	1546439	✓	✓	✓	✓
Calf support with foot support left	1546705	✓	✓	✓	✓
Lateral	1535077	✓	✓	✓	✓
Ergonomic headrest holder	1531613	✓	✓	✓	✓
Ergonomic headrest	1531465	✓	✓	✓	✓
Ergonomic neckrest	1531608	✓	✓	✓	✓
Toilet pan	10230	✓	✓	✓	✓
Toilet pan holder	AP1602078	✓	✓	✓	✓
Front anti-tipper Ocean Ergo models	AP1601986	✓	✓	✓	✓
Comfort soft backrest Ocean Ergo models	AP1604136	✓	✓	✗	✗
Comfort soft backrest Ocean Ergo XL models	AP1604016	✗	✗	✓	✓

* For more information on cross compatibility across the Ocean ranges, please contact safepatienthandling@invacare.com.

Invacare Limited
Pencoed Technology Park - Pencoed
Bridgend CF35 5AQ - United Kingdom
Tel: +44 1 656 776 222
Fax: +44 1656 776 220
E-mail: uk@invacare.com
Sales Order E-mail: ordersuk@invacare.com
www.invacare.co.uk

Invacare Ireland Ltd
Unit 5 - Seatown Business Campus
Seatown Road - SWORDS
County Dublin - Ireland
Tel. +353 1 8107084
Fax +353 1 8107085
Email: Ireland@invacare.com
www.invacare.ie



Safe Patient Handling